

Aesthetic Innovation

JetCare by TavTech Solutions & Treatment Guidelines

Presentation by: Laura Masini, International Trainer, TavTech

January 2022



After a full year of research and development
TavTech renewed and upgraded the entire cosmetic
line under its brand: **JetCare by TavTech**

Formulas improved and upgraded with the latest
cutting-edge materials and ingredients

Formulas carefully crafted with ingredients
sourced from the **highest quality cosmetic
suppliers**

JetCare solutions are the only solutions
sanctioned and **approved by TavTech** for use
with its Jet Pressure energy devices and unique
JetPeel handpieces



Following the upgrade of the JetPeel device to JetPro, there was an important need to use the latest biochemical and dermatological research and the latest developments in active cosmetic ingredients **in order to upgrade JetPeel's line of cosmetics**

Formulation guidelines included:

- **Optimization of trans-epidermal** transport by adjusting **concentrations and viscosity to the treatment goal**: which skin layer should be reached and where should the active ingredients achieve a sustained effect
- **Optimization and reformulation** of the active ingredients based on the latest research and the most advanced supply sources
- **Reduction of frictional resistance in the handpieces** resulting in a constant flow and an increase in the efficacy of the handpiece

JetPeel by TavTech – the perfect trio combining:

- the enduring **JetPro device**
- the original patented **JetPeel handpieces**
- the new and improved **JetCare solutions**

designed exclusively for use together to deliver outstanding results



TRIPLE



DOUBLE



MAGIC



Welcome to the next level of JetPeel Treatment

New Treatment Guidelines


1. We included in the treatment guidelines different **skin types** (normal, dry, oily and sensitive) with varied needs

2. We have guidelines for **young and mature skin**

3. We have **scalp guidelines** to treat **itchy, dry, oily** and **flaky scalp**

We have almost **40 treatment guidelines!**

JetCare by TavTech: specially suited to different skin ages and types

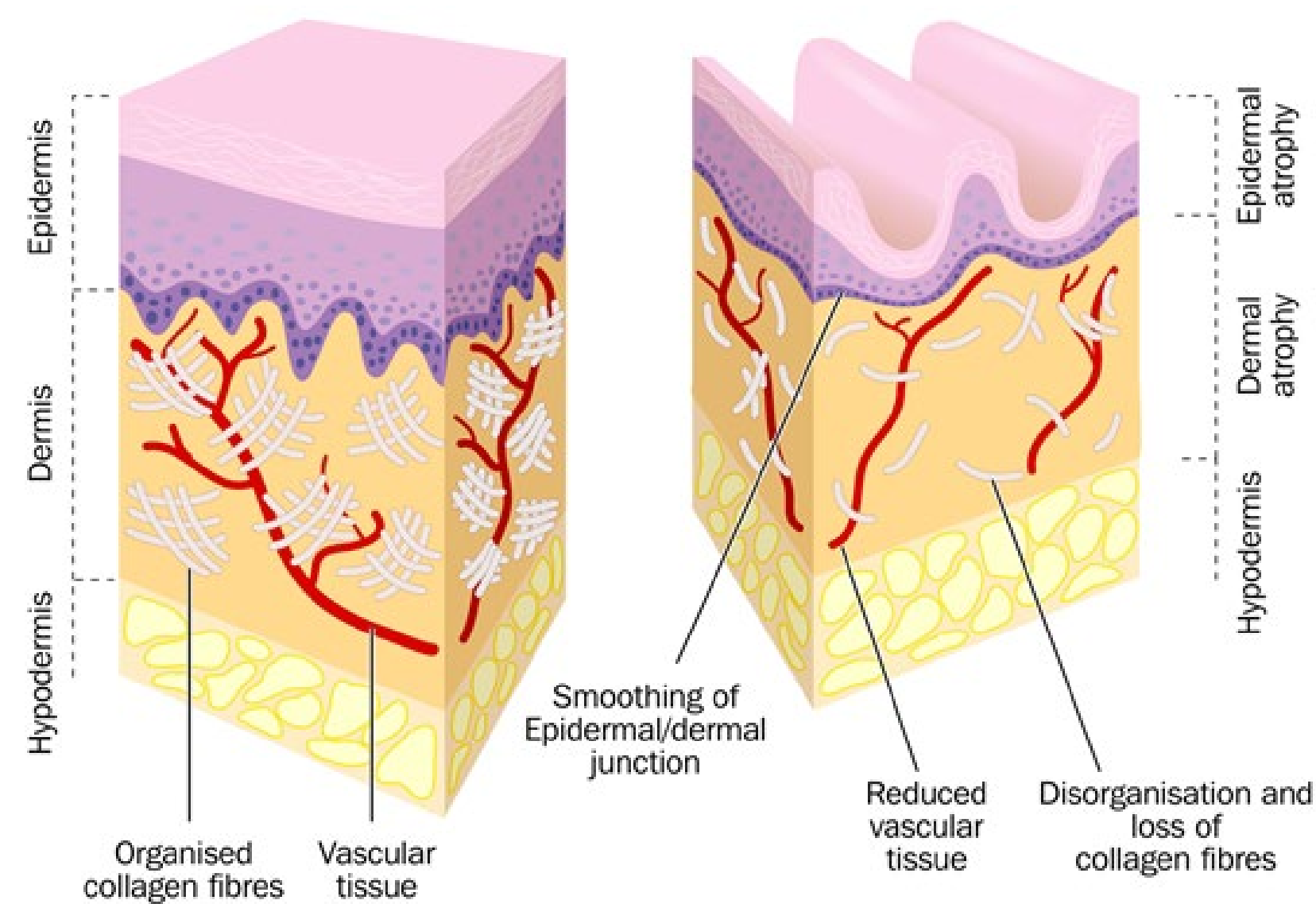

Skin Age

	 <p>Normal Skin</p>	 <p>Dry Skin</p>	 <p>Oily Skin</p>	 <p>Sensitive Skin</p>	 <p>Combination Skin</p>
Appearance	<ul style="list-style-type: none"> • Radiant complexion • No imperfections • Barely visible pores • Soft apperance • Not too oily, no too dry 	<ul style="list-style-type: none"> • Dull, rough complexion • Red patches • Desquamation • Cracked skin • Less elasticity 	<ul style="list-style-type: none"> • Shini or dull skin • Humid and bright apperance 	<ul style="list-style-type: none"> • Redness (itching, burning) • Fragile • Dryness • Feeel discomfort • Loose the skin barrier protection 	<ul style="list-style-type: none"> • Both dry and oily skin appreance • Oily (forehead, nose and chin) • Dry on the cheeks
Texture	<ul style="list-style-type: none"> • Barely visible pores 	<ul style="list-style-type: none"> • Almost invisible pores • More visible lines 	<ul style="list-style-type: none"> • Enlarge pores • Blackheads, pimples or other blemishes 	<ul style="list-style-type: none"> • Almost invisible pores • Dryness • Peeling 	<ul style="list-style-type: none"> • Pores that look larger than normal, blackheads are visible
Secretion	<ul style="list-style-type: none"> • Perfect balance of oil and moisture 	<ul style="list-style-type: none"> • Minimal sebaceous secretion 	<ul style="list-style-type: none"> • Overactive sebaceous glands 	<ul style="list-style-type: none"> Minimal sebaceous secretion 	<ul style="list-style-type: none"> On the T zone more production of sebum

Skin Types

Anti-Aging Treatments by Skin Characteristics

ANTI-AGING - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost - Glow	Treatment Guideline 13 with JetCare Boost - Regenerate	Treatment Guideline 4 with JetCare Boost - Tight
Dry	Treatment Guideline 4 with JetCare Boost - Glow	Treatment Guideline 13 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight
Oily	Treatment Guideline 4 with JetCare Boost - Glow	Treatment Guideline 14 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight
Sensitive	Treatment Guideline 4 with JetCare Boost - Glow	Treatment Guideline 13 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight

ANTI-AGING – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost - Glow	Treatment Guideline 15 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight
Dry	Treatment Guideline 4 with JetCare Boost - Glow	Treatment Guideline 14 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight
Oily	Treatment Guideline 2 with JetCare Boost - Glow	Treatment Guideline 15 with JetCare Boost - Regenerate	Treatment Guideline 2 with JetCare Boost - Tight
Sensitive	Treatment Guideline 4 with JetCare Boost - Glow	Treatment Guideline 14 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight







4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost - Hydrate	Treatment Guideline 8 with JetCare Boost - Glow	Treatment Guideline 9 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight
Treatment Guideline 16 with JetCare Boost - Hydrate	Treatment Guideline 8 with JetCare Boost - Glow	Treatment Guideline 29 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight
Treatment Guideline 14 with JetCare Boost - Hydrate	Treatment Guideline 6 with JetCare Boost - Glow	Treatment Guideline 10 with JetCare Boost - Regenerate	Treatment Guideline 2 with JetCare Boost - Tight
Treatment Guideline 16 with JetCare Boost - Hydrate	Treatment Guideline 8 with JetCare Boost - Glow	Treatment Guideline 29 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 10 with JetCare Boost - Hydrate	Treatment Guideline 29 with JetCare Boost - Glow	Treatment Guideline 6 with JetCare Boost - Regenerate	Treatment Guideline 2 with JetCare Boost - Tight
Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost - Glow	Treatment Guideline 6 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight
Treatment Guideline 10 with JetCare Boost - Hydrate	Treatment Guideline 29 with JetCare Boost - Glow	Treatment Guideline 7 with JetCare Boost - Regenerate	Treatment Guideline 2 with JetCare Boost - Tight
Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost - Glow	Treatment Guideline 8 with JetCare Boost - Regenerate	Treatment Guideline 1 with JetCare Boost - Tight

Treatment Guideline No. 1

Indications: Anti-Aging
Loss of Tone & Wrinkles
Acne
Post-Acne
Couperose & Sensitive Skin
Lightening & Pigmentation







Glycolic Acid 5% + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 5%		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight/Regenerate/Glow	
 Remarks			Leave on the skin for 5-8 minutes and then wash with ~8ml of JetCare Hydro. Triple handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 2

Indications: Anti-Aging
Loss of Tone & Wrinkles
Acne
Post-Acne
Couperose & Sensitive Skin
Lightening & Pigmentation







Glycolic Acid 10% + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight/Regenerate/Glow	
 Remarks			Leave on the skin for 5-8 minutes and then wash with ~8ml of JetCare Hydro. Triple handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 3

Indications: Lightening & Pigmentation







Glycolic Acid 16% + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight	
 Remarks			Leave on the skin for 5-8 minutes and then wash with ~8ml of JetCare Hydro. Triple handpiece at 45°- 60°					

Treatment Guideline No. 4

Indications: Anti-Aging
Loss of Tone & Wrinkles
Couperose & Sensitive Skin
Lightening & Pigmentation
Acne

Mandelic & Salicylic + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Mandelic & Salicylic		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight/Regenerate/Glow	
 Remarks			Wash with ~8ml of JetCare Hydro if necessary. Triple handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

The Treatment Sequence

STEP
01

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
02

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
03


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP
04

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

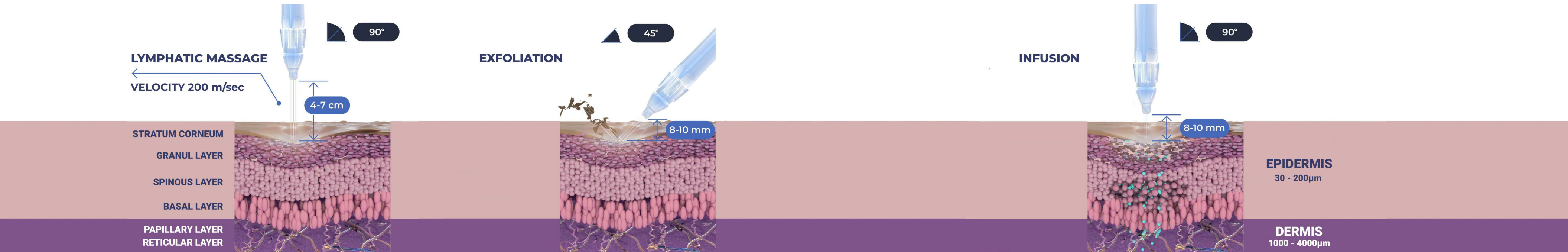

JetCareHYDRO

(Replaces Jet Detox Water)

- All JetPeel sessions begin with JetCare Hydro
- Gentle solution formulated for the initial lymphatic massage, and to prep the skin for the deeper JetCare treatments
- **Hyaluronic Acid concentration reformulated, significantly higher** than in former version
- Enriched with **higher concentrations of organic concentrated Aloe Vera**
- Solid plastic bottle, won't puncture during transportation
- **300 ml - 20% more than in former version**



Renewal Care Series

- The ultimate exfoliating treatment line enriched with **AHA's and BHA's**
- The Renewal Care Series is adjusted to **pH of 3.5**, the ideal range for rejuvenating the skin, activating skin enzymes and barrier peptides, and for balancing the Natural Moisturizing Factor (NMF)



Glycolic Acid Series

(Replaces Renewal Complex 1, 2, 3)

- Personalized exfoliation containing Glycolic Acid, **Organic Aloe Vera** and **Pro-Vitamin B₅ Panthenol**
- Available in 3 concentrations:
- **5% Glycolic acid** – for delicate and normal skin
- **10% Glycolic acid** – for normal to oily skin
- **16% Glycolic acid** – for mature skin



Mandelic & Salicylic Series

(Replaces Renewal Complex 4)

- The gentlest way to achieve a deep exfoliating and resurfacing effect
- Solution enriched with the ideal **blend of AHA & BHA** for maximum benefit
- **5% Mandelic – 2% Salicylic – 1% Malic – 0.7% Glycolic – 0.7% Latic**



Anti Aging Care Series



Wide Range Hyaluronic Acid

(Replaces Anti-Aging Complex 1)

- Unique wide-range complex containing **five molecular weights of HA** that enable simultaneous infusion into different layers of the skin
- **Low-** and **medium-weight HA molecules** can go deeper into the skin, increasing hydration over time
- **High-weight HA molecules** lie on the surface level, for instant hydration
- Combined with **Regu-age®**, a **synergistic blend of patented peptides and proteins** that demonstrates fast action anti-aging benefits. Containing purified soy and rice peptide, blended with bio-tech produced yeast protein, helps to boost tone and radiance





Peptide Complex

(Replaces Anti-Aging Complex 2)

- **Formulated for skin showing signs of aging** and loss of elasticity
- **Concentrated amino-rich peptide blend** helps restore firmness and a youthful appearance

Reinforced with:

Replexium™ - a fusion of two patented skin bioavailable peptides and proteins. Reduce wrinkles and improves dermal density.

Juveleven™ - a powerful skin protecting hexapeptide.

Pentavitin® - 100% natural, plant- based carbohydrate complex

Regu-age® - synergistic blend of patented purified soy and rice peptides and biotech produced yeast protein



Witch Hazel & Peptides

(Replaces Anti-Aging Complex 3)

- Purifying solution for radiant complexion
- Selected peptides blend **helps to fortify the skin's firmness and elasticity**
- Richly concentrated with **Witch Hazel extract**, a strong astringent for all skin types and powerful natural calming agent

Reinforced with:

Regu-age® - synergistic blend of patented purified soy and rice peptides and biotech produced yeast

Replexium™ - a fusion of two patented skin bioavailable peptides and proteins

Betaine – a natural humectant, helps retain hydration



Selective Care Series



Growth Factor

- **New addition** to the JetCare line designed to meet the biochemistry of the skin
- **Advanced skin rejuvenation technology** with new active ingredients adapted to galenics
- Harnesses the **power of biotechnologically produced 3% EGF** (Epidermal Growth Factor) modern technology made with 5 different growth factors + **Vitamin B9, Niacinamide** and **Betaine**
- Powerful skin firming and anti-aging agent



**NEW
PRODUCT**



Argireline® Complex

(Replaces ATX-Nat)

- **5% Argireline®** is a hexapeptide that reduces fine wrinkles and lines
- Contains Argireline® peptide solution, resulting in significantly more acetyl hexapeptide than former version
- Combined with **Vitamin B5** and a **triple molecular weight Hyaluronic Acid** and **Betaine**.
- Extremely effective anti-aging treatment



Lightening

(Replaces Whitening Complex 1)

- The key to achieving the appearance of brighter skin and an even toned complexion
- **2% Contains alpha-arbutin** - a natural derivative of hydroquinone


Niacinamide (Vit B3)

1% Beta-white™

1% Salicylic acid

Stay C ®



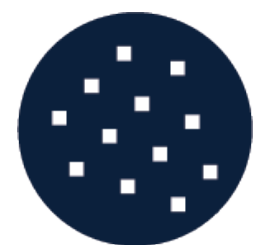

Clear

(Replaces Pure - Acne)

- Specially formulated to achieve clear, even, calm skin
- Rich in **Neutrazen™** three amino acids advance biomimetic soothing action and **Salicylic Acid** to help purify clogged pores
- Fortified with **Niacinamide (Vit B3)**
- Added **Allantoin** (soothing agent)
- **Acnacido-10-HAD** this omaga-hydroxy acid is naturally derived from Royal Jelly.



Soothing

(Replaces Couperose and Sensitive Skin)

- Contains significantly higher concentration of **8% Azeoglicina®** (azelaic+glycine)
- Fortified with **Nordic cotton extract**, **aloe vera** known as a soothing agent.
- **Ronacare® – Troxerutin**, improves the capillary resistance, flight the oxidative stress
- **Organic Aloe**





Scalp & Hair

- For men and women
- Creates the conditions for enhanced fullness and volume
- Formulated with innovative targeted ingredients
- Rich in Biotin Vitamin B7 that helps rejuvenate the scalp

Contains:

Root Biotech™ - deeply nourishing for a fuller, healthy look

Vitamin B combination- niacinamide, panthenol, biotin

Redensyl™ - amino acids and antioxidants for healthier hair

Capixyl™ - a Biomimetic peptide with red clover extract (Biochanin A) which helps create thicker, fuller and healthier hair



Recommended to be used with
NEW DOUBLE Handpiece




JetCare Boost Series

- New and improved booster formulas designed to amplify the power of JetPeel infusions



Jet Boost Hydrate (HA)


- State-of-the-art HA formula for the ultimate filler effect and beautifully smooth skin
- **Maximum concentration of PrimalHyal™ Ultrafiller**, an innovative HA derivative
- **Niacinamide** and **Betaine**

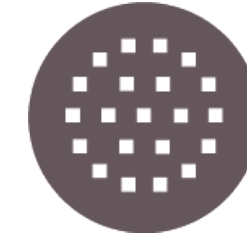

Regenerate (Vit A + E)


- Serum booster enriched with **vitamins A and E for high performance anti-aging action**
- Specially formulated to regenerate the skin with the antioxidant power
- **Regu-age®** and **Roman Chamomile Extract**



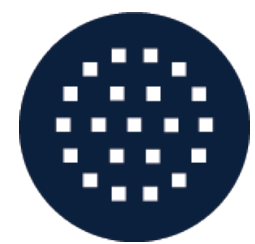

Jet Boost Glow (Vit C)

- Excellent booster for a beautiful glow effect
- Updated formula contains stable vitamin C derivative - this is clearly seen by its yellow color
- Enriched with ACB fruit mix, natural extract of vegetative sources of AHA
- Stable Vitamin C
- Natural fruit mix extract – source of AHAs

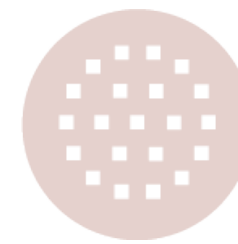

Tight (Vit B Complex)

- Power booster delivers effective tightening and hydrating action
- **Complex of 3 vitamin B derivatives - B3, B5, B7**
- Synergetic effect for a wide range of skin rejuvenation processes



JetCare Boost Set

- Set of 4 JetCare Boost varieties, for every skin type and need
- Amplify JetPeel sessions with the perfect finish:



JetCare Boost Hydrate (HA)

State-of-the-art HA for ultimate filler and volumizing effect




JetCare Boost Regenerate (Vit A + E)

Anti-oxidant power of Vitamins A and E for high-performance anti-aging



JetCare Boost Glow (Vit C)

Delivers the glow benefits of Vitamin C directly to the skin

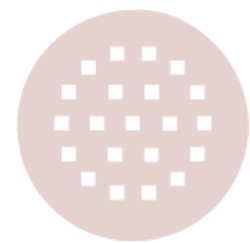

JetCare Boost Tight (Vit B complex)

The firming power of Vitamins B3, B5, and B7



- A solid research & development platform for on-going JetCare innovation
 - continually to be upgraded
 - always based on new research
- The new improved JetCare solutions guarantee treatment success and customer satisfaction due to the calibration, coherence and correlation between the device, handpieces and reagents

A new world of
JetCare awaits...

Thank you

