



Skin Age Reversal

Presentation by: Laura Masini, International Trainer, TavTech

April 2022



Humans have two different ages



CHRONOLOGICAL AGE


The actual age of a person. The age that is listed in your passport.

BIOLOGICAL AGE

It is the age of our cells. It tells our real age depending on how the aging process has affected us.

- **Chronological Age** which is the number of years a person has been alive
- **Biological Age** which is how old a person seems.
- When we talk about **reversing aging**, it is the **biological age** that we want to slow down and improve on.

Causes of cellular aging

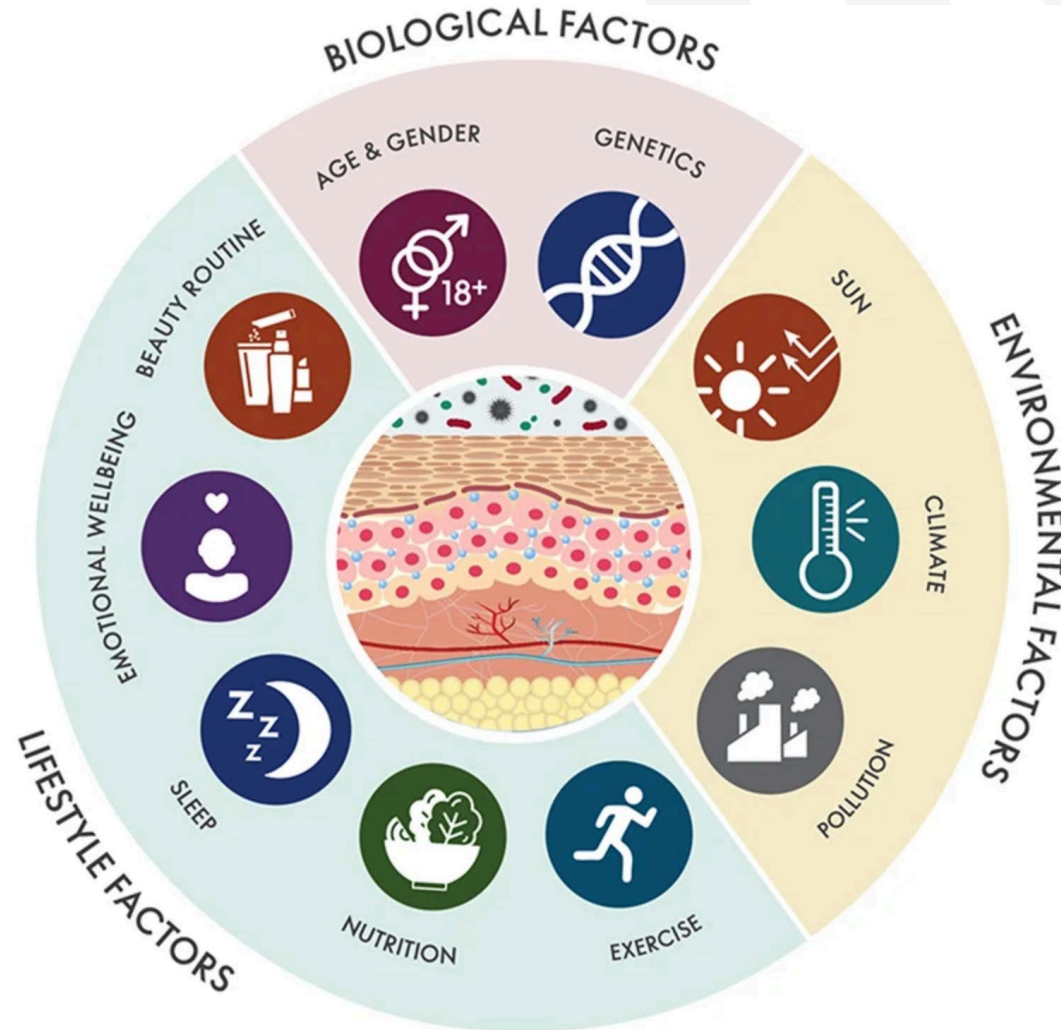

Different lifestyle and environmental factors such as diet, sleep, exercise, smoking, and drinking alcohol can also affect the cells aging.

Signs of ageing skin



- **Thinning** – the epidermis and dermis become thinner
- **Sagging** – the skin produces less elastin and collagen
- **Wrinkles** – reduced elastin and collagen
- **Age spots** – pigment cells (melanocytes) tend to increase in number in certain areas
- **Dryness** – the skin more prone to dryness
- **Broken blood vessels**– blood vessels are more likely to break and bruise



The effect of aging on facial attractiveness

We cannot deny that attractiveness is still a very powerful tool.

This is why skin rejuvenation aesthetic procedures are the most requested treatments worldwide.

People want to look and feel better.

Age-Less

Maintain your skin healthy for longer



Reversal of time effects is possible when you:

- improve cell regeneration
- stimulate collagen production
- improve skin tone

This will prevent wrinkles, uneven skin tone, dry skin, age spots, rough skin texture, visible pores, loss of tone, and saggy skin.

Growth Factor Can Reverse Skin Age



Growth Factor from JetCare by TavTech is the latest BioTech formulation developed for skin rejuvenation.

It is based on advanced technology, and fights the effects of time, reverses skin age, enhances skin elasticity and contributes to youthful skin appearance.

Aging is caused by loss of Growth Factors (GF)

Our skin produces lower quantities of growth factors, starting around 18 years old.




Fetus

Fetus obtains all the nutrients and GF made by placenta. GF will stimulate the expansion of cells in the body system.


0~3 years old

Infants produce nutritional values and GF on their own.

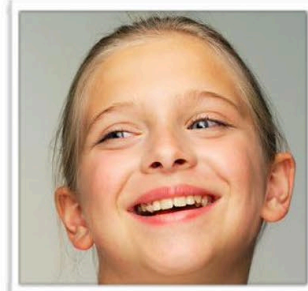

10~13 years old

GF production decreases in body and growth rate starts to slow down.

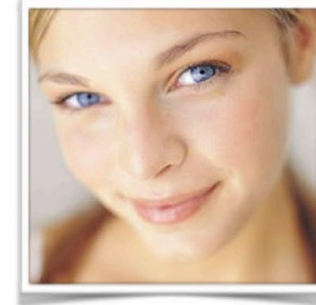

More than 18 years old

GF production and the growth process decreases rapidly. Minimal amount is produced for metabolism.



GF decrease and absent of cell replacement leads to aging.

Aging is caused by loss of Growth Factors(GF)

The next frontier in
the anti-aging treatment



What is Growth Factor for skin?

GF's (Growth Factors) are natural substances produced by the skin's cells.

Growth Factors

- maintain the skin healthy and strong
- repair damaged skin
- give the skin firmness and elasticity
- protect the skin functions
- regulate cellular proliferation and regeneration

JetPeel Age Reverse



JetPeel Treatment:

- improves cell renewal
- stimulates collagen and elastin production
- counteracts free-radicals
- stimulates fibroblasts
- deeply hydrates and nourishes the skin
- improves micro-circulation
- detoxifies the skin
- stimulates cell epithelization
- adds to skin plump and glow





JetPeel Treatments Guidelines


Skin Age Reverse



The Treatment Sequence

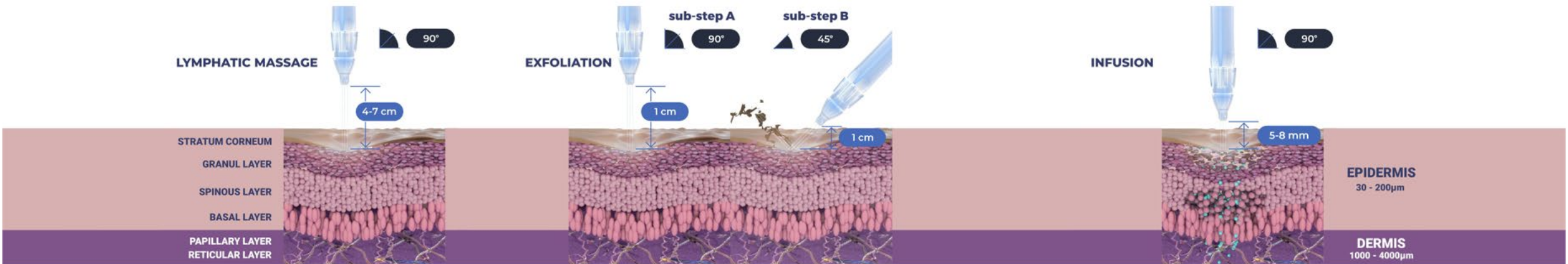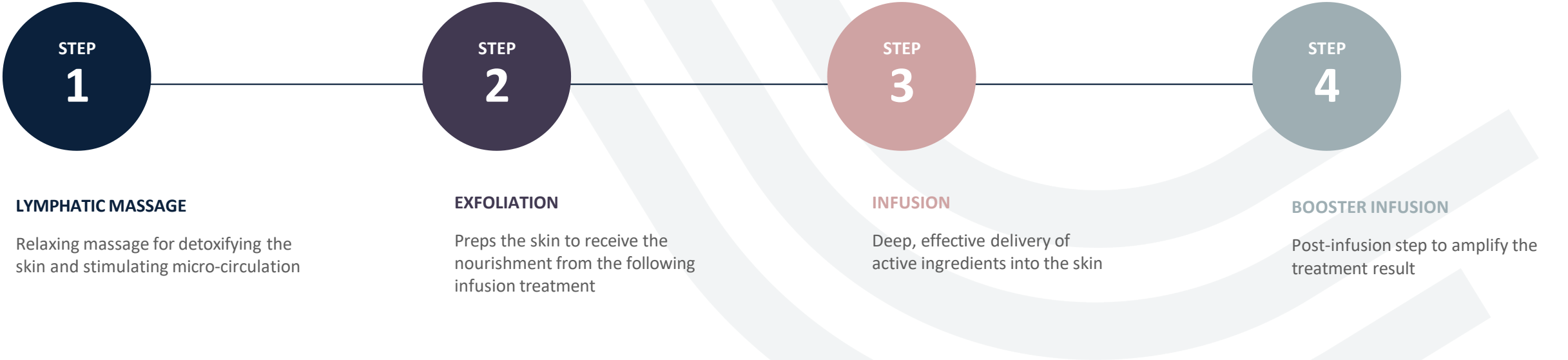

Step 1: Lymphatic Massage

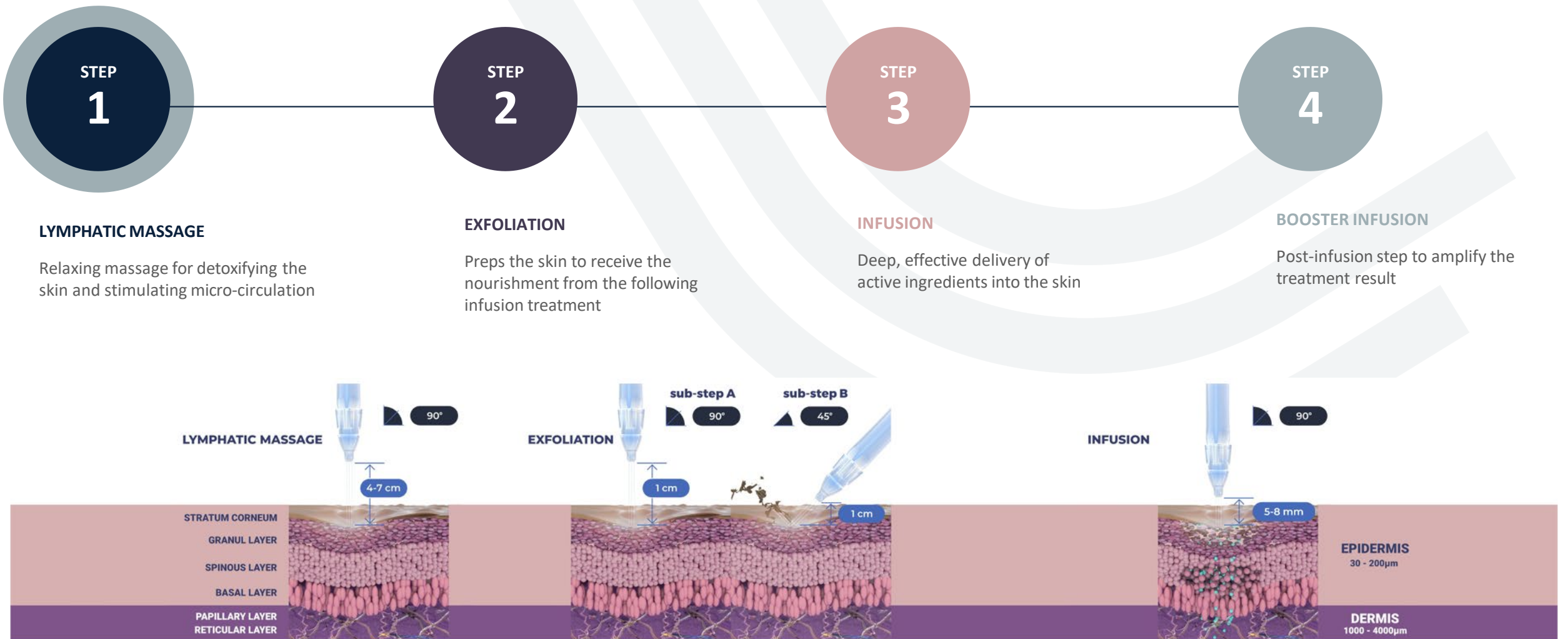



JetCareHYDRO

(Replaces Jet Detox Water)

- All JetPeel sessions begin with JetCare Hydro
- Gentle solution formulated for the initial lymphatic massage, and to prep the skin for the deeper JetCare treatments
- Hyaluronic Acid concentration reformulated, significantly higher than in former version
- Enriched with higher concentrations of organic concentrated Aloe Vera
- Solid plastic bottle, won't puncture during transportation
- 300 ml - 20% more than in former version



Step 2: Exfoliation

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP
4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

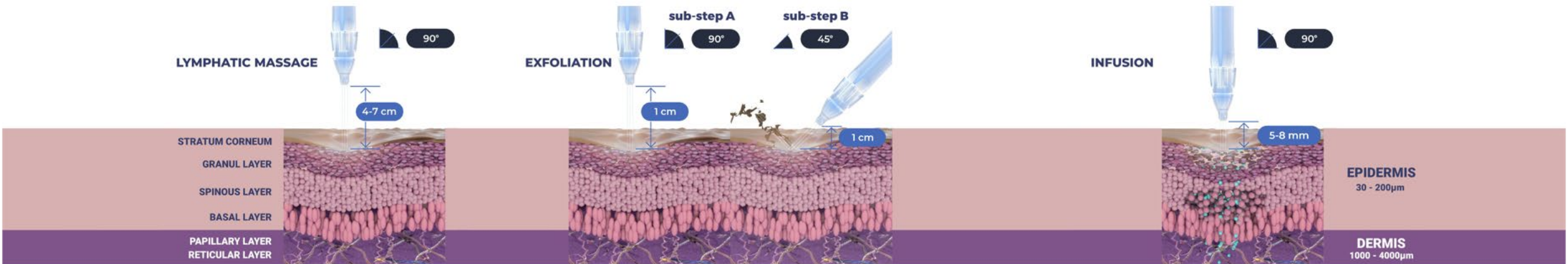

Renewal Care Series

- The ultimate exfoliating treatment line enriched with AHA's and BHA's
- The Renewal Care Series is adjusted to pH of 3.5, the ideal range for rejuvenating the skin, activating skin enzymes and barrier peptides, and for balancing the Natural Moisturizing Factor (NMF)



Glycolic Acid Series

(Replaces Renewal Complex 1, 2, 3)

- Personalized exfoliation containing Glycolic Acid, Organic Aloe Vera and Pro-Vitamin B5 Panthenol
- Available in 3 concentrations:
 - 5% Glycolic acid – for delicate and normal skin
 - 10% Glycolic acid – for normal to oily skin
 - 16% Glycolic acid – for mature skin



Mandelic & Salicylic Series

(Replaces Renewal Complex 4)

- The gentlest way to achieve a deep exfoliating and resurfacing effect
- Solution enriched with the ideal blend of AHA & BHA for maximum benefit



Step 3: Infusion



LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation



EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment




INFUSION

Deep, effective delivery of active ingredients into the skin



BOOSTER INFUSION

Post-infusion step to amplify the treatment result

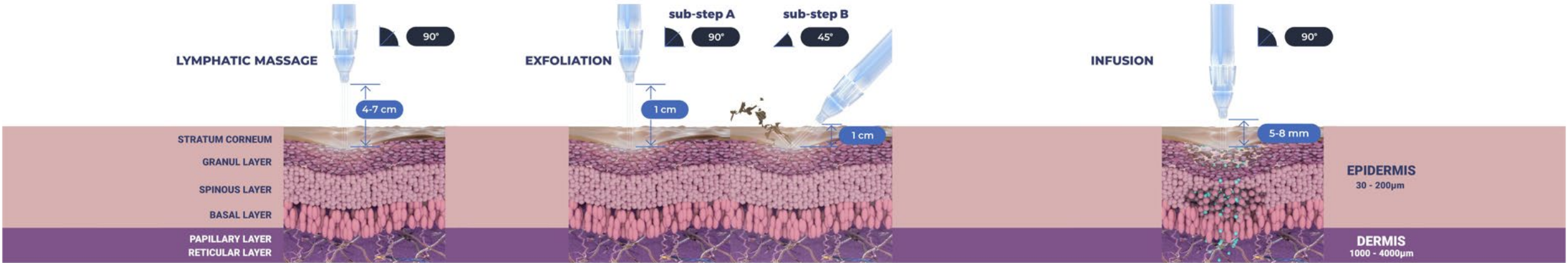

Selective Care Series



Growth Factor

- **New addition** to the JetCare line designed to meet the biochemistry of the skin
- **Advanced skin rejuvenation technology with new active ingredients adapted to galenics**
- **Harnesses the power of biotechnologically produced EGF (Epidermal Growth Factor) complex**
- **Optimal active ingredient concentrations formulated to get into the skin layer in which they are most effective**
- **Powerful skin firming and anti-aging agent**



**NEW
PRODUCT**

Growth Factor Ingredients

BIO PLACENTA	EFFECT
EGF – Epidermal Growth Factor	Skin rejuvenation Wound healing effect, growth differentiation and migration of cells
IGF-1 Insulin-like Growth Factor	Anti-wrinkles Restoration on phospholipid, wound healing and growth stimulation of fibroblast cell
Acidic FGF – Fibroblast Growth Factor	Anti-wrinkles Stimulation of skin regeneration, stimulation of collagen and elastin, wound healing
Basic FGF - Fibroblast Growth Factor	Elasticity enhance
VEGF – Vascular Endothelial Growth Factor	Nutritional supplement Induction of nutrient supplements to fibroblast cells, growth and migration of cells, wound healing effect
Vitamin B₉	Cell vitalization
Amino acids: Acetyl Glutamine	Energy supplement
Niacinamide	Antioxidants, renew and protect the skin
Betaine	Enhance skin texture, anti-aging

Step 4: Booster Infusion



LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation



EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment




INFUSION

Deep, effective delivery of active ingredients into the skin



BOOSTER INFUSION

Post-infusion step to amplify the treatment result

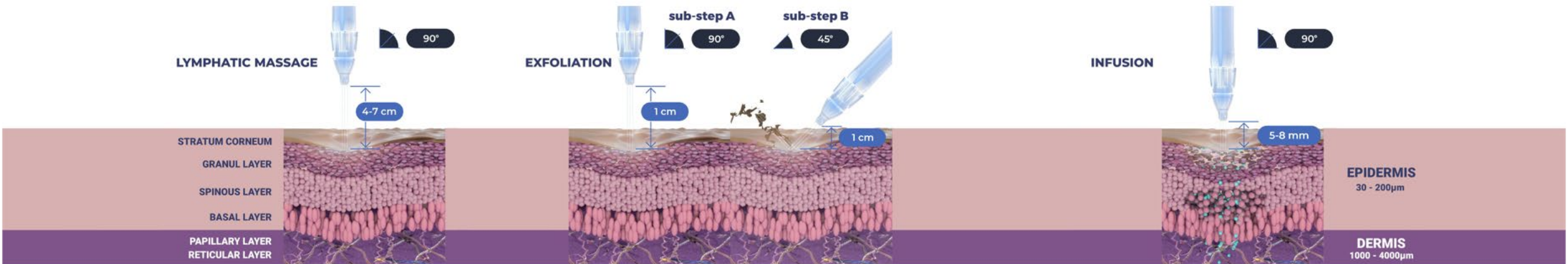

JetCare Boost Set



JetCare Boost Hydrate (HA)

State-of-the-art HA for ultimate filler and volumizing effect



JetCare Boost Regenerate (Vit A + E)

Anti-oxidant power of Vitamins A and E for high-performance anti-aging



JetCare Boost Glow (Vit C)

Delivers the glow benefits of Vitamin C directly to the skin



JetCare Boost Tight (Vit B complex)

The firming power of Vitamins B3, B5, and B7









Growth Factor Treatment Guidelines

Treatment Guideline No. 29

Indications: Anti-Aging
Loss of Tone & Wrinkles Acne
Post-Acne Couperose & Sensitive Skin
Lightening & Pigmentation







Glycolic Acid 5% + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 5%		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost: Regenerate / Glow / Hydrate / Tight	
 Remarks			Leave on the skin for 5-8 minutes and then wash with ~8ml of JetCare Hydro. Triple handpiece at 45° - 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 30

Indications: Acne Post-Acne







Glycolic Acid 10% + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost: Regenerate / Tight	
 Remarks			Leave on the skin for 5-8 minutes and then wash with ~8ml of JetCare Hydro. Triple handpiece at 45° - 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 31

Indications: Acne Lightening & Pigmentation







Glycolic Acid 16% + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost: Hydrate / Tight	
 Remarks			Leave on the skin for 5-8 minutes and then wash with ~8ml of JetCare Hydro. Triple handpiece at 45° - 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 32

Indications: Acne

Mandelic & Salicylic + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	Triple		Triple		Triple or Magic		Triple or Magic	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Mandelic & Salicylic		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost:Tight	
 Remarks			Wash with ~8ml of JetCare Hydro if necessary. Triple handpiece at 45° - 60°					

JetPeel by TavTech – the perfect trio combining:

- the enduring JetPro device,
- the original patented JetPeel handpieces, and
- the new and improved JetCare solutions

designed exclusively for use together to deliver outstanding results



TripleJet




DoubleJet



SingleJet
(Magic)



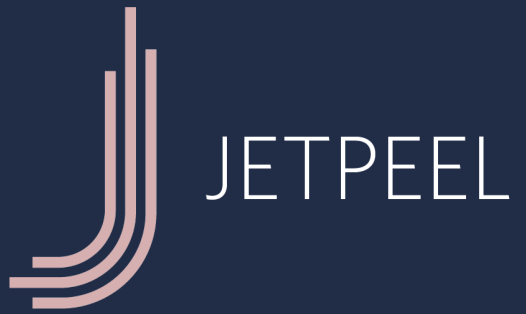

Thank you

