

JetPeel by TavTech Academy



We provide people with an
opportunity to become the best
version of themselves

**We believe in
making people
feel good ...**



... by delivering
a powerful,
instant-result
treatment

Suitable for all skin types
and all year-round





JetPeel by TavTech Training Program

1. JetPeel Technology and Solutions: Training Introduction
2. Lymphatic Massage with JetPeel
 - 2.1. Lymphatic Massage with JetPeel – Face & Neck
 - 2.2. Lymphatic Massage with JetPeel – Body
3. Exfoliation with JetPeel
 - 3.1. Exfoliation with JetPeel – Face & Neck
4. Infusion with JetPeel
 - 4.1. Infusion with JetPeel – Face & Neck
5. Scalp & Hair with JetPeel
6. JetPeel Handpieces






2.1. Lymphatic Massage with JetPeel Face & Neck



The versatility of JetPeel: Face and Body

Unique jet pressure energy

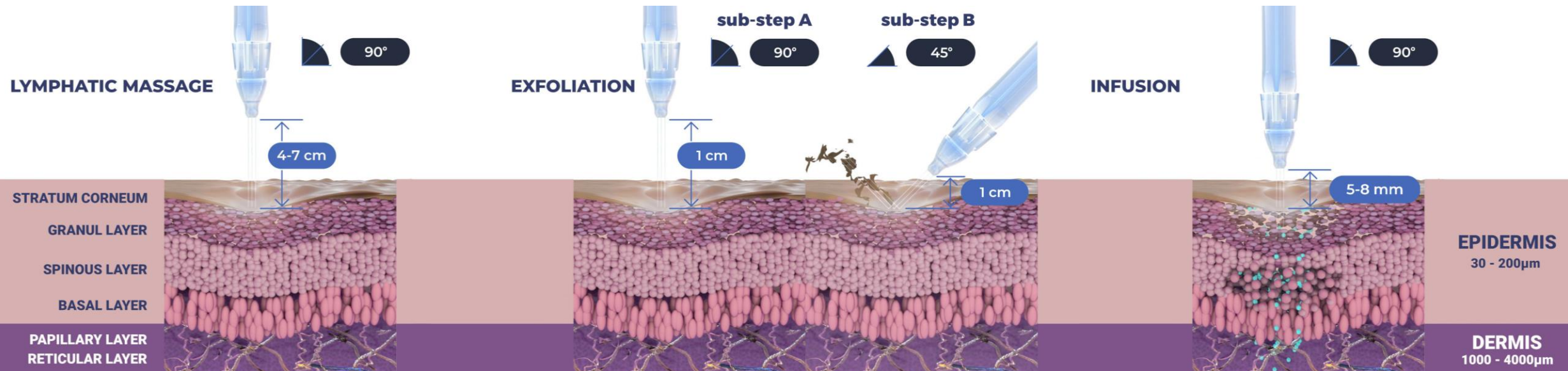

The treatment sequence



LYMPHATIC MASSAGE

A pleasant and relaxing massage for detoxifying the skin and stimulating micro-circulation



GENTLE EXFOLIATION

Removing dead skin cells prepares the skin to receive the nourishment from the following Infusion Step



EFFECTIVE INFUSION

This step provides effective and deep delivery of active ingredients into the skin



BOOSTER INFUSION

Post-infusion step to amplify treatment result

The treatment sequence



LYMPHATIC MASSAGE

A pleasant and relaxing massage for detoxifying the skin and stimulating micro-circulation



GENTLE EXFOLIATION

Removing dead skin cells prepares the skin to receive the nourishment from the following Infusion Step



EFFECTIVE INFUSION

This step provides effective and deep delivery of active ingredients into the skin



BOOSTER INFUSION

Post-infusion step to amplify treatment result

Lymphatic massage with JetPeel

- Lymphatic massage can be easily performed to improve circulation, soothing and hydration.
- The energy-pressure derived jet stream of microdroplets applied by the non-contact handpiece delivers nutrients to the skin.
- The special solution used for the lymphatic massage, JetCare Hydro contains: organic Aloe Vera, low molecular Hyaluronic Acid and Saline.
- Lymphatic massage is a therapeutic jet massage treatment. The massage uses gentle pressure and long, gentle, rhythmic strokes to increase the flow of lymph and reduce toxins in your body. The lymph system is part of your body's immune system and helps fight infection.
- Lymphatic massage can improve the skin texture by reducing swelling, puffiness and blotches providing clean, healthy pores.






JETPEEL
by TavTech

Benefits of the cooling effect of the JetPeel lymphatic massage


The cooling jet sensation causes the lymph vessels to contract, forcing the lymphatic system to pump lymph fluids throughout the body, flushing the waste out of the area. It also activates the body's natural healing powers that can relieve the symptoms of many medical conditions and promotes a sense of health and well-being, even providing long-lasting changes to your body's immune, lymphatic, circulatory and digestive systems that enhance the overall quality of your life.



Facial lymph nodes

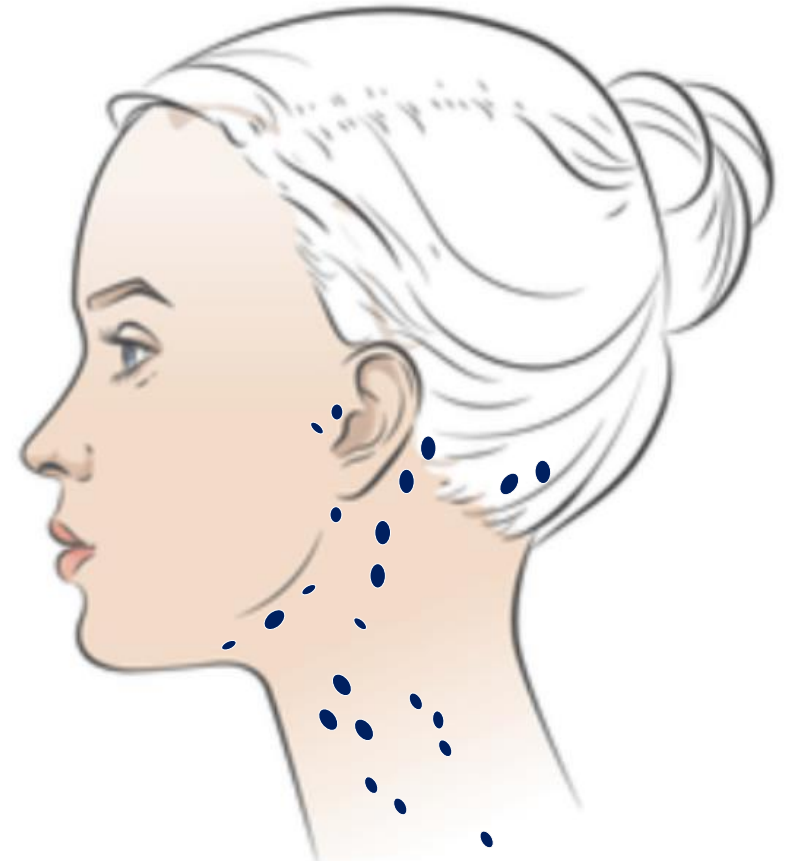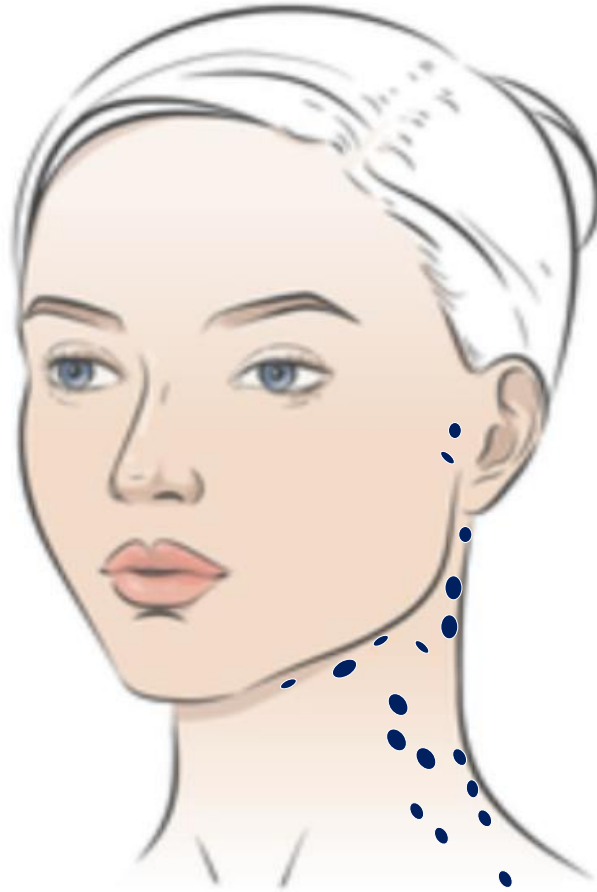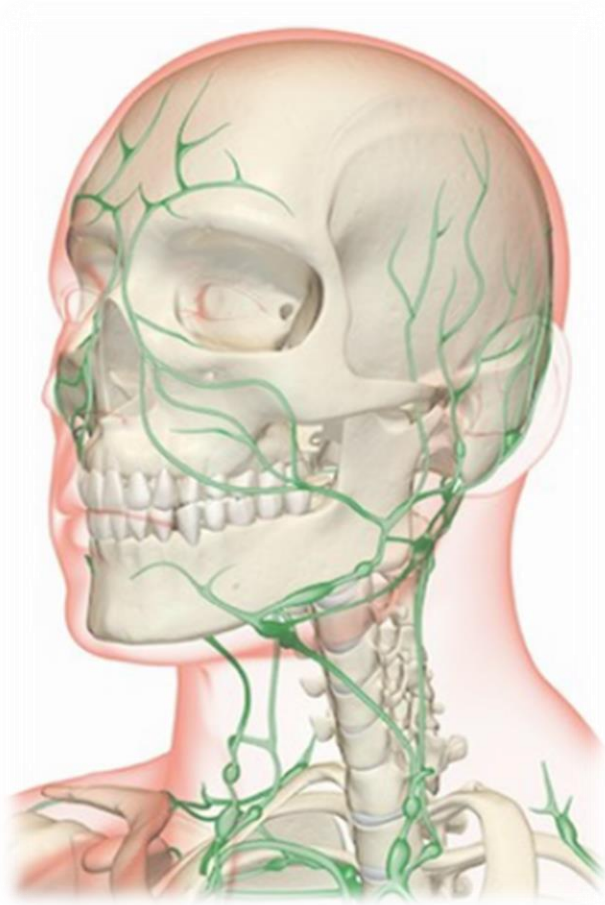

Facial lymph nodes

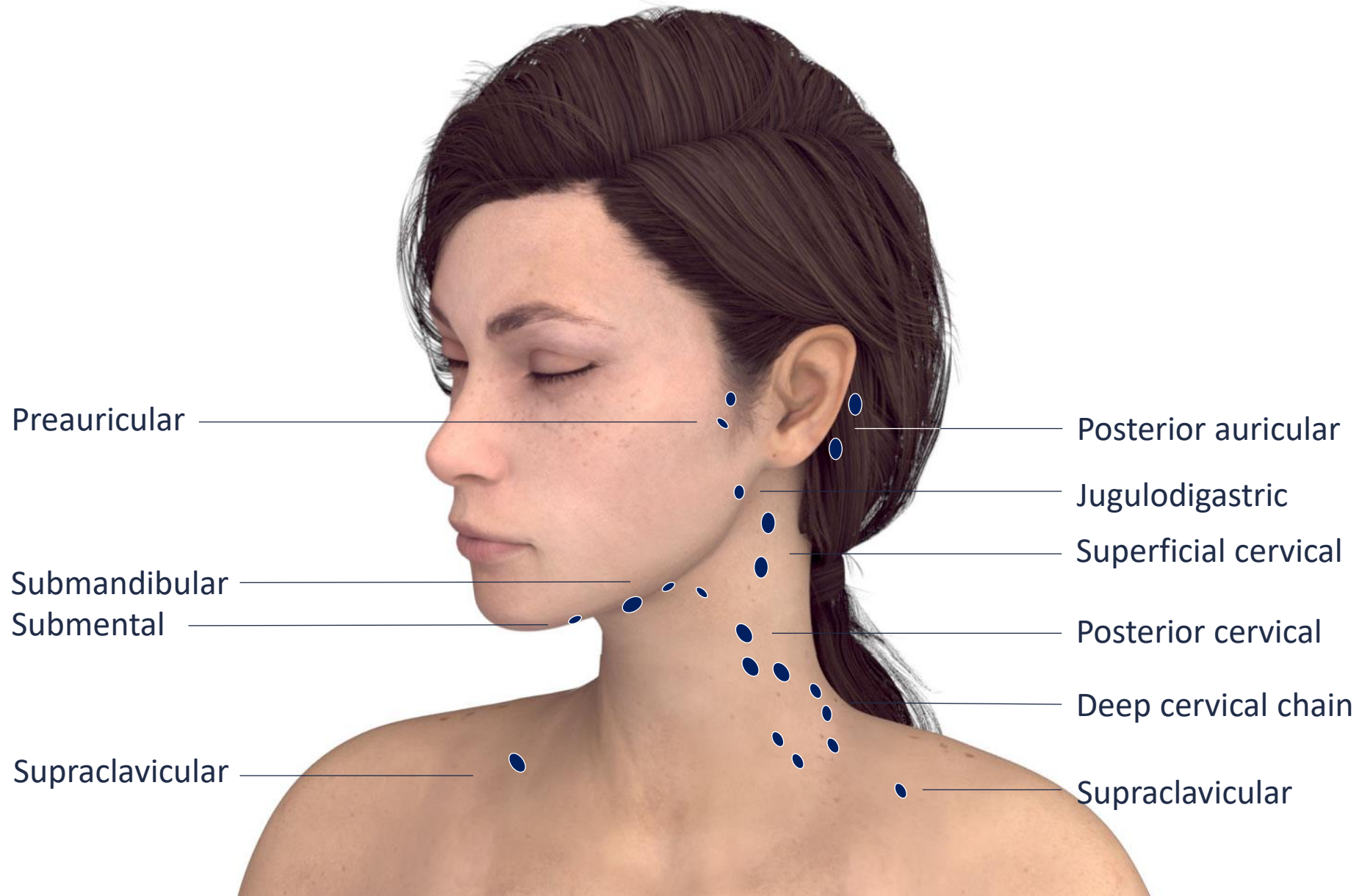



- All JetPeel sessions begin with JetCare Hydro
- Gentle solution formulated for the initial lymphatic massage, and to prep the skin for the deeper JetCare treatments
- Hyaluronic Acid concentration reformulated, significantly higher than in former version
- Enriched with higher concentrations of organic concentrated Aloe Vera
- Solid plastic bottle, won't puncture during transportation
- 300 ml - 20% more than in former version




Face & neck lymphatic massage

Technique & Sequence



Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin

Use JetCare
Hydro

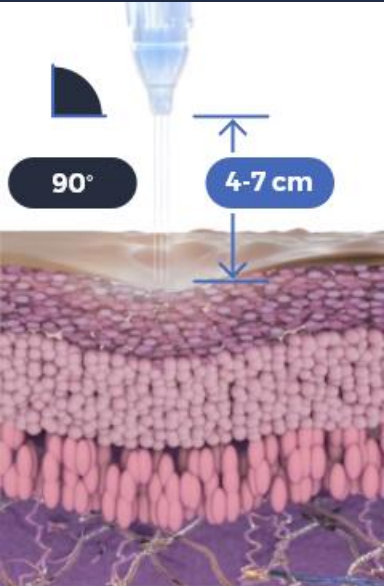

Start and complete
treatment on the left
side, only then
proceed to treat the
right side


Try to keep a
consistent pace
during the massage

Repeat 3 times



Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin


Use JetCare
Hydro

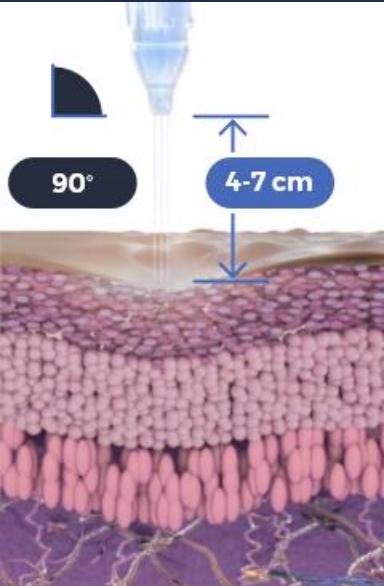

Start and complete
treatment on the left
side, only then
proceed to treat the
right side

Try to keep a
consistent pace
during the massage


Repeat 3 times

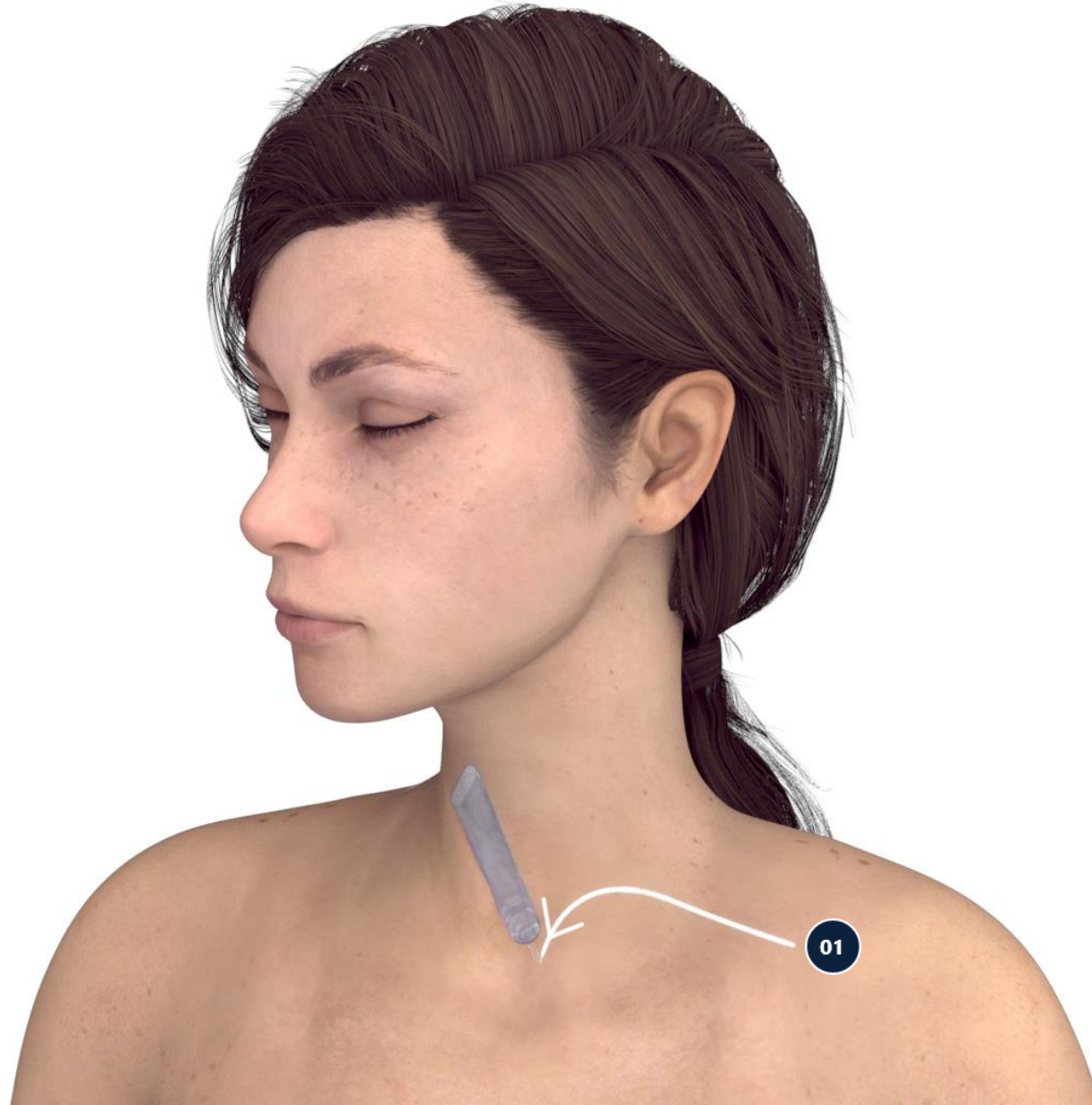

02

Use the TripleJet handpiece held at 90° to the skin and at 4-7 cm from the skin


Use JetCare Hydro

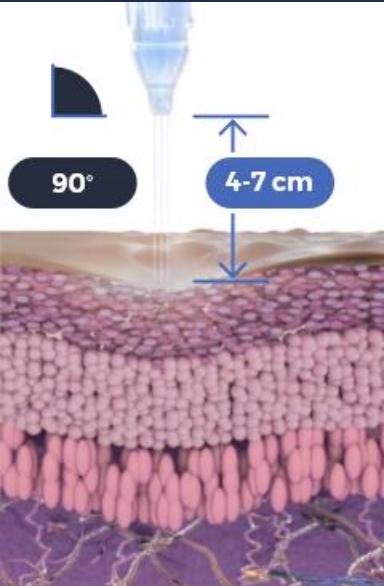

Start and complete treatment on the left side, only then proceed to treat the right side

Try to keep a consistent pace during the massage


Repeat 3 times

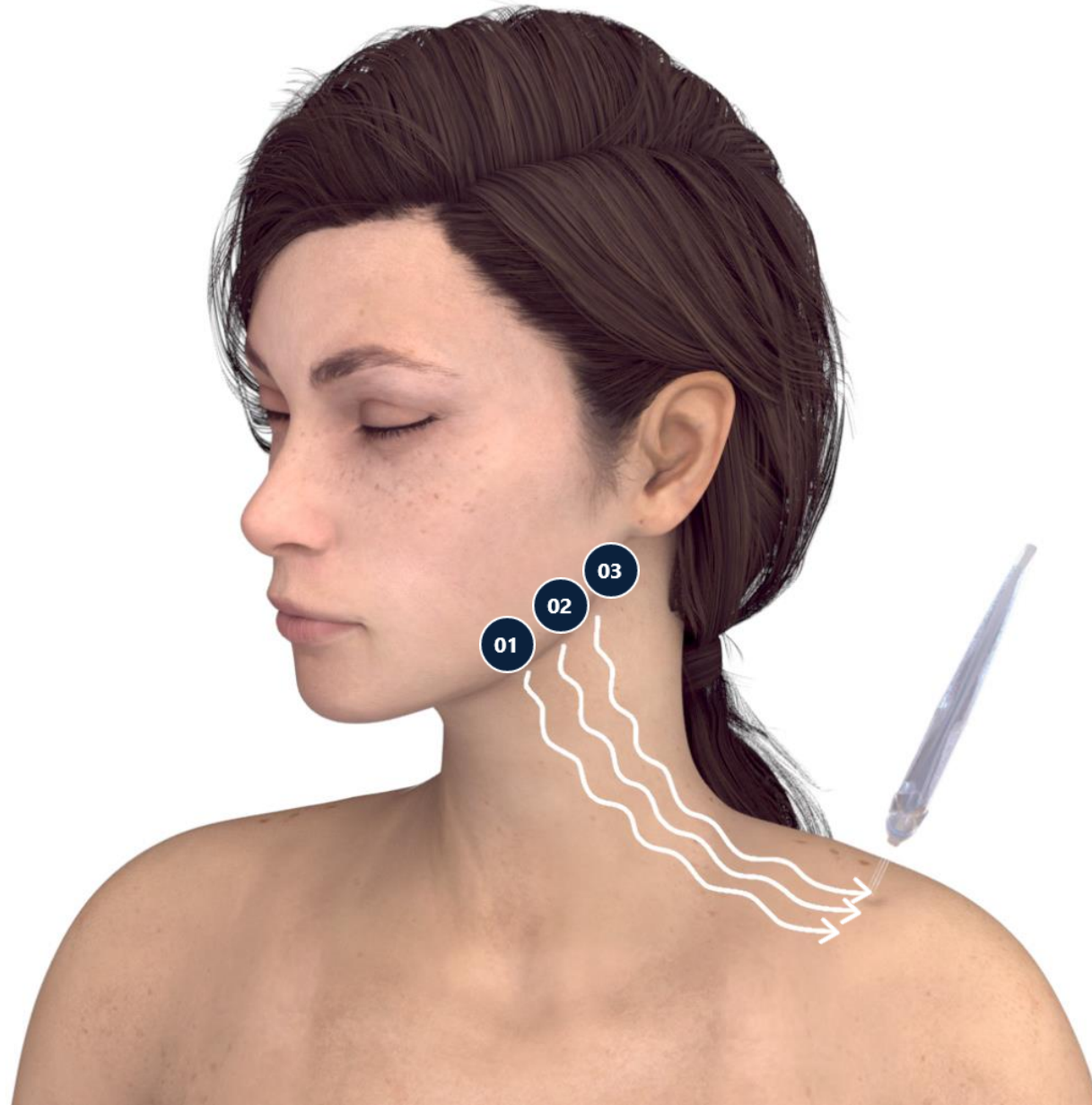

03

Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin


Use JetCare
Hydro

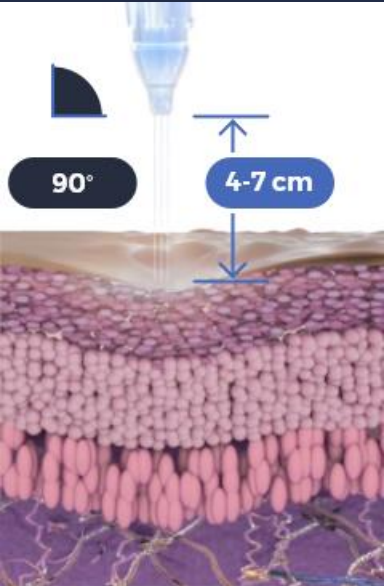

Start and complete
treatment on the left
side, only then
proceed to treat the
right side


Try to keep a
consistent pace
during the massage

Repeat 3 times

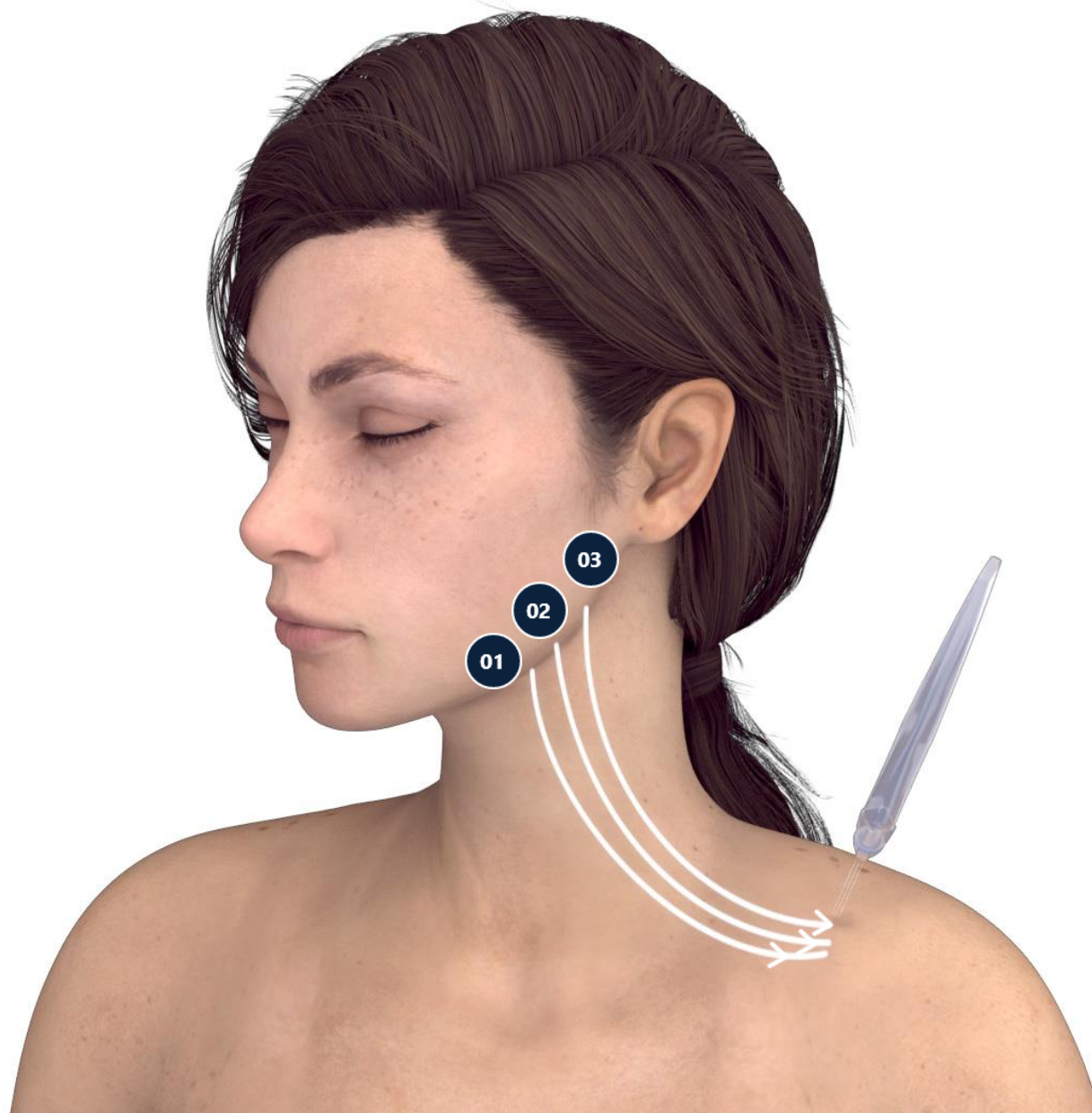

Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin


Use JetCare
Hydro

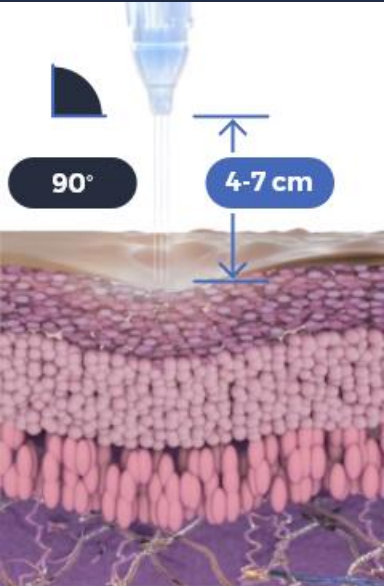

Start and complete
treatment on the left
side, only then
proceed to treat the
right side

Try to keep a
consistent pace
during the massage


Repeat 3 times

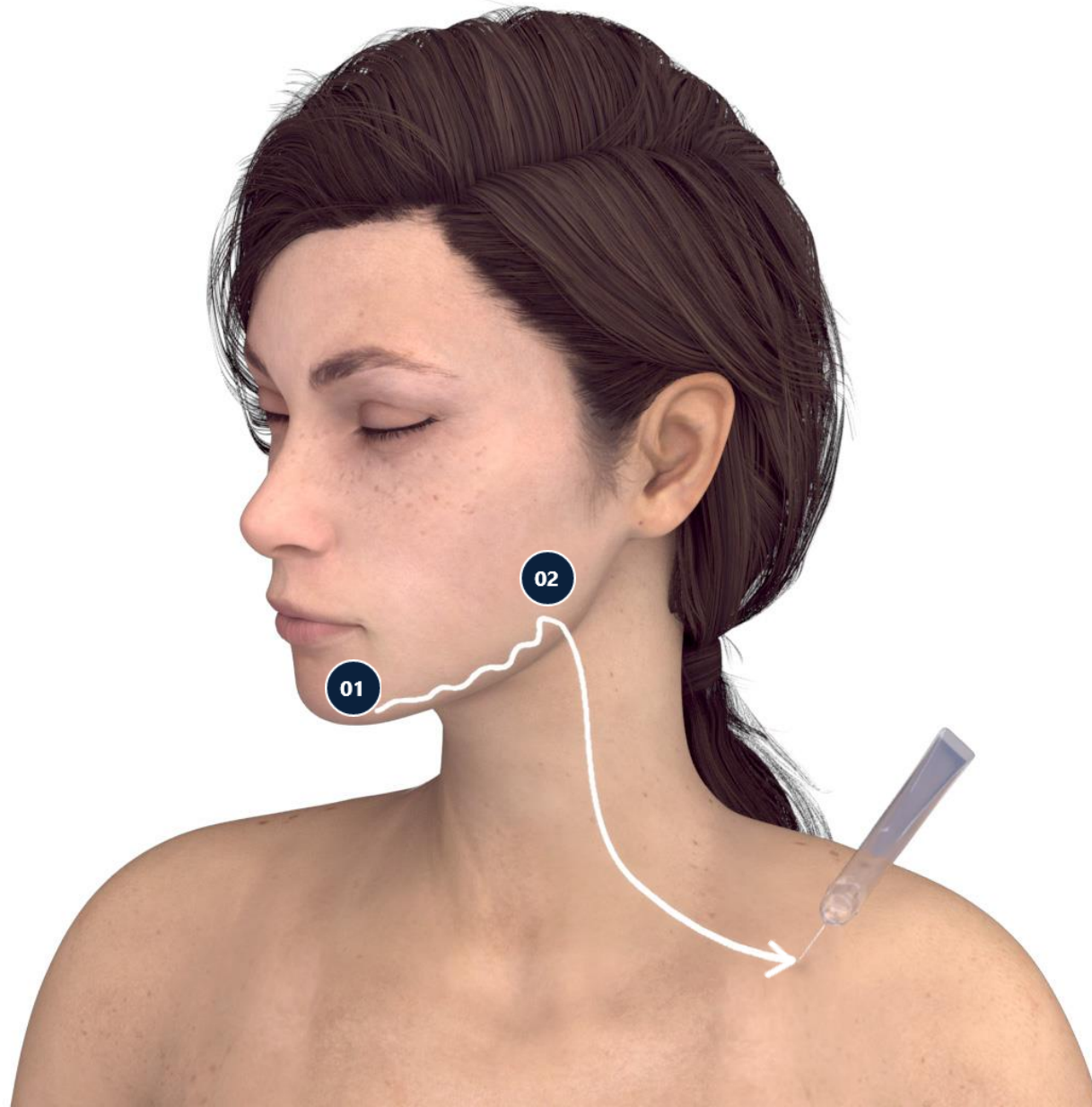

05

Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin


Use JetCare
Hydro

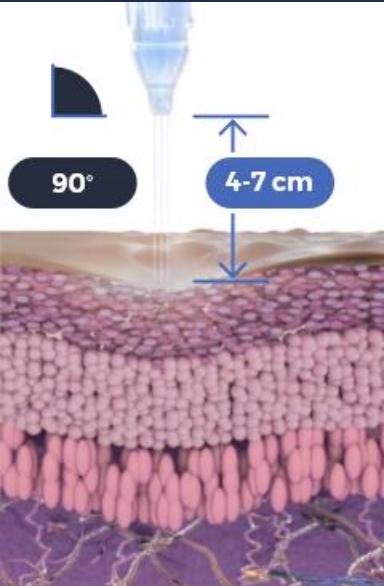

Start and complete
treatment on the left
side, only then
proceed to treat the
right side


Try to keep a
consistent pace
during the massage

Repeat 3 times

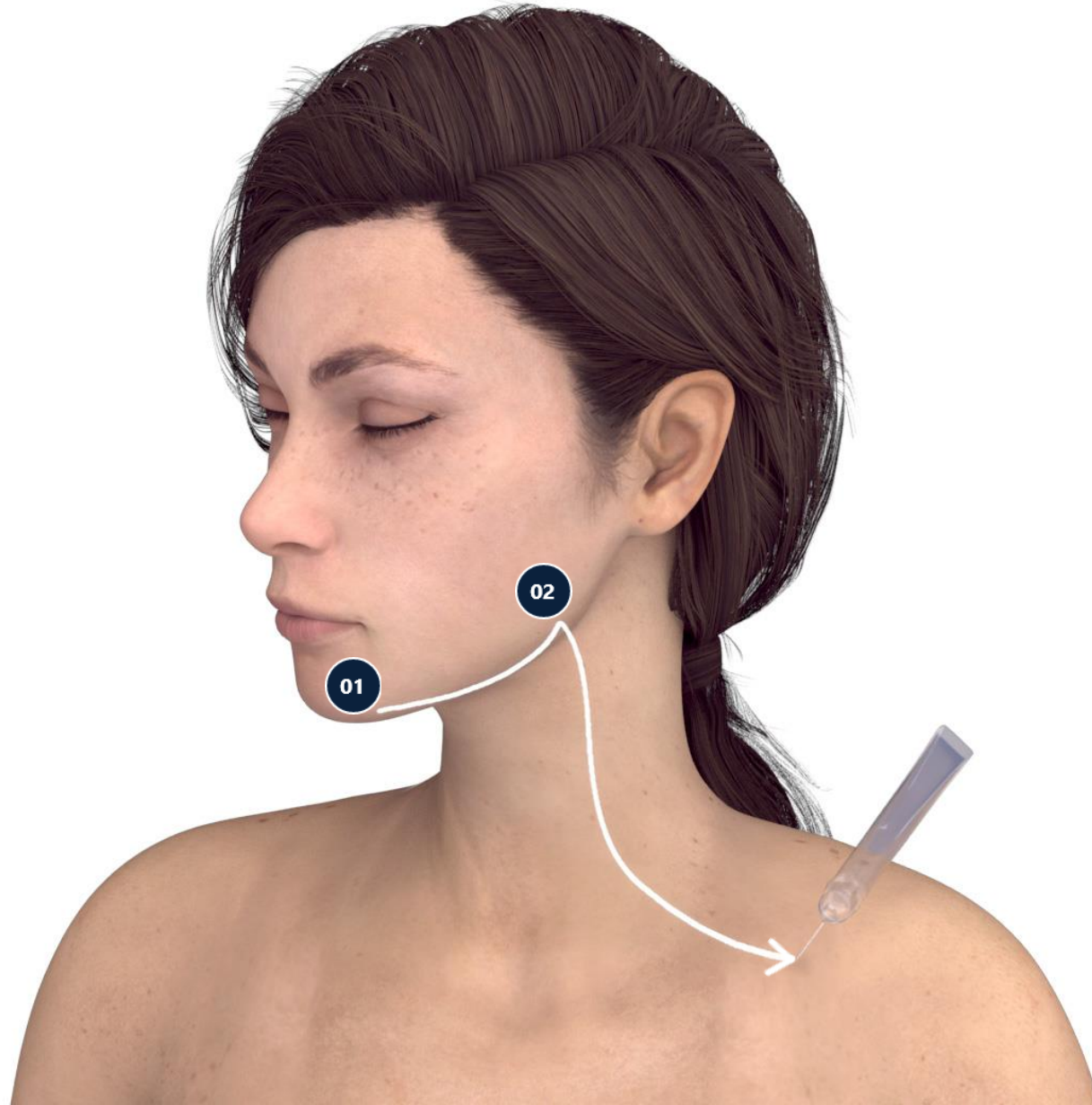

Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin


Use JetCare
Hydro

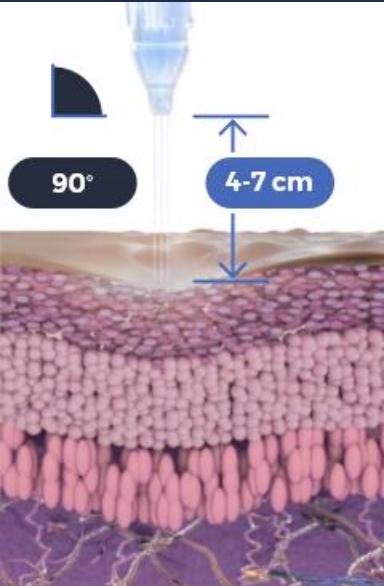

Start and complete
treatment on the left
side, only then
proceed to treat the
right side


Try to keep a
consistent pace
during the massage

Repeat 3 times

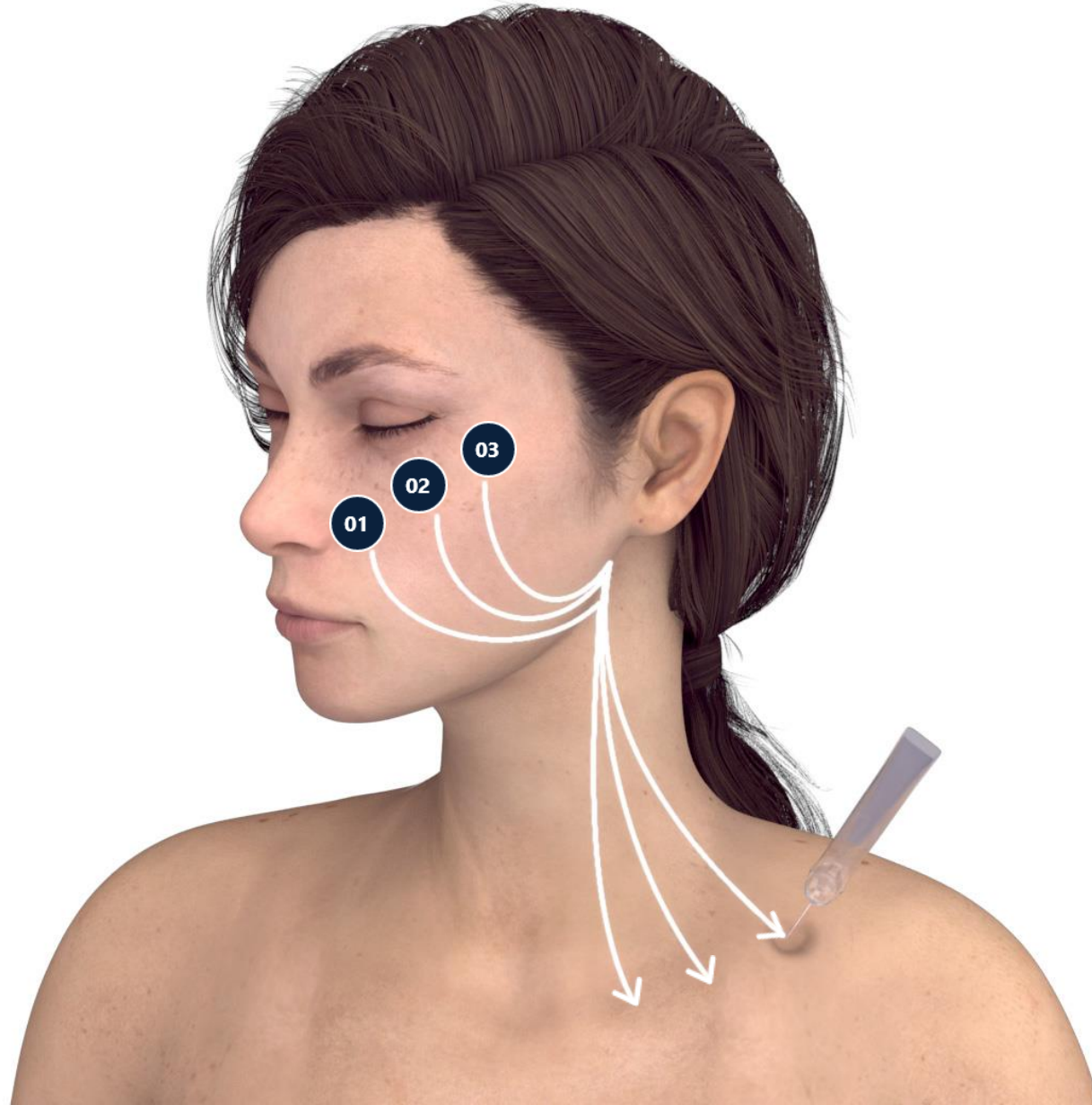

Use the TripleJet handpiece held at 90° to the skin and at 4-7 cm from the skin


Use JetCare Hydro

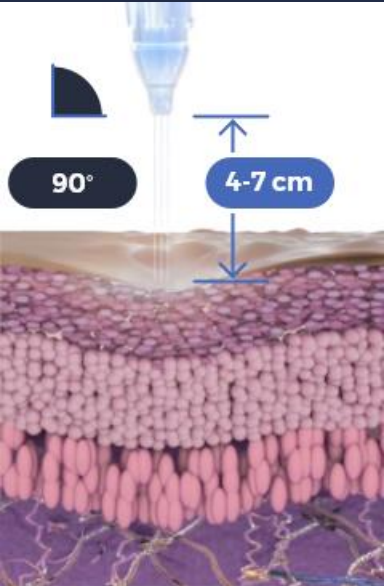

Start and complete treatment on the left side, only then proceed to treat the right side


Try to keep a consistent pace during the massage

Repeat 3 times

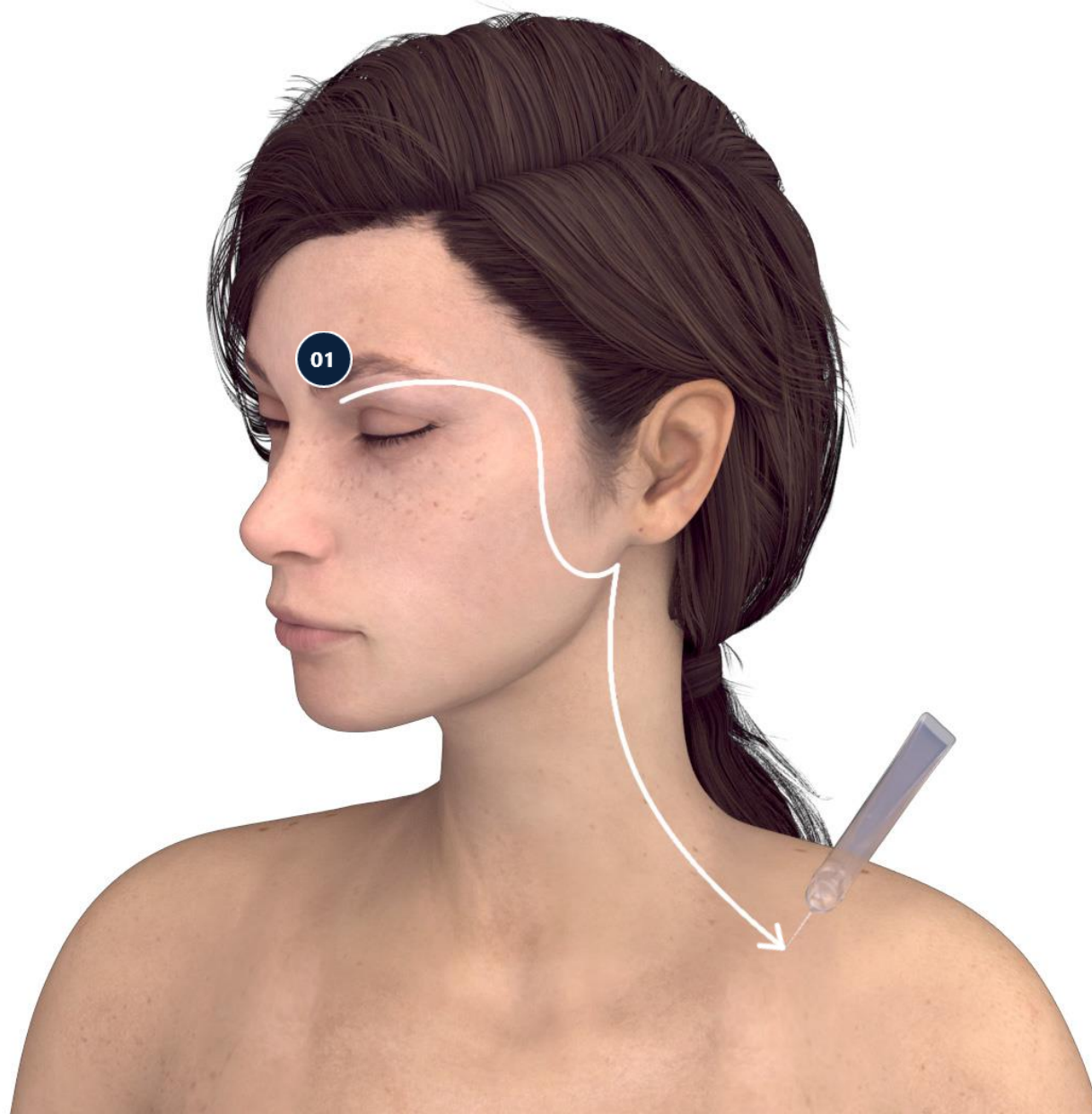

Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin


Use JetCare
Hydro

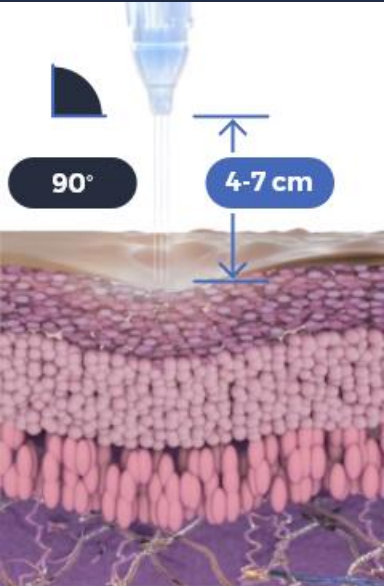

Start and complete
treatment on the left
side, only then
proceed to treat the
right side


Try to keep a
consistent pace
during the massage

Repeat 3 times

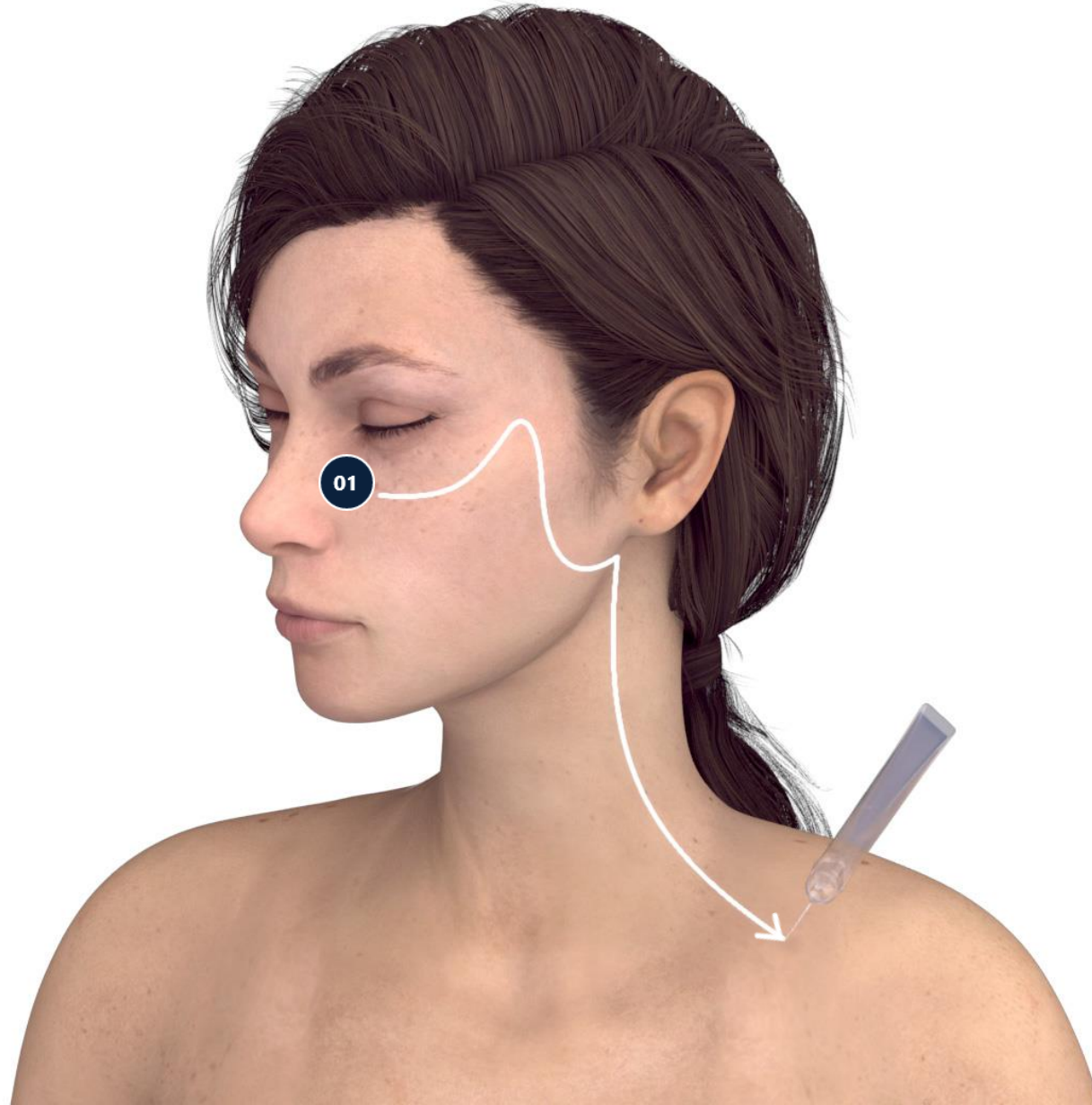

Use the TripleJet handpiece held at 90° to the skin and at 4-7 cm from the skin


Use JetCare Hydro

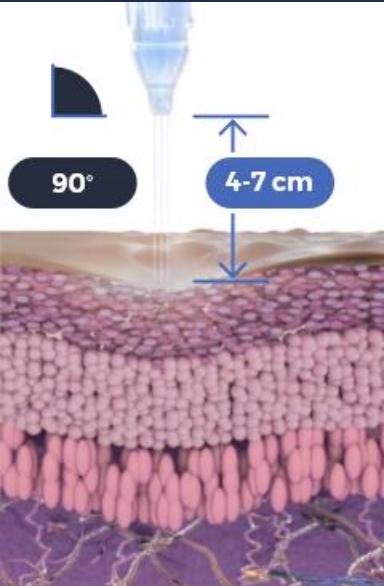

Start and complete treatment on the left side, only then proceed to treat the right side


Try to keep a consistent pace during the massage

Repeat 3 times

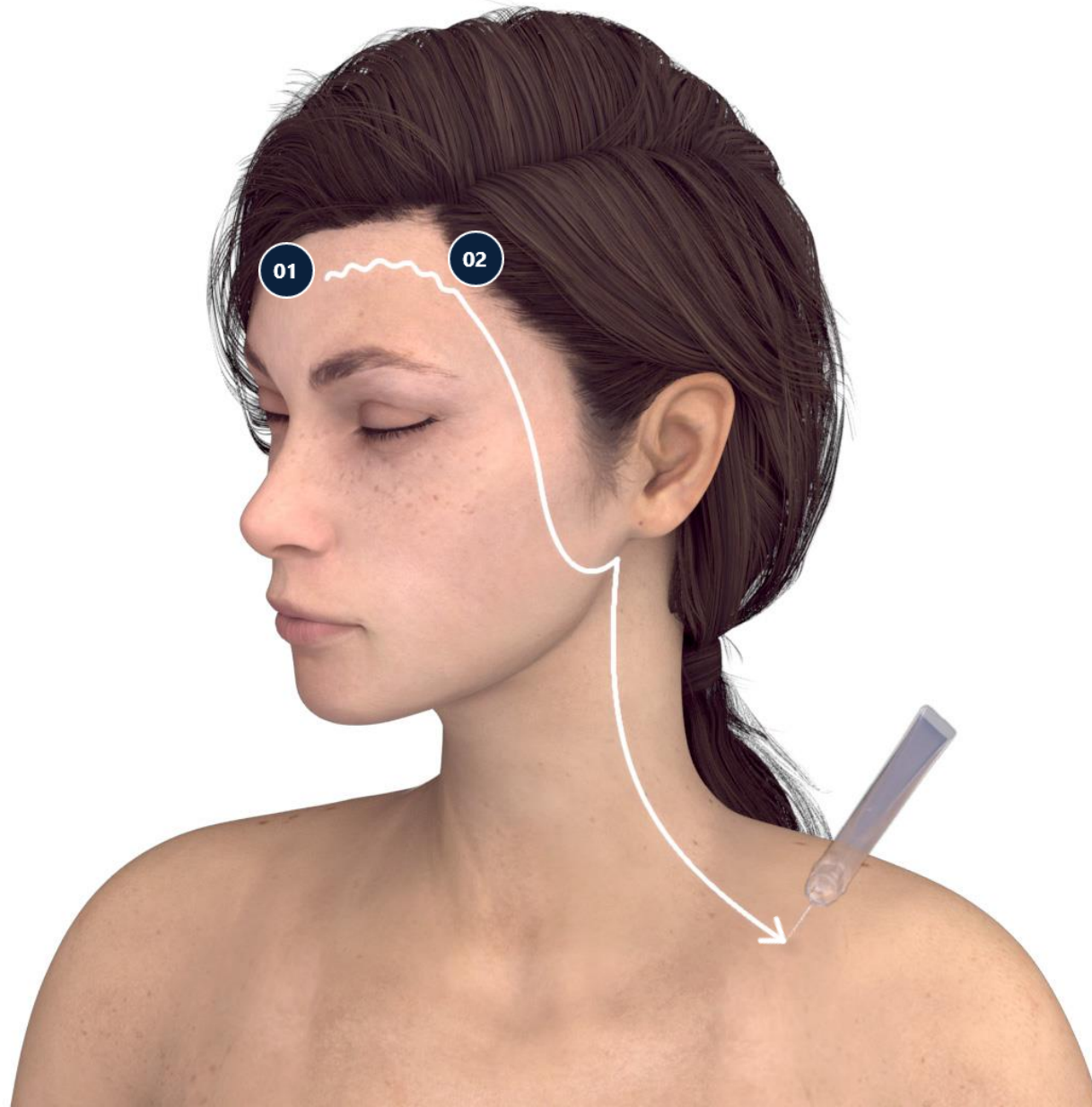

Use the TripleJet handpiece held at 90° to the skin and at 4-7 cm from the skin

Use JetCare Hydro

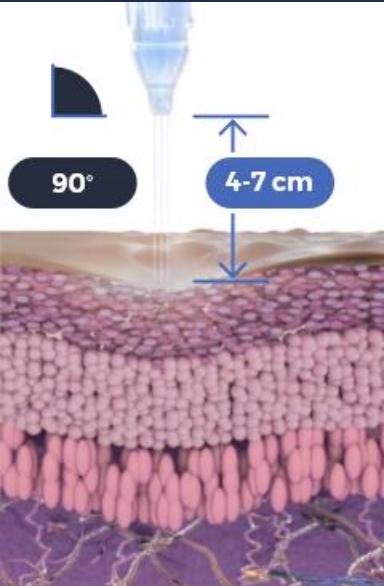

Start and complete treatment on the left side, only then proceed to treat the right side


Try to keep a consistent pace during the massage

Repeat 3 times



Use the TripleJet handpiece held at 90° to the skin and at 4-7 cm from the skin


Use JetCare Hydro

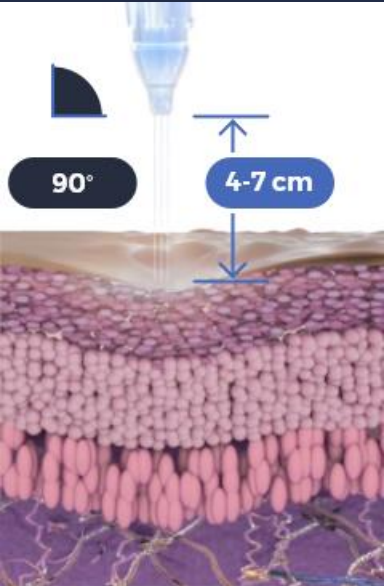

Start and complete treatment on the left side, only then proceed to treat the right side


Try to keep a consistent pace during the massage

Repeat 3 times

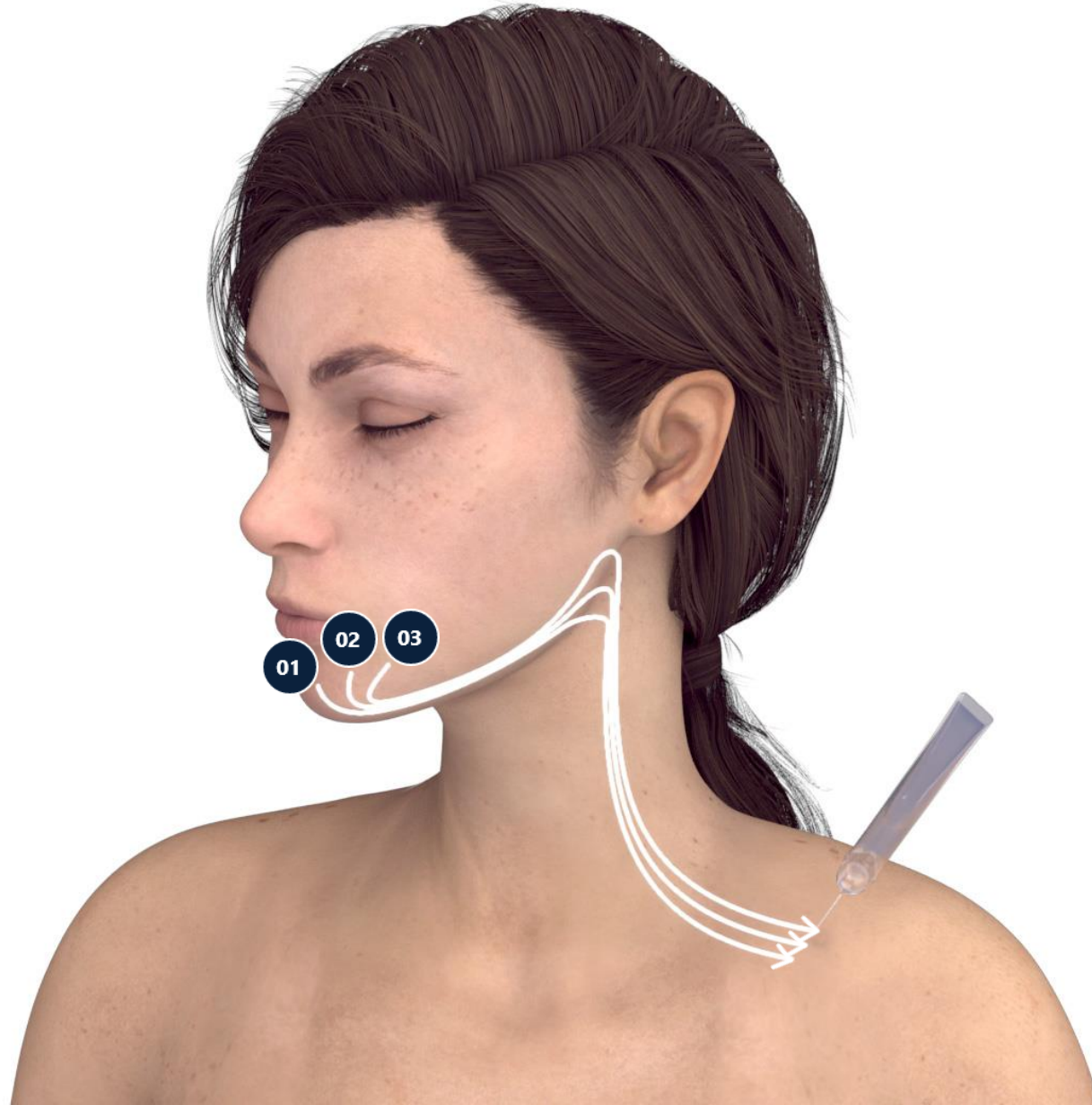

Use the TripleJet handpiece held at 90° to the skin and at 4-7 cm from the skin


Use JetCare Hydro

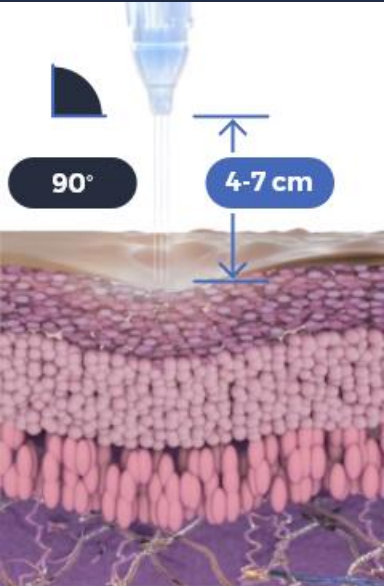

Start and complete treatment on the left side, only then proceed to treat the right side


Try to keep a consistent pace during the massage

Repeat 3 times

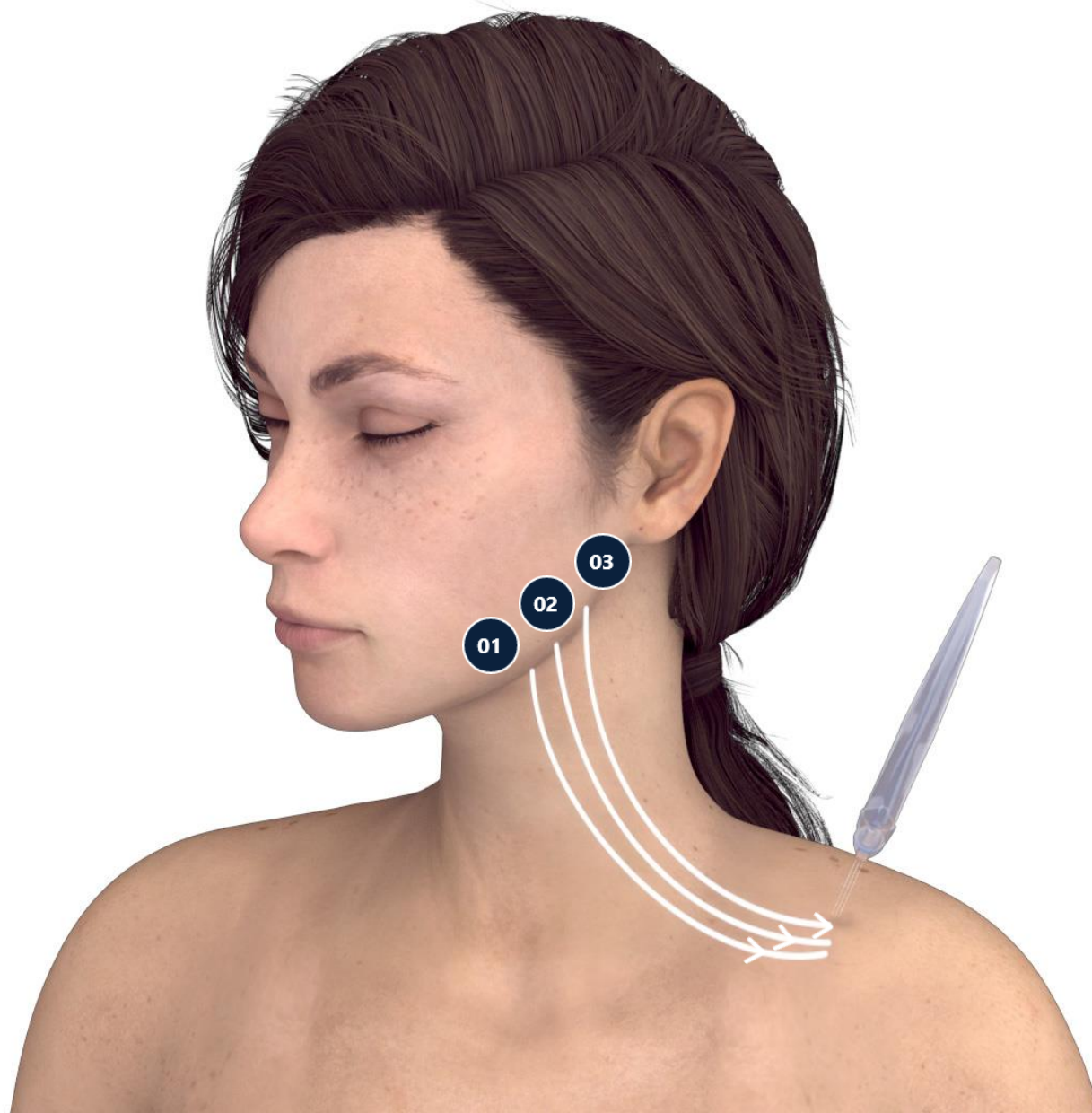

Use the TripleJet
handpiece held at
90° to the skin
and at 4-7 cm
from the skin


Use JetCare
Hydro

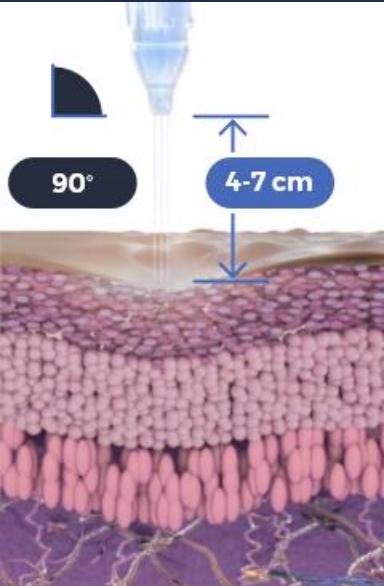

Start and complete
treatment on the left
side, only then
proceed to treat the
right side

Try to keep a
consistent pace
during the massage

Repeat 3 times

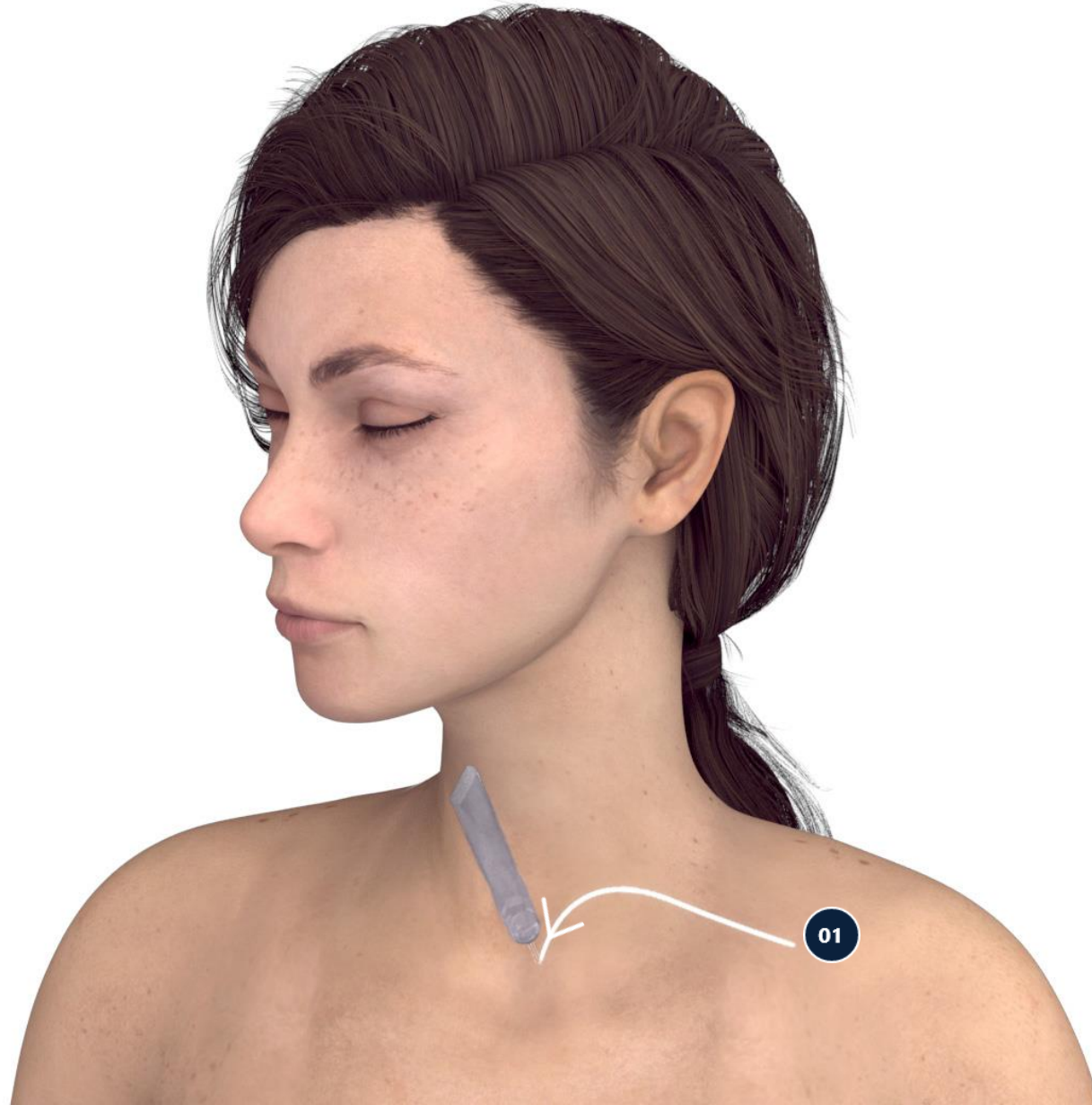

Contraindications

- Open wounds
- Severe sensitivity to cold
- Skin neoplasms
- Active cancer
- Trigeminal nerve inflammation
- Active sinusitis or laryngitis
- Active herpes
- Active bronchial asthma
- Thrombosis
- The system has not been tested or evaluated on pregnant or lactating women, nor on children and infants, therefore is not intended for use on these groups of users

JetPeel by TavTech

The perfect trio combining:



**The enduring
JetPro device**



**The original patented
JetPeel handpieces**



**The premium
JetCare solutions**

Designed exclusively for use together to deliver outstanding results



For more information on Lymphatic Massage Techniques with JetPeel check out the following presentations:

- **Lymphatic Massage with
JetPeel – Body**





JetPeel by TavTech Training Program

1. JetPeel Technology and Solutions: Training Introduction

2. Lymphatic Massage with JetPeel

2.1. Lymphatic Massage with JetPeel – Face & Neck

2.2. Lymphatic Massage with JetPeel – Body

3. Exfoliation with JetPeel

3.1. Exfoliation with JetPeel – Face & Neck

4. Infusion with JetPeel

4.1. Infusion with JetPeel – Face & Neck

5. Scalp & Hair with JetPeel

6. JetPeel Handpieces





THANK YOU!



JetPeel by TavTech Academy

