

# JetPeel by TavTech Academy



We provide people with an  
opportunity to become the best  
version of themselves

**We believe in  
making people  
feel good ...**





... by delivering  
a powerful,  
instant-result  
treatment

Suitable for all skin types  
and all year-round





# JetPeel by TavTech Training Program

1. JetPeel Technology and Solutions: Training Introduction
2. Lymphatic Massage with JetPeel
  - 2.1. Lymphatic Massage with JetPeel – Face & Neck
  - 2.2. Lymphatic Massage with JetPeel – Body
3. Exfoliation with JetPeel
  - 3.1. Exfoliation with JetPeel – Face & Neck
4. Infusion with JetPeel
  - 4.1. Infusion with JetPeel – Face & Neck
5. Scalp & Hair with JetPeel
6. JetPeel Handpieces






## **3.1. Exfoliation with JetPeel**

### **Face & Neck**



## The versatility of JetPeel: Face and Body

Unique jet pressure energy

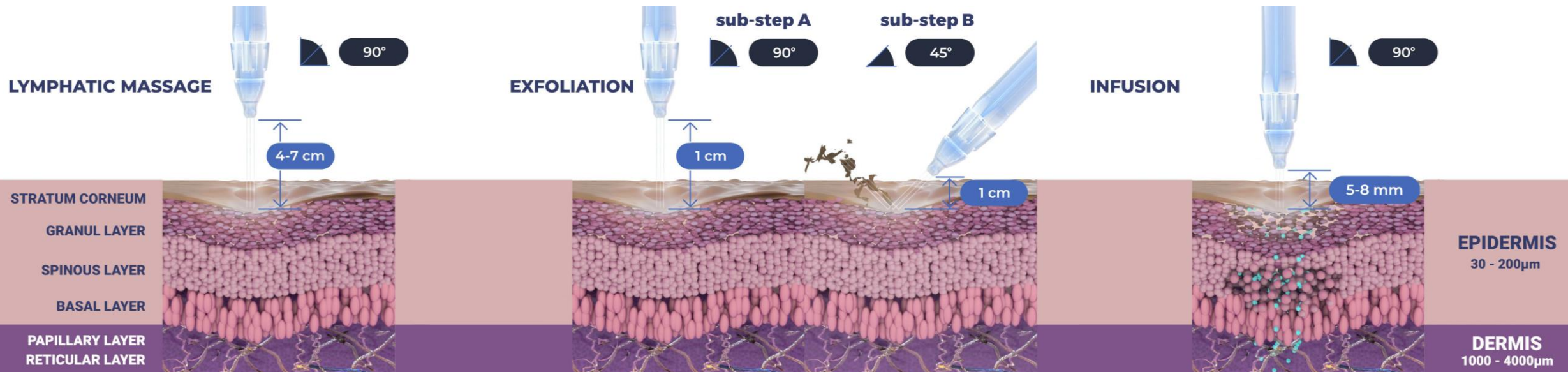

# The treatment sequence



## LYMPHATIC MASSAGE

A pleasant and relaxing massage for detoxifying the skin and stimulating micro-circulation



## GENTLE EXFOLIATION

Removing dead skin cells prepares the skin to receive the nourishment from the following Infusion Step



## EFFECTIVE INFUSION

This step provides effective and deep delivery of active ingredients into the skin



## BOOSTER INFUSION

Post-infusion step to amplify treatment result

# The treatment sequence



## LYMPHATIC MASSAGE

A pleasant and relaxing massage for detoxifying the skin and stimulating micro-circulation



## GENTLE EXFOLIATION

Removing dead skin cells prepares the skin to receive the nourishment from the following Infusion Step



## EFFECTIVE INFUSION

This step provides effective and deep delivery of active ingredients into the skin



## BOOSTER INFUSION


Post-infusion step to amplify treatment result



## Step 2 - Exfoliation with JetPeel

- Exfoliation is a form of microdermabrasion using the JetPeel jet pressure energy system with JetCare solutions to hydrate the skin throughout the treatment
- Exfoliation clears the dead skin cells (stratum corneum of the epidermis) revealing a smoother, more even texture of the skin and improving the skin appearance

## Step 2 - Exfoliation

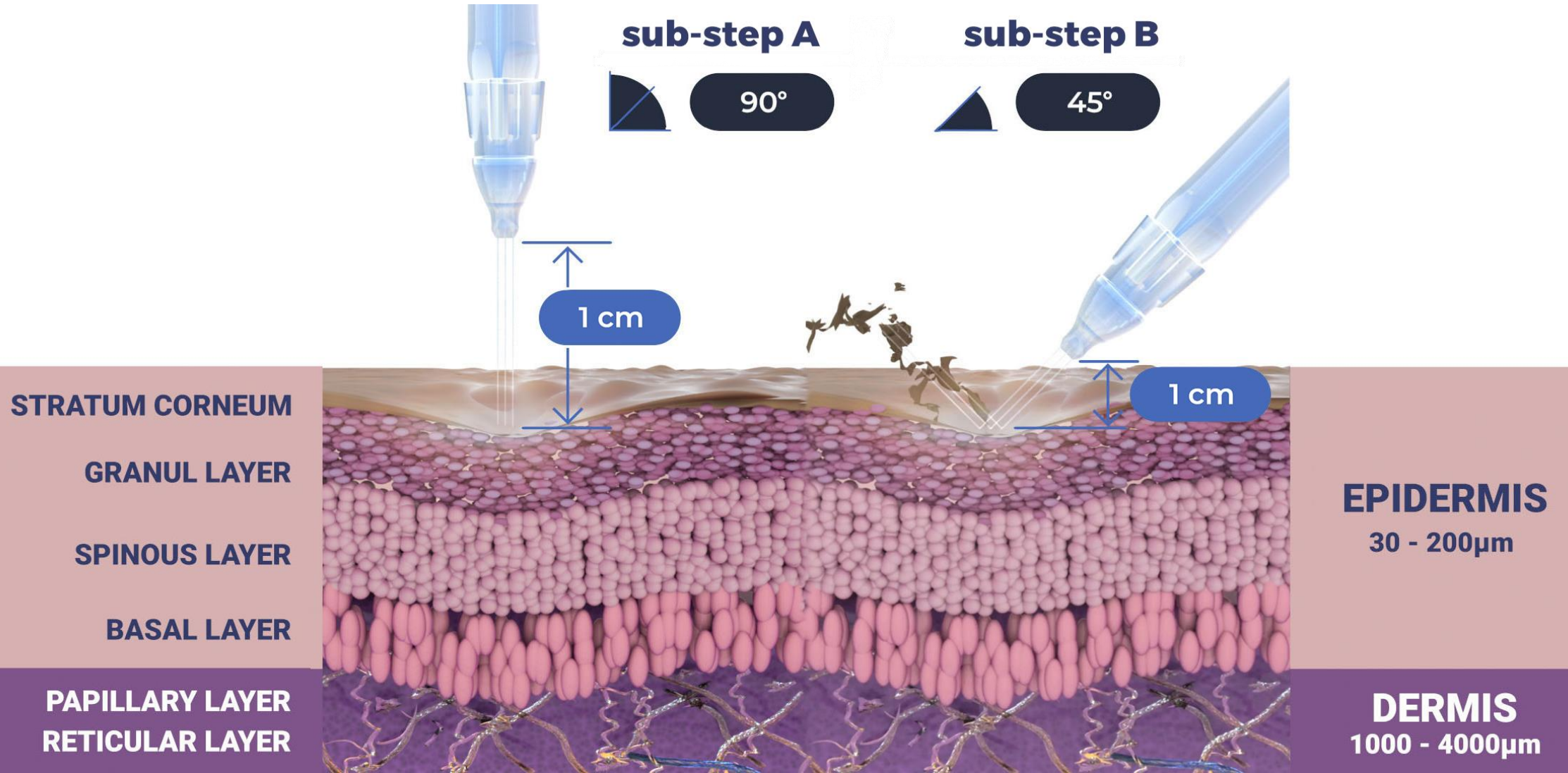

# The Renewal Care Series

Glycolic Acid Series  
Mandelic & Salicylic





# Exfoliation

Technique & Sequence






- According to the skin condition and desired exfoliation, use ~ 5 ml of either:
  - Renewal Care Glycolic Acid 5%, or
  - Renewal Care Glycolic Acid 10%, or
  - Renewal Care Glycolic Acid 16%, or
  - Renewal Care Mandelic & Salicylic Acids
- If exfoliating with Glycolic acid, leave on the skin for 5-8 minutes and only after that, wash with ~8ml of JetCare Hydro.
- If exfoliating with Mandelic & Salicylic acids, then immediately after that, wash with ~8ml JetCare Hydro if necessary.
- It is recommended to use the TripleJet handpiece.
- For the exfoliation step - keep the handpiece at a 90° angle relative to the skin surface and keep 1cm from it.
- For the washing step – keep the handpiece at a 45° - 60° angle relative to the skin surface 8-10mm from it.
- Move in zigzag motions, covering the entire treatment area.
- Duration of a complete facial exfoliation is about 5-7 minutes.

### Sub-Step A


Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

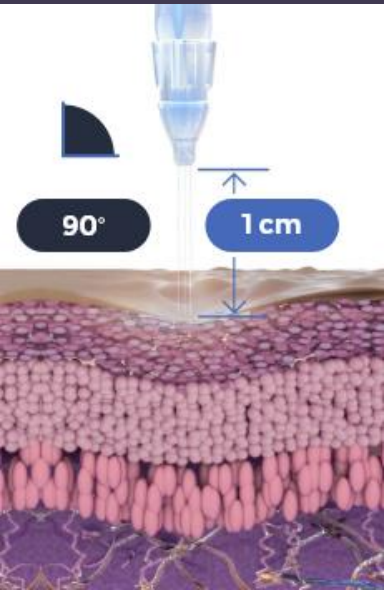

Move in zigzag motions covering the entire area


# Repeat 3 times

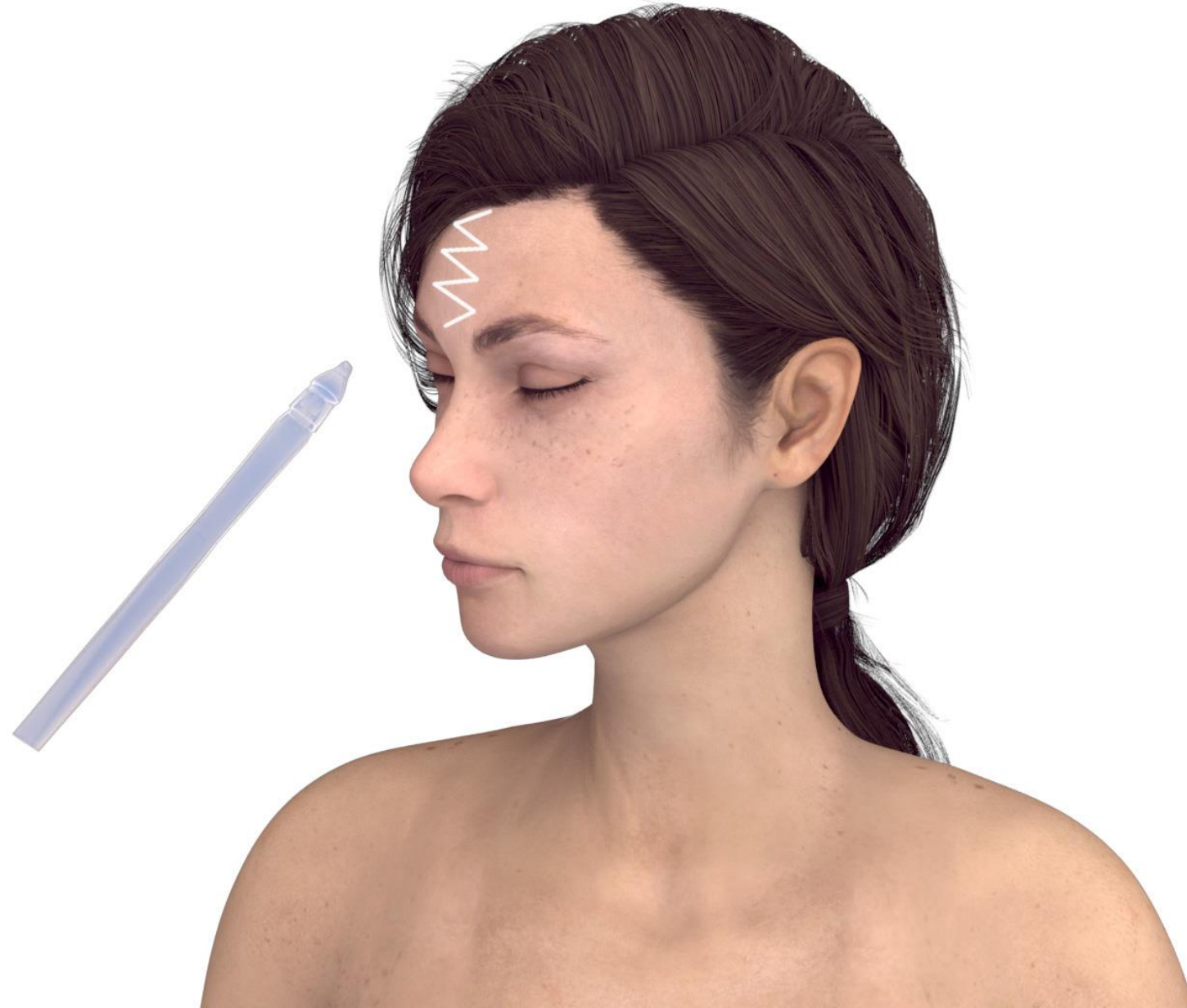

### Sub-Step A

Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

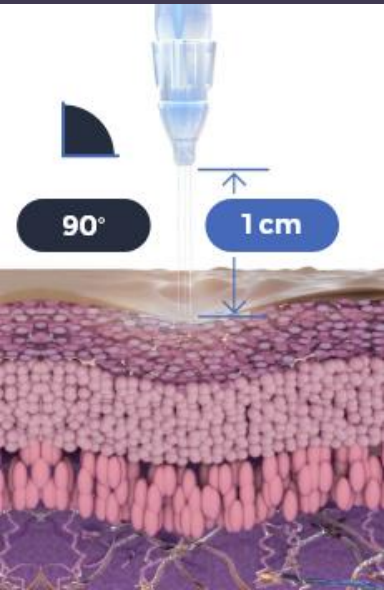

Move in zigzag motions covering the entire area


# Repeat 3 times



### Sub-Step A

Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

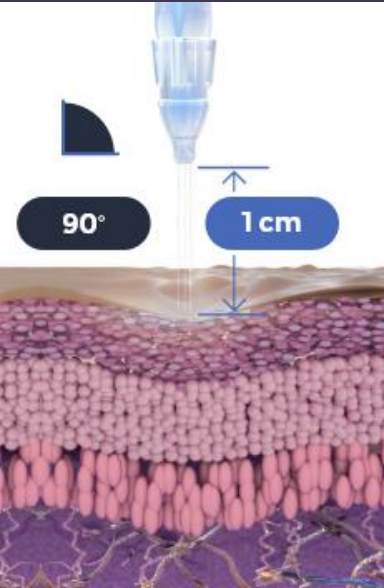

Move in zigzag motions covering the entire area

# Repeat 3 times






### Sub-Step A

Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

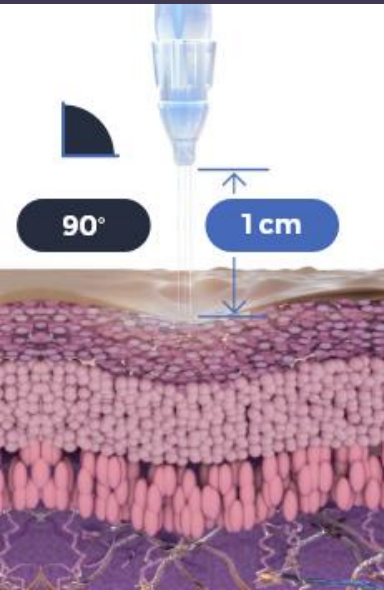

Move in zigzag motions covering the entire area


# Repeat 3 times



### Sub-Step A

Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

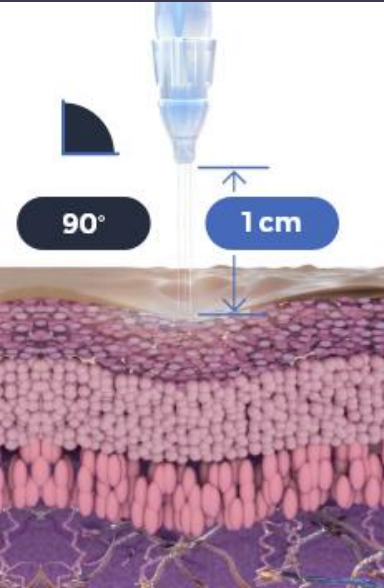

Move in zigzag motions covering the entire area


# Repeat 3 times



### Sub-Step A

Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

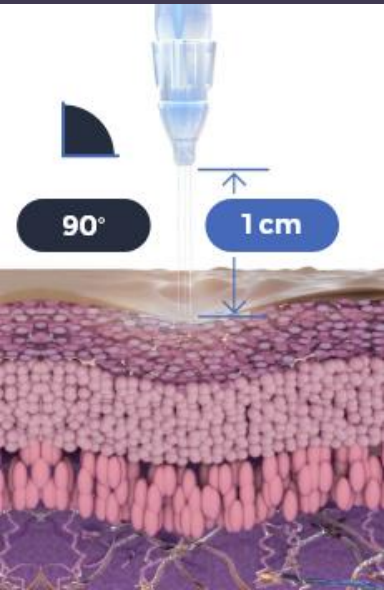

Move in zigzag motions covering the entire area

## Repeat 3 times






### Sub-Step A


Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

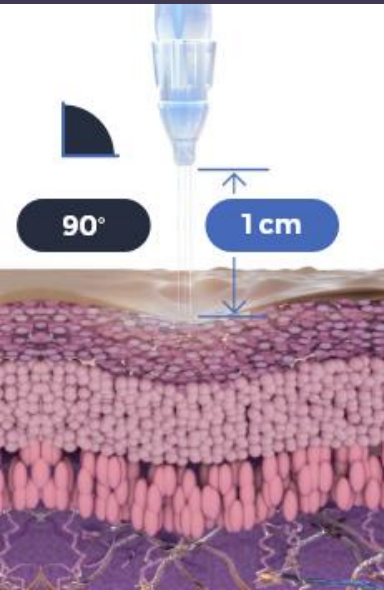

Move in zigzag motions covering the entire area

# Repeat 3 times

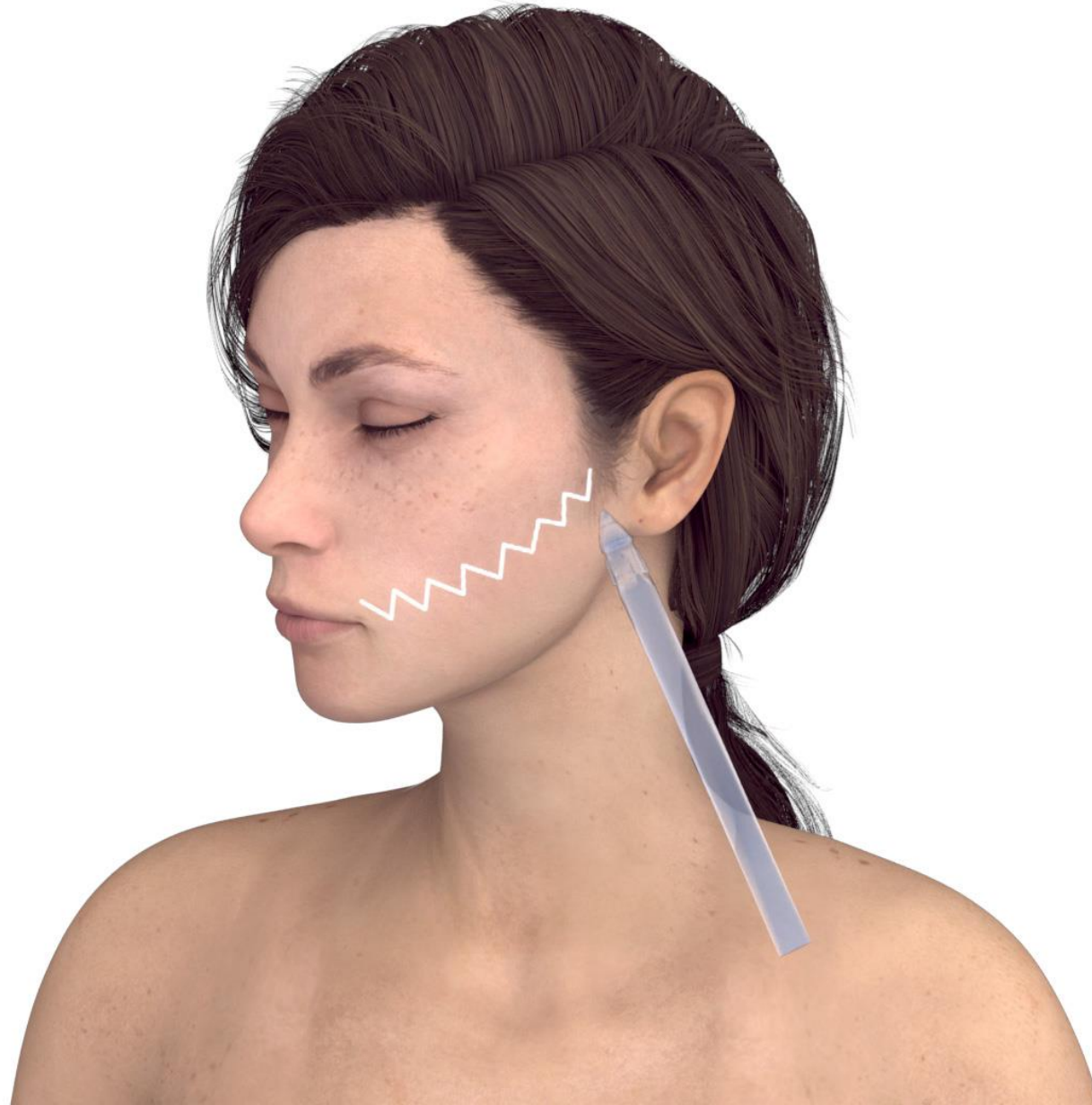




### Sub-Step A


Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

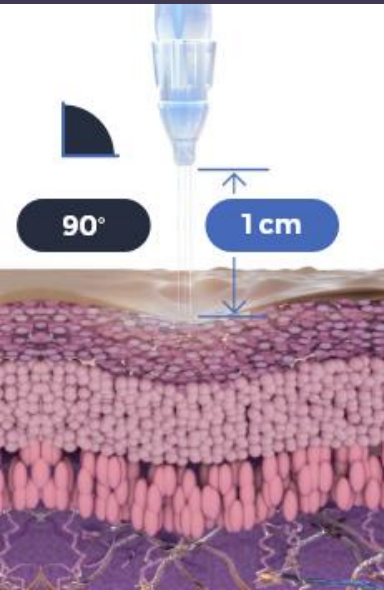

Move in zigzag motions covering the entire area


## Repeat 3 times

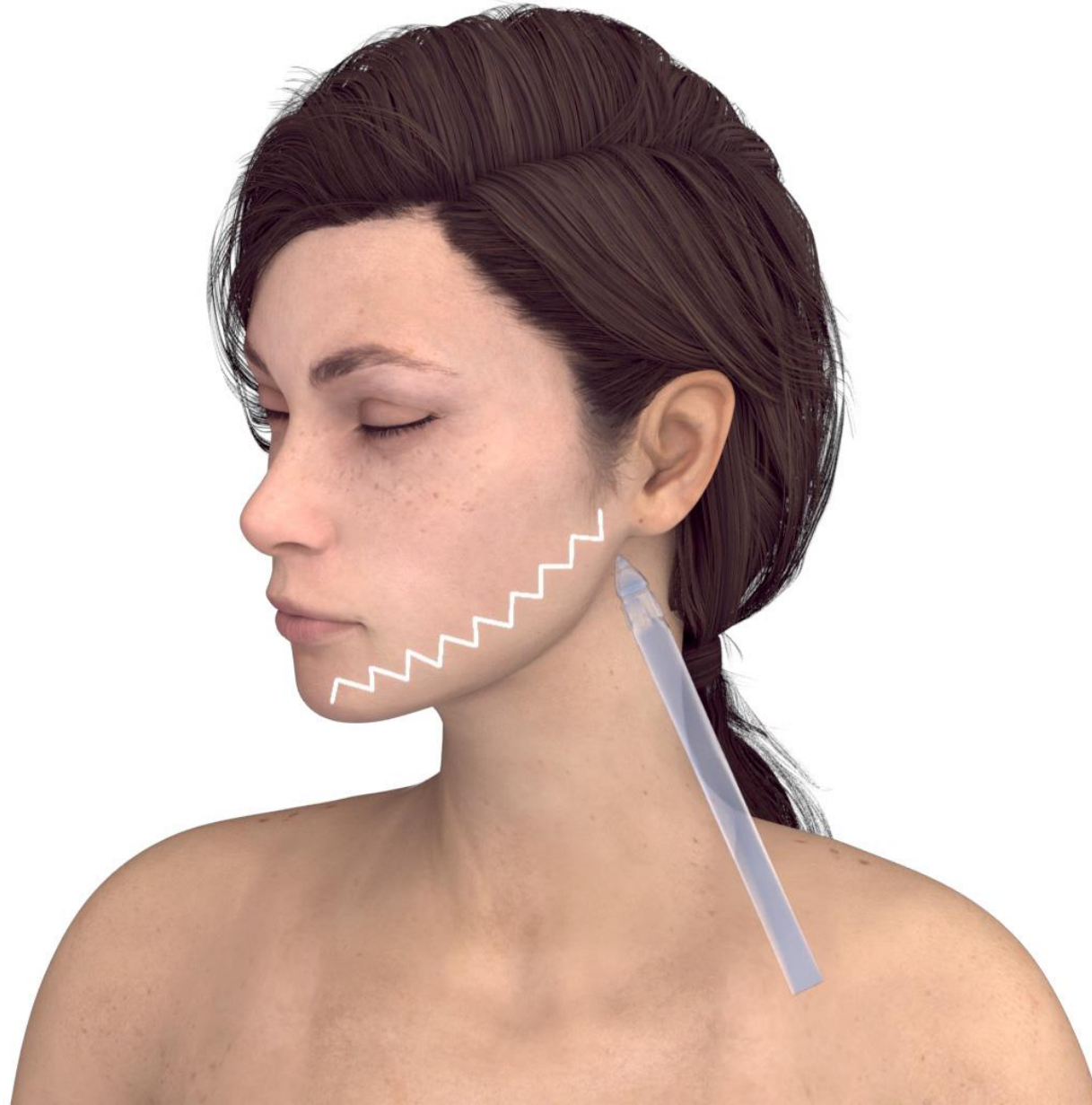

### Sub-Step A


Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

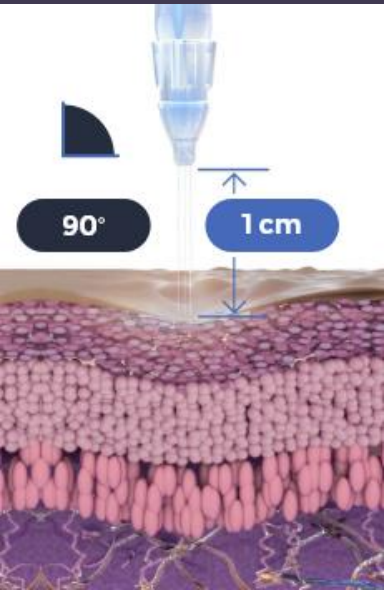

Move in zigzag motions covering the entire area


# Repeat 3 times

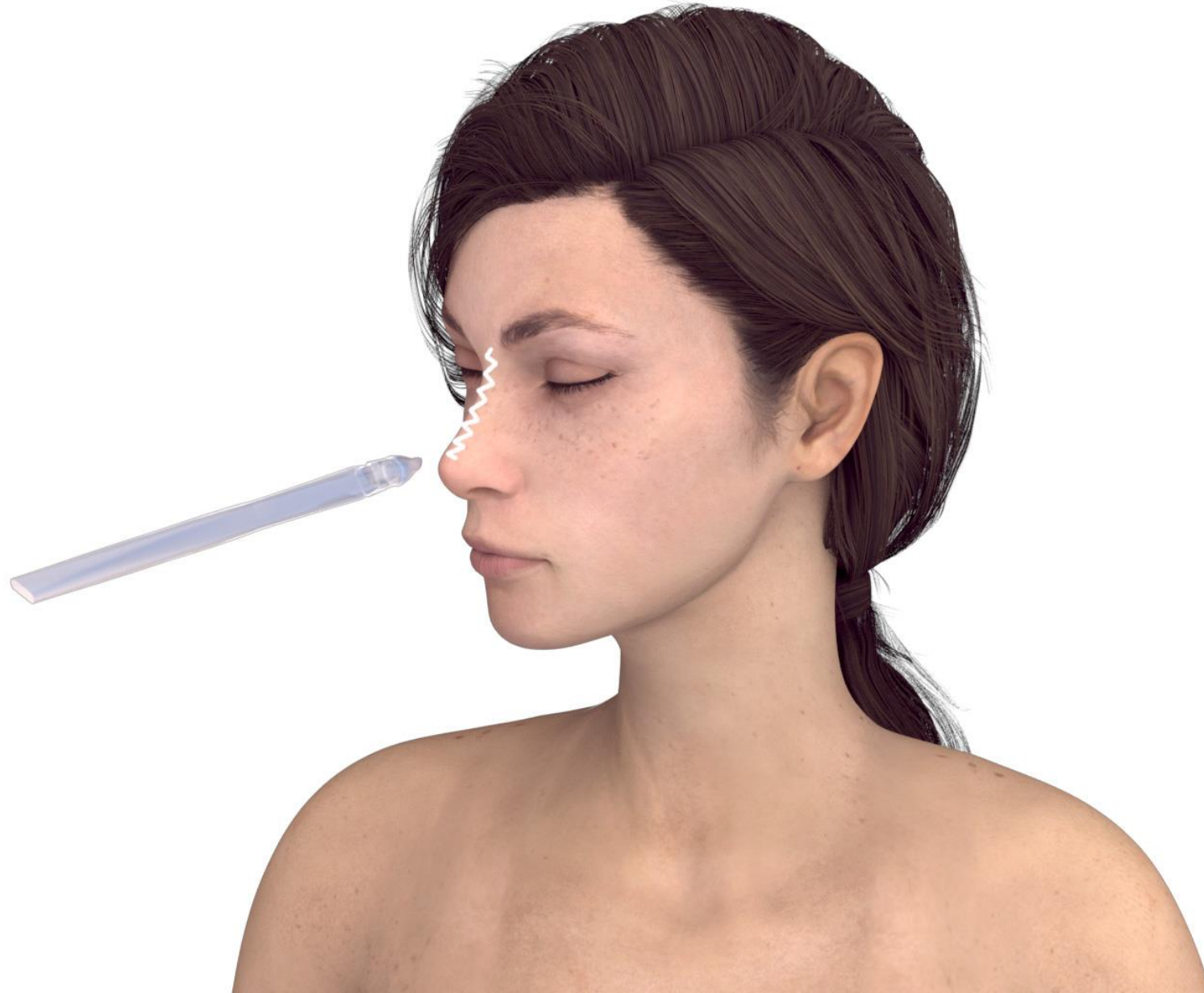

### Sub-Step A

Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

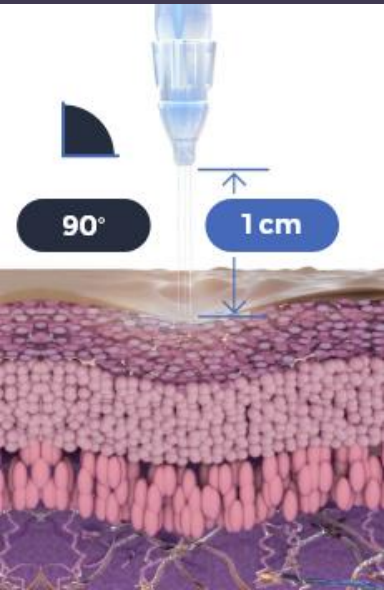

Move in zigzag motions covering the entire area

## Repeat 3 times






### Sub-Step A


Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

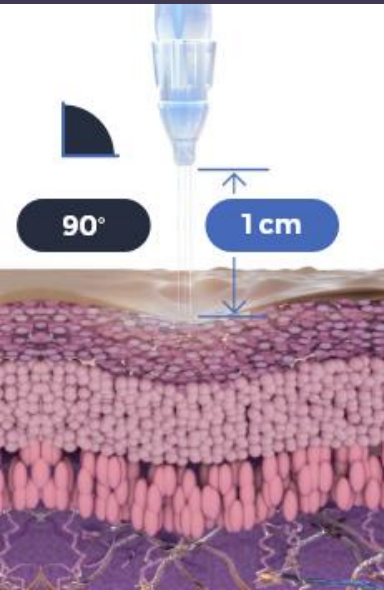

Move in zigzag motions covering the entire area

# Repeat 3 times

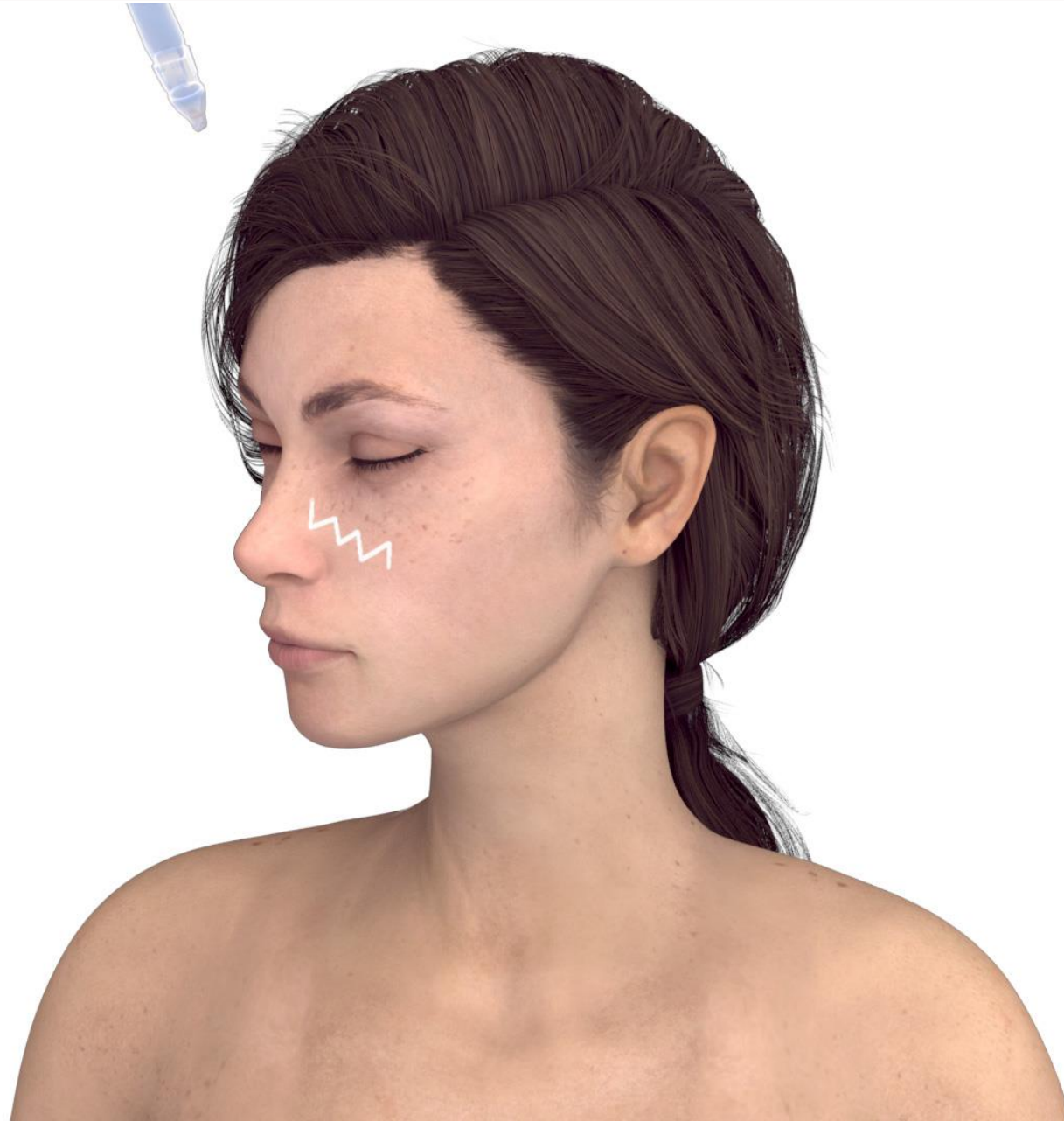




### Sub-Step A


Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

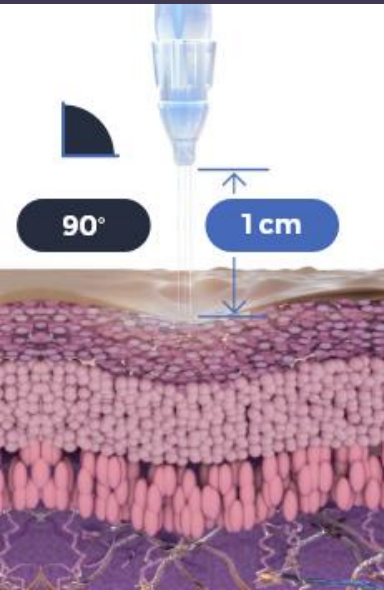

Move in zigzag motions covering the entire area


## Repeat 3 times

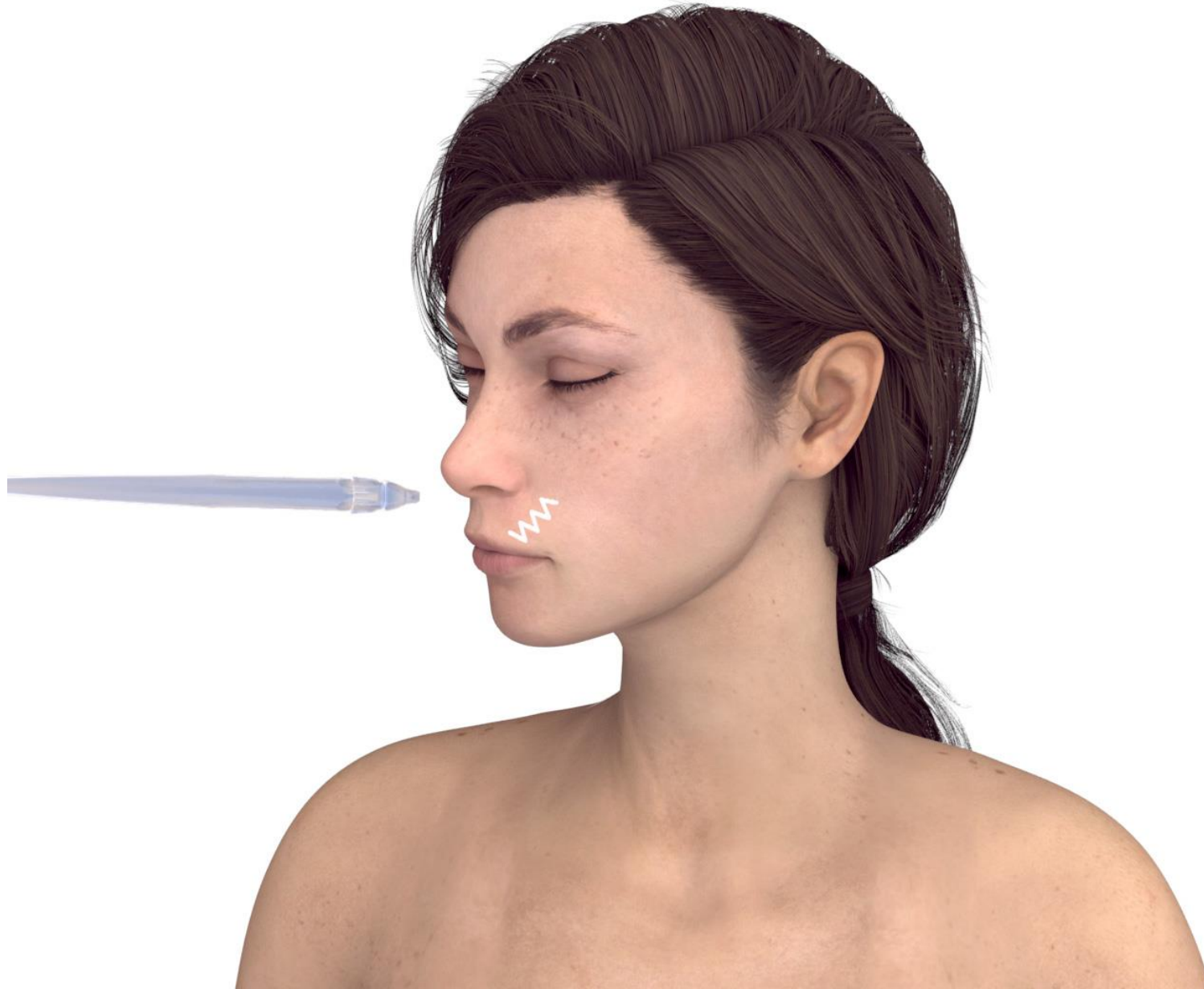

### Sub-Step A

Use the TripleJet handpiece held at 90° to the skin at 1 cm from the skin

Use one of the JetCare Renewal Care solutions

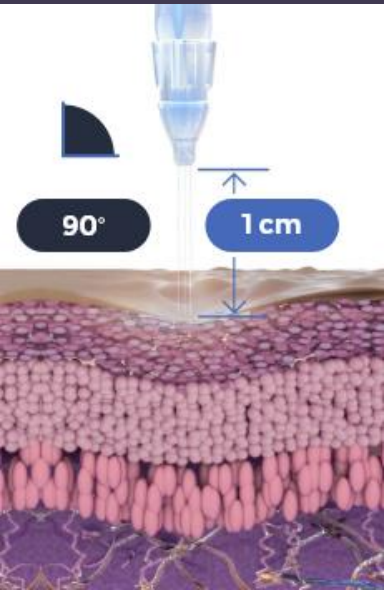

Move in zigzag motions covering the entire area

# Repeat 3 times




### Sub-Step B


Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

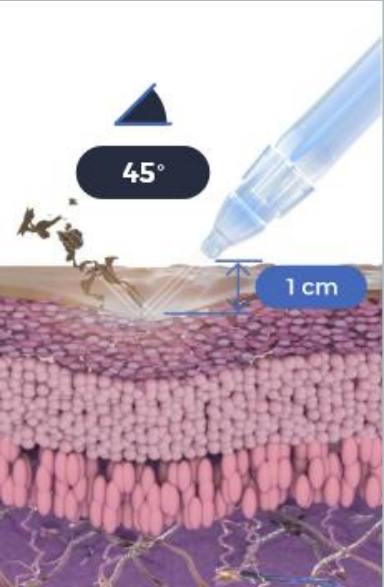

Move in zigzag motions covering the entire area

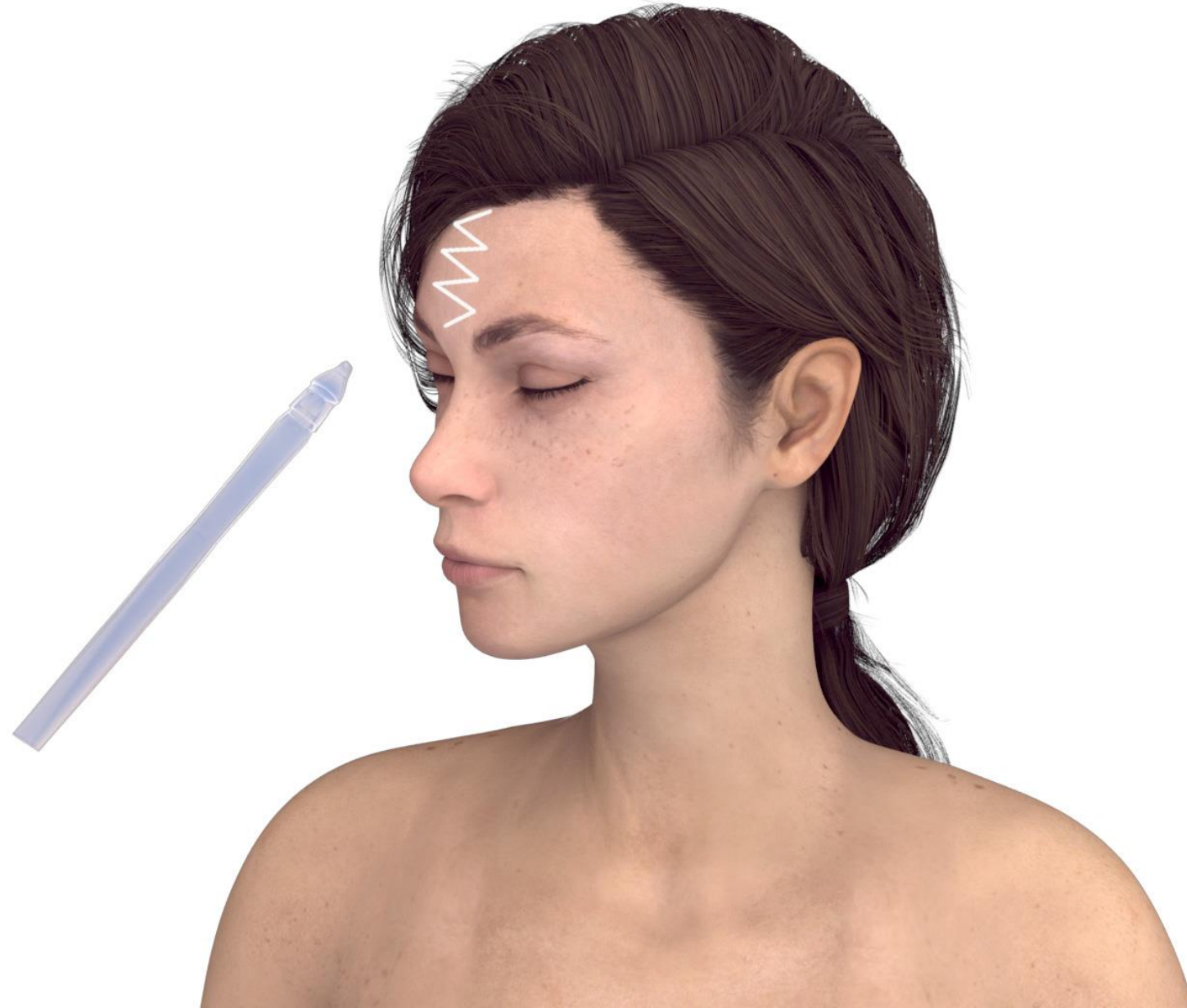




### Sub-Step B

Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

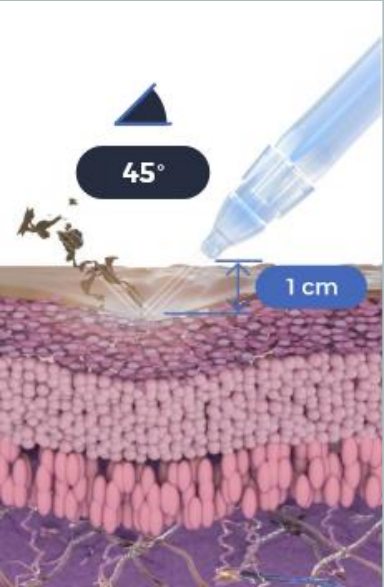

Move in zigzag motions covering the entire area






### Sub-Step B

Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

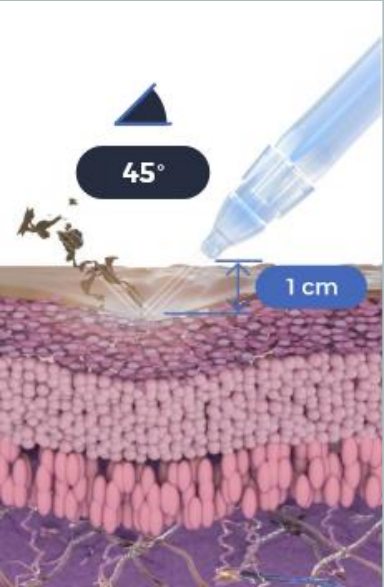

Move in zigzag motions covering the entire area




### Sub-Step B

Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

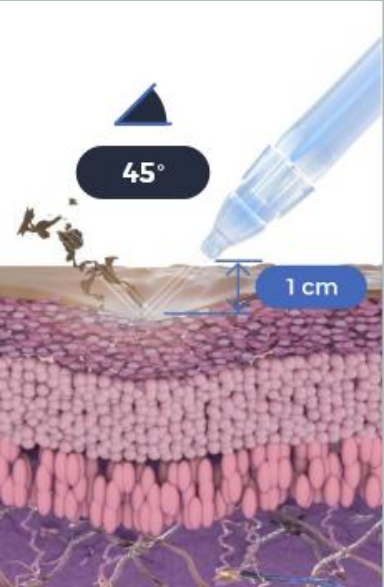

Move in zigzag motions covering the entire area




### Sub-Step B

Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

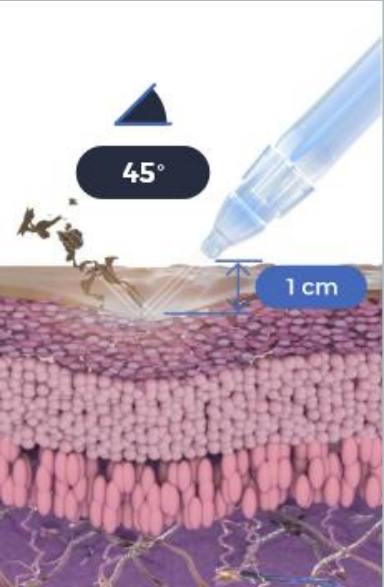

Move in zigzag motions covering the entire area






### Sub-Step B


Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

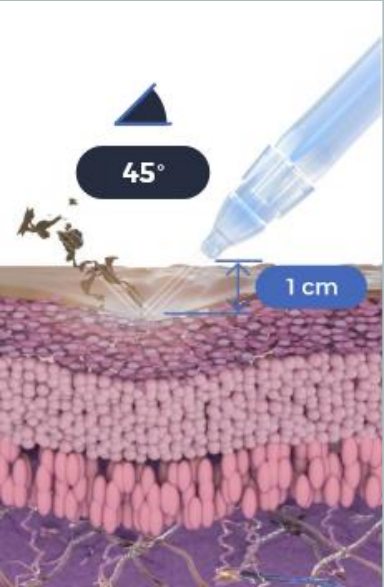

Move in zigzag motions covering the entire area

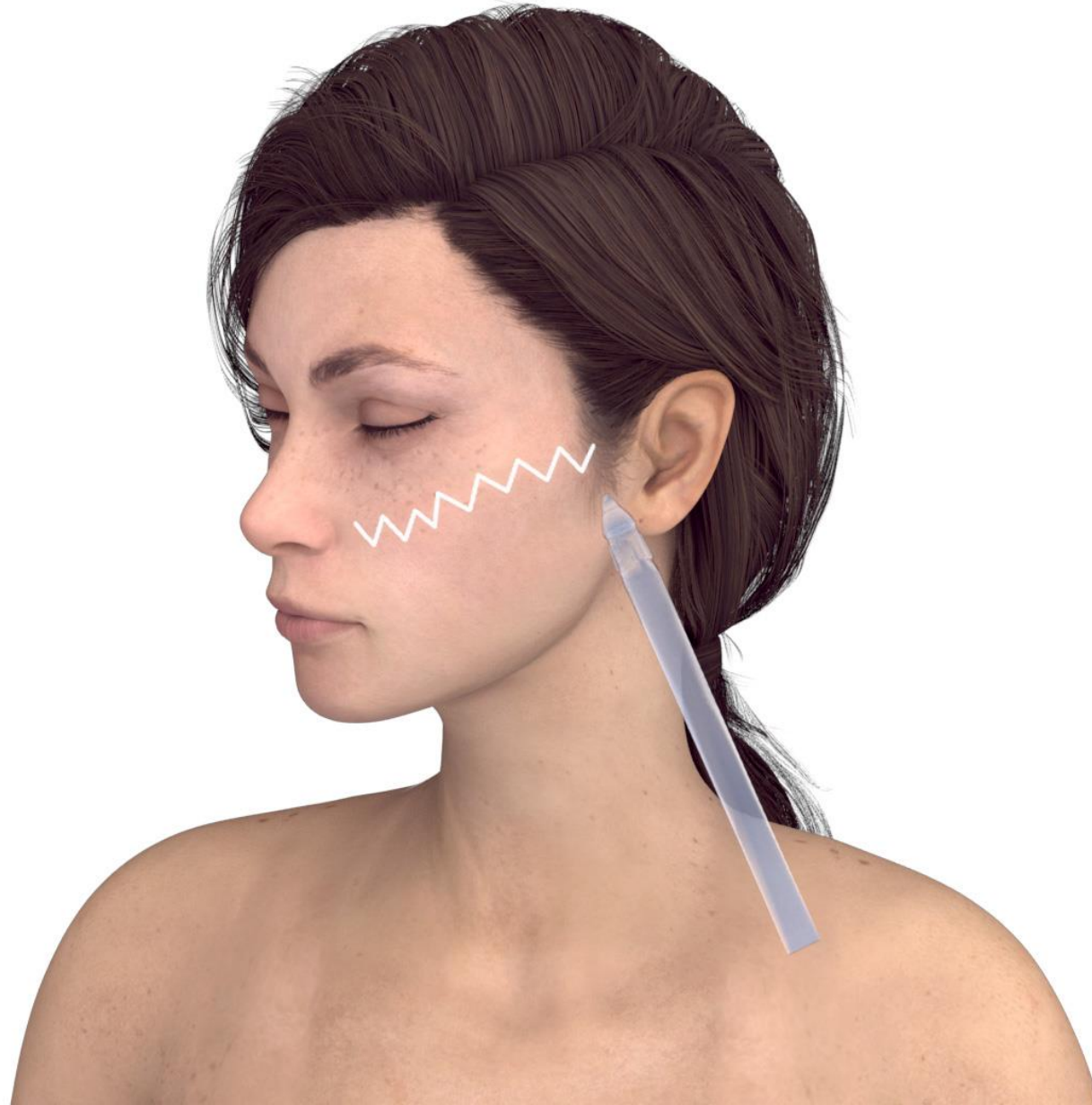




### Sub-Step B


Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

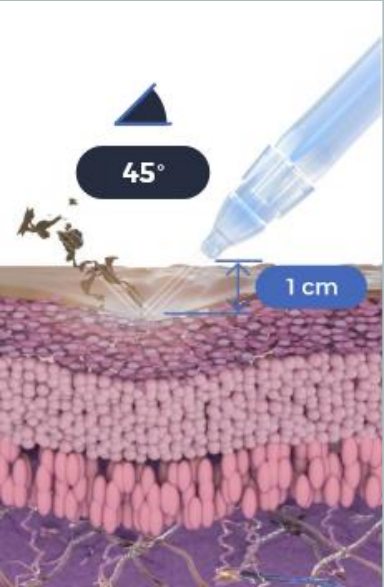

Move in zigzag motions covering the entire area

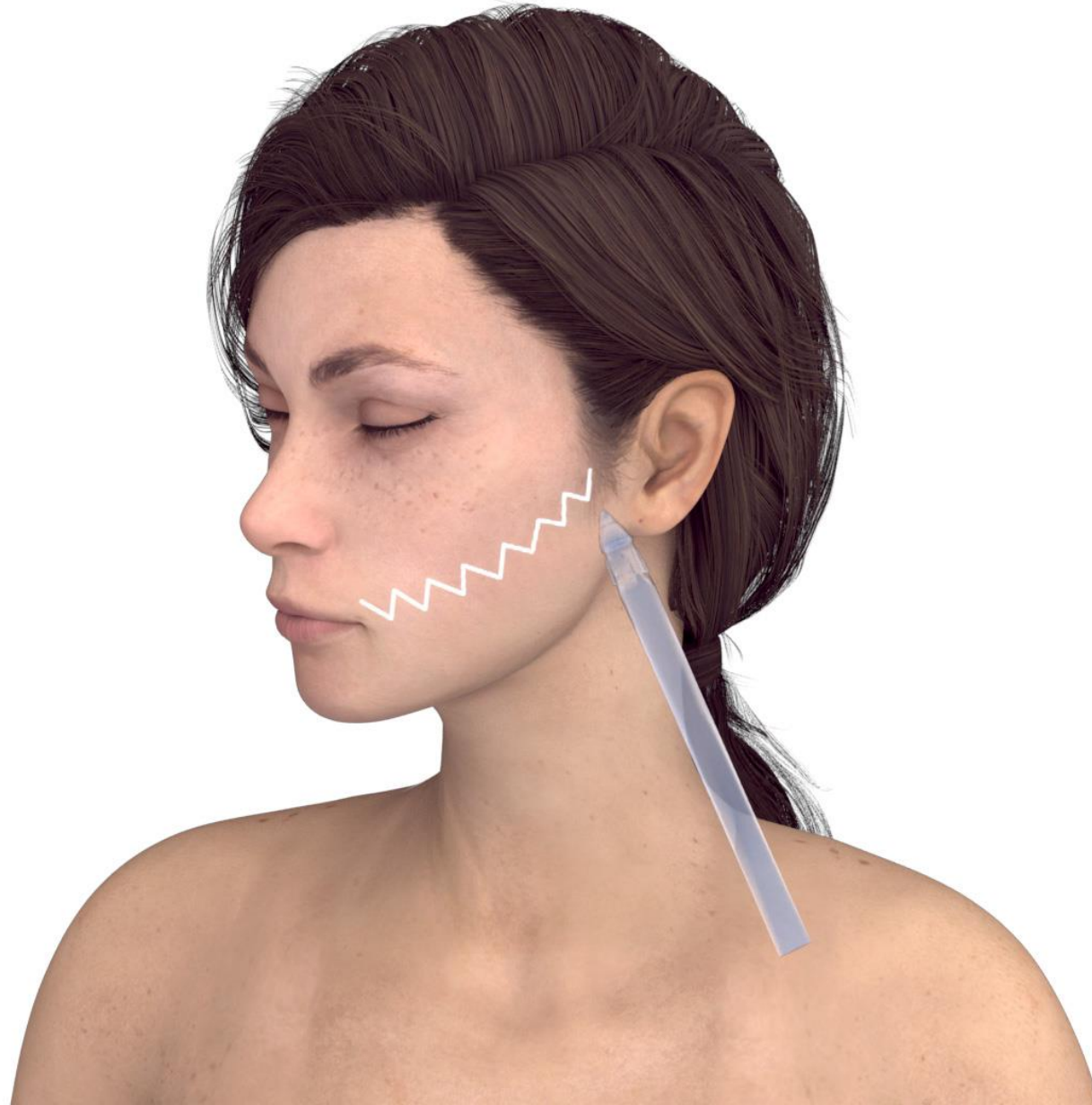


### Sub-Step B


Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

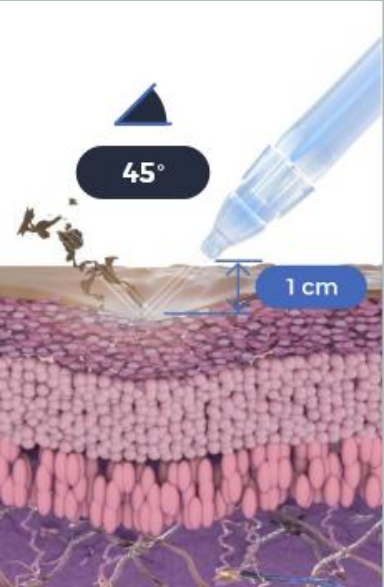

Move in zigzag motions covering the entire area

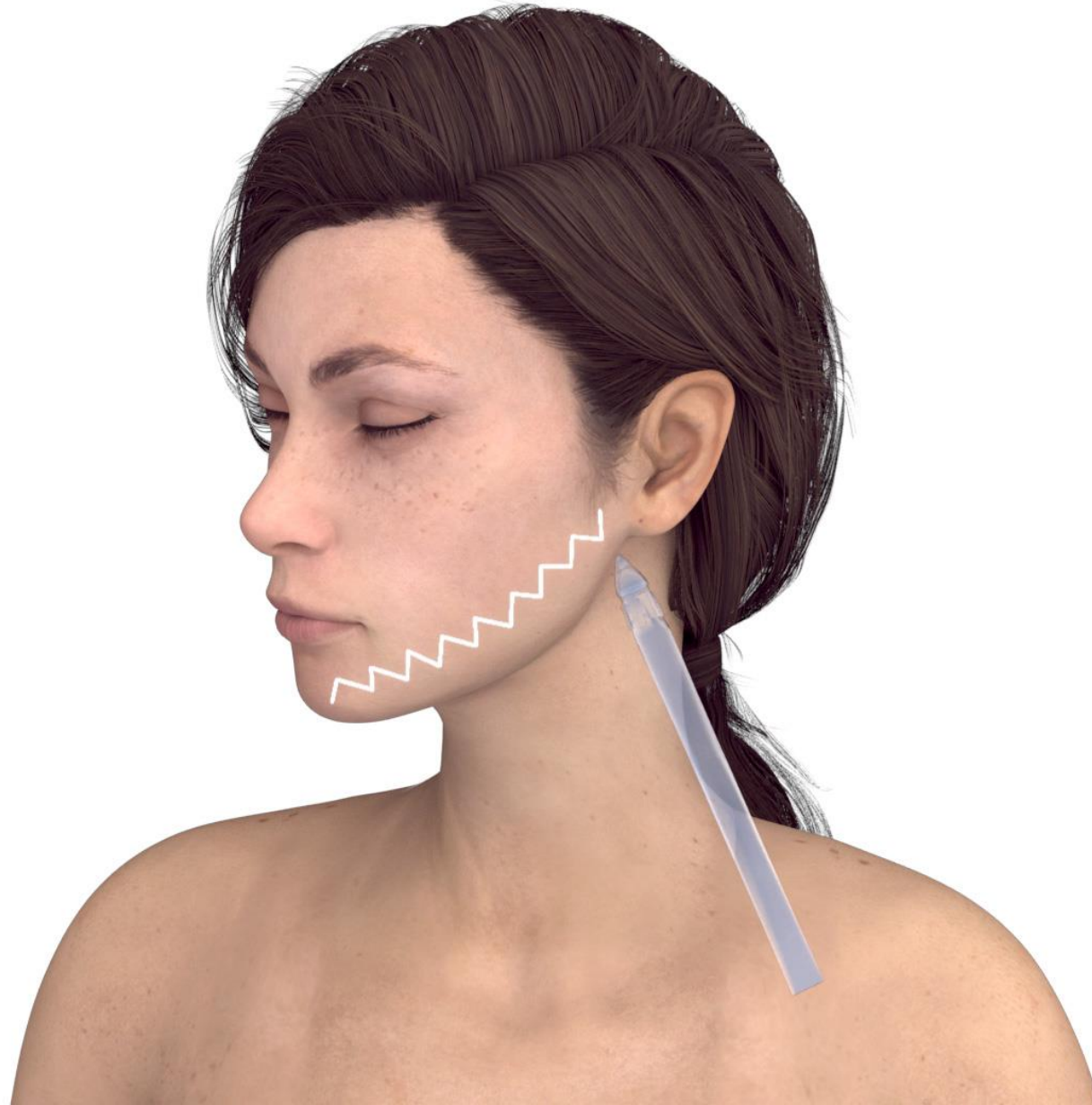


### Sub-Step B


Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

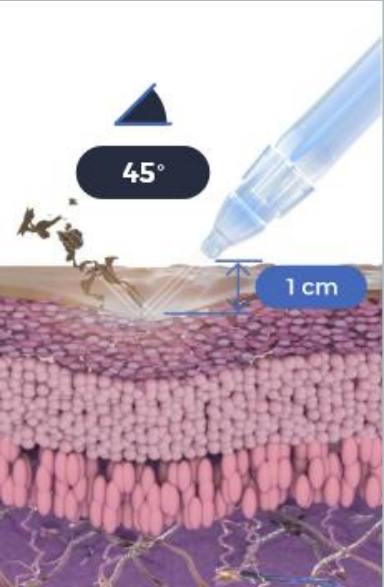

Move in zigzag motions covering the entire area

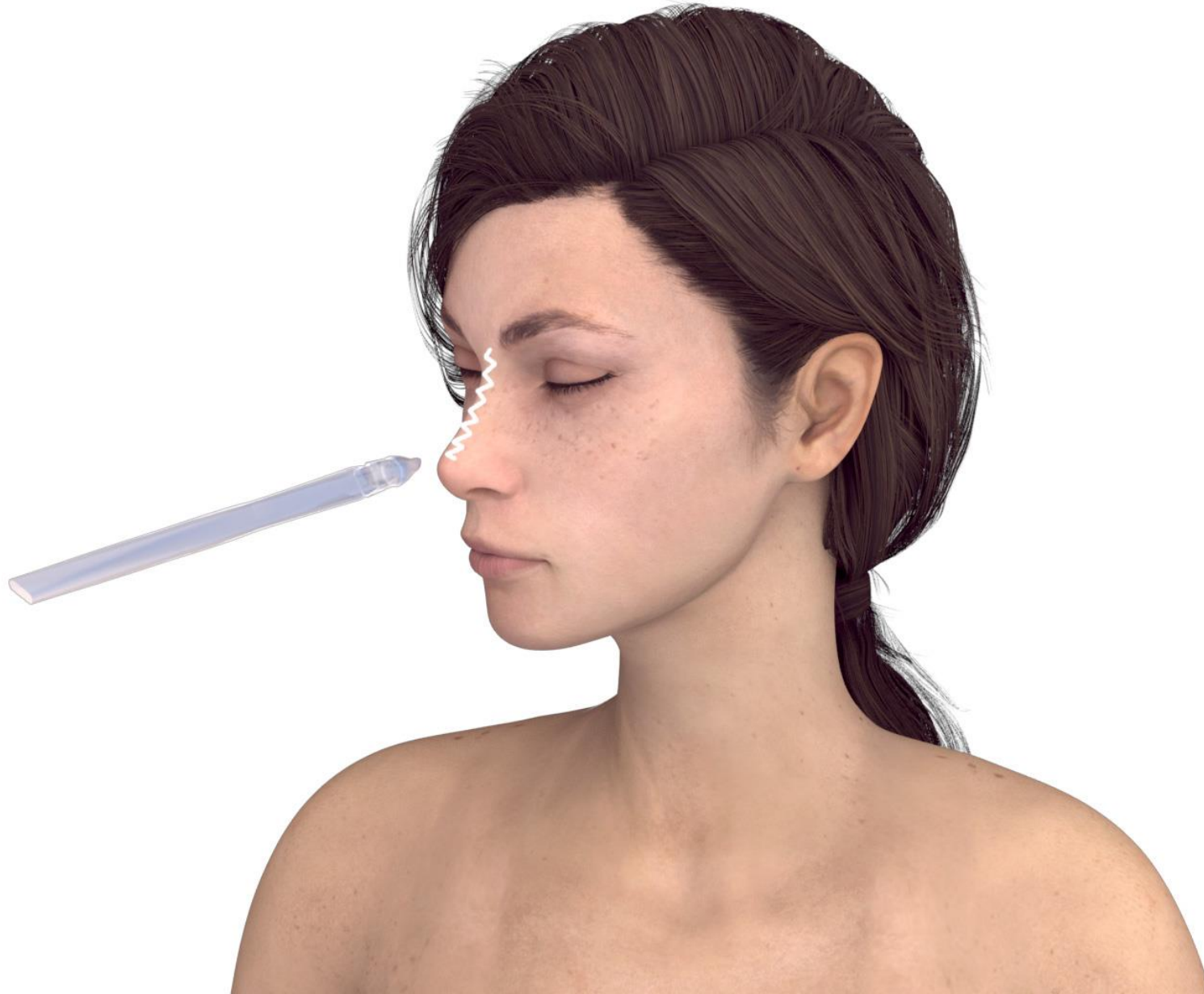




### Sub-Step B

Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

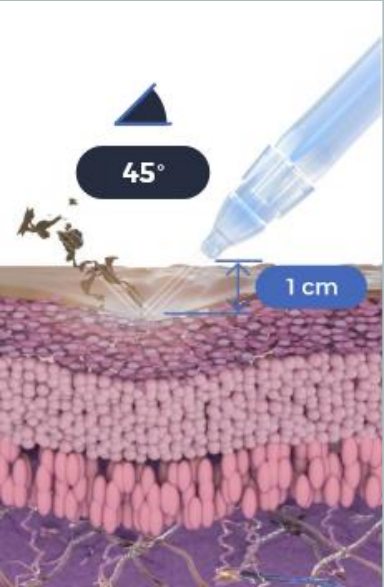

Move in zigzag motions covering the entire area




### Sub-Step B


Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

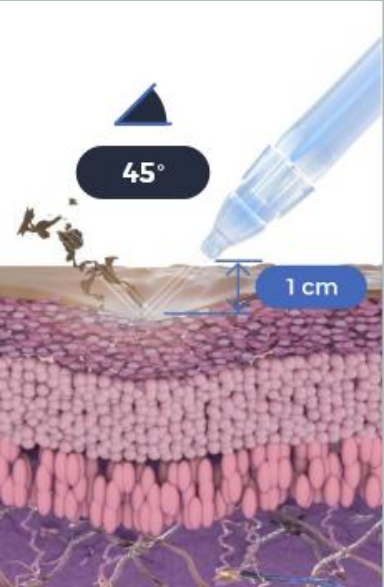

Move in zigzag motions covering the entire area

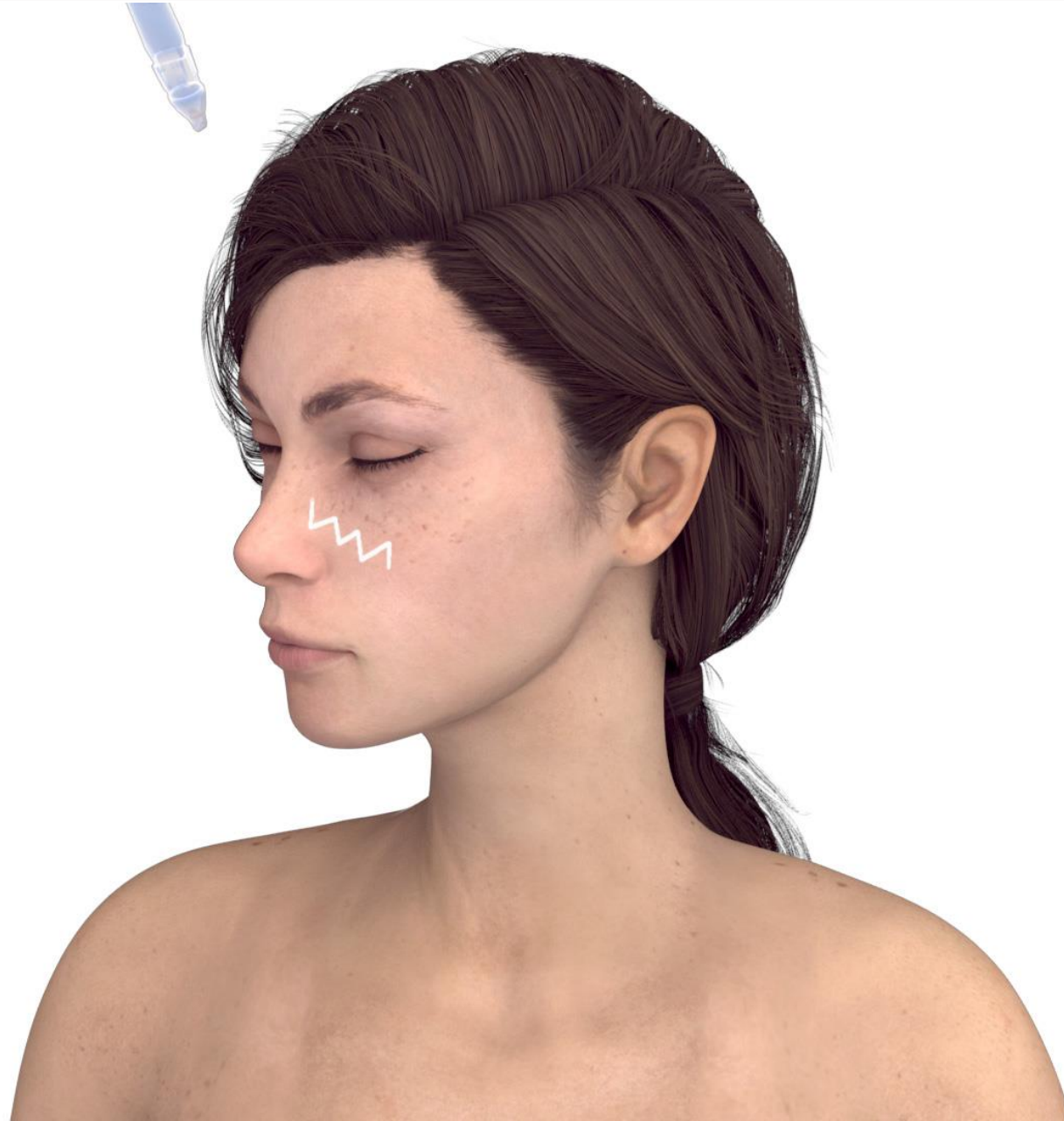


### Sub-Step B


Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro

# Repeat 3 times

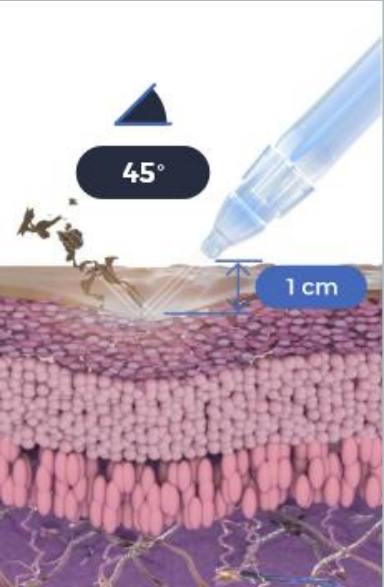

Move in zigzag motions covering the entire area

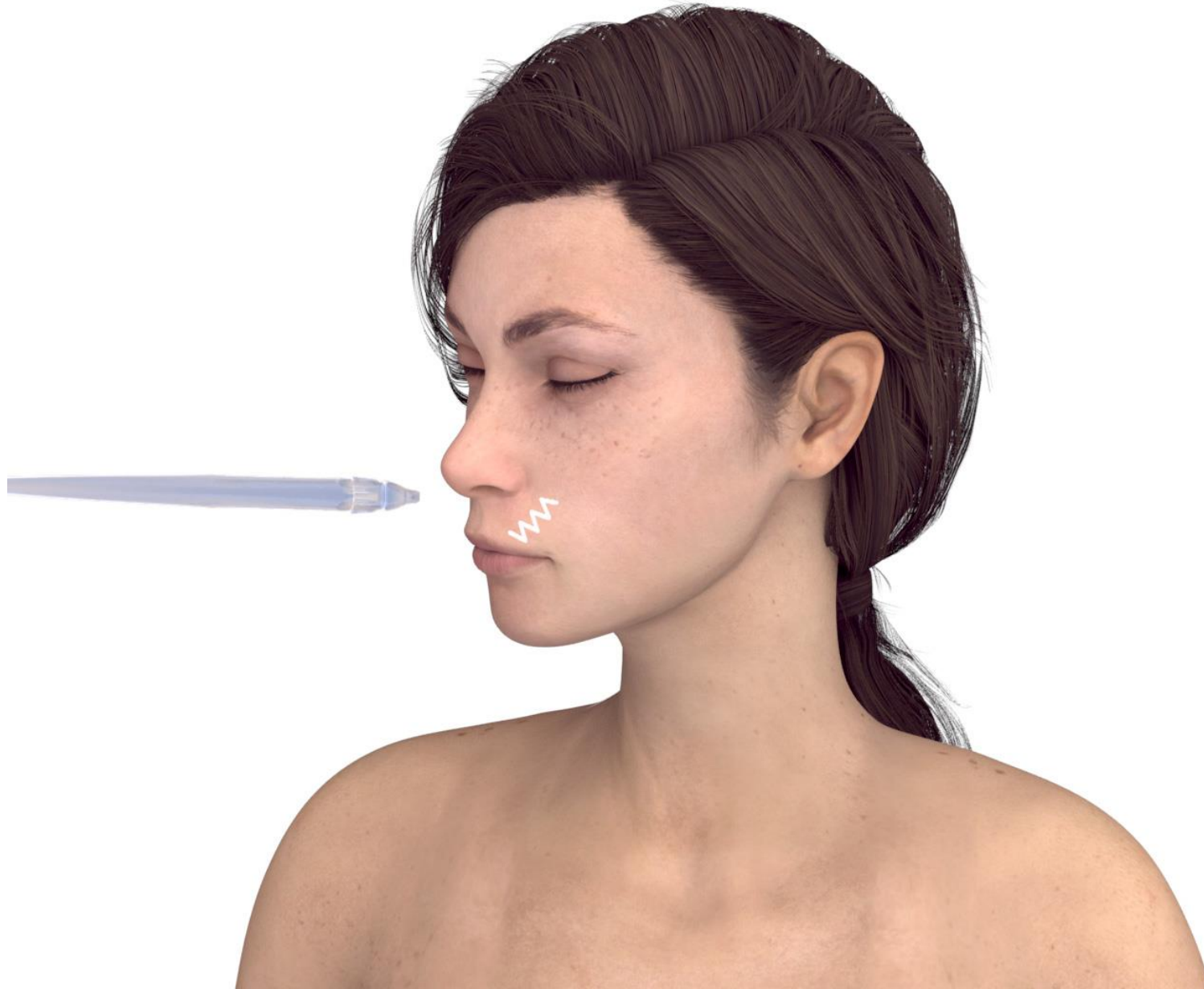




### Sub-Step B

Use the TripleJet handpiece held at 45° to the skin at 1 cm from the skin

Use JetCare Hydro


# Repeat 3 times

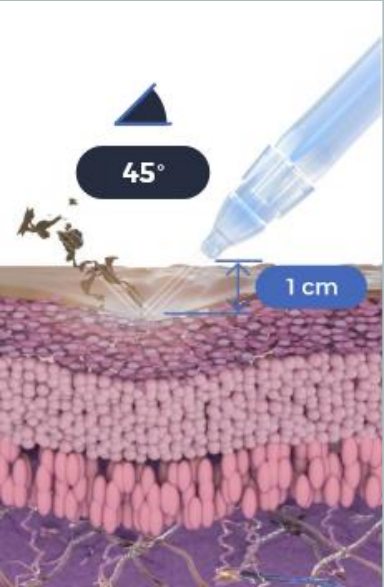

Move in zigzag motions covering the entire area



# Exfoliation with JetPeel

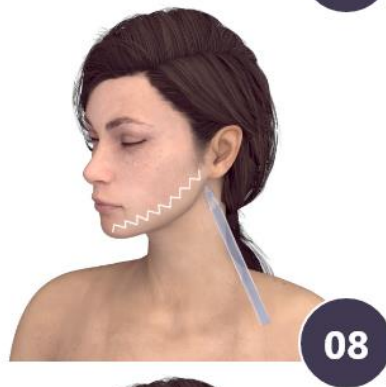

1

Stretch the skin gently with the operator's fingers to define the skin imperfections especially close to the eyes, forehead, and lips where naturally we can have more lines and wrinkles

2

Treat the wrinkles by following the horizontal and vertical direction of the wrinkle itself

3

For other imperfections such as scars, mature skin, stretch marks, spots, large pores, treat the area more than once

## Can you do Exfoliation to sensitive skin?

Yes, you can!

This treatment is recommended also for those who have sensitive or very sensitive skin, unlike other exfoliation treatments.

Exfoliation with the JetPeel exfoliates the skin without peeling or breaking the skin.



# Contraindications

- Open wounds
- Severe sensitivity to cold
- Skin neoplasms
- Active cancer
- Trigeminal nerve inflammation
- Active sinusitis or laryngitis
- Active herpes
- Active bronchial asthma
- Thrombosis
- The system has not been tested or evaluated on pregnant or lactating women, nor on children and infants, therefore is not intended for use on these groups of users

# JetPeel by TavTech

## The perfect trio combining:



**The enduring  
JetPro device**



**The original patented  
JetPeel handpieces**



**The premium  
JetCare solutions**

**Designed exclusively for use together to deliver outstanding results**



# JetPeel by TavTech Training Program

1. JetPeel Technology and Solutions: Training Introduction

2. Lymphatic Massage with JetPeel

2.1. Lymphatic Massage with JetPeel – Face & Neck

2.2. Lymphatic Massage with JetPeel – Body

3. Exfoliation with JetPeel

3.1. Exfoliation with JetPeel – Face & Neck

4. Infusion with JetPeel

4.1. Infusion with JetPeel – Face & Neck

5. Scalp & Hair with JetPeel


6. JetPeel Handpieces







**THANK YOU!**

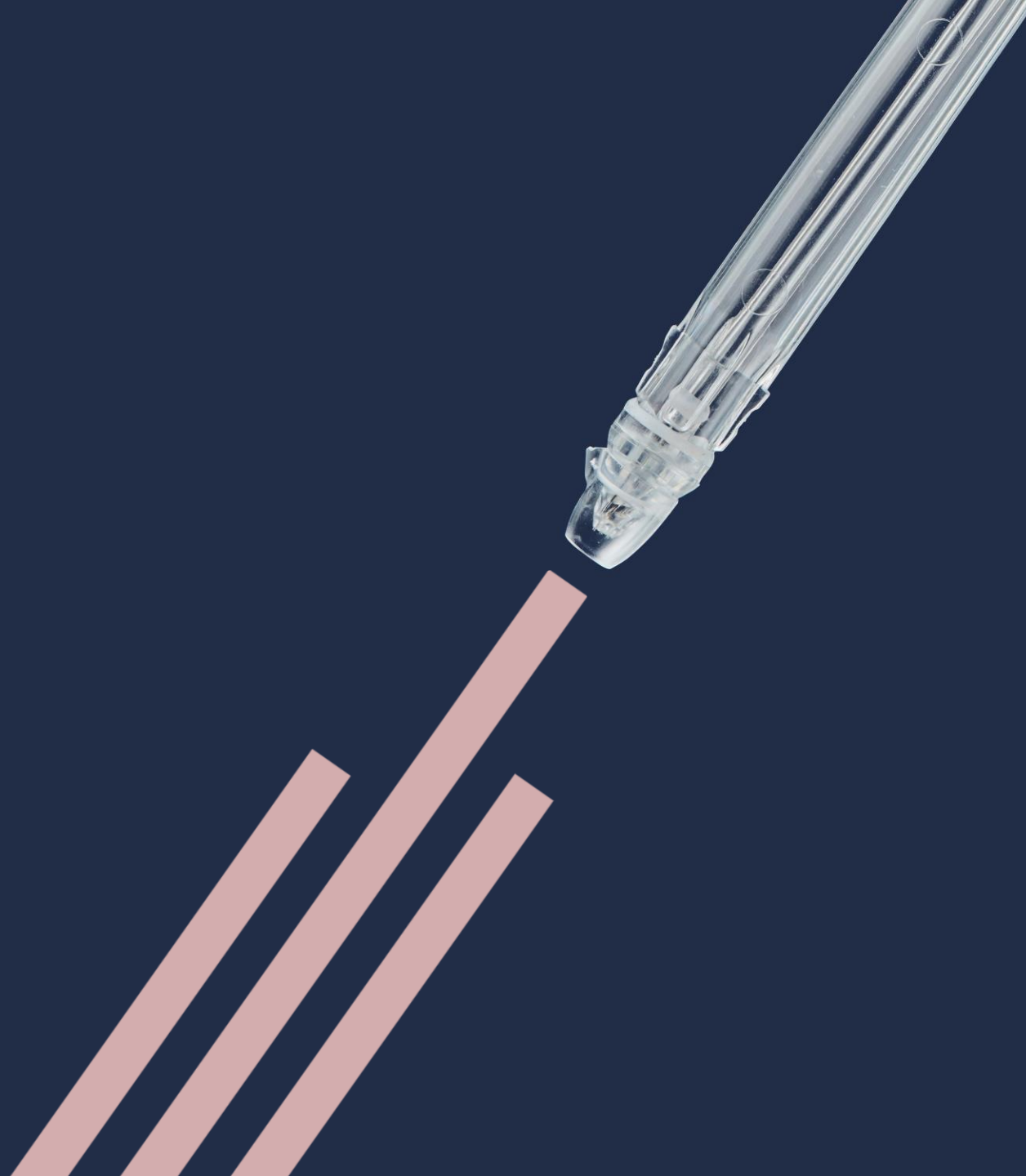

# JetPeel by TavTech Academy

