

Beauty and Prevention

In summer with JetPeel

Presentation by: Laura Masini, International Trainer, TavTech

July 2022





Excess heat, sunlight and air conditioning cause damage to the skin

- **Photoaging**
- **Sunburn**
- **Dehydration**
- **Premature ageing signs**

are just some of the most visible consequences on the skin during summer time

Photoaging is Skin Aging


Photoaging refers to **premature skin changes** attributed to prolonged exposure to UV radiation.

Photoaging appears with:

- Widespread wrinkles
- Dry and flaking skin
- Visible capillaries
- Sunspots
- Loss of skin tone

Photoaging damages the skin

At a young age there is a balance between synthesis of new collagen and the degradation of existing collagen by some molecules.

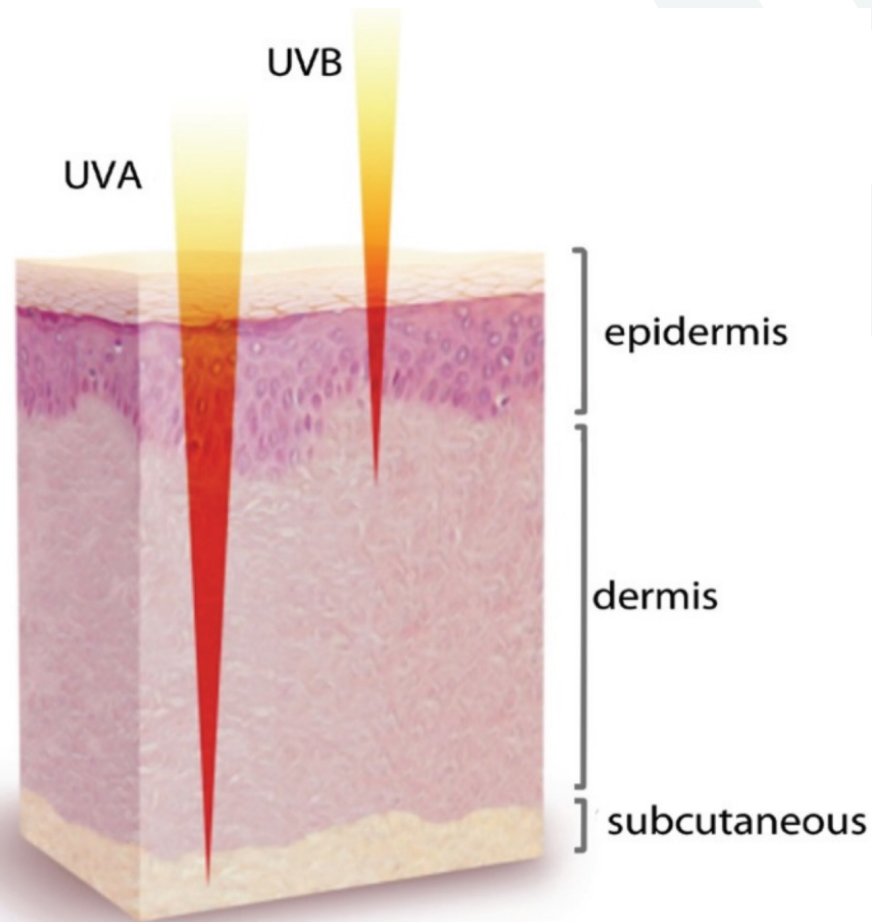

Ultraviolet radiation damages the structures of the skin by:

- Increasing oxidative stress
- Causing instability and imbalance in collagen production
- Accelerating skin aging process

Exposure to sunlight and heat can cause quick skin dehydration



Water is absolutely essential for the normal functioning of the skin cells.

- The dermis contains about 70% water
- The stratum corneum contain about 20-35% water

When our skin is dry, it's because the dermal layer isn't holding enough water to keep our connective tissues hydrated.


Skin cells need water for maintain the skin healthy and young



Skin hydration is important for:

- Rejuvenated, soft, and smooth skin
- Wrinkles prevention
- Protection against skin irritation
- Improved skin complexion
- Reduced puffiness
- Faster healing
- Maintaining sebum production in balance
- Slowing signs of aging
- Tightening the skin
- Rebalancing the skin pH

Sunburn

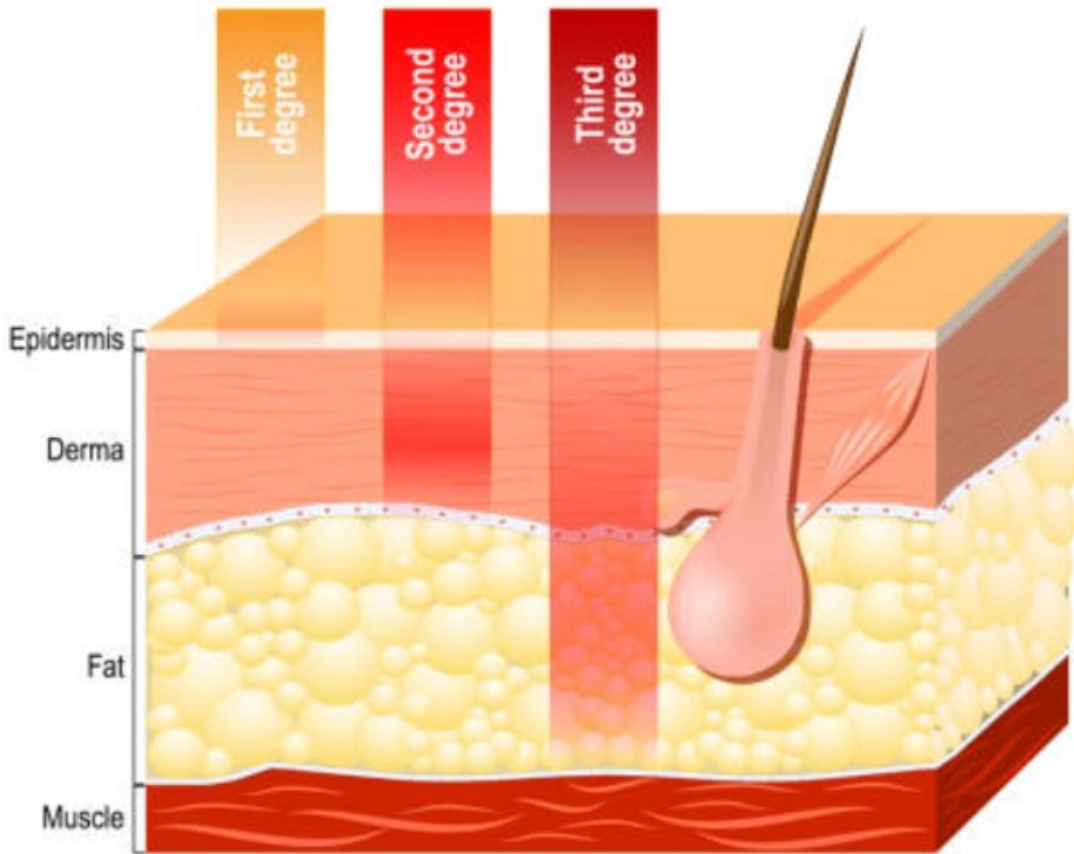

Sunburn is an inflammatory reaction to UV exposure.

The danger goes far beyond any short-term pain, redness and discomfort, because after the sunburn fades away, lasting damage remains.

Sunburn accelerates skin aging and skin cancer.


Light pink is still bad

No matter how mild, every burn is a sign of injury to your skin that can result in premature aging




“Inflammaging” Is Inflammation + Aging

Aging signs are accelerated by Inflammatory state


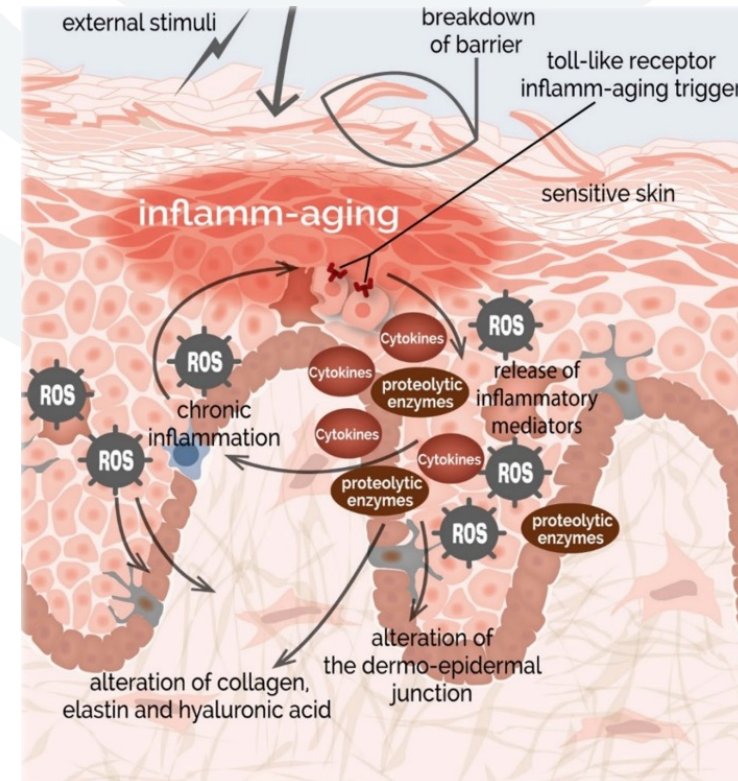



Inflammation exhausts the skin's defense and damages the fundamental skin structure with oxidative stress

Excessive UV exposure can lead to inflammation that increases skin cells damage or oxidative stress, resulting in collagen and elastin degradation, faster aging, wrinkles, and a weak skin barrier.

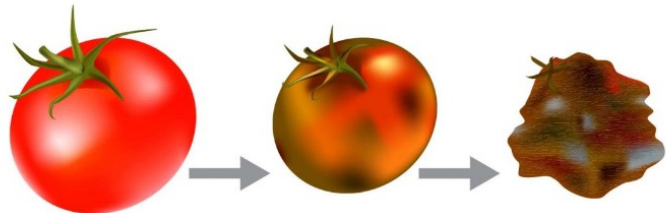

OXIDATIVE STRESS

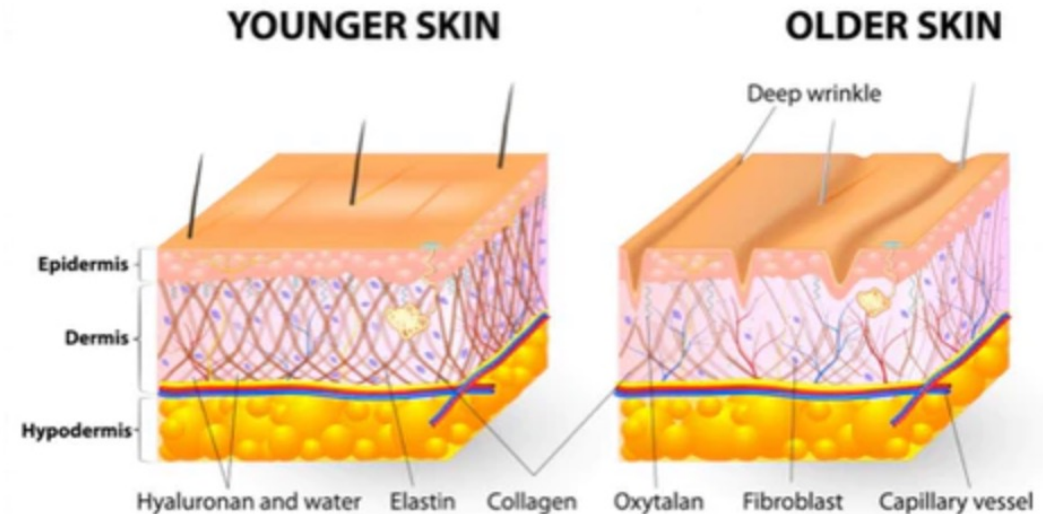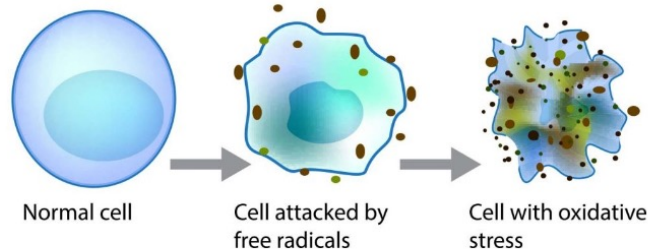



Beauty and prevention
in summer with JetPeel

JetPeel treatment

- Detoxifies
- Recharges and revitalizes the skin
- Regenerates the skin cells
- Moisturizes & hydrates
- Produces NEW collagen,
- Rejuvenation the kin



PROTECTION ANTIOXIDANT

Infusion Jetpeel Treatment

**This is the season for
radiant, hydrated and
protected skin with
JetPeel**

A remarkable cocktail with anti-inflammatory, regenerating, and moisturizing properties contributing to normal skin pigmentation, and protecting the skin from sun damage.



JetPeel Treatment Sequence

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin


STEP
4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

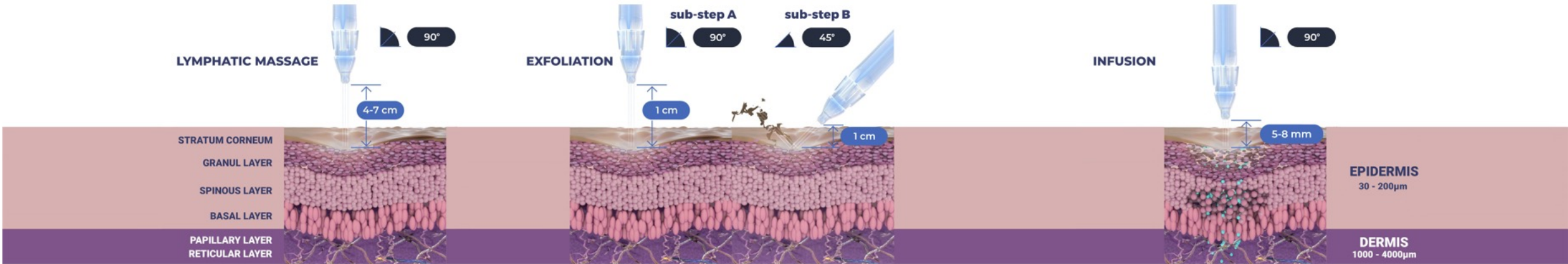

Step 1: Lymphatic Massage

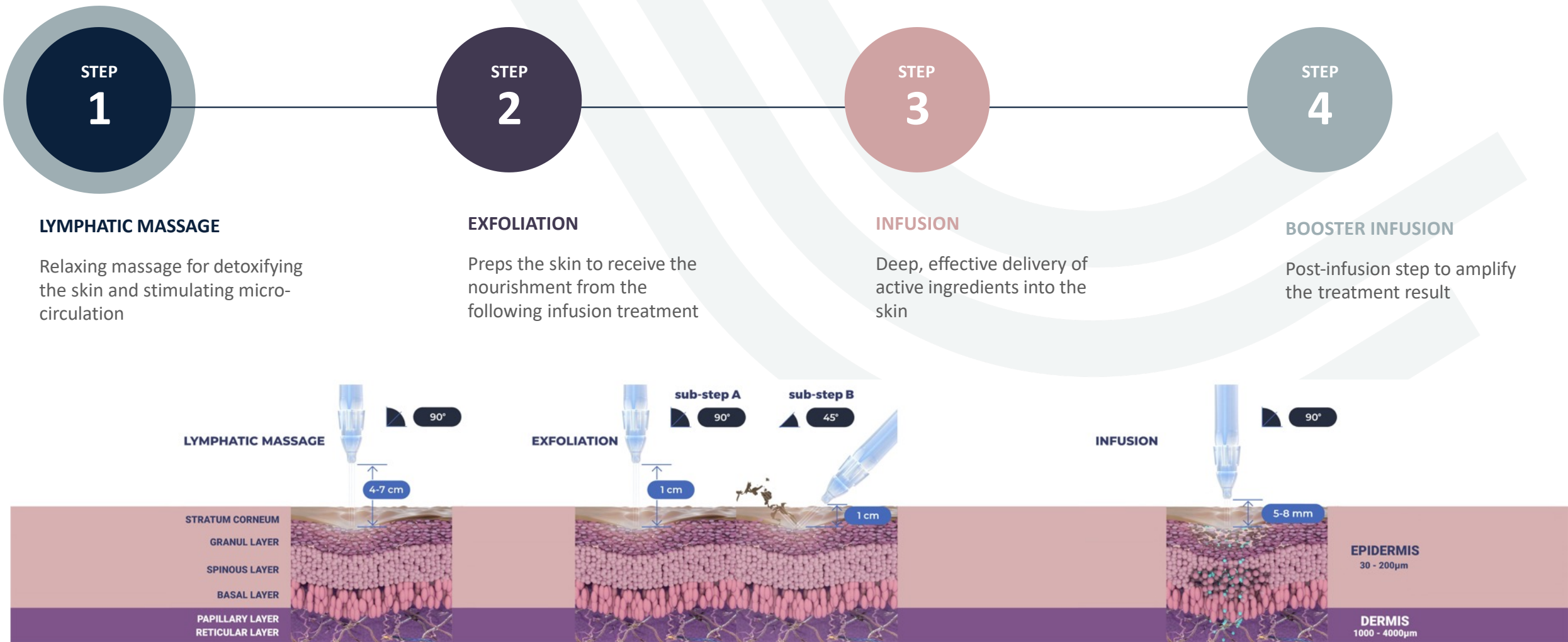



JetCare HYDRO

The skin is healthier, brighter and less prone to aging when it is well hydrated.

JetCare Hydro contains: a high concentration of Hyaluronic Acid and Organic Aloe Vera.

The solution has a powerful effect on:

- anti-redness
- soothing
- moisturising
- hydrating
- antioxidant properties

Step 2: Exfoliation

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP
4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

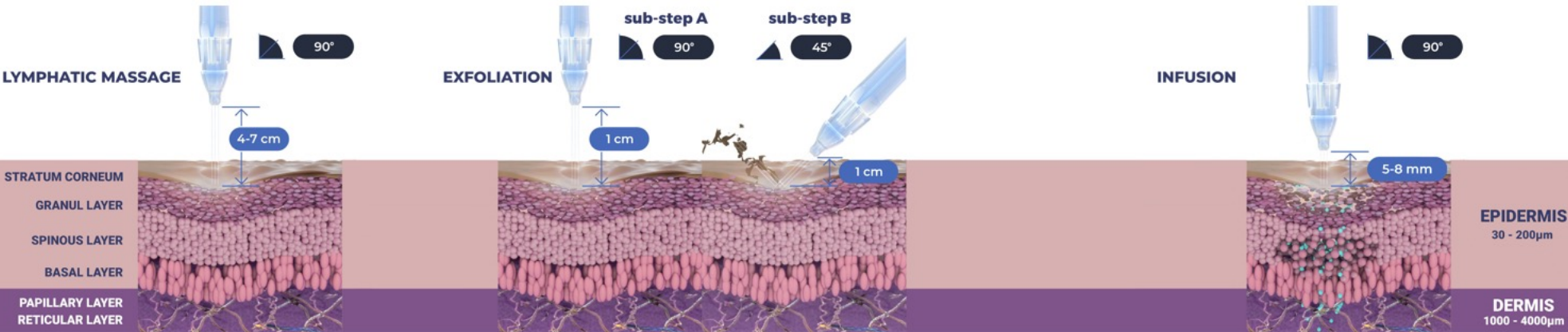



JetCare Renewal Care MANDELIC & SALICYLIC

The gentlest way to achieve an exfoliating and resurfacing effect with a blend of **AHA & BHA**

Contains: Mandelic, Salicylic, Malic, Glycolic, and Lactic acids.

The result is radiant, fresher, and healthy looking skin.

Step 3: Infusion

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP
4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

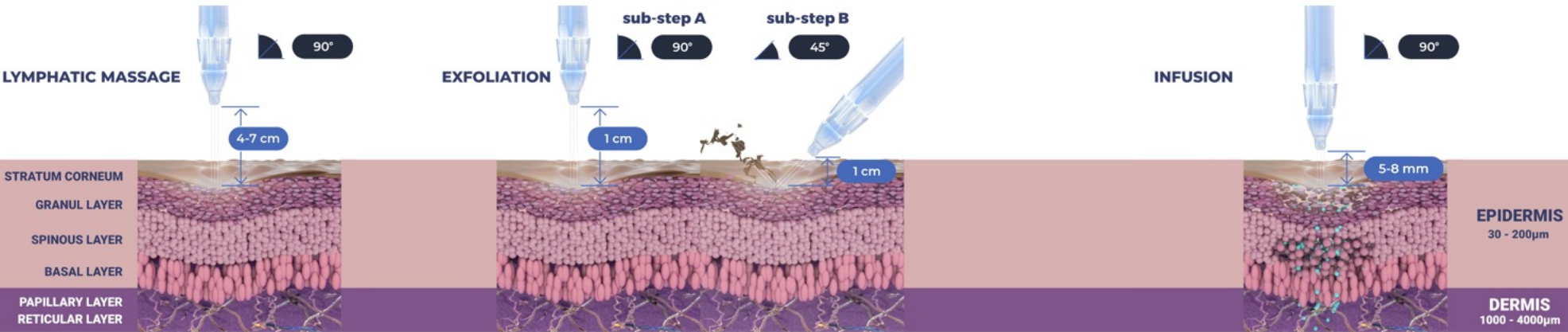



JetCare Selective Care LIGHTENING

Prevent: hyperpigmentation,
skin imperfections

Stimulate: collagen production

Contains: Alpha Arbutin
(α -Arbutin), Niacinamide
(Vit. B₃), Beta-White (β -White)
Alpaflor[®] Gigawhite, Stay –C[®]
and Salicylic Acid



JetCare Anti-Aging Care WIDE RANGE HYALURONIC ACID

Contains: 5 molecular weights of Hyaluronic Acid, as well as Regu-Age® peptide and protein in order to:

- Work as a cell regenerator
- Stimulate the skin's defences against sun damage
- Give the skin tone and elasticity
- Provide natural skin rejuvenation

Step 4: Booster Infusion

1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP 2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP 3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP 4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

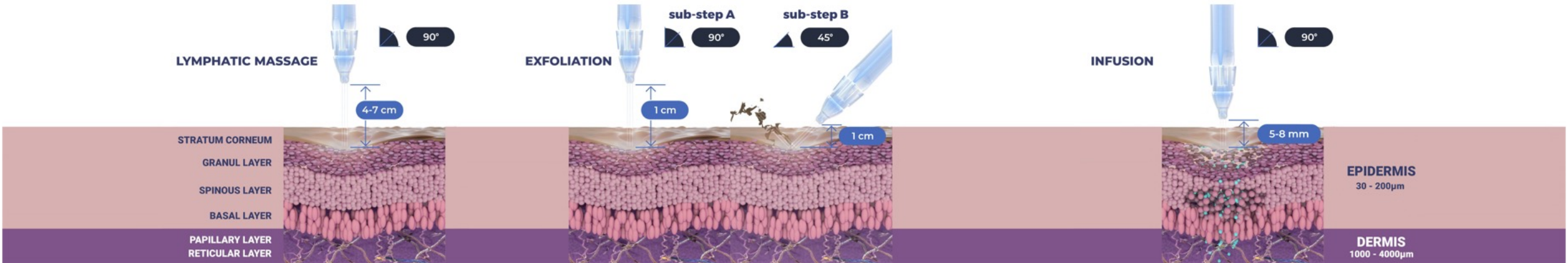



JetCare Boost HYDRATE, REGENERATE, GLOW, TIGHT

JetCare Boost amplifies the treatment results for skin hydration, regeneration, glowing and tightening effects.

Hydrate: Primalhyal™ Ultrafiller, Niacinamide, Betaine

Regenerate: Vitamin A+E, Pentavitin® and Regu-Age®

Glow: Stay-C®, Fruit Extract AHAs

Tight: Vitamin B₃-B₅-B₇

JetPeel by TavTech

The perfect trio combining:



**The enduring
JetPro device**



**The original patented
JetPeel handpieces**



**The premium
JetCare solutions**

Designed exclusively for use together to deliver outstanding results



JETPEEL

Thank you

