

# Glow & Go JetPeel Treatment

Presentation by: **Laura Masini, Global Trainer, TavTech**

September 2022





# Glow & Go Jet Infusion Treatment for immediate results

(in just 15 to 20 minutes)

6 painless JetPeel express treatments, designed to refresh and improve skin quality, with immediately results



# What does glowing skin mean?

The skin looks healthier and vibrant

Makes us look more attractive, sexier, and happier



# The skin “glows” because it is

- Exfoliated
- Hydrated
- Regenerated
- Rejuvenated





Glow your skin all year round, ideal for every age and for all skin types

- Cleanse and protect your skin
- Boost Collagen
- Smooth skin imperfections

Result: flawless skin appearance



Today's customers desire non-invasive and non-ablative aesthetic treatments that improve the skin quality in a natural way, and with immediately results

- Glow & Go is a great opportunity to achieve this in a short time
- Ideal for very dynamic people, special events and for every day life





# Glow & Go treatments focus on:

- Minimizing treatment time and maximizing results
- Boosting collagen and skin rejuvenation
- Infusing the highest quality ingredients into the skin to achieve the desired skincare goals


# Glow & Go is ideal for:



- Lunch break, before a date, interview, important meeting or event
- For anyone and everyone wanting to improve skin texture, to smooth skin imperfections, and to brighten the skin



Jet Infusion is an advanced treatment to infuse nutrients into the skin without needles

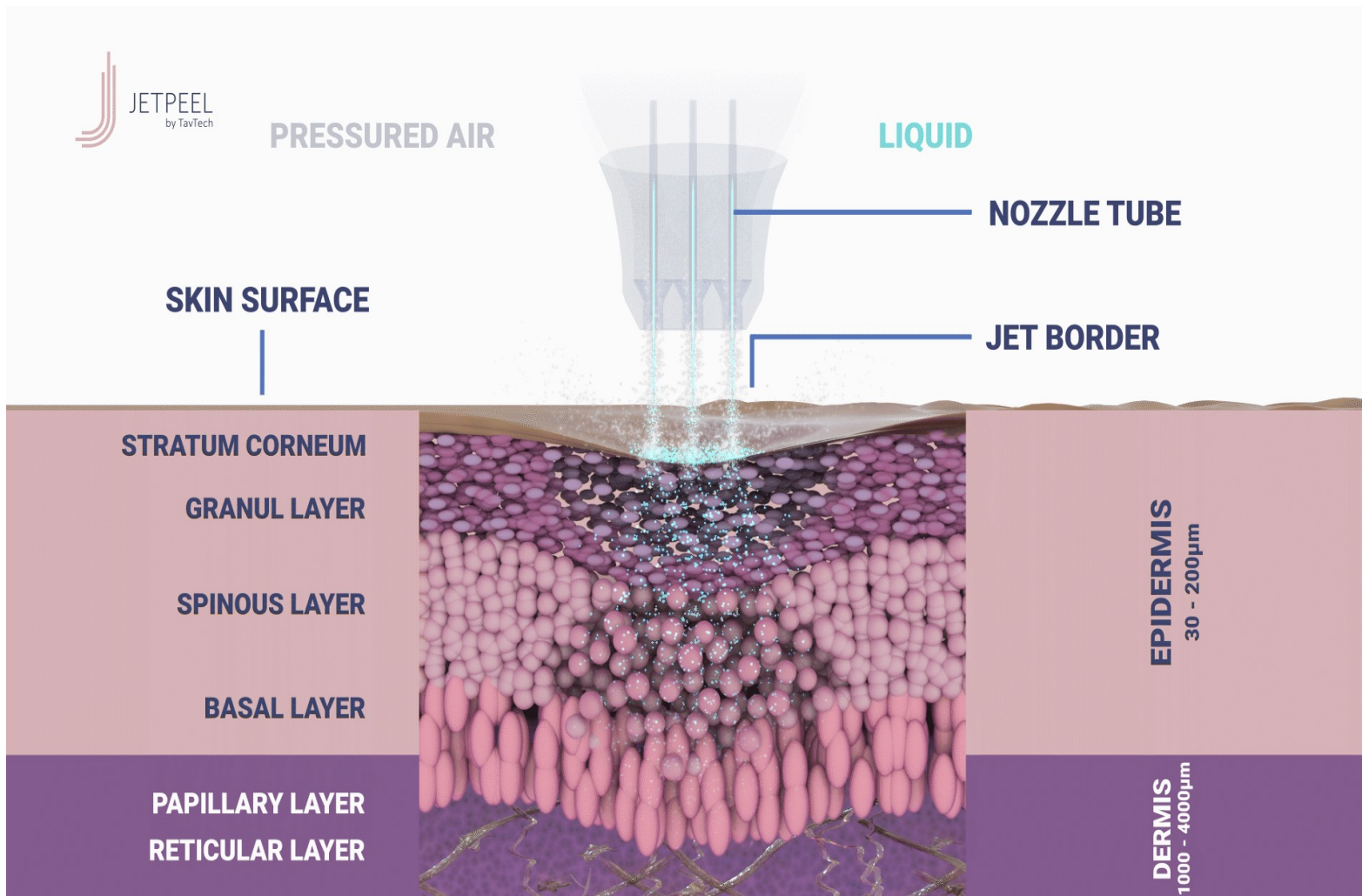

JetPeel hydrates, exfoliates, oxygenates and nourishes the skin without touching it (non-contact)



# How do Glow & Go JetPeel treatments work?

- The jet stream removes dead skin cells, cleans pores, improves micro-circulation and stimulates cellular renewal
- This is followed by the infusion and booster infusion of high concentration active ingredients into the skin for intense skin nourishing

# Glow & Go – Treatment Sequence

STEP  
1

## SKIN MASSAGE & GENTLE EXFOLIATION

Gently remove the dead skin cells and improve the micro-circulation.

STEP  
2

## JET INFUSION

Deep, effective delivery of active ingredients into the skin

STEP  
3


## BOOSTER INFUSION

Post-infusion step to amplify the treatment result





## STEP 1: MASSAGE & GENTLE EXFOLIATION Use: JetCare Hydro

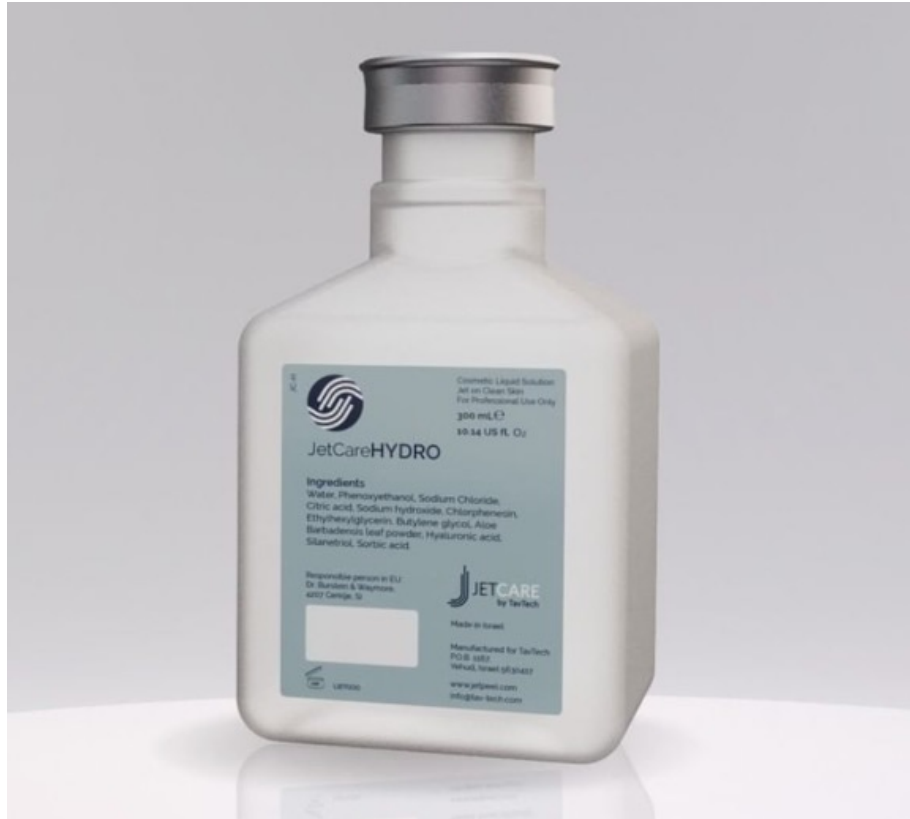

Thanks to the high concentration of **Organic Aloe Vera** and **Hyaluronic Acid**, this step:

- Hydrates
- Stimulates micro-circulation
- Removes dead skin cells
- Exfoliates
- Improves and smooths skin texture
- Reduces inflammation
- Improves skin radiance

## STEP 2: JET INFUSION – Use: JetCare Anti-Aging Care and JetCare Selective Care

### Anti-Aging Care Witch Hazel & Peptides



- Improve the skin tightness
- Improve firmness
- Anti-aging

Use for Glow & Go JetPeel treatments:  
Face Lift-Up Jet Infusion

### Selective Care Argireline® Complex



- Reduce the appearance of wrinkles
- Stimulate fibroblast cells
- Nourishing the skin cells
- Smooth the skin texture

Use for Glow & Go JetPeel treatments:  
Wrinkles Filler Jet Infusion

Argireline® is a trademark of Lipotec S.A. or its affiliates

### Anti-Aging Care Wide Range Hyaluronic Acid



- Keep the skin looking younger
- Reduce inflammation
- Promote the skin's natural barrier function
- Stimulate collagen synthesis
- Provides significant firming

Use for Glow & Go JetPeel treatments:  
Neck rejuvenation Jet infusion  
Skin Hydration Jet infusion

### Selective Care Growth Factor



- Producing collagen and elastin
- Healing cells
- Dividing cells
- Preventing scar tissue formation
- Restoring even coloration in skin
- Reducing unwanted inflammation
- Remodelling tissue

Use for Glow & Go JetPeel Treatment:  
Celebrity Skin Rejuvenation Jet Infusion

## STEP 3: BOOSTER INFUSION – Use: JetCare Boost

### JetCare Boost Hydrate



- Reduce deep wrinkles (on upper and lower part of the face)
- Improve skin texture and skin rejuvenation

Use for Glow & Go JetPeel treatments:  
Wrinkle Filler Jet Infusion  
Silky Skin Hydrate Jet Infusion

### JetCare Boost Glow



- Enhances youthful appearance
- Degreases the appearance of age spots
- Boost photoprotection
- Promote collagen

Use for Glow & Go JetPeel treatments:  
Neck Rejuvenation Jet Infusion

### JetCare Boost Regenerate



- Revitalise the skin
- Protect the cells from damage
- Antioxidant
- Increase skin firmness
- Reduce wrinkles

Use for Glow & Go JetPeel treatments:  
Neck Rejuvenation Jet Infusion

### JetCare Boost Tight



- Improve the skin firmness
- Anti-free radicals
- Nourish
- Leaving it revitalised, hydrated, and plump

Use for Glow & Go JetPeel treatments:  
Face Lif-Up Jet Infusion



# GLOW & GO TREATMENT GUIDELINES

---



| TREATMENT GUIDELINES   | TREATMENT INDICATION  | SKIN MASSAGE & GENTLE EXFOLIATION                                   | JET INFUSION   | BOOSTER INFUSION  | TIME / NOS. OF SESSIONS & MAINTENANCE   |
|--|---|---|--|---|---|
| <b>FACE LIFT-UP</b><br>JETPEEL INFUSION<br><br>TRANS-EPIDERMAL INFUSION OF <b>WITCH HAZEL &amp; PEPTIDES</b> AND <b>VITAMIN B COMPLEX</b>                | SKIN TIGHTENING AND LIFT-UP   | <b>JET CARE HYDRO</b><br>WITH TRIPLEJET HP<br>AT 45° FOR 10 MINUTES | <b>WITCH HAZEL &amp; PEPTIDES (5ML) FROM THE JETCARE ANTI-AGING CARE SERIES</b><br>USE TRIPLEJET HP<br>AT 90° FOR 5 MINUTES  | <b>JETCARE BOOST TIGHT (3ML)</b><br>USE SINGLEJET HP<br>AT 90° FOR 5 MINUTES      | 20 MINUTES<br>NOS. OF SESSIONS: 6 (ONCE OR TWICE A WEEK)<br><b>MAINTENANCE: 1</b><br>TREATMENT ONCE A MONTH |
| <b>NECK REJUVENATION</b><br>JETPEEL INFUSION<br><br>TRANS-EPIDERMAL INFUSION OF <b>HYALURONIC ACID, VITAMIN A+E</b> AND <b>PEPTIDES</b>                  | PREMATURE NECK AGING SIGNS, SKIN TIGHTENING AND LIFT-UP   | <b>JET CARE HYDRO</b><br>WITH TRIPLEJET HP<br>AT 45° FOR 5 MINUTES  | <b>WIDE RANGE HYALURONIC ACID (5ML) FROM THE JETCARE ANTI-AGING CARE SERIES</b><br>USE TRIPLEJET HP<br>AT 90° FOR 5 MINUTES  | <b>JETCARE BOOST REGENERATE (3ML)</b><br>USE TRIPLEJET HP<br>AT 90° FOR 5 MINUTES | 15 MINUTES<br>NOS. OF SESSIONS: 6 (ONCE OR TWICE A WEEK)<br><b>MAINTENANCE: 1</b><br>TREATMENT ONCE A MONTH |
| <b>WRINKLE FILLER</b><br>JETPEEL INFUSION<br><br>TRANS-EPIDERMAL INFUSION OF <b>ARGIRELINE®,</b> AND <b>HYALURONIC ACID</b>                              | REDUCE THE VOLUME OF WRINKLES IN THE UPPER AND LOWER PARTS OF THE FACE  | <b>JET CARE HYDRO</b><br>WITH TRIPLEJET HP<br>AT 45° FOR 10 MINUTES | <b>ARGIRELINE® COMPLEX (5ML) FROM THE JETCARE ANTI-AGING SERIES</b><br>USE TRIPLEJET HP<br>AT 90° FOR 5 MINUTES<br><small>Argireline® is a trademark of Lipotec S.A. or its affiliates</small> | <b>JETCARE BOOST HYDRATE (3ML)</b><br>USE SINGLEJET HP<br>AT 90° FOR 5 MINUTES    | 20 MINUTES<br>NOS. OF SESSIONS: 6 (ONCE OR TWICE A WEEK)<br><b>MAINTENANCE: 1</b><br>TREATMENT ONCE A MONTH |
| <b>LIP PLUMPING</b><br>JETPEEL INFUSION<br><br>TRANS-EPIDERMAL INFUSION OF <b>PRIMALHYALU™</b><br><b>ULTRAFILLER</b> THE ULTIMATE <b>HYALURONIC ACID</b> | LIP VOLUMISING, PLUMPING AND FILLER EFFECT  | <b>JET CARE HYDRO</b><br>WITH TRIPLEJET HP<br>AT 45° FOR 1 MINUTE   |  | <b>JETCARE BOOST HYDRATE (1ML)</b><br>USE SINGLEJET HP<br>AT 90° FOR 4 MINUTES    | 5 MINUTES<br>NOS. OF SESSIONS: 6 (ONCE OR TWICE A WEEK)<br><b>MAINTENANCE: 1</b><br>TREATMENT ONCE A MONTH  |
| <b>CELEBRITY SKIN REJUVENATION</b><br>JETPEEL INFUSION<br><br>TRANS-EPIDERMAL INFUSION OF <b>GROWTH FACTOR</b> AND <b>STAY-C® VITAMIN</b>                | FLAWLESS, GLOWING SKIN, PERFECT FOR YOUTUBERS, BLOGGERS, AND FOR THOSE WHO WORK WITH SOCIAL MEDIA AND ON CAMERA | <b>JET CARE HYDRO</b><br>WITH TRIPLEJET HP<br>AT 45° FOR 10 MINUTES | <b>GROWTH FACTOR (5ML) FROM THE JETCARE SELECTIVE CARE SERIES</b><br>USE TRIPLEJET HP<br>AT 90° FOR 5 MINUTES  | <b>JETCARE BOOST GLOW (3ML)</b><br>USE SINGLEJET HP<br>AT 90° FOR 5 MINUTES       | 20 MINUTES<br>NOS. OF SESSIONS: 6 (ONCE OR TWICE A WEEK)<br><b>MAINTENANCE: 1</b><br>TREATMENT ONCE A MONTH |
| <b>SILKY SKIN HYDRATION</b><br>JETPEEL INFUSION<br><br>TRANS-EPIDERMAL INFUSION OF <b>HYALURONIC ACID</b>  | ENHANCES MOISTURE, REVITALISES, SMOOTHS AND HYDRATES  | <b>JET CARE HYDRO</b><br>WITH TRIPLEJET HP<br>AT 45° FOR 5 MINUTES  | <b>WIDE RANGE HYALURONIC ACID (5ML) FROM THE JETCARE ANTI-AGING CARE SERIES</b><br>USE TRIPLEJET HP<br>AT 90° FOR 5 MINUTES  | <b>JETCARE BOOST HYDRATE (3ML)</b><br>USE SINGLEJET HP<br>AT 90° FOR 5 MINUTES    | 15 MINUTES<br>NOS. OF SESSIONS: 6 (ONCE OR TWICE A WEEK)<br><b>MAINTENANCE: 1</b><br>TREATMENT ONCE A MONTH |

# JetPeel by TavTech

## The perfect trio combining:



The enduring  
JetPro device



The original patented  
JetPeel handpieces



The premium  
JetCare solutions

Designed exclusively for use together to deliver outstanding results





JETPEEL

Thank you

