

Get Ready For Winter with JetPeel Treatments

Presentation by: **Laura Masini, Global Trainer, TavTech**

November 2022



Winter: threats and damage to the skin

Keep your skin in good shape
through the cold months with
special winter **Glow & Go
JetPeel Treatments**

Ideal for all skin types and all skin ages





During Winter

Cold wind and dry air remove the natural oils from the epidermis, making the skin dry and prone to irritation.

Thermal shock due to the indoor - outdoor temperature differentials damage the skin

Consequences:

- Frequent and permeant skin redness due to continuous vasodilatation and vasoconstriction of the blood vessels in the skin
- Vascular skin disorder such as rosacea may be accentuated in winter






**Cold weather reduces
skin blood flow and
oxygenation**




Less skin nutrition

Slows natural skin regeneration

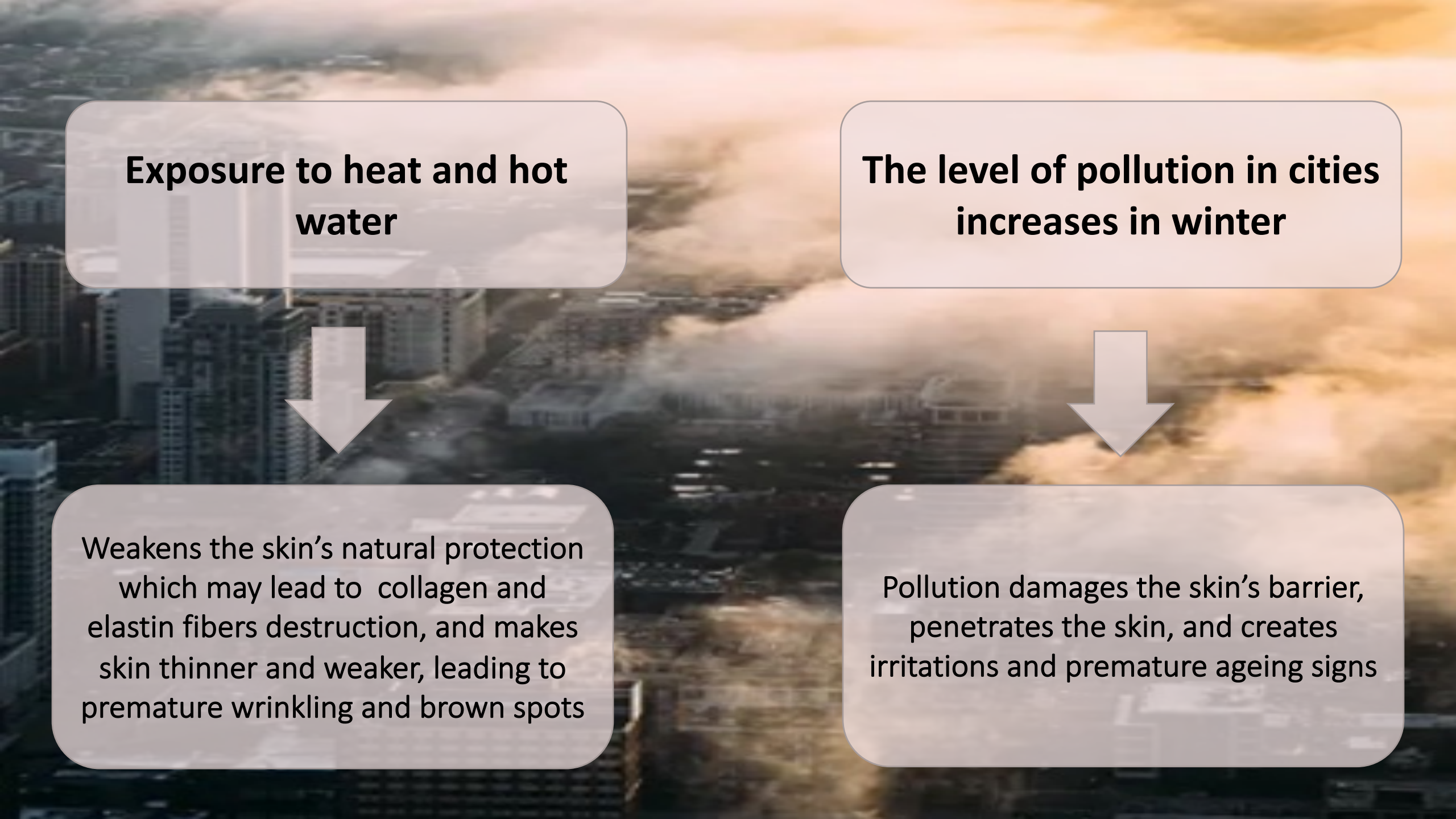


Exposure to heat and hot water

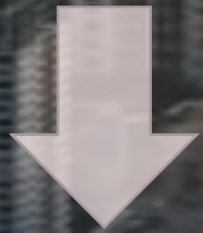

Weakens the skin's natural protection which may lead to collagen and elastin fibers destruction, and makes skin thinner and weaker, leading to premature wrinkling and brown spots

The level of pollution in cities increases in winter

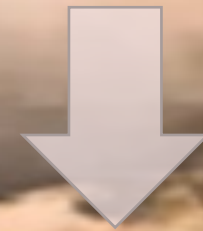

Pollution damages the skin's barrier, penetrates the skin, and creates irritations and premature ageing signs



Winter sunlight

The winter sun is powerful enough to damage the skin. Especially with snow, when the sun's light reflects up to 80 per cent more. This means that the sun's rays have almost double the impact on exposed skin (usually face).

Winter sports

Skiing, jogging and cycling in extreme cold deplete the skin's natural moisture and weaken the skin barrier, leading to dry, flaky, and irritation-prone skin.



In **preparation**
for Winter,
3 Glow & Go
short JetPeel
treatments to
protect your
skin



Get ready for winter with
JetPeel's short treatments

Glow & Go



Winter In The City

Anti-Pollution



Reduce the harmful effects of pollution

Main active ingredients:

Glycolic Acid, safeguards against pollutants, gently exfoliates and cleans the skin

Alpha-Arbutin counteracts hyper-pigmentation as a consequence of pollution

Stay-C Vitamin protects the skin from free-radicals, works as anti-wrinkles and anti-dark spots

Vitamin B complex (B₃, B₅, B₇) protects the skin from the environment

Soothing Remedy

Extra attention to sensitive skin



Protect sensitive skin in cold weather

Main active ingredients:

Azeoglicina® improves skin radiance, elasticity and hydration

Ronacare® Troxerutin offers powerful soothing properties

Organic Aloe with instant calming and healing effect

Nordic cotton extract hydrates and nourishes


Betaine enhances the skin texture with an anti-aging effect

Niacinamide improves skin tone, fights antioxidants

PrimalHyal™ Ultrafiller improves skin appearance and counteract aging signs

Enjoy Your Skii

Treat your skin while you enjoy outdoor sports

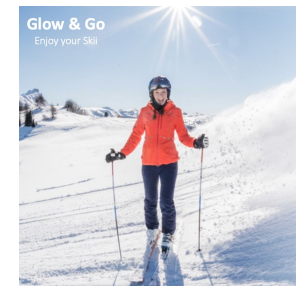

Hydrate the skin & protect it from extreme cold

Main active ingredients:

Hyaluronic Acid instantly hydrates and moisturizes the skin, provides anti-aging benefits, and boosts overall skin health

Ragu-age® protects and reinforces the skin

Vitamin A+E antioxidant and anti-inflammatory, protects and stimulates the immune system

Pentavitin® relieves itchy, flaky, dry skin, prevents dehydration

Roman Chamonile Extract & Aloe Vera reduces inflammation and repairs skin damage

STEP 1

Step 1

Skin Massage & Gentle Exfoliation

Gently remove the dead skin cells and improve the micro-circulation

STEP 2

Step 2

Jet Infusion


Deep, effective delivery of active ingredients into the skin

STEP 3

Step 3

Booster Infusion

Post-infusion step to amplify the treatment result

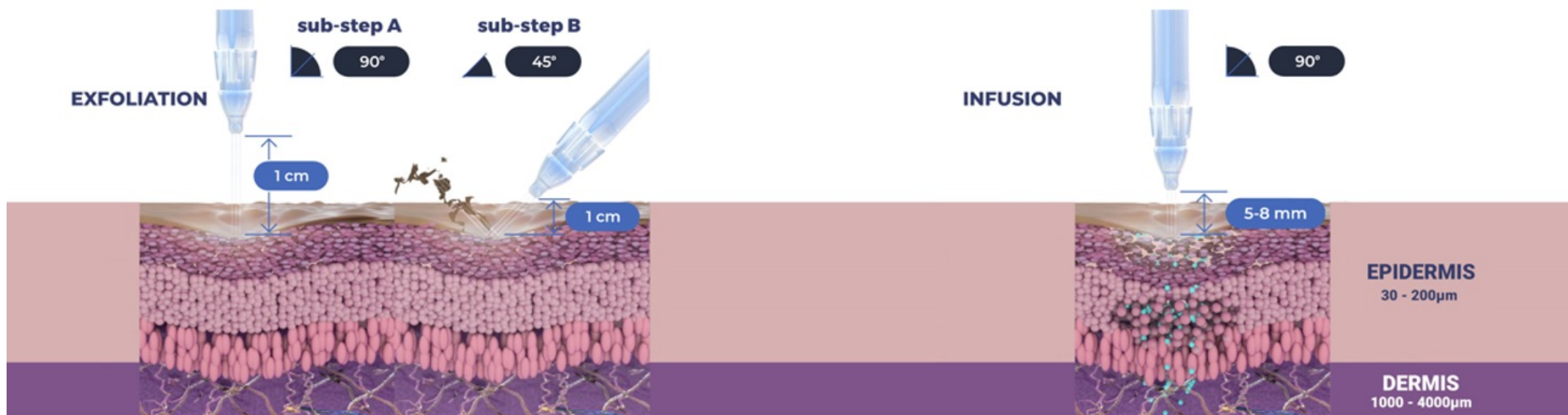

Winter in the city

Glow & Go JetPeel Treatment

Just imagine... the feeling of
stepping into winter with
roaring confidence

Glow & Go

Winter in the City treatment



STEP 1

Step 1
Skin Massage & Gentle Exfoliation
with **GLYCOLIC ACID 5% or 10% + HYDRO**
Gently remove the dead skin cells and
improve the micro-circulation
Time: 15 minutes



Glycolic Acid, Organic Aloe Vera, Pro-vitamin
B5, Hyaluronic Acid

Ideal for: all skin types and skin ages
Maintenance: once a month

STEP 2

Step 2
Jet Infusion
with **LIGHTENING**
Deep, effective delivery of active ingredients into
the skin
Time: 5 minutes



Alpha-Arbutin, Beta-White,
Alpaflor® Gigawhite, Stay-C ®

Treatment Cycle: 6 treatments sessions, once a week
Treatment time: 25 Minutes

STEP 3

Step 3
Booster Infusion
with **TIGHT**
Post-infusion step to amplify the
treatment result
Time: 5 minutes



Pro-Vitamin B5, Niacinamide, Biotin-Vitamin B7

Soothing Remedy

Glow & Go JetPeel Treatment



In **PREPARATION** for winter,

Glow & Go !

with a soothing remedy
short treatment



STEP 1

Step 1 Skin Massage & Gentle Exfoliation with **HYDRO**

Remove the dead skin cells and improve the micro-circulation
Time: 5 minutes



Hyaluronic Acid, Aloe Vera

STEP 2

Step 2 Jet Infusion with **SOOTHING**

Deep, effective delivery of active ingredients into the skin.
Time: 5 minutes



Azeloglicina®, Ronacare® Troxerutin,
Crodamor Nordic Cotton, Organic Aloe Vera

STEP 3

Step 3 Booster Infusion with **HYDRATE**

Post-infusion step to amplify the treatment result
Time: 5 minutes



PrimalHyal™, Niacinamide, Betaine

Ideal for: all skin types and skin ages
Maintenance: once a month

Treatment Cycle: 6 treatments sessions, once a week
Treatment time: 15 Minutes

Enjoy Your Skii

Glow & Go JetPeel Treatment

Glow & Go



STEP 1

Step 1 Skin Massage & Gentle Exfoliation with **HYDRO**

Remove the dead skin cells and improve the micro-circulation
Time: 10 minutes



Hyaluronic Acid, Aloe Vera

STEP 2

Step 2 Jet Infusion with **WIDE RANGE HYALURONIC ACID** Deep, effective delivery of active ingredients into the skin. Time: 5 minutes



Hyaluronic Acid, Ragu-age®

STEP 3

Step 3 Booster Infusion with **REGENERATE** Post-infusion step to amplify the treatment result Time: 5 minutes



Vitamin A, Vitamin E, Pentavitin®, Ragu-age®, Roman Chamomile Extract

Ideal for: all skin types and skin ages
Maintenance: once a month

Treatment Cycle: 6 treatment sessions, once a week
Treatment time: 20 Minutes

JetPeel by TavTech

The perfect trio combining:



The enduring
JetPro device




The original patented
JetPeel handpieces



The premium
JetCare solutions

Designed exclusively for use together to deliver outstanding results

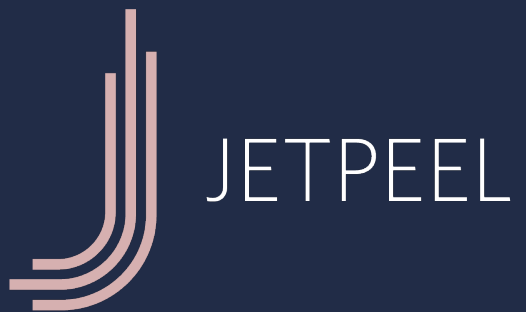

Thank you

