


Winter Holiday Skin Preparation

Presentation by: Laura Masini, Global Trainer, TavTech

December 2022

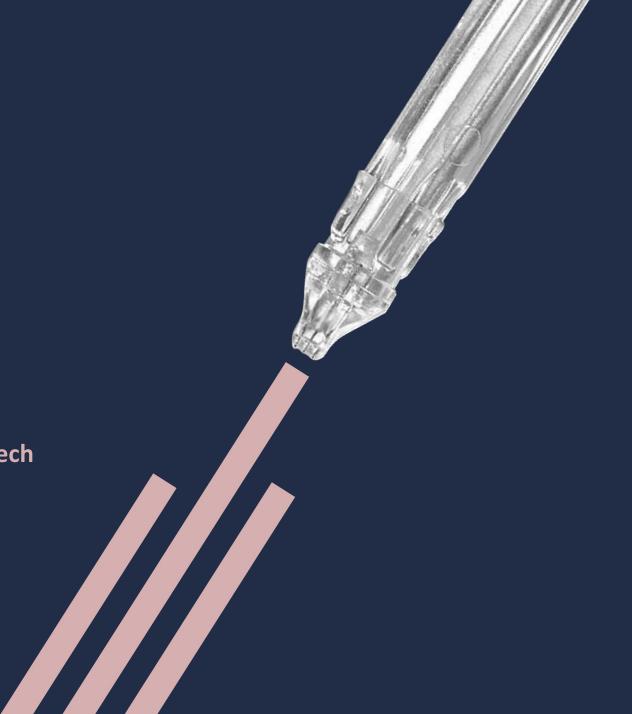



December is the month of excess

- Many social events
- Intense parties
- Too much food and drink
- Not enough sleep

Plus it's <u>winter</u>: your skin is prone to becoming dry, flaky, and dull looking.





Excess of food and drinks damage the skin, resulting in:

- Dehydration
- Sweeling and puffiness
- Dark circles
- Dull and grey skin
- Wrinkles, and loss of tone



Be ready for New Year's Eve

- With healthy, glowing skin
- Without skin imperfections
- With perfect skin texture





Prepare your skin with 3 on-the-go
Red Carpet
JetPeel Glowing
Treatments


Start with TREATMENT 1: Glowing Detox

Continue with TREATMENT 2: Glowing Lift

Finish with TREATMENT 3: Glowing Wrinkles Free



Red Carpet JetPeel Glowing Treatments Steps

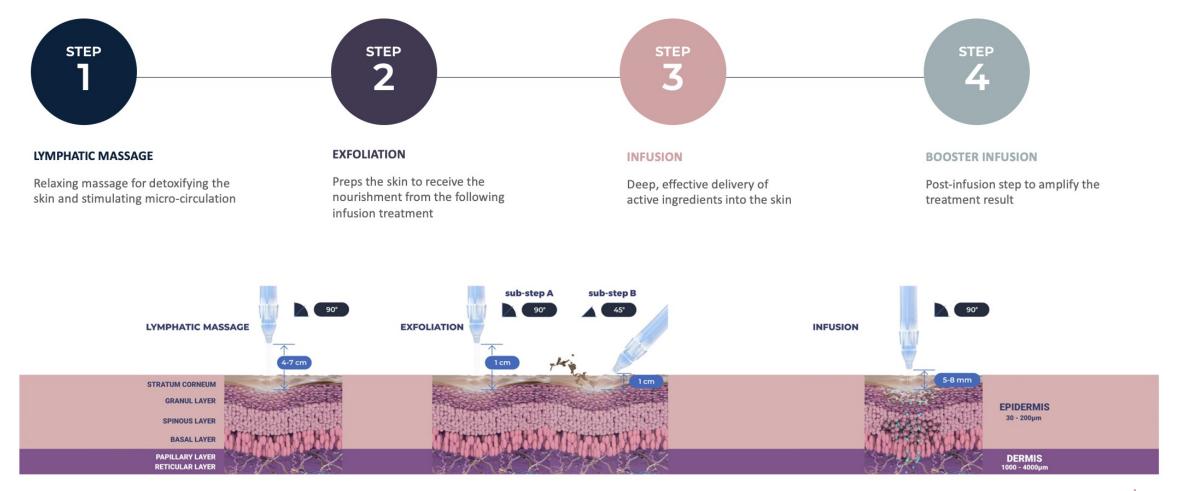





Start with Glowing Detox

Skin benefits:

- Refreshes
- Reduces imperfections
- Removes impurities
- Improves skin complexion
- Glows

Ideal for all skin types and all skin ages.





Glowing Detox

STEP 1

STEP 2 STEP 3

STEP \

Lymphatic Massage

The skin appears brighter, healthier and hydrated.

Use: JetCare HYDRO

Contains: Organic Aloe Vera and Hyaluronic Acid.

Time: 10 minutes

Exfoliation

The skin appears cleaner, brighter and softer.

Use: GLYCOLIC ACID 5% or

10% or 16%

Contains: Glycolic Acid, Organic Aloe Vera and Pro-Vitamin B₅ Panthenol

Time: 10 minutes

Infusion

The skin appears lighter, and brigther

Use: LIGHTENING

Contains: Stay C[®],

Alpha-Arbutin, Niacinamide

(Vit B₃), and Betawhite™, Salicylic acid,

Time: 5 minutes

Booster Infusion

The skin appears stronger, and heathier

Use: GLOW

Contains: Stable Vitamin C,

Fruit extract AHAs **Time:** 5 minutes

Total Treatment time: 30

Minutes













Continue with Glowing Lift

Skin benefits:

- Restores skin volume
- Youthful appearance
- Tightens & lifts-up
- Improves the facial features

Ideal for all skin types and all skin ages.

Glowing Lift

STEP 1

Lymphatic Massage

The skin appears brighter, healthier and hydrated.

Use: JetCare HYDRO Contains: Organic Aloe

Vera and Hyaluronic Acid.

Time: 10 minutes

STEP 2

Exfoliation

The skin appears softer and smoother

Use: MANDELIC &

SALICYLIC

Contains: a blend of AHA & BHA Mandelic & Salicylic Acids and Fruit Extract

AHAs

Time: 10 minutes

STEP 3

Infusion

The skin appears tighter

Use: WITCH HAZEL & PEPTIDES Contains: Witch Hazel, Regu-

age®, Replexium™ and Betaine

Time: 5 minutes

STEP

4

Booster Infusion

The skin appears hydrated and nourished

Use: JetCare Boost TIGHT

Contains: Complex of 3 vitamin B derivatives - B₃, B₅, B₇ Pro-

Vitamin B₅

Time: 5 minutes

Total Treatment Time: 30 minutes











Finish with Glowing Wrinkles Free

Skin benefits:

- Reduces fine lines and wrinkles
- Fills and volumizes
- Refresh tired skin
- Rejuvenates

Ideal for all skin types and all skin ages.



Glowing Wrinkles Free

.

STEP 1

STEP 2

STEP 3

STEP 4

Lymphatic Massage

The skin appears brighter, healthier and hydrated.

Use: JetCare HYDRO

Contains: Organic Aloe Vera and Hyaluronic Acid.

Time: 10 minutes

Exfoliation

The skin appears softer and smoother.

Use: MANDELIC &

SALICYLIC

Contains: a blend of AHA & BHA Mandelic & Salicylic Acids and Fruit Extract

AHAs

Time: 10 minutes

Infusion

The skin appears wrinkle-free **Use: ARGIRELINE® COMPLEX Contains:** Argireline® , Prinlayhal

-3KTM, Betaine and Pro-Vitamin

B₅ Panthenol **Time:** 5 minutes

Booster Infusion

The skin appears plump

Use: HYDRATE

Contains: PrimalHyal Ultrafiller™, Niacinamide,

and Betaine

Time: 5 minutes

Total Treatment Time:

30 minutes


















JetPeel by TavTech The perfect trio combining:



The enduring JetPro device

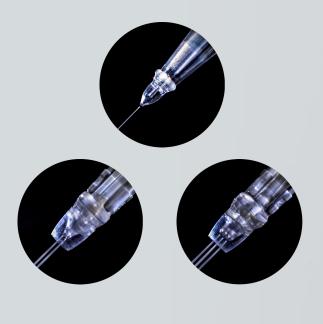

The original patented JetPeel handpieces



The premium
JetCare solutions

Designed exclusively for use together to deliver outstanding results



Thank you

