



# Skin Health and Wellness with JetPeel

Presentation by: Laura Masini, Global Trainer, TavTech

February 2023





## Today's consumers desire:

- **Holistic aesthetics approach:** improving internal skin health while simultaneously showing visible (external) results
- **Prevention + Rejuvenation = Pre-juvenation:** preventing loss of tone and improving skin elasticity

# Beauty

“Reflects how you feel inside”



## *Looking Good, Feeling Good with JetPeel*

### **Holistic treatments focus on:**

- Improving skin quality
- Regenerating collagen & elastin fibres
- Encouraging skin cells longevity
- Reinforcing the skin's barrier







**When it comes to skin health**  
the mantra is *“Be Gentle, Use JetPeel”*

**JetPeel:**

- Provides a **delicate** treatment
- Is **non-invasive** and **non-ablative**
- Is **non-contact**
- Offers a **pleasant patient experience**
- Is **safe** and **effective** with **zero-downtime**

**Ideal for all skin types, ages, and all year-round.**

# Healthy skin

starts with a strong skin barrier



## Benefits are:

- Keeping skin hydrated, smooth and elastic
- Blocking bacteria and pollution that can damage the skin
- Improving the skin's glow and appearance

# Prejuvenation: The New Approach to Anti-Aging

Younger consumers are looking towards delaying the signs of aging, thereby:

Preventing + Rejuvenating = Prejuvenation

## Prejuvenation for “Well-Aging”

*“New Aesthetic Trend & Life Style”*

Today’s younger consumers begin active skin care in their late teens and early 20’s by preserving their skin’s natural youth well before requiring anti-aging treatments in later years





# To Stay Healthy

The skin needs 3 fundamental compounds:

- Collagen
- Elastin
- Hyaluronic acid





# What is needed to support your skin?

- Amino acids
- Peptides
- Vitamins
- Minerals
- Growth Factors
- Hyaluronic acids



PRESSURED AIR

LIQUID

NOZZLE TUBE

JET BORDER

SKIN SURFACE

STRATUM CORNEUM

GRANUL LAYER

SPINOUS LAYER

BASAL LAYER

PAPILLARY LAYER

RETICULAR LAYER

EPIDERMIS  
30 - 200µm

DERMIS  
1000 - 4000µm

## Feed your skin:

- To support skin cells' activities; and
- To improve skin quality, elasticity, and longevity

with the JetPeel's trans-epidermal infusion.



**Our skin can display external signs indicating that we are not healthy on the inside when the skin's surface shows:**


- redness, inflammation,
- dry, flaky,
- skin breakout, acne,
- wrinkled, saggy skin,
- uneven skin tone, dark spots





## **Maintain and support your skin all year round**

- Defend against the environment
- Pollution
- Dehydration
- Free-radicals
- Damage from the sun

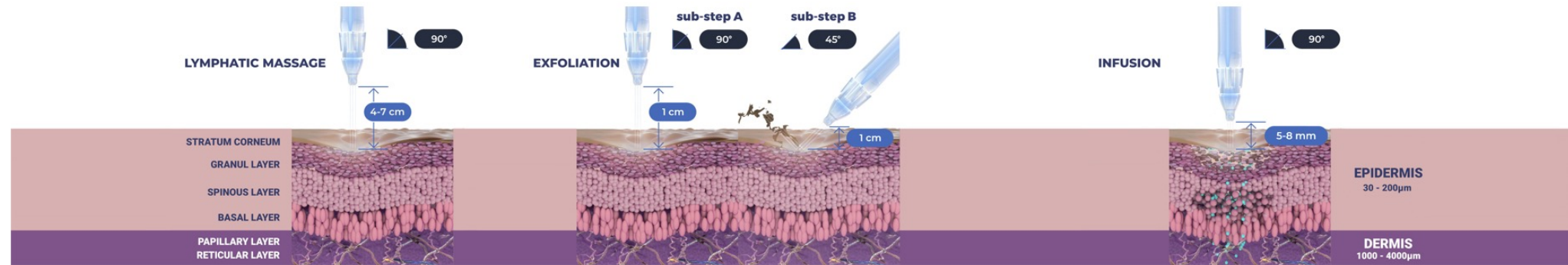

## JetPeel Beauty Concept

1. **Improve your skin's micro-circulation:** lymphatic drainage, exchange nutrients and metabolites between blood and tissue, boost circulation.
2. **Gentle exfoliation of the skin surface:** remove dirty and dead skin cells and bacteria, increase the skin's radiance and youthfulness, unclog pores, prevent skin imperfections, even skin tone.
3. **Infusion of state-of-the art ingredients, nutrients and vitamins into the skin:** essential for skin cell longevity, boost collagen and elastin fibres, protect the skin barrier and protect from environmental aggressions.



JETPEEL  
by TavTech

## 3 Skin Health & Skin Wellness JetPeel Treatments

To be performed all year-round





# The JetPeel Treatment Sequence

STEP  
1

## LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP  
2

## EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP  
3


## INFUSION

Deep, effective delivery of active ingredients into the skin

STEP  
4

## BOOSTER INFUSION

Post-infusion step to amplify the treatment result

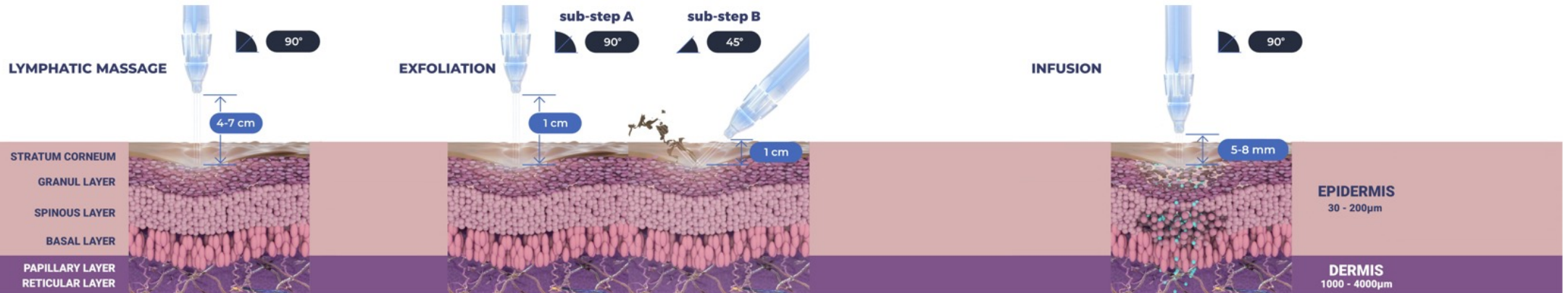

# Start with - **BETTER SKIN**

Improve skin defence, skin regeneration and rejuvenation

Total treatment time: **40 minutes**

**Start with 2 sessions of BETTER SKIN (once a week)**

STEP  
1

## LYMPHATIC MASSAGE

Detoxify the skin, stimulate micro-circulation, improve lymphatic system, remove puffiness and dark circles under the eyes. Use **Hydro**



STEP  
2

## EXFOLIATION

Remove dead skin cells, improve skin radiance, unclog pores, reduce hyperpigmentation. Keep your skin looking tight, plump and younger. Alternate between **Glycolic Acid 10%** and **Mandelic Acid**



STEP  
3

## INFUSION

Stimulate collagen and elastin, regenerate and rejuvenate the skin cells.. Alternate between **Growth Factor** and **Peptide Complex**



STEP  
4

## BOOSTER INFUSION

Micro infusion to enhance the skin's health, hydration, elasticity, and to improve overall skin appearance. Alternate between **Tight** and **Regenerate**



# Continue with - **HEALTHY SKIN**

Improve your skin's natural glow, smoothness, brightness and hydration

Total treatment time: **40 minutes**

**Continue with 2 sessions of HEALTHY SKIN (once a week)**

STEP  
1

## LYMPHATIC MASSAGE

Detoxify the skin, stimulate micro-circulation, improve lymphatic system, remove puffiness and dark circles under the eyes. Use **Hydro**



STEP  
2

## EXFOLIATION

Remove dead skin cells, improve skin radiance, unclog pores, reduce hyperpigmentation. Keep your skin looking tight, plump and younger. Alternate between **Glycolic Acid 16%** and **Mandelic Acid**



STEP  
3

## INFUSION

Stimulate collagen and elastin, regenerate and rejuvenate the skin cells.. Alternate between **Wide Range Hyaluronic Acid** and **Witch Hazel & Peptides**



STEP  
4

## BOOSTER INFUSION

Micro infusion to enhance the skin's health, hydration, elasticity, and to improve overall skin appearance. Alternate between **Glow** and **Hydrate**





# Finish with - **NATURAL BEAUTY**

Transforms your skin appearance making it glow from the inside out

Total treatment time: **40 minutes**

Finish with 2 sessions of NATURAL BEAUTY SKIN (once a week)

STEP  
1

## LYMPHATIC MASSAGE

Detoxify the skin, stimulate micro-circulation, improve lymphatic system, remove puffiness and dark circles under the eyes. Use **Hydro**



STEP  
2

## EXFOLIATION

Remove dead skin cells, improve skin radiance, unclog pores, reduce hyperpigmentation. Keep your skin looking tight, plump and younger. Use **Mandelic Acid**



STEP  
3

## INFUSION

Stimulate collagen and elastin, regenerate and rejuvenate the skin cells.. Alternate between **Argireline® Complex** and **Lightening**



STEP  
4

## BOOSTER INFUSION

Micro infusion to enhance the skin's health, hydration, elasticity, and to improve overall skin appearance. Alternate between **Hydrate** and **Glow**



Argireline® is a trademark of Lipotec S.A. or its affiliates

# RENEW & REJUVENATE REGENERATE



## **Maintenance**

One treatment every month

- Maintain skin health
- Support cellular aging process
- Prevent skin imperfections

# JetPeel by TavTech – the perfect trio combining:

- the enduring JetPro device,
- the original patented JetPeel handpieces, and
- premium JetCare solutions

**designed exclusively for use together to deliver outstanding results**





# Thank You

