



Post–Pregnancy JetPeel by TavTech Treatments

Presentation by: Laura Masini, Global Trainer, TavTech



WARNING

The system has not been tested or evaluated on pregnant or lactating women, therefore is not intended for use on this group of users




During pregnancy women go through emotional and physical changes

- Change in skin volume and shape
- The skin looks tired
- Hyperpigmentation
- Loss of hair
- Puffiness
- Sensitized skin
- Loss of skin elasticity





**After giving birth,
mums they want to
return to their pre-baby
life, and they want to
feel and look good**

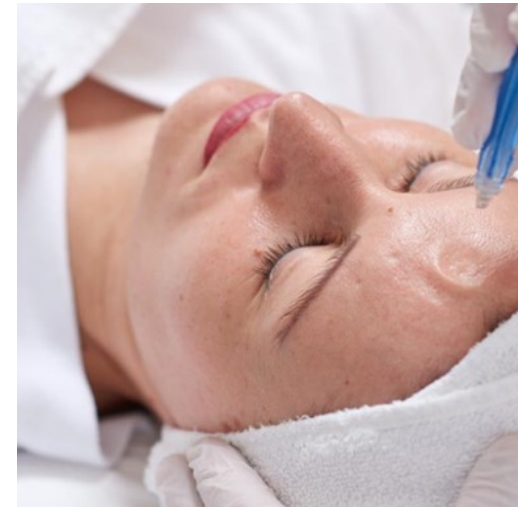

**Post-partum mothers who
are not breastfeeding can
receive JetPeel treatments**

***JetPeel by TavTech** is a relaxing and pleasant skin procedure that improves skin appearance and texture.*



JetPeel is a non-invasive treatment

which uses pressurized Jet streams to penetrate the skin with vitamins and nutrients without needles, without downtime, without pain and without skin contact.

The Jet Streams:

- Clean and exfoliate the outermost layer of the epidermis (stratum corneum)
- Improve micro-circulation and lymphatic system
- Infuse active ingredients into the skin
- Improve skin appearance
- Regenerate and rejuvenate
- Smooth skin imperfections
- Improve skin quality
- Delay signs of aging

4 Postpartum skin problem and 4 JetPeel Treatment solutions

Healthy Skin JetPeel Treatment Ideal for: dry, puffy and sensitive skin	Supercharged Skin JetPeel Treatment Ideal for: Tired skin, loss of elasticity, and lined skin	Radiant Skin Tone JetPeel Treatment Ideal for: hyperpigmentation and uneven skin tone	Hair Growth JetPeel Treatment Ideal for: hair loss
Benefits are: Protect Defend Sooth Hydrate	Benefits are: Boost collagen Delay aging signs Regenerate Refresh	Benefits are: Decrease dark spots Bright Renew Light	Benefits are: Detox Oxygenate Reinforce hair bulb Regrowth hair

The JetPeel Treatment Sequence

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP
4



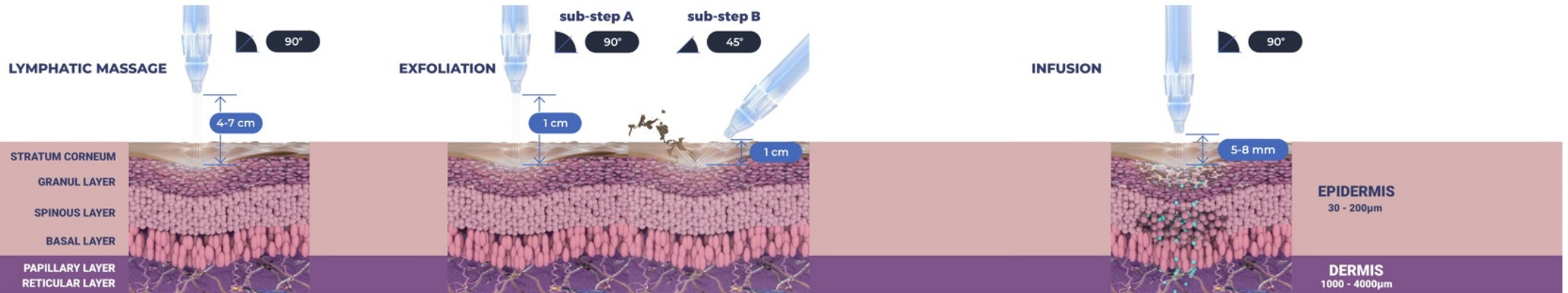

BOOSTER INFUSION

Post-infusion step to amplify the treatment result



Improve Skin Health
with JetPeel by TavTech

Improve Skin Health

DETOX with JetCare <u>Hydro</u>	RENEW with JetCare <u>Mandelic & Salicylic</u>	DEFEND with JetCare <u>Soothing</u>	REINFORCE with JetCare <u>Regenerate</u>
			
<ul style="list-style-type: none">• Improve skin barrier function• Reduce redness• Hydrate the skin• Detoxify the skin	<ul style="list-style-type: none">• Gently exfoliate• Promote cell turnover• Replenish moisture and nourish the skin	<ul style="list-style-type: none">• Highly moisturising• Anti-inflammatory• Antioxidant• Stimulate the extracellular matrix	<ul style="list-style-type: none">• Protect skin cells• Strengthen skin barrier function• Prevent aging signs

Hyaluronic Acid, Organic Aloe Vera

AHA & BHA blend
Mandelic, Salicylic, Malic, Glycolic and Lactic Acids

Azeoglicina® (azelaic+glycine). Nordic cotton extract,
Ronacare®— Troxerutin, Organic Aloe

Vitamins A + E, Pentavitin ® , Ragu-age®,
Roman Chamomile Extract

IMPROVE HEALTHY SKIN – TREATMENT SEQUENCE

Ideal for: dry, sensitized, and puffy skin conditions

Total treatment time: 40 minutes

Treatment Sessions: 6 once or twice a week

Maintenance: once a month

STEP 1

LYMPHATIC MASSAGE

Cleans inside, and makes your skin glow, hydrates, smooths and improves skin appearance.

Use **JetCare Hydro**



STEP 2

EXFOLIATION

Renews, accelerates the cell turnover, and acts as anti-inflammatory.

Use **JetCare Mandelic & Salicylic**



STEP 3

INFUSION

Improves skin radiance, skin quality and skin hydration, soothes, and protects the skin barrier.

Use: **JetCare Soothing**



STEP 4

BOOSTER INFUSION

Speeds up healing, promotes natural moisturising, assists maintaining a healthy dermis and epidermis, protects the skin from damage.

Use **JetCare Regenerate**





Supercharged Tired Skin
with JetPeel by TavTech

Supercharged Tired Skin

Refresh with JetCare Hydro



- Improve lines
- Increase skin elasticity
- Reduce puffiness
- Stimulate skin cells

Hyaluronic Acid, Organic Aloe Vera

Rejuvenate with JetCare Glycolic acid 5%, 10% or 16%



- Exfoliate dead skin and oils
- Boost collagen
- Regenerate new skin cells
- Reduce aging signs

Glycolic Acid, Organic Aloe Vera, Pro-Vitamin B5 Panthenol

Smooth with JetCare Argireline® Complex



- Delay the appearance of wrinkles
- Promote collagen
- Improve skin hydration
- Increase skin elasticity

Argireline®, Prinlayhal-3KTM, Beatine, Pro-Vitamin B5-Panthenol

Boost with JetCare Hydrate



- Wrinkle filler
- Hydrate
- Plumping effect
- Energize the skin

PrimalHyalTM Ultrafiller, Niacinamide, Betaine

Argireline® is a trademark of Lipotec S.A. or its affiliates

SUPERCARGED TIRED SKIN – TREATMENT SEQUENCE

Ideal for: tired skin, uneven skin texture, loss of elasticity & lined skin

Total treatment time: 40 minutes

Treatment Sessions: 6 once or twice a week

Maintenance: once a month

STEP 1

LYMPHATIC MASSAGE

Detoxifies and brightens the skin, reduces lines, and improves the skin metabolism.

Use **JetCare Hydro**



STEP 2

EXFOLIATION

Improves skin tone, elasticity, and appearance.

Choose between JetCare Glycolic Acid 5%, 10% or 16%



STEP 3

INFUSION

Relaxes and smooths the skin, promotes collagen production, moisturises, and assists keeping the skin strong and firm.

Use: **JetCare Argireline® Complex**



STEP 4

BOOSTER INFUSION

Boosts skin rejuvenation, makes the skin healthier and hydrated, with invisible skin protection.

Use **JetCare Hydrate**



Argireline® is a trademark of Lipotec S.A. or its affiliates

Radiant even skin tone
with JetPeel by TavTech



Radiant Even Skin Tone

Bright with JetCare Hydro



- Decongests the skin
- Adds glow to the skin
- Improves the skin appearance

Hyaluronic Acid, Organic Aloe Vera

Regenerate with JetCare Mandelic & Salicylic



- Evens the skin tone
- Reduces signs of photo-aging and dark spots
- Assists with hyperpigmentation

AHA & BHA blend
Mandelic, Salicylic, Malic, Glycolic and Lactic Acids

Light with JetCare Lightening



- Reduces dark spots
- Reduces the formation of free-radicals
- Brightens the skin

Alpha-Arbutin, Niacinamide, Beta-White™, Salicylic acid, Stay-C

Radiance With JetCare Glow



- Reduces sign of photo-aging and dark spots
- Lightens the skin

Stable vitamin C, and ACB fruit mix, natural extract of vegetative sources of AHA

RADIANT EVEN SKIN TONE – TREATMENT SEQUENCE

Ideal for: hyperpigmentation and uneven skin tone

Total treatment time: 40 minutes

Treatment Sessions: 6 once or twice a week

Maintenance: once a month

STEP 1

LYMPHATIC MASSAGE

Detox the skin, slow down hyperpigmentation, and lighten the skin.

Use **JetCare Hydro**



STEP 2

EXFOLIATION

Lighten and reduce hyperpigmentation

Use: **JetCare Mandelic & Salicylic**



STEP 3

INFUSION

Decrease pigmentation and dark patches, improve skin evening.

Use: **JetCare Lightening**



STEP 4

BOOSTER INFUSION


Lighten the skin, protect from photo damage, decreases spots, and assists skin tone balancing.

Use **JetCare Glow**



Hair Growth with JetPeel by TavTech

Hair Growth

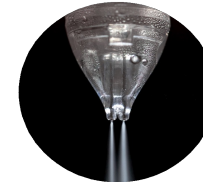

Recommended to be used with
the JetPeel DoubleJet Handpiece

Massage and Exfoliate with JetCare Hydro

Start
with deep clean, exfoliate and detox the scalp



Hyaluronic Acid, Organic Aloe Vera

Infuse with JetCare Scalp & Hair

Peptides infusion, boosting and nourishing,
to increase proteins necessary to maintain hair growth.



Root Biotech™, Redensyl™, Capixyl™, Biotin

HAIR GROWTH - JETPEEL TREATMENT SEQUENCE

Ideal for: thin hair and hair loss

Total treatment time: 20 minutes

Treatment Sessions: 10 once a week

Maintenance: once a month

STEP

1

MASSAGE & EXFOLIATION

A pleasant and relaxing massage to exfoliate the scalp, while improving micro-circulation

Use **JetCare Hydro**



STEP

2

INFUSION

This step provides effective and deep delivery of active ingredients into the scalp

Use: **JetCare Scalp & Hair**



JetPeel by TavTech – the perfect trio combining:

- the enduring JetPro device,
- the original patented JetPeel handpieces, and
- premium JetCare solutions

designed exclusively for use together to deliver outstanding results





THANK YOU!

