



Prepare Your Skin For Summer

Presentation by: Laura Masini, Global Trainer, TavTech





Our skin may react to the rising temperature and humidity with:

- Acne breakouts
- Inflammation
- Sunburn
- Dehydration
- Premature aging signs



In Summer, skin break-out's occur more easily due to high temperature

There is an increase in:

- Sebum production
- Sweating

This condition can:

- Trap bacteria
- Increase skin inflammation
- Create blackheads
- Cause pimples

JetPeel effectively assists with acne break-outs:

- Cools the skin down
- Rebalances sebum production
- Soothes the skin
- Exfoliates dead skin cells



UV rays damage the skin barrier function and increase water loss

In summer the skin dehydrates due to:

- Sunlight absorbing the moisture from the skin
- The hot temperature and air conditioning leave the skin dehydrated

Effectively alleviate skin dehydration with JetPeel:

- Increase the water level in the skin
- Reinforce the skin barrier
- Elevate skin protection



Prolonged sun exposure irritates & inflames the skin

The skin becomes:

- Red
- Warm
- Sore
- Tingles
- Blisters or peels
- Ages

JetPeel effectively:

- Refreshes
- Soothes
- Regenerates
- Boosts the healing process



Photoaging

Premature aging due to prolonged exposure to sunlight. When UV rays enter skin cells, there is a detrimental effect on the skin's appearance.

Photoaging signs:

- Wrinkles
- Pigmentation
- Loss of skin tone
- Broken capillaries
- Redness

To alleviate photo-damage, JetPeel:

- Reduces oxidative stress
- Strengthens the collagen and elastin fibres
- Rejuvenates skin cells

Prepare your skin for summer: start with keeping your skin cool!

JetPeel's unique combination of cool air and JetCare liquid solutions in the form of a high-velocity, high-pressure, but gentle jet stream, is used to perform:

- Lymphatic massage
- Exfoliation
- Infusion
- Booster infusion

JetPeel quenches thirsty skin and provides it with protection from the sun's powerful UV rays by:

- Cooling the skin down
- Purifying and regenerating skin cells
- Hydrating thirsty skin
- Boosting skin protection
- Providing protection from sun damage



The JetPeel Treatment Sequence

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP
4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

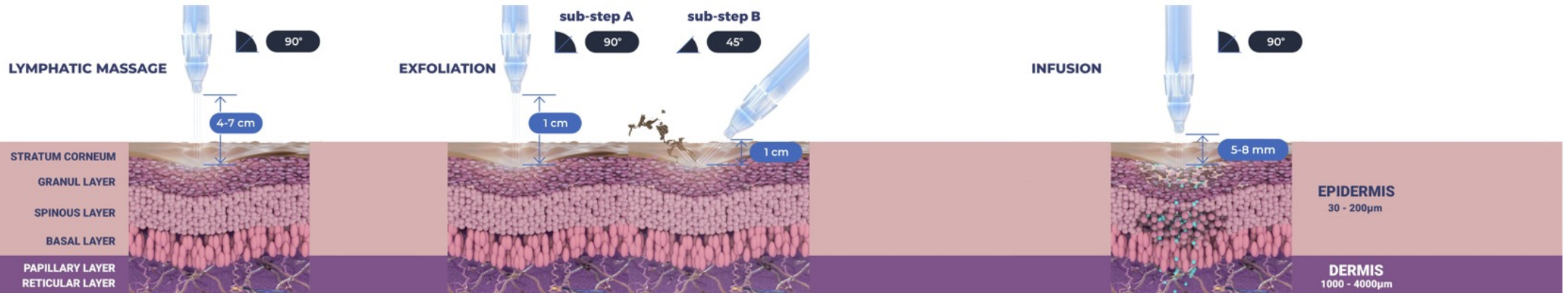



4 JetCare by TavTech Products *you must use* during Summertime

HYDRO

Hydrates, Soothes and Protects

MANDELIC & SALICYLIC

Exfoliates, Purifies, Renews and Regenerates

WIDE RANGE HYALURONIC ACID

Smooths, Rejuvenates, Plumps and Reinforces

GLOW

Strengthens, Brightens and Glows



PURIFY & REGENERATE SKIN CELLS – *Special Summer*

Alleviate Acne & Pimples – Avoid Breakout

Total treatment time: 40 minutes

Treatment cycle: 6 treatment sessions (once or twice a week)

Maintenance: once a month

STEP

1

LYMPHATIC MASSAGE

- Bring oxygen to the skin
- Reduce puffiness and redness
- Reduce acne appearance

Use **JetCare Hydro**

Contains: **Organic Aloe Vera** and **Hyaluronic Acid**

STEP

2

EXFOLIATION

- Wash off impurities and bacteria
- Remove dead skin cells
- Even out skin tone

Use **JetCare Mandelic & Salicylic**

Contains: **Mandelic, Salicylic, Fruit Extract** and other **AHA's**.

STEP

3

INFUSION

- Decreases the sebaceous gland activity
- Inhibition bacteria proliferation
Reduce inflammation

Use **Clear**

Contains: **Salicylic Acid, Acnacidol,** and **Niacinamide**

STEP

4

BOOSTER INFUSION

- Reduce redness
- Sebum-rebalance and reduce post-inflammatory hyperpigmentation
- Brighten

Use: **Glow**

Contains: **Stay-C** and **ACB Fruit Mix**



COOL THE SKIN DOWN – *Special Summer*

Treat Skin Inflammation and Accelerate the Healing Process

Total treatment time: 40 minutes

Treatment cycle: 6 treatment sessions (once or twice a week)

Maintenance: once a month

STEP

1

LYMPHATIC MASSAGE

- Reduce skin inflammation
- Cool the skin down and reduce swelling
- Promote healing

Use **JetCare Hydro**

Contains: **Organic Aloe Vera** and **Hyaluronic Acid**

STEP

2

EXFOLIATION

- Gently remove dead skin cells
- Lighten the skin
- Prevent post-inflammation hyperpigmentation

Use **JetCare Mandelic & Salicylic**

Contains: **Mandelic, Salicylic, Fruit Extract** and other **AHA's**

STEP

3

INFUSION

- Gently moisturize
- Restore the skin
- Heal
- Reduce itching, skin irritation

Use **Soothing**

Contains: **Azeoglicina, Ronacare–Troxerutin, Organic Aloe** and **Nordic Cotton Extract**

STEP

4

BOOSTER INFUSION

- Antioxidant
- Nourish
- Protect
- Calm

Use: **Regenerate**

Contains: **Vitamin A+E, Pentavitin, Regu-age, Roman Chamomile Extract**



HYDRATE THIRSTY SKIN – *Special Summer*

Skin Protection and Alleviate Skin Dehydration

Total treatment time: 40 minutes

Treatment cycle: 6 treatment sessions (once or twice a week)

Maintenance: once a month

STEP

1

LYMPHATIC MASSAGE

- Hydrate the skin
- Strengthen the skin barrier
- Improve the skin's appearance

Use **JetCare Hydro**

Contains: **Organic Aloe Vera** and **Hyaluronic Acid**

STEP

2

EXFOLIATION

- Keep the skin clean
- Renew skin tissue
- Increase skin hydration

Use **JetCare Mandelic & Salicylic**

Contains: **Mandelic, Salicylic, Fruit Extract** and other **AHA's**

STEP

3

INFUSION

- Hydrate
- Reduce the formation of wrinkles
- Keep the skin soft and elastic
- Stimulate skin regeneration

Use **Wide Range Hyaluronic Acid**

Contains: **5 molecular weights of HA** and **Regu-Age**

STEP

4

BOOSTER INFUSION

- Filler lines
- Plump skin
- Youthful
- Strangeness skin surface

Use: **Hydrate**

Contains: **PrimalHyal Ultrafiller, Niacinamide, Betaine**



BOOST SKIN PROTECTION – *Special Summer*

Reduce Premature Aging Signs & Skin Oxidative Stress

Total treatment time: 40 minutes

Treatment cycle: 6 treatment sessions (once or twice a week)

Maintenance: once a month

STEP

1

LYMPHATIC MASSAGE

- Bring oxygen to the skin
- Reduce puffiness and redness
- Reduce acne appearance

Use **JetCare Hydro**

Contains: **Organic Aloe Vera** and **Hyaluronic Acid**

STEP

2

EXFOLIATION

- Wash off impurities and bacteria
- Remove dead skin cells
- Even out skin tone

Use **JetCare Mandelic & Salicylic**

Contains: **Mandelic, Salicylic, Fruit Extract** and other **AHA's**

STEP

3

INFUSION

- Anti-oxidant
- Boost collagen and elastin fibres
- Prevent photoaging
- Prevent dark spots

Use: Alternate between **Lightening** and **Peptide Complex**

Contains: **Alpha-Arbutin, Beta-White, Alpaflor, Stay-C** and **Peptide Complex**

STEP

4

BOOSTER INFUSION

- Reduce redness
- Sebum-rebalance
- Prevent hyperpigmentation
- Boost Collagen production

Use: Alternate between **Glow** and **Regenerate**

Contains: **Stay-C, ACB Fruit Mix, Pentavitin, Vitamin A & E, Regu-age**, and **Roman Chamomile Extract**



JetPeel by TavTech – the perfect trio combining:

- the enduring JetPro device,
- the original patented JetPeel handpieces, and
- premium JetCare solutions

designed exclusively for use together to deliver outstanding results





JETPEEL
by TavTech

THANK YOU!

