




JetPeel

Jet-LED Combination Therapies

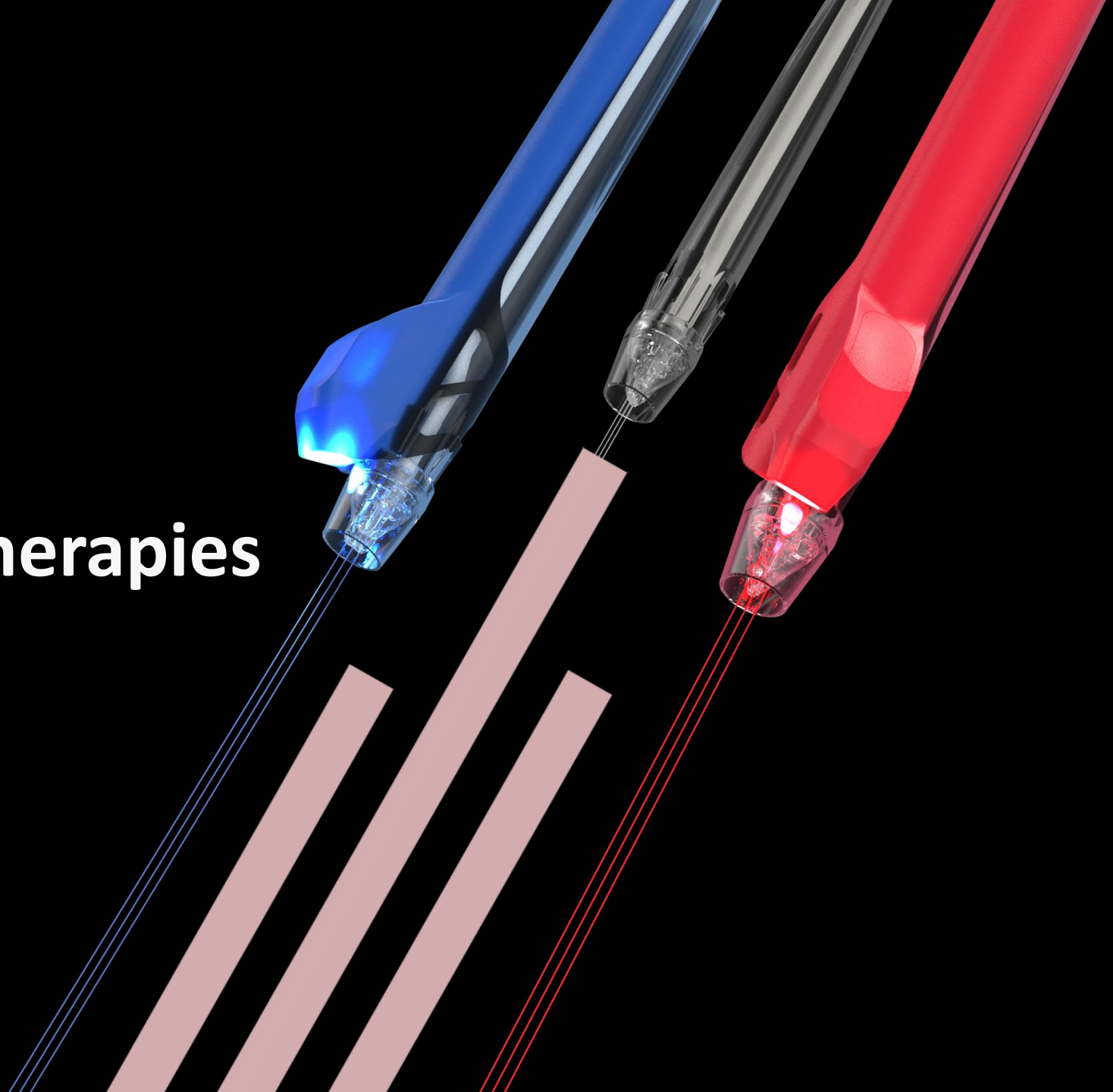

The JetPeel jet pressure energy system for:

- **Lymphatic Massage**
- **Exfoliation**
- **Trans-epidermal Infusion of a rich variety of nutrients and solutions**



Anti-Aging Care



Scalp & Hair



Lightening



Clear



Soothing



Growth Factor







Renewal Care

The science behind the patented handpieces

Researched and perfected over several decades:

- Unique nozzle
- Pressure
- Flow rate
- Spray pattern
- Droplet size & speed
- Reach
- Impact

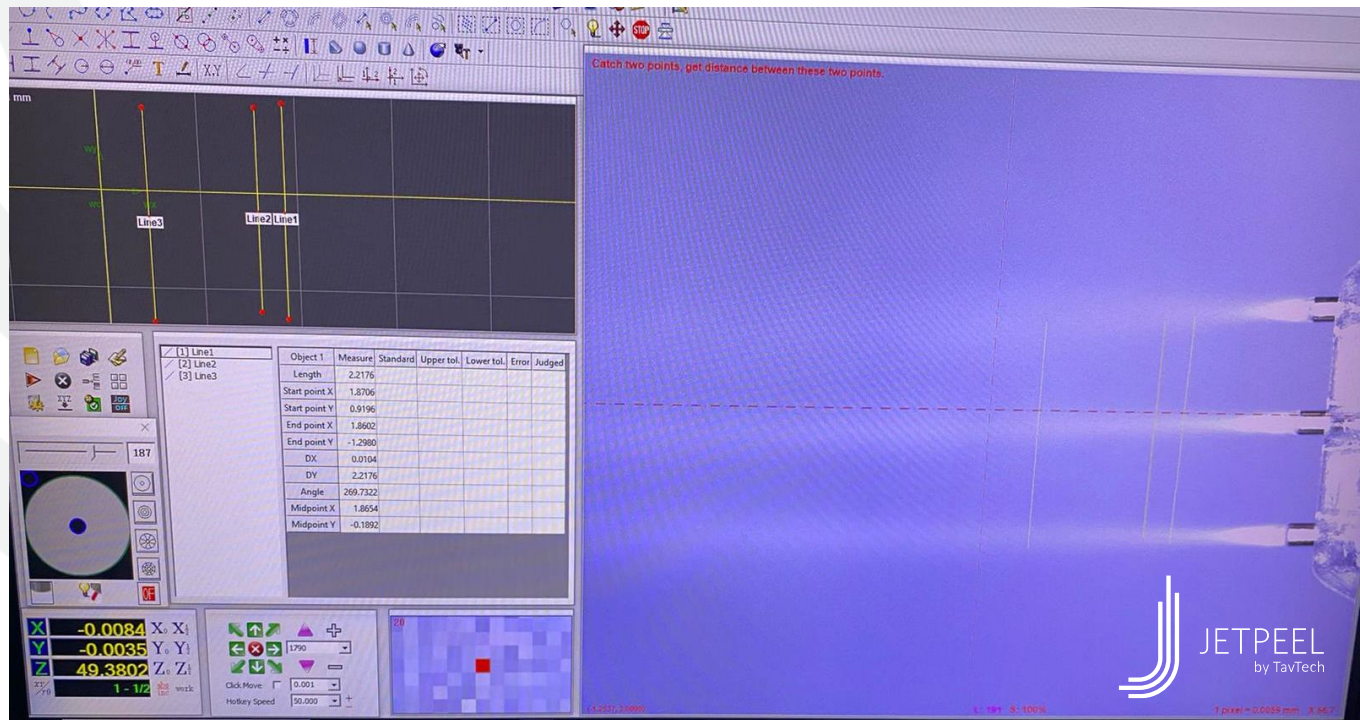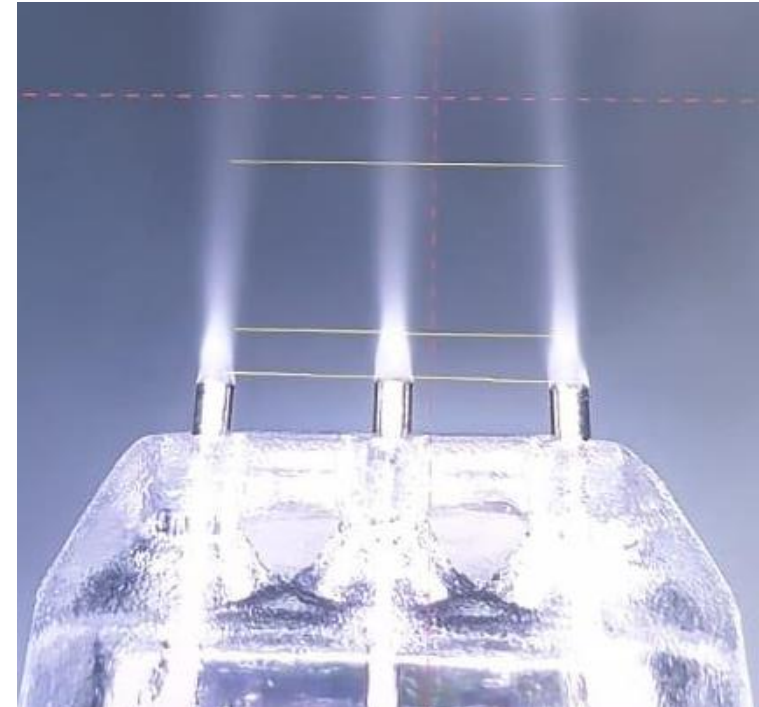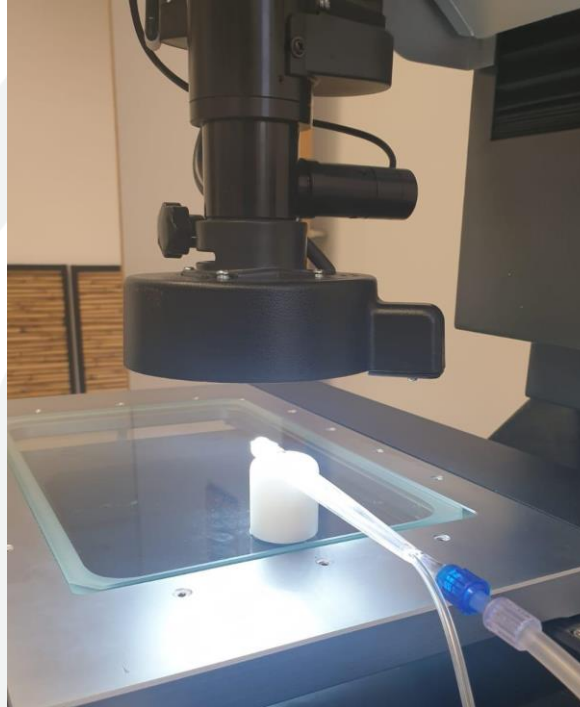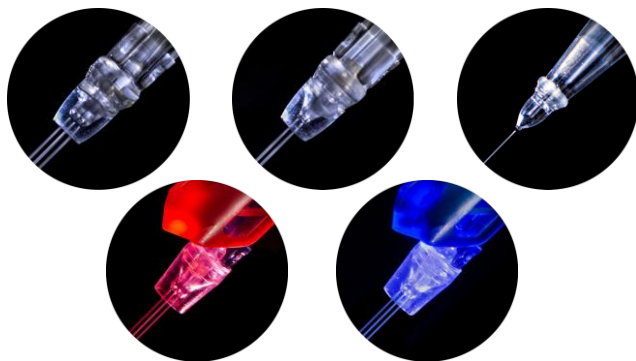

A variety of solutions, vitamins, minerals and acids



and an array of treatment guidelines

Combined and Simultaneous Therapy



Jet + LED

Designed to be used simultaneously



Unique combination of

2 POWERFUL ENERGIES

Outstanding results



Without any additional treatment time or consumables



Super powerful treatment

BIO SYNERGY JET + LED

PROMOTES **SKIN REJUVENATION** & **REGENERATION** FOR HEALTHY SKIN FUNCTIONS



JET STREAMS

Deliver nutrients
into the dermis without needles.

The jet streams rejuvenate
the skin at a cellular level



RED LED ADD-ON

Simultaneously boost
collagen and elasticity, and
speed up cell renewal



BLUE LED ADD-ON

Extra powerful results.
Kill acne-causing bacteria
and reduce inflammation

Jet pressure and the cavitation effect with LED boost



PRESSURED AIR

LIQUID

NOZZLE TUBE

JET BORDER

SKIN SURFACE

STRATUM CORNEUM

GRANUL LAYER

SPINOUS LAYER



BASAL LAYER

PAPILLARY LAYER

RETICULAR LAYER

EPIDERMIS
30 - 200µm

DERMIS
1000 - 4000µm

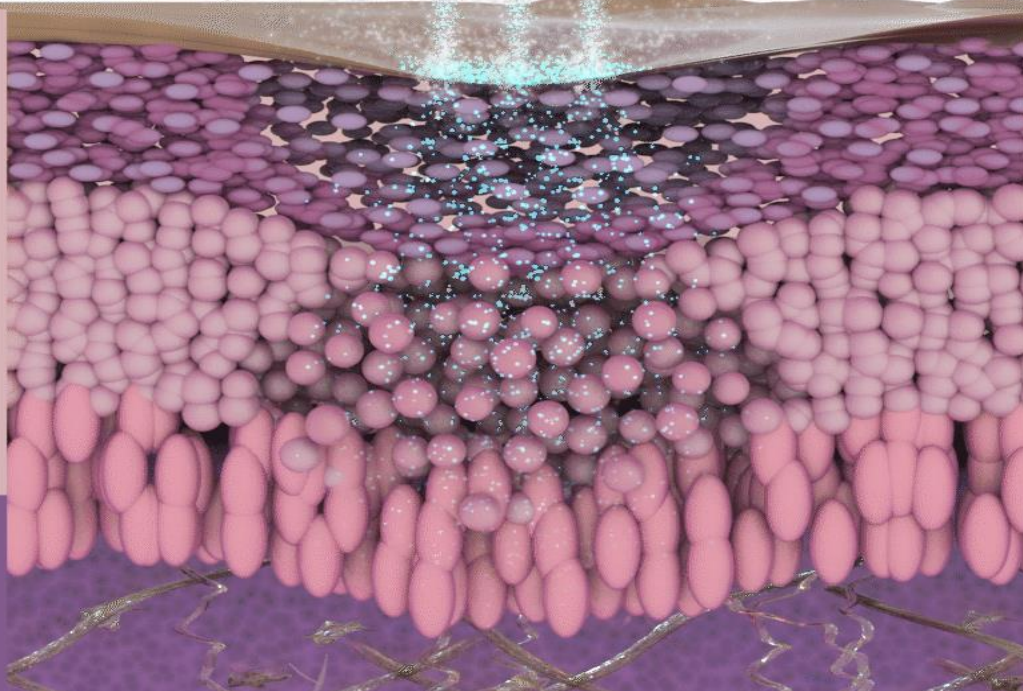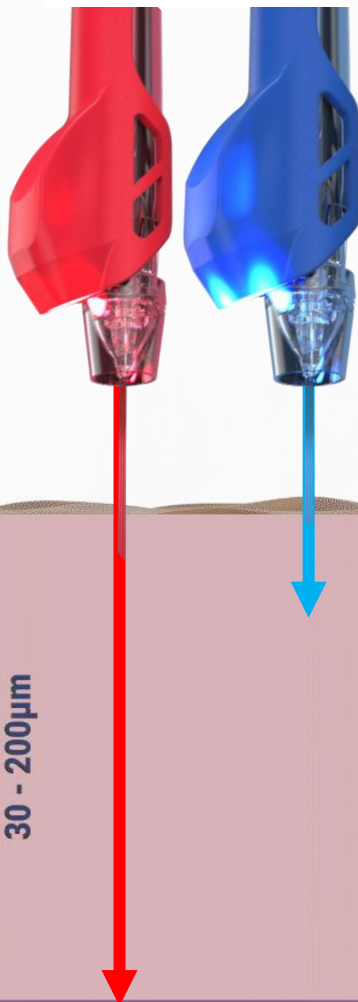

The positive light therapy

LED Therapy stimulates natural cell regeneration

LED Light recharges your skin cells' batteries

LED light therapy is based on the principle that living cells are able to absorb light and are influenced by it.

Photo-biomodulation:

- Boosts collagen production
- Stimulates blood flow
- Increases cell turnover
- Calms skin
- Reduces blemishes

*LED stimulates cell functions
for a healthier, radiant and younger looking skin*

JetPro Duo with LED add-ons

Red light: 620-630 nm

Recommended for:

Anti Aging, Loss of Tone & Wrinkles, Post-Acne, Lightening & Pigmentation.

Blue light: 460-470 nm

Recommended for:

Mild-Moderate Acne, Couperose & Sensitive Skin, Itchy & Sensitive Scalp.



Red LED light affects mitochondrial function to produce ATP

- Red light penetrates the deeper skin layers where it is clinically proven to stimulate fibroblast cells and boost collagen production.
- Helps to smooth the appearance of fine lines and firm and lift the skin.
- 90% reported softer skin and fewer fine lines.*


* Weiss RA, McDaniel DH, Geronemus RG et al. Clinical experience with light-emitting diode (LED) photomodulation. Dermatol Surg. 2005 Sep; 31 (9 Pt 2): 1199-1205




Blue LED targets acne bacteria, reduces skin breakouts and reduces inflammation

- Blue light penetrates to a maximum skin depth of 1mm, soothing irritation in the upper skin layers.
- Known for its ability to target the P.acnes bacteria strain that causes acne.
- Studies show a decrease in inflammatory lesions in cases of mild to moderate acne.*


* Daniel R. Opel, MD, Erika Hagstrom, MD, MA, Aaron K. Pace, MD, Krissane Sisto, MD, Stefanie A. Hirano-Ali, MD, Shraddha Desai, MD, James Swan, MD. Light Emitting Diodes. J Clin Aesthet Dermatol. 2015 Jun; 8 (6): 36-44

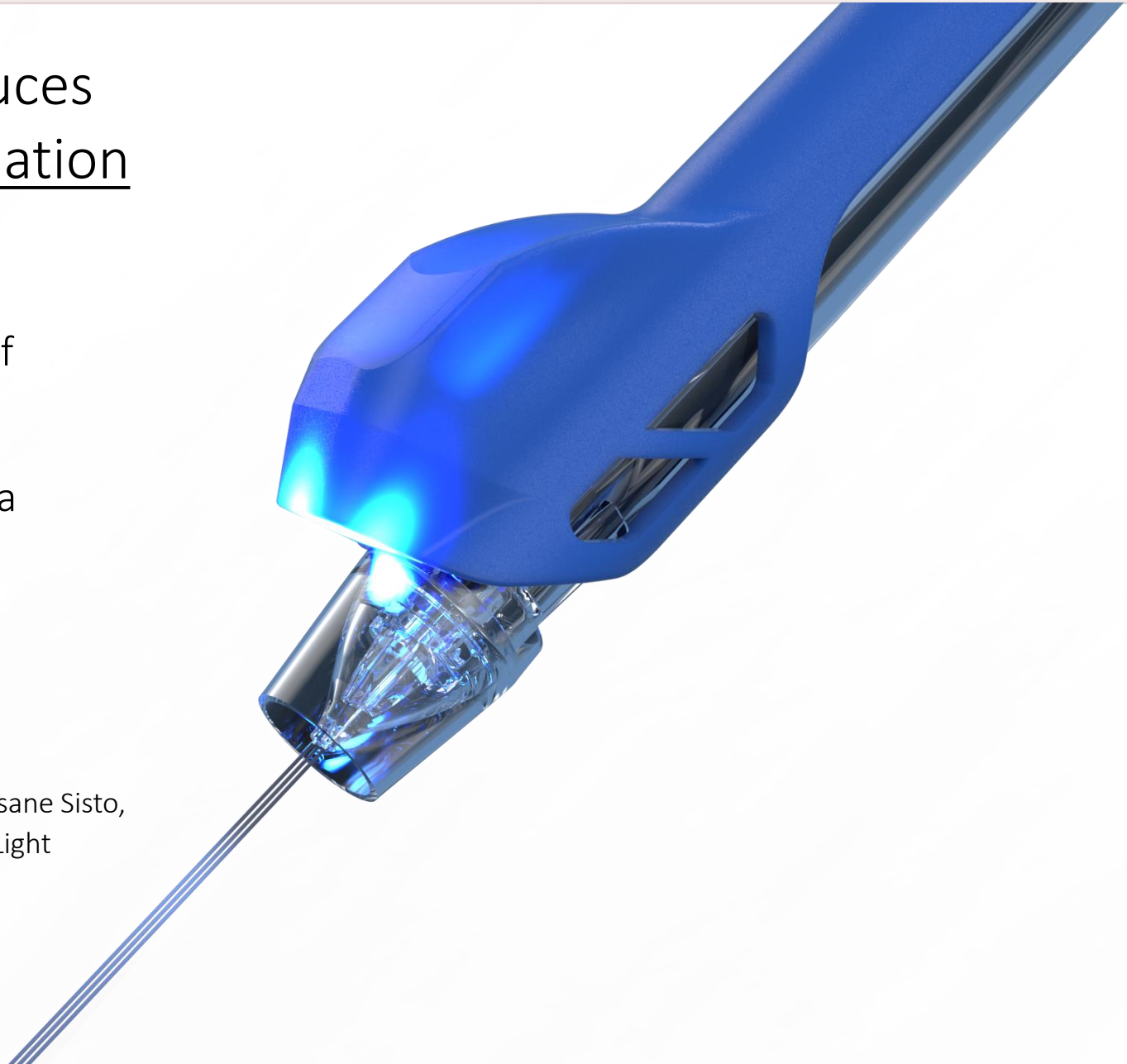

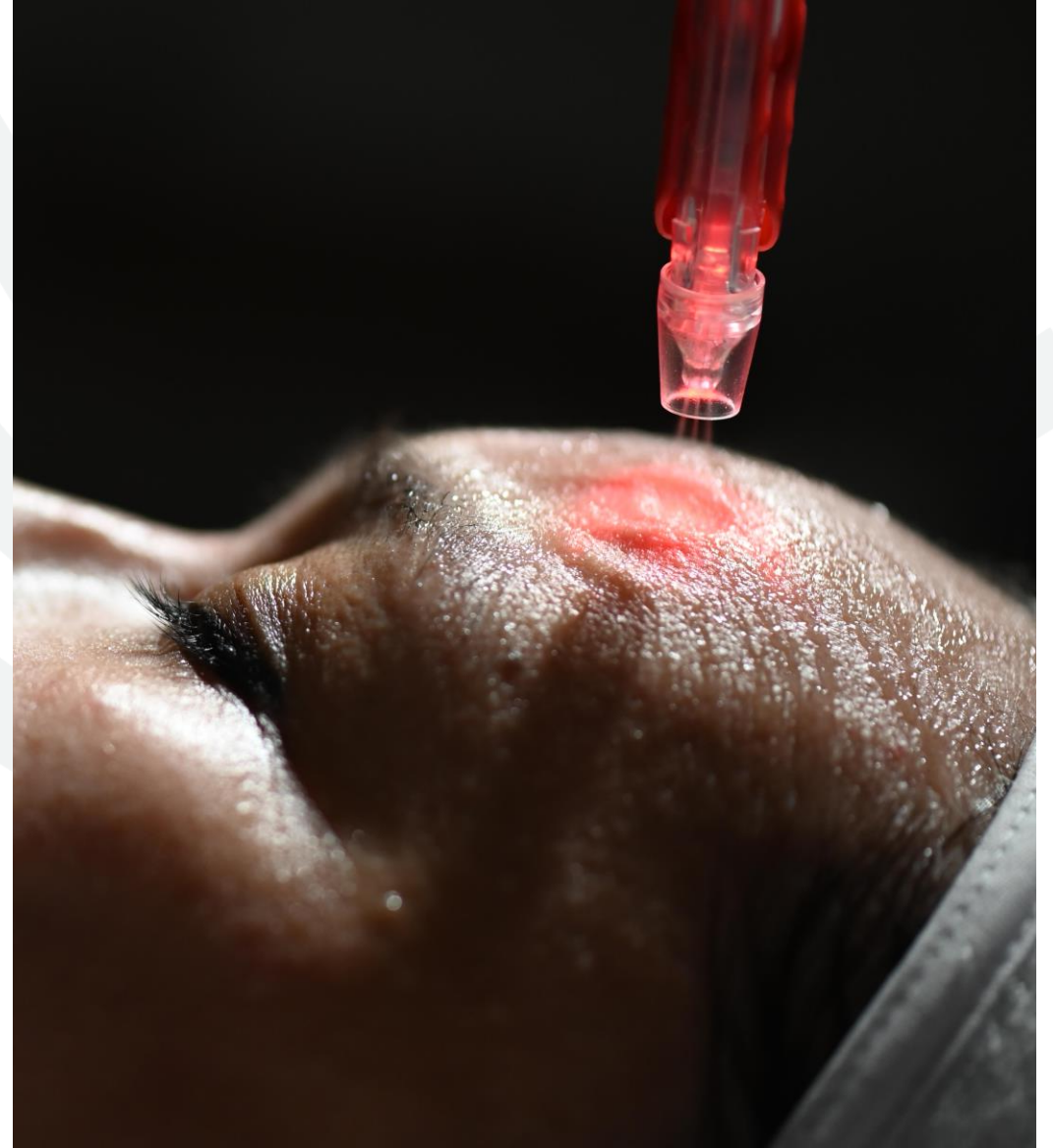



JetPeel **BEFORE**
other aesthetic procedures

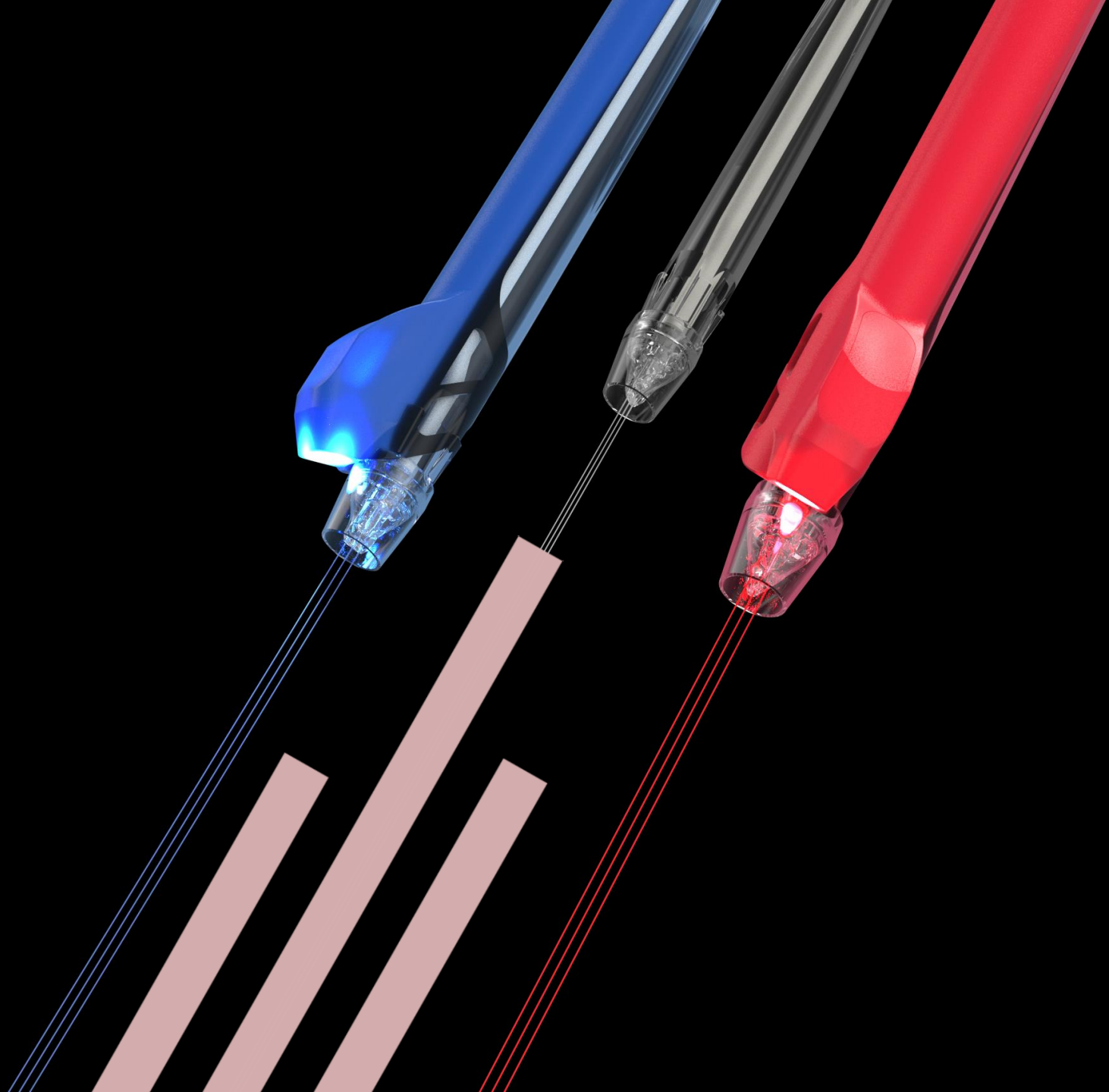

JetPeel treatment **2 DAYS BEFORE** other aesthetic procedures helps stimulate cutaneous collagen, elastin, hyaluronic acid, and assists the skin during the healing process

JetPeel helps to:

Oxygenate

- Increase blood flow
- Increase skin nutrition
- Prevent swelling

Defend

- Reduce bacteria proliferation
- Reduce the risk of infection
- Prevent inflammation

Hydrate

- Improve skin cell activities
- Protect skin barrier
- Accelerate healing process

Regenerate

- Increase skin elasticity
- Increase collagen and dermis density
- Stimulate skin rejuvenation

JetPeel Treatment Sequence

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin


STEP
4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

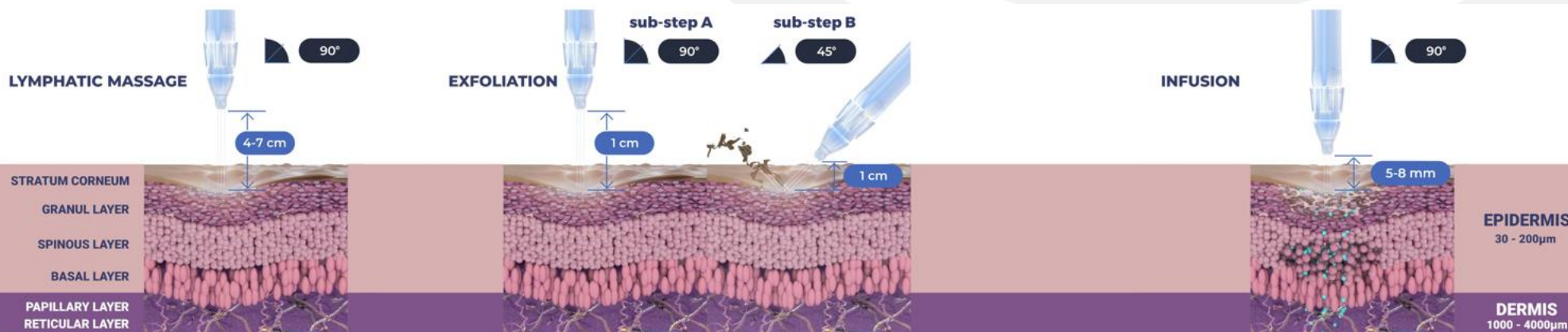

Step 1 - Lymphatic Massage

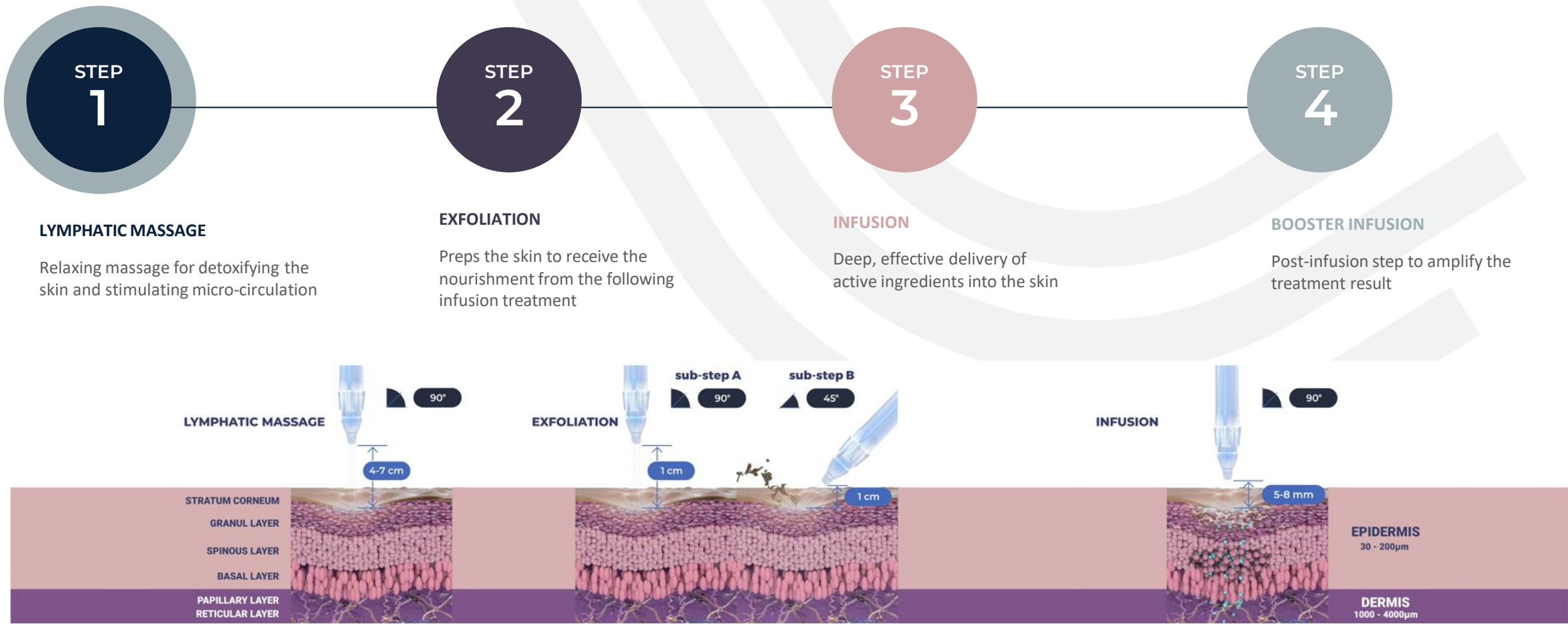



JetCareHYDRO

Powerful JetPeel treatments begin with JetCare Hydro

JC-H



Used for indications:

Lymphatic Massage, All Indications.

Treatment guideline numbers:

All.

All JetPeel sessions are initiated with JetCare Hydro. This gentle water-based solution is formulated especially for the initial lymphatic massage, and to prep the skin for the deeper JetCare treatments which follow.

Prepare the skin for beautiful results

Most of our facial muscles are not usually used. The JetPeel lymphatic massage with JetCare Hydro effectively stimulates facial muscles and blood microcirculation for a young, healthy appearance. The JetCare Hydro comprises a very mild concentration of sodium chloride solution, combined with soothing organic aloe vera, and reinforced with a relatively high concentration of hyaluronic acid.



Key ingredients

Organic Aloe Vera

This enduring extract, known for its calming effect, is a significant ingredient in JetCare Hydro. Aloe Vera is proven to be an antioxidant emollient that instantly softens and soothes the skin, providing antimicrobial benefits.

Hyaluronic Acid

This unique formulation contains low-molecular HA that allows for improved penetration and impact within the skin. The relatively high concentration of HA in this ingredient boosts moisture, helps to protect the skin against dehydration and ensures high-performance while supporting a gentle, yet powerful lymphatic massage.



JetPeel's **Lymphatic
Massage** before and
after other aesthetic
procedures help to:

- Detoxify the skin
- Decrease inflammation
- Optimize skin health
- Decrease skin pain
- Promote healing process
- Clear out bacteria and toxins
- Increase skin hydration

Step 2 - Exfoliation

STEP 1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP 2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP 3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP 4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

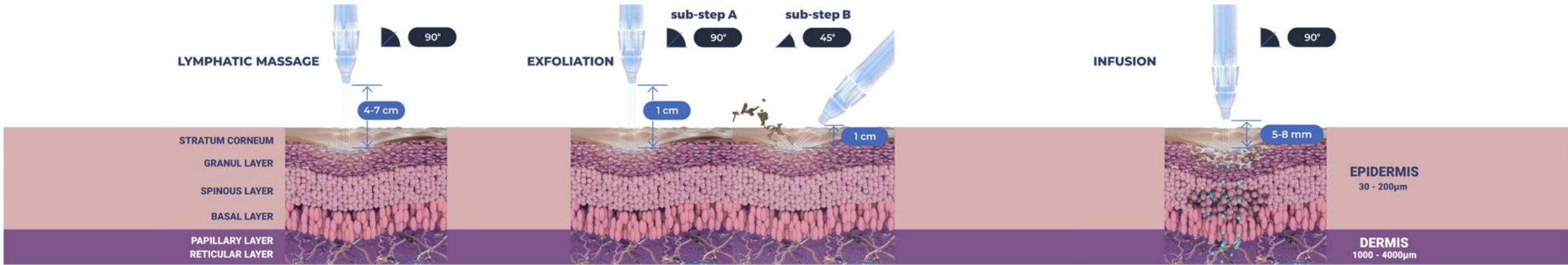

The Renewal Care Series

Glycolic Acid Series
Mandelic & Salicylic



The importance of skin **exfoliation** in medical aesthetic procedures:

- Eliminates dead skin cells, dirt and oil that can cause clogged pores
- Improves skin texture and discolorations
- Diminishes skin imperfections
- Stimulates cells turnover and skin regeneration
- Improves deep wrinkles and photoaging
- Stimulates collagen production

Step 3 – Infusion

STEP 1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP 2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP 3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP 4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

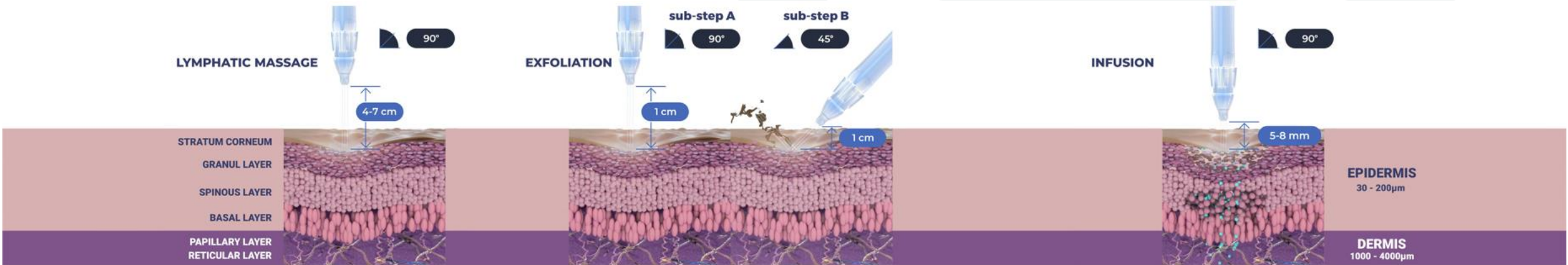

The Anti-Aging Care Series

**Wide Range Hyaluronic Acid
Peptide Complex
Witch Hazel & Peptides**



The Selective Care Series

Argireline® Complex
Lightening
Soothing
Clear
Scalp & Hair
Growth Factor



Infusion supports skin cell activities

- Promote keratinocyte proliferation
- Improve skin pigmentation
- Support extracellular matrix and fibroblast
- Increase hydration
- Promote cell growth and collagen
- Support skin structure
- Prevent oxidative stress

Step 4 – Booster Infusion

STEP
1

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation

STEP
2

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment

STEP
3


INFUSION

Deep, effective delivery of active ingredients into the skin

STEP
4

BOOSTER INFUSION

Post-infusion step to amplify the treatment result

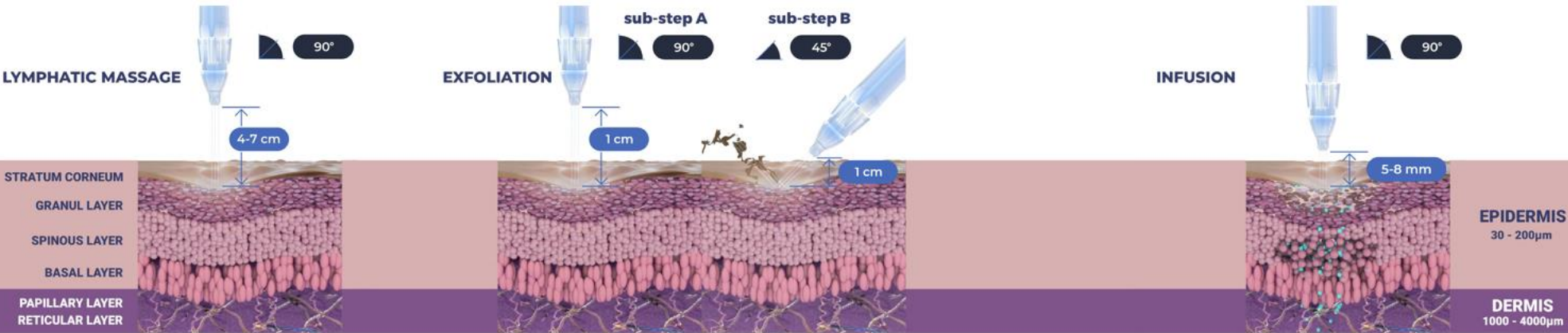

The JetCare Boost Series

Hydrate
Regenerate
Glow
Tight
Boost set



The **Booster** **Infusion** amplifies results

This step targets multiple skin concerns by delivering a high concentration of hyaluronic acid and vitamins which:

- Improve texture
- Increase elasticity
- Hydrate and plump
- Restore radiance and overall skin appearance

JETPEEL **2 DAYS BEFORE** OTHER AESTHETIC PROCEDURES

STEP
1

LYMPHATIC MASSAGE

USE: JetCare Hydro

- Smooth skin texture
- Moisturize
- Support skin barrier and reduce water loss
- Decrease dryness and skin irritation



Hyaluronic Acid, organic Aloe Vera

STEP
2

EXFOLIATION

USE: JetCare Glycolic Acid 5%

- Increase skin turnover
- Collagen synthesis
- Increase skin elasticity and firmness
- Improve skin texture
- Increase skin hydration



Glycolic acid 5% - pH 3.5%, organic Aloe Vera , Pro-Vitamin B₅ Panthenol, Allantoin

STEP
3

INFUSION

USE: JetCare Wide Range Hyaluronic Acid

- Increase skin hydration
- Regulate tissue repair
- Reduce breakout of collagen and elastin matrix
- Strengthen connective tissue



5 molecular weights of Hyaluronic Acid, Regu-Age®

STEP
4

BOOSTER INFUSION

USE: JetCare Boost Regenerate

- Fortify the functioning of the skin's natural barrier
- Rejuvenate skin tissue
- Repair skin damage




Vitamins A and E, Pentavitin®, Ragu-age®, Roman Chamomile Extract

JetPeel Protocol

	Step 1 Lymphatic Massage	Step 2 Exfoliation	Step 3 Infusion	Step 4 Booster Infusion
Handpiece	Triple	Triple	Triple or Magic	Triple or magic
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
Use	18ml of JetCare Hydro	5ml of JetCare Renewal Care Glycolic Acid 5%	5ml of JetCare Anti-Aging Wide Range Hyaluronic Acid	3ml of JetCare Boost Hydrate
Remarks		Leave on the skin for 5-8 minutes, and then wash with 8ml of JetCare Hydro with Triple handpiece at 45°-60°		



JetPeel AFTER
other aesthetic procedures

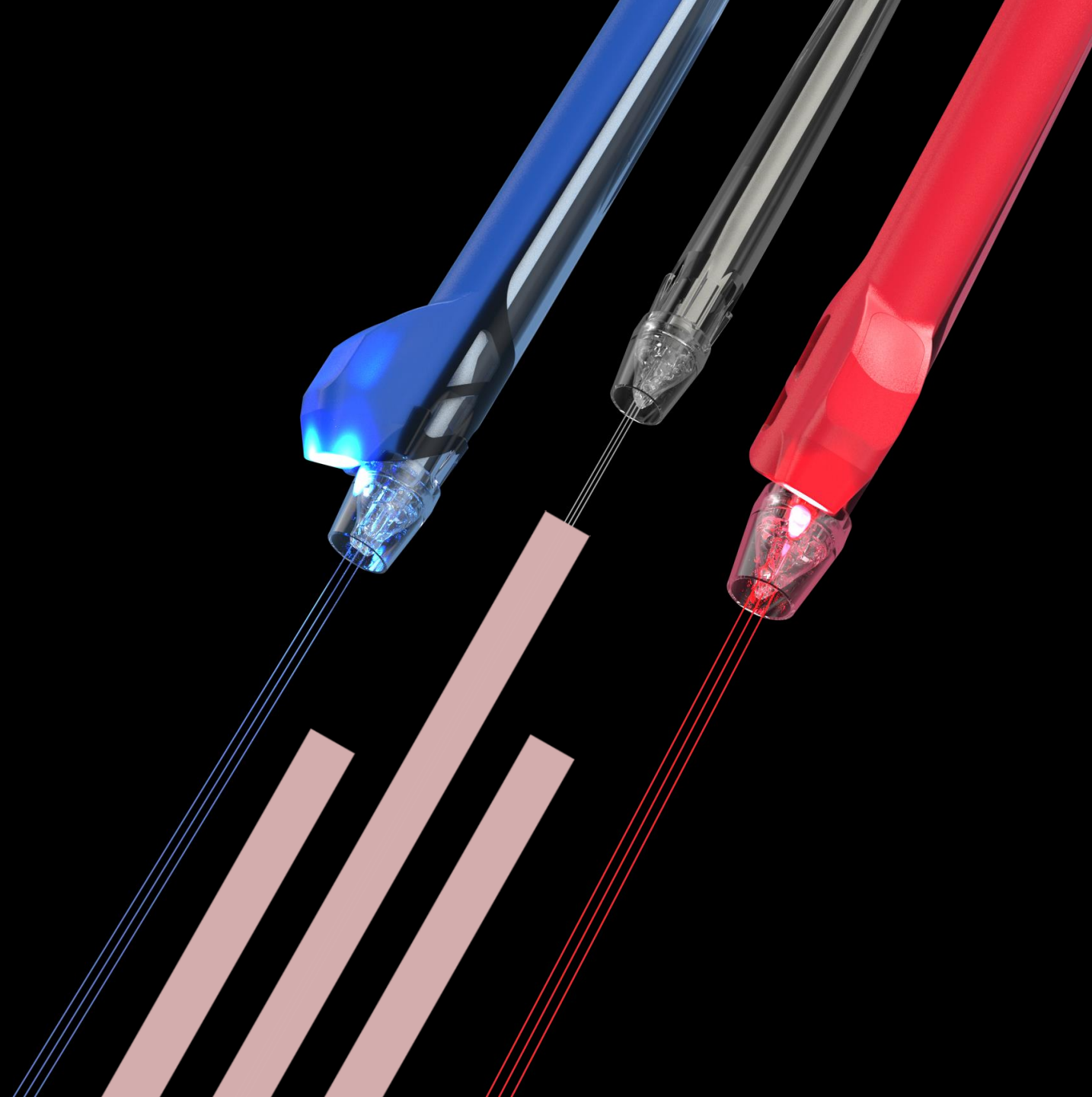

When should a JetPeel treatment be performed **AFTER** other aesthetic procedures?

- Botox - 1 month after* Important note: do not use the JetCare Boost TIGHT solution for approximately 4 months after a botox treatment. Vitamin B can make the botox disappear in a short time.
- Filler - 1 month after
- RF - can be performed on the same day
- IPL - 2 weeks after
- Ablative laser - 2-3 months after* Important note: sometimes it can be done earlier but without performing the exfoliation.
- Non-ablative laser - 2 weeks after* Important note: JetPeel treatment can help rid the dots from the skin quickly.
- Microneedling – 1-2 weeks after

***IMPORTANT NOTE:** The times indicated above can change. This depends on the intensity of different aesthetic procedures and on individual skin healing process. **DO NOT perform any JetPeel treatment on open skin.**

Protocol A: Post-medical aesthetic procedures

STEP
1

LYMPHATIC MASSAGE

USE: JetCare Hydro

- Smooth skin texture
- Moisturize
- Support skin barrier and reduce water loss
- Prevent dryness and skin irritation



Hyaluronic Acid, organic Aloe Vera

STEP
2

EXFOLIATION

USE: JetCare Glycolic Acid 5%

- Increase skin turnover
- Collagen synthesis
- Increase skin elasticity and firmness
- Improve skin texture
- Increase skin hydration



Glycolic acid 5% - pH 3.5%, organic Aloe Vera, Pro-vitamin B₅ Panthenol, Allantoin

STEP
3

INFUSION

USE: JetCare Growth Factor

- Replenish natural growth factors
- Accelerate skin repair
- Increase skin renewal
- Boost collagen



3% EGF (Epidermal Growth Factor) complex of 5 different growth factors, Vitamin B₉, Niacinamide, Betaine

STEP
4

BOOSTER INFUSION

USE: JetCare Boost Hydrate

- Protect the extracellular matrix from collagen degradation
- Restore the antioxidant defences
- Immediate anti-ageing and wrinkles filler effect
- Smooths skin texture



PrimalHyal™ Ultrafiller, Niacinamide, Betaine

Protocol A – AFTER
aesthetic procedures

Recommended Treatment Cycle

- **First 6 treatments:** 1-2 times every week for 3-6 weeks
- **After 6 treatments:** Maintenance treatments every 3-4 weeks
- Note: alternate between Protocol A and Protocol B



	Step 1 Lymphatic Massage	Step 2 Exfoliation	Step 3 Infusion	Step 4 Booster Infusion
Handpiece	Triple	Triple	Triple or Magic	Triple or magic
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
Use	18ml of JetCare Hydro	5ml of JetCare Renewal Care Glycolic Acid 5%	5ml of JetCare Selective Care Growth Factor	3ml of JetCare Boost Hydrate
Remarks		Levae on the skin for 5-8 minutes and then wash with 8ml of JetCare Hydro. Use triple handpiece at 45°-60°		

Protocol B: Post medical aesthetic procedures

STEP
1

LYMPHATIC MASSAGE

USE: JetCare Hydro

- Smooth skin texture
- Moisturize
- Support skin barrier and reduce water loss
- Prevent dryness and skin irritation



Hyaluronic Acid, organic Aloe Vera

STEP
2

EXFOLIATION

USE: JetCare Mandelic & Salicylic

- Remove dirt, bacteria, clear pores, purify unclogged pores
- Brighten skin and smoothen skin texture.
- Improve cell turnover
- Improve skin hydration
- Reduce wrinkles and aging signs



Mandelic 4% (AHA), Salicylic 2% (BHA), Malic 1% (AHA), Glycolic 0.7% (AHA), Lactic 0.7%, ABC fruit extract, D-Phanthenol, Roman Chamomile Extract, Aloe

STEP
3

INFUSION

USE: JetCare Peptide Complex

- Stimulate epidermal and dermal activity
- Increase collagen production
- Increase dermis density and skin firmness.



Replexium™, Juveleven™, Pentavitin®, Regu-Age®, Niacinamide, Betaine

STEP
4

BOOSTER INFUSION

USE: JetCare Boost Regenerate

- Fortify the functioning of the skin's natural barrier
- Rejuvenate skin tissue
- Repair skin damage



Vitamins A and E, Pentavitin®, Ragu-age®, Roman Chamomile Extract

Protocol B – AFTER aesthetic procedures

Recommended Treatment Cycle

- **First 6 treatments:** 1-2 times every week for 3-6 weeks
- **After 6 treatments:** Maintenance treatments every 3-4 weeks
- **Note:** alternate between Protocol B and Protocol A



	Step 1 Lymphatic Massage	Step 2 Exfoliation	Step 3 Infusion	Step 4 Booster Infusion
Handpiece	Triple	Triple	Triple or Magic	Triple or magic
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
Use	18ml of JetCare Hydro	5ml of JetCare Renewal Care Mandelic & Salicylic	5ml of JetCare Anti- Aging Peptide Complex	3ml of JetCare Boost Regenerate
Remarks		Wash with 8ml of JetCare Hydro if necessary. Triple handpiece at 45°-60°		

TREATMENT GUIDELINES



Anti-Aging Treatments by Skin Characteristics

ANTI-AGING - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 4 with JetCare Boost Tight
Dry	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Sensitive	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 6 with JetCare Boost Glow	Treatment Guideline 10 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

ANTI-AGING – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Dry	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 2 with JetCare Boost Glow	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Sensitive	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 10 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 6 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 6 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Treatment Guideline 10 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 7 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 8 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight



Loss of Tone and Wrinkles Treatments by Skin Characteristics

LOSS OF TONE & WRINKLES - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 4 with JetCare Boost Regenerate
Dry	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 1 with JetCare Boost Regenerate
Oily	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 2 with JetCare Boost Regenerate
Sensitive	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 1 with JetCare Boost Regenerate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 5 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 6 with JetCare Boost Glow	Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 20 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA

LOSS OF TONE & WRINKLES - MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 2 with JetCare Boost Tight	Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 6 with JetCare Boost Regenerate
Dry	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 5 with JetCare Boost Regenerate
Oily	Treatment Guideline 2 with JetCare Boost Tight	Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 7 with JetCare Boost Regenerate
Sensitive	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 5 with JetCare Boost Regenerate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 10 with JetCare Boost Hydrate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 11 with JetCare Boost Hydrate	Treatment Guideline 19 Boost with Wide Range HA
Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA



Acne Treatments by Skin Characteristics

ACNE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 4 with JetCare Boost Tight
Dry	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 4 with JetCare Boost Tight
Oily	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 1 with JetCare Boost Tight
Sensitive	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 4 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 28 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 32 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 32 with JetCare Boost Tight	Treatment Guideline 28 with JetCare Boost Hydrate

ACNE - MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 26 with JetCare Boost Glow	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 27 with JetCare Boost Glow	Treatment Guideline 2 with JetCare Boost Tight
Sensitive	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 30 with JetCare Boost Tight	Treatment Guideline 26 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 31 with JetCare Boost Tight	Treatment Guideline 27 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate



Post-Acne Treatments by Skin Characteristics

POST-ACNE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Dry	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Oily	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Sensitive	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA

POST-ACNE – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Dry	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA
Oily	Treatment Guideline 2 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 19 Boost with Wide Range HA
Sensitive	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 9 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 19 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA



Couperose and Sensitive Skin Treatments by Skin Characteristics

COUPEROSE AND SENSITIVE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Dry	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Oily	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Sensitive	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 4 with JetCare Boost Glow

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 21 with JetCare Boost Hydrate

COUPEROSE & SENSITIVE – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 21 with JetCare Boost Regenerate	Treatment Guideline 22 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Dry	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Oily	Treatment Guideline 22 with JetCare Boost Regenerate	Treatment Guideline 23 with JetCare Boost Tight	Treatment Guideline 2 with JetCare Boost Glow
Sensitive	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 24 with JetCare Boost Glow	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 11 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 24 with JetCare Boost Glow	Treatment Guideline 21 with JetCare Boost Hydrate



Lightening and Pigmentation Treatments by Skin Characteristics

LIGHTENING AND PIGMENTATION - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Dry	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 4 with JetCare Boost Tight
Oily	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Sensitive	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 4 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 13 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 10 with JetCare Boost Regenerate	Treatment Guideline 14 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate

LIGHTENING & PIGMENTATION – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 3 with JetCare Boost Tight
Dry	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Sensitive	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 10 with JetCare Boost Regenerate	Treatment Guideline 14 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 31 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 15 with JetCare Boost Hydrate
Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate



Scalp and Hair Treatment Guidelines

SCALP TYPE	1st Treatment	2nd Treatment	3rd Treatment	4th Treatment	5th Treatment	6th Treatment	7th Treatment	8th Treatment	9th Treatment	10th Treatment	Maintenance Treatments (every 3-4 weeks)
 Itchy, Sensitive	Treatment Guideline 33 with JetCare Hydro with Boost Regenerate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 35 with JetCare Hydro with Boost Hydrate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 36 with JetCare Hydro with Boost Tight	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp and Hair
Dry	Treatment Guideline 33 with JetCare Hydro with Boost Regenerate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 38 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp and Hair
Oily	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 40 with Glycolic Acid 10% with Boost Hydrate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 39 with Glycolic Acid 5% with Scalp and Hair
Flaky	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 33 with JetCare Hydro with Boost Regenerate	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 40 with Glycolic Acid 10% with Boost Hydrate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 41 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 39 with Glycolic Acid 5% with Scalp and Hair




JetPeel by TavTech

The perfect trio combining:



The enduring
JetPro devices

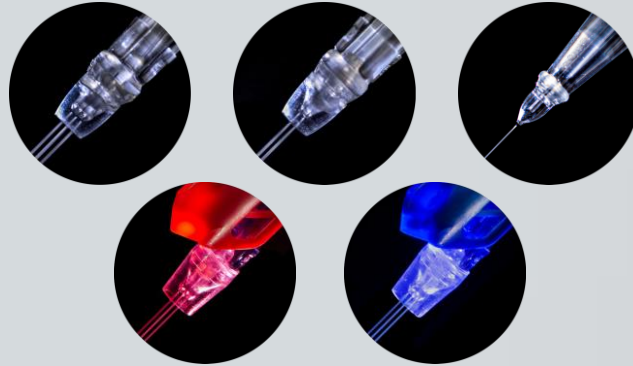

The original patented
JetPeel handpieces



The premium
JetCare solutions

Designed exclusively for use together to deliver outstanding results



Thank you

