TREATMENT GUIDELINES







Feels More Beautiful Than Ever

Welcome to JetCare Treatment Guidelines.

2

JetPeel's entire line of cosmetic solutions has been renewed, with each formula improved and upgraded with the latest cutting-edge materials and ingredients, all from the highest quality cosmetic suppliers, and all especially suited to JetPeel's unique and patented handpieces, and to its enduring jet pressure-energy device.

Concentrations have been improved to optimally treat a wide range of skin types, body areas and customer concerns.

JetPeel brings this new and vastly improved line of JetCare solutions to its customers after many months of R&D and testing to ensure that it is the most compatible line of solutions to the JetPeel handpieces and devices, and after ensuring enhanced performance.

Developed exclusively by JetPeel in conjunction with a leading, renowned cosmetics formulation laboratory, JetCare products deliver powerfully effective JetPeel treatments in the clinic, spa and cosmetic salon, and are the only solutions sanctioned and approved by JetPeel for use with its handpieces and devices.

Together with these advances, JetPeel by TavTech has suited Treatment Guidelines to its exclusive line of JetCare solutions for the best treatment outcomes and customer satisfaction.

JetCare by TavTech is the first choice for cosmeticians and specialists worldwide, and the only choice for the ultimate treatment experience by JetPeel.

With JetCare formulas and Treatment Guidelines, the skin feels smoother, brighter and more beautiful than ever before.

CONTENTS

JetPeel Treatment Sequence	4
Skin	6-35
Skin Characteristics	6
Anti-Aging Treatments by Skin Characteristics	8
Noss of Tone and Wrinkle Treatments by Skin Characteristics	10
Acne Treatments by Skin Characteristics	12
Nost-Acne Treatments by Skin Characteristics	14
Ouperose and Sensitive Skin Treatments by Skin Characteristics	16
Lightening and Pigmentation Treatments by Skin Characteristics	18
Skin Treatment Guidelines	20
Scalp & Hair	36-43
Scalp & Hair Characteristics	37
Nreatment Guidelines	38





The Treatment Sequence



STEP 02

STEP 03

STEP 04

LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation.

EXFOLIATION


Preps the skin to receive the nourishment from the following infusion treatment.


INFUSION

Deep, effective delivery of active ingredients into the skin.

BOOSTER INFUSION

Post-infusion step to amplify the treatment result.

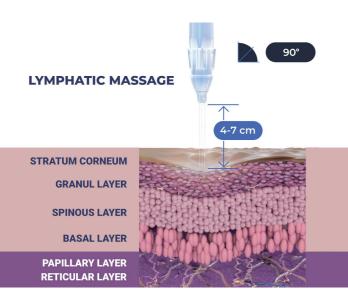

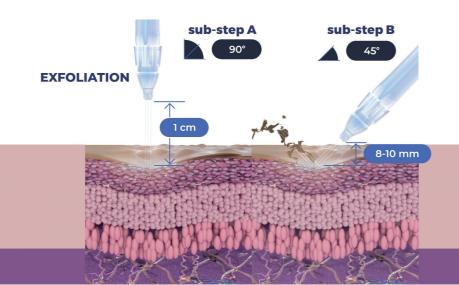

INFUSION

EPIDERMIS

30 - 200µm

DERMIS 1000 - 4000μm





SKIN

Recommended Treatment Cycle



First 6 treatments:

1-2 times every week for 3-6 weeks



After 6 treatments:

Maintenance treatments every 3-4 weeks

Skin Characteristics

Maturity in the skin occurs when the skin's ability to retain its youthful glow is detrimentally affected by the natural process of aging and by adverse environmental influences such as pollution and sun exposure. This leads to a decrease in the production of collagen, elastin and sebum, and increased risk of dehydration, which may affect skin hyperpigmentation, dark spots and appearance.

Normal Skin Type

Normal skin, which is neither too dry nor too oily, has:

- · No or few imperfections
- · No severe sensitivity
- · Barely visible pores
- A radiant complexion

Oily Skin Type

- Enlarged pores
- · Dull or shiny, thick complexion
- Blackheads, pimples, or other blemishes

Dry Skin Type

- Almost invisible pores
- Dull, rough complexion
- Red patches
- Less elasticity
- More visible lines

Sensitive Skin Type

- Redness
- Itching
- BurningDryness

Combination Skin Type

Skin can be dry or normal in some areas and oily in others, such as the T-zone (nose, forehead, and chin). Many people have this skin type. It may need different care in different areas

Combination skin can have:

- Pores that look larger than normal since they are more open
- Blackheads
- Shiny skin

JETCARE TAVTECH LTD.

Anti-Aging Treatments by Skin Characteristics

ANTI-AGING - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4	Treatment Guideline 13	Treatment Guideline 4
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Dry	Treatment Guideline 4	Treatment Guideline 13	Treatment Guideline 1
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Oily	Treatment Guideline 4	Treatment Guideline 14	Treatment Guideline 1
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Sensitive	Treatment Guideline 4	Treatment Guideline 13	Treatment Guideline 1
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight

ANTI-AGING – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Dry	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 2 with JetCare Boost Glow	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Sensitive	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8	Treatment Guideline 9	Treatment Guideline 1
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8	Treatment Guideline 29	Treatment Guideline 1
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Treatment Guideline 14	Treatment Guideline 6	Treatment Guideline 10	Treatment Guideline 2
with	with	with	with
JetCare Boost Hydrate	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8	Treatment Guideline 29	Treatment Guideline 1
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 10	Treatment Guideline 29	Treatment Guideline 6	Treatment Guideline 2
with	with	with	with
JetCare Boost Hydrate	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Treatment Guideline 18	Treatment Guideline 29	Treatment Guideline 6	Treatment Guideline 1
Boost with	with	with	with
Wide Range HA	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Treatment Guideline 10 with JetCare Boost Hydrate	Treatment Guideline 29	Treatment Guideline 7	Treatment Guideline 2
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight
Treatment Guideline 17	Treatment Guideline 29	Treatment Guideline 8	Treatment Guideline 1
Boost with	with	with	with
Wide Range HA	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Tight



Loss of Tone and Wrinkles Treatments by Skin Characteristics

LOSS OF TONE & WRINKLES - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4	Treatment Guideline 17	Treatment Guideline 4
	with	Boost with	with
	JetCare Boost Tight	Wide Range HA	JetCare Boost Regenerate
Dry	Treatment Guideline 4	Treatment Guideline 17	Treatment Guideline 1
	with	Boost with	with
	JetCare Boost Tight	Wide Range HA	JetCare Boost Regenerate
Oily	Treatment Guideline 1	Treatment Guideline 17	Treatment Guideline 2
	with	Boost with	with
	JetCare Boost Tight	Wide Range HA	JetCare Boost Regenerate
Sensitive	Treatment Guideline 4	Treatment Guideline 17	Treatment Guideline 1
	with	Boost with	with
	JetCare Boost Tight	Wide Range HA	JetCare Boost Regenerate

LOSS OF TONE & WRINKLES - MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 2	Treatment Guideline 19	Treatment Guideline 6
	with	Boost with	with
	JetCare Boost Tight	Wide Range HA	JetCare Boost Regenerate
Dry	Treatment Guideline 1	Treatment Guideline 18	Treatment Guideline 5
	with	Boost with	with
	JetCare Boost Tight	Wide Range HA	JetCare Boost Regenerate
Oily	Treatment Guideline 2	Treatment Guideline 19	Treatment Guideline 7
	with	Boost with	with
	JetCare Boost Tight	Wide Range HA	JetCare Boost Regenerate
Sensitive	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 5 with JetCare Boost Regenerate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 5	Treatment Guideline 17	Treatment Guideline 12	Treatment Guideline 17
with	Boost with	with	Boost with
JetCare Boost Glow	Wide Range HA	JetCare Boost Hydrate	Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17	Treatment Guideline 17	Treatment Guideline 17
	Boost with	Boost with	Boost with
	Wide Range HA	Wide Range HA	Wide Range HA
Treatment Guideline 6	Treatment Guideline 18	Treatment Guideline 12	Treatment Guideline 18
with	Boost with	with	Boost with
JetCare Boost Glow	Wide Range HA	JetCare Boost Hydrate	Wide Range HA
Treatment Guideline 8	Treatment Guideline 20	Treatment Guideline 17	Treatment Guideline 17
with	Boost with	Boost with	Boost with
JetCare Boost Glow	Wide Range HA	Wide Range HA	Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 19	Treatment Guideline 29	Treatment Guideline 10	Treatment Guideline 18
Boost with	with	with	Boost with
Wide Range HA	JetCare Boost Glow	JetCare Boost Hydrate	Wide Range HA
Treatment Guideline 18	Treatment Guideline 29	Treatment Guideline 17	Treatment Guideline 17
Boost with	with	Boost with	Boost with
Wide Range HA	JetCare Boost Glow	Wide Range HA	Wide Range HA
Treatment Guideline 19	Treatment Guideline 29	Treatment Guideline 11	Treatment Guideline 19
Boost with	with	with	Boost with
Wide Range HA	JetCare Boost Glow	JetCare Boost Hydrate	Wide Range HA
Treatment Guideline 17	Treatment Guideline 29	Treatment Guideline 17	Treatment Guideline 17
Boost with	with	Boost with	Boost with
Wide Range HA	JetCare Boost Glow	Wide Range HA	Wide Range HA





Acne Treatments by Skin Characteristics

ACNE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 28	Treatment Guideline 25	Treatment Guideline 4
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Glow	JetCare Boost Tight
Dry	Treatment Guideline 28	Treatment Guideline 25	Treatment Guideline 4
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Glow	JetCare Boost Tight
Oily	Treatment Guideline 28	Treatment Guideline 25	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Glow	JetCare Boost Tight
Sensitive	Treatment Guideline 28	Treatment Guideline 25	Treatment Guideline 4
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Glow	JetCare Boost Tight

ACNE – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 28	Treatment Guideline 26	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Glow	JetCare Boost Tight
Oily	Treatment Guideline 28	Treatment Guideline 27	Treatment Guideline 2
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Glow	JetCare Boost Tight
Sensitive	Treatment Guideline 28	Treatment Guideline 25	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Glow	JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 28 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 32 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 32 with JetCare Boost Tight	Treatment Guideline 28 with JetCare Boost Hydrate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14	Treatment Guideline 28	Treatment Guideline 30	Treatment Guideline 26
with	with	with	with
JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Hydrate
Treatment Guideline 14	Treatment Guideline 28	Treatment Guideline 31	Treatment Guideline 27
with	with	with	with
JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Hydrate
Treatment Guideline 14	Treatment Guideline 28	Treatment Guideline 29	Treatment Guideline 25
with	with	with	with
JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Hydrate



Post-Acne Treatments by Skin Characteristics

POST-ACNE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 1	Treatment Guideline 14	Treatment Guideline 18
	with	with	Boost with
	JetCare Boost Tight	JetCare Boost Regenerate	Wide Range HA
Dry	Treatment Guideline 1	Treatment Guideline 13	Treatment Guideline 18
	with	with	Boost with
	JetCare Boost Tight	JetCare Boost Regenerate	Wide Range HA
Oily	Treatment Guideline 1	Treatment Guideline 14	Treatment Guideline 18
	with	with	Boost with
	JetCare Boost Tight	JetCare Boost Regenerate	Wide Range HA
Sensitive	Treatment Guideline 1	Treatment Guideline 13	Treatment Guideline 17
	with	with	Boost with
	JetCare Boost Tight	JetCare Boost Regenerate	Wide Range HA

POST-ACNE – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Dry	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA
Oily	Treatment Guideline 2 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 19 Boost with Wide Range HA
Sensitive	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12	Treatment Guideline 30	Treatment Guideline 18
	with	with	Boost with
	JetCare Boost Hydrate	JetCare Boost Regenerate	Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 18	Treatment Guideline 29	Treatment Guideline 18
	Boost with	with	Boost with
	Wide Range HA	JetCare Boost Regenerate	Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12	Treatment Guideline 30	Treatment Guideline 18
	with	with	Boost with
	JetCare Boost Hydrate	JetCare Boost Regenerate	Wide Range HA
Treatment Guideline 8	Treatment Guideline 17	Treatment Guideline 29	Treatment Guideline 17
with	Boost with	with	Boost with
JetCare Boost Glow	Wide Range HA	JetCare Boost Regenerate	Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 8	Treatment Guideline 12	Treatment Guideline 30	Treatment Guideline 18
with	with	with	Boost with
JetCare Boost Glow	JetCare Boost Hydrate	JetCare Boost Regenerate	Wide Range HA
Treatment Guideline 8	Treatment Guideline 17	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 17
with	Boost with		Boost with
JetCare Boost Glow	Wide Range HA		Wide Range HA
Treatment Guideline 8	Treatment Guideline 9	Treatment Guideline 30	Treatment Guideline 19
with	with	with	Boost with
JetCare Boost Glow	JetCare Boost Hydrate	JetCare Boost Regenerate	Wide Range HA
Treatment Guideline 8	Treatment Guideline 17	Treatment Guideline 30	Treatment Guideline 17
with	Boost with	with	Boost with
JetCare Boost Glow	Wide Range HA	JetCare Boost Regenerate	Wide Range HA



Couperose and Sensitive Skin Treatments by Skin Characteristics

COUPEROSE AND SENSITIVE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 24	Treatment Guideline 21	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow
Dry	Treatment Guideline 24	Treatment Guideline 21	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow
Oily	Treatment Guideline 24	Treatment Guideline 21	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow
Sensitive	Treatment Guideline 24	Treatment Guideline 21	Treatment Guideline 4
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow

COUPEROSE & SENSITIVE - MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 21	Treatment Guideline 22	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow
Dry	Treatment Guideline 24	Treatment Guideline 21	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow
Oily	Treatment Guideline 22	Treatment Guideline 23	Treatment Guideline 2
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow
Sensitive	Treatment Guideline 24	Treatment Guideline 21	Treatment Guideline 1
	with	with	with
	JetCare Boost Regenerate	JetCare Boost Tight	JetCare Boost Glow

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16	Treatment Guideline 8	Treatment Guideline 9	Treatment Guideline 21
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8	Treatment Guideline 29	Treatment Guideline 21
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Hydrate	JetCare Boost Hydrate
Treatment Guideline 14	Treatment Guideline 8	Treatment Guideline 9	Treatment Guideline 21
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8	Treatment Guideline 29	Treatment Guideline 21
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Hydrate	JetCare Boost Hydrate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14	Treatment Guideline 29	Treatment Guideline 9	Treatment Guideline 21
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Hydrate
Treatment Guideline 14	Treatment Guideline 29	Treatment Guideline 24	Treatment Guideline 21
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Glow	JetCare Boost Hydrate
Treatment Guideline 14	Treatment Guideline 29	Treatment Guideline 11	Treatment Guideline 21
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Hydrate
Treatment Guideline 13	Treatment Guideline 29	Treatment Guideline 24	Treatment Guideline 21
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Glow	JetCare Boost Hydrate



Lightening and Pigmentation Treatments by Skin Characteristics

LIGHTENING AND PIGMENTATION - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 16	Treatment Guideline 13	Treatment Guideline 1
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight
Dry	Treatment Guideline 16	Treatment Guideline 13	Treatment Guideline 4
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight
Oily	Treatment Guideline 16	Treatment Guideline 14	Treatment Guideline 1
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight
Sensitive	Treatment Guideline 16	Treatment Guideline 13	Treatment Guideline 4
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight

LIGHTENING & PIGMENTATION – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 13	Treatment Guideline 14	Treatment Guideline 3
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight
Dry	Treatment Guideline 16	Treatment Guideline 14	Treatment Guideline 1
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight
Oily	Treatment Guideline 14	Treatment Guideline 15	Treatment Guideline 2
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight
Sensitive	Treatment Guideline 16	Treatment Guideline 13	Treatment Guideline 1
	with	with	with
	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8	Treatment Guideline 9	Treatment Guideline 13
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8	Treatment Guideline 13	Treatment Guideline 13
	with	with	with
	JetCare Boost Glow	JetCare Boost Hydrate	JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 8	Treatment Guideline 10	Treatment Guideline 14
	with	with	with
	JetCare Boost Glow	JetCare Boost Regenerate	JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8	Treatment Guideline 13	Treatment Guideline 13
	with	with	with
	JetCare Boost Glow	JetCare Boost Hydrate	JetCare Boost Hydrate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14	Treatment Guideline 29	Treatment Guideline 10	Treatment Guideline 14
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Regenerate	JetCare Boost Hydrate
Treatment Guideline 14	Treatment Guideline 29	Treatment Guideline 14	Treatment Guideline 14
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Hydrate	JetCare Boost Hydrate
Treatment Guideline 14	Treatment Guideline 31	Treatment Guideline 14	Treatment Guideline 15
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Hydrate	JetCare Boost Hydrate
Treatment Guideline 13	Treatment Guideline 29	Treatment Guideline 13	Treatment Guideline 13
with	with	with	with
JetCare Boost Regenerate	JetCare Boost Hydrate	JetCare Boost Hydrate	JetCare Boost Hydrate



Loss of Tone & Wrinkles

Post-Acne

Couperose & Sensitive Skin Lightening & Pigmentation

Glycolic Acid 5% + Wide Range Hyaluronic Acid

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Anti-Aging Care Wide Range Hyaluronic Acid	~3ml of JetCare Boost: Tight/ Regenerate/Glow
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 2

Loss of Tone & Wrinkles

Post-Acne

Couperose & Sensitive Skin Lightening & Pigmentation

Glycolic Acid 10% + Wide Range Hyaluronic Acid

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
日 Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Anti-Aging Care Wide Range Hyaluronic Acid	~3ml of JetCare Boost: Tight/ Regenerate/Glow
! Remarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 3

Indications: Lightening & Pigmentation

Glycolic Acid 16% + Wide Range Hyaluronic Acid

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Anti-Aging Care Wide Range Hyaluronic Acid	~3ml of JetCare Boost: Tight
! Remarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		

Treatment Guideline No. 4

Loss of Tone & Wrinkles

Couperose & Sensitive Skin Lightening & Pigmentation

Mandelic & Salicylic + Wide Range Hyaluronic Acid

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Anti-Aging Care Wide Range Hyaluronic Acid	~3ml of JetCare Boost: Tight/ Regenerate/Glow
! Remarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics



Indications: Loss of Tone & Wrinkles

Glycolic Acid 5% + Peptide Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Anti-Aging Care Peptide Complex	~3ml of JetCare Boost: Regenerate/ Glow
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 6

22

Indications:

Anti-Aging

Loss of Tone & Wrinkles

Glycolic Acid 10% + Peptide Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Anti-Aging Care Peptide Complex	~3ml of JetCare Boost: Regenerate/ Glow
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 7

Indications: Anti-Aging

Glycolic Acid 16% + Peptide Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Anti-Aging Care Peptide Complex	~3ml of JetCare Boost: Regenerate
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		

Treatment Guideline No. 8

Loss of Tone & Wrinkles

Couperose & Sensitive Skin Lightening & Pigmentation

Post-Acne

Mandelic & Salicylic + Peptide Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Anti-Aging Care Peptide Complex	~3ml of JetCare Boost: Regenerate/ Glow/ Hydrate
! Remarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics



Lightening & Pigmentation

Post-Acne Couperose & Sensitive Skin

Glycolic Acid 5% + Witch Hazel & Peptides

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Anti-Aging Care Witch Hazel & Peptides	~3ml of JetCare Boost: Regenerate/ Glow/Hydrate
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 10

Indications:

Lightening & Pigmentation Loss of Tone & Wrinkles

Glycolic Acid 10% + Witch Hazel & Peptides

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Anti-Aging Care Witch Hazel & Peptides	~3ml of JetCare Boost: Regenerate/ Glow/Hydrate
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 11

Indications: Loss of Tone & Wrinkles Couperose & Sensitive Skin

Glycolic Acid 16% + Witch Hazel & Peptides

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP INFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(B) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Anti-Aging Care Witch Hazel & Peptides	~3ml of JetCare Boost: Glow / Hydrate
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 12

Loss of Tone & Wrinkles

Post-Acne

Mandelic & Salicylic + Witch Hazel & Peptides

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Anti-Aging Care Witch Hazel & Peptides	~3ml of JetCare Boost: Hydrate
! Remarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°-60°		



Indications:

Anti-Aging Post-Acne

Couperose & Sensitive Skin Lightening & Pigmentation

Glycolic Acid 5% + Lightening

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Selective Care Lightening	~3ml of JetCare Boost: Regenerate / Glow / Hydrate
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 14

26

Indications:

Acne Post-Acne

Anti-Aging

Couperose & Sensitive Skin Lightening & Pigmentation

Glycolic Acid 10% + Lightening

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Selective Care Lightening	~3ml of JetCare Boost: Regenerate / Glow / Hydrate
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 15

Indications:

Anti-Aging Lightening & Pigmentation

Glycolic Acid 16% + Lightening

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
(L) Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Selective Care Lightening	~3ml of JetCare Boost: Regenerate / Hydrate
! Remarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 16

Indications:

Anti-Aging

Couperose & Sensitive Skin Lightening & Pigmentation

27

Mandelic & Salicylic + Lightening

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP INFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Selective Care Lightening	~3ml of JetCare Boost: Regenerate / Glow / Hydrate
! Remarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

JETCARE by TayTech TAVTECH LTD.

Indications: Anti-Aging

Post-Acne

Loss of Tone & Wrinkles

Glycolic Acid 5% + Argireline® Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Selective Care Argireline® Complex	~3ml of Anti-Aging Care Wide Range Hyaluronic Acid
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60'		

Treatment Guideline No. 18

Indications:

Anti-Aging

Post-Acne

Loss of Tone & Wrinkles

Glycolic Acid 10% + Argireline® Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(B) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Selective Care Argireline® Complex	~3ml of Anti-Aging Care Wide Range Hyaluronic Acid
Pemarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		

Treatment Guideline No. 19

Post-Acne

Loss of Tone & Wrinkles

Glycolic Acid 16% + Argireline® Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP INFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Selective Care Argireline® Complex	~3ml of Anti-Aging Care Wide Range Hyaluronic Acid
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°		

Treatment Guideline No. 20

Indications:

Loss of Tone & Wrinkles

Mandelic & Salicylic + Argireline® Complex

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION O2	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(B) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Selective Care Argireline® Complex	~3ml of Anti-Aging Care Wide Range Hyaluronic Acid
! Remarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45 - 60°		

28



Indications: Couperose & Sensitive Skin

Glycolic Acid 5% + Soothing

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Selective Care Soothing	~3ml of JetCare Boost: Regenerate / Glow / Hydrate / Tight
(!) Remarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 22

30

Indications:

Couperose & Sensitive Skin

Glycolic Acid 10% + Soothing

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Selective Care Soothing	~3ml of JetCare Boost: Regenerate / Glow / Tight
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45*- 60*		As per Treatments by Skin Characteristics

Treatment Guideline No. 23

Indications: Couperose & Sensitive Skin

Glycolic Acid 16% + Soothing

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP INFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Selective Care Soothing	~3ml of JetCare Boost: Tight
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°		

Treatment Guideline No. 24

Indications: Couperose & Sensitive Skin

31

Mandelic & Salicylic + Soothing

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 1NFUSION O4
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Selective Care Soothing	~3ml of JetCare Boost: Regenerate / Glow
! Remarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics



Indications: Acne

Glycolic Acid 5% + Clear

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 04	
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow	
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm	
Handpiece Angle	90°	90°	90°	90°	
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes	
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Selective Care Clear	~3ml of JetCare Boost: Glow / Hydrate	
! Remarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare. Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics	

Treatment Guideline No. 26

32

Indications:

Glycolic Acid 10% + Clear

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Selective Care Clear	~3ml of JetCare Boost: Glow / Hydrate
Premarks	Leave on skin for 5-8 min.		As per Treatments by Skin Characteristics	

Treatment Guideline No. 27

Glycolic Acid 16% + Clear

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Selective Care Clear	~3ml of JetCare Boost: Glow / Hydrate
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 28

Mandelic & Salicylic + Clear

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Selective Care Clear	~3ml of JetCare Boost: Regenerate / Hydrate
Premarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics



Loss of Tone & Wrinkles Post-Acne

Couperose & Sensitive Skin Lightening & Pigmentation

Glycolic Acid 5% + Growth Factor

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%	~5ml of Selective Care Growth Factor	~3ml of JetCare Boost: Regenerate / Glow / Hydrate / Tight
! Remarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 30

Indications:

Post-Acne

Glycolic Acid 10% + Growth Factor

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(A) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%	~5ml of Selective Care Growth Factor	~3ml of JetCare Boost: Regenerate / Tight
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 31

Indications: Acne Lightening & Pigmentation

Glycoloic Acid 16% + Growth Factor

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP 1NFUSION 04
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm
Handpiece Angle	90°	90°	90°	90°
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes
(B) Use	~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%	~5ml of Selective Care Growth Factor	~3ml of JetCare Boost: Hydrate / Tight
Premarks		Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°-60°		As per Treatments by Skin Characteristics

Treatment Guideline No. 32

Mandelic & Salicylic + Growth Factor

	LYMPHATIC STEP MASSAGE 01	EXFOLIATION STEP 02	INFUSION 03	BOOSTER STEP NFUSION 04	
Handpiece	TripleJet	TripleJet	TripleJet or SingleJet Narrow	TripleJet or SingleJet Narrow	
Distance from skin	4-7 cm	1 cm	0.5-0.8 cm	0.5-0.8 cm	
Handpiece Angle	90°	90°	90°	90°	
① Time	10-15 minutes	5-7 minutes	5-7 minutes	5-7 minutes	
☐ Use	~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic	~5ml of Selective Care Growth Factor	~3ml of JetCare Boost:Tight	
Premarks		Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45'- 60°			

TAVTECH LTD.

JETCARE by TayTech



SCALP & HAIR

Recommended Treatment Cycle



10 Treatments

once every week for 10 weeks



After 10 treatments:

Maintenance treatments every 3-4 weeks

Scalp & Hair Characteristics

Itchy, Sensitive Scalp

The main reason for itchiness of scalp is inflammation and sensitivity. This may be due to environmental causes, such as pollution or sun exposure. Itchy scalp can also be caused by emotional issues such as stress and anxiety. In some cases, itchiness is caused by buildup of dry skin or from the use of harsh products that irritate the delicate skin of the scalp.

Dry Scalp

Dry scalp is sometimes genetically caused, and those who suffer from dry scalp are also often prone to other skin conditions, such as eczema. In other cases, dry skin of the scalp is caused by external or emotional factors, such as indoor heating or stress. The scalp has numerous oil glands, however, those with a tendency towards dry skin often suffer from dry scalp. Dry scalp may create a sensation of tightness and may also be accompanied by unwanted flaking.

Oily Scalp

Skin and scalp have a Hydrolipidic film consisting of sebum, sweat and moisture that forms a natural barrier to protect the skin from external elements. In those prone to oily skin, the Hydrolipidic film may over-perform, causing oiliness of the scalp. Oily skin can result from stress, hormonal changes, exposure to pollution or harsh products. An oily scalp can cause the hair roots to become greasy very quickly, which may cause acne spots.

Flaky Scalp

Flaky scalp is highly visible, as it produces small dry flakes that sit on the scalp or fall onto the clothing. The causes of flaky scalp are varied. It may occur in dry skin, oily and irritated skin, or sensitive skin. Flakiness may also be a result of contact dermatitis, which is caused by sensitivity to products used on the hair and scalp.



Scalp and Hair Treatment Guidelines

SCALP TYPE	1st Treatment	2nd Treatment	3rd Treatment	4th Treatment	5th Treatment
Itchy, Sensitive	Treatment Guideline 33 with JetCare Hydro with Boost Regenerate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 35 with JetCare Hydro with Boost Hydrate
Dry	Treatment Guideline 33	Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 37
	with JetCare Hydro	with JetCare Hydro	with JetCare Hydro	with JetCare Hydro	with Glycolic Acid 5%
	with Boost Regenerate	with Scalp & Hair	with Scalp & Hair	with Scalp & Hair	with Boost Hydrate
Oily	Treatment Guideline 37	Treatment Guideline 39	Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 40
	with Glycolic Acid 5%	with Glycolic Acid 5%	with JetCare Hydro	with JetCare Hydro	with Glycolic Acid 10%
	with Boost Hydrate	with Scalp & Hair	with Scalp & Hair	with Scalp & Hair	with Boost Hydrate
Flaky	Treatment Guideline 37	Treatment Guideline 33	Treatment Guideline 39	Treatment Guideline 34	Treatment Guideline 40
	with Glycolic Acid 5%	with JetCare Hydro	with Glycolic Acid 5%	with JetCare Hydro	with Glycolic Acid 10%
	with Boost Hydrate	with Boost Regenerate	with Scalp & Hair	with Scalp & Hair	with Boost Hydrate

Treatment Guideline No. 33

	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
(L) Time	10 minutes	5-7 minutes
(B) Use	~12ml of JetCare Hydro	~6-7ml of Boost Regenerate

6th Treatment	7th Treatment	8th Treatment	9th Treatment	10th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 36 with JetCare Hydro with Boost Tight	Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 34
with JetCare Hydro	with JetCare Hydro		with JetCare Hydro	with JetCare Hydro	with JetCare Hydro
with Scalp & Hair	with Scalp & Hair		with Scalp & Hair	with Scalp & Hair	with Scalp and Hair
Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 38	Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 34
with JetCare Hydro	with JetCare Hydro	with Glycolic Acid 5%	with JetCare Hydro	with JetCare Hydro	with JetCare Hydro
with Scalp & Hair	with Scalp & Hair	with Boost Hydrate	with Scalp & Hair	with Scalp & Hair	with Scalp and Hair
Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 37	Treatment Guideline 39	Treatment Guideline 34	Treatment Guideline 39
with JetCare Hydro	with JetCare Hydro	with Glycolic Acid 5%	with Glycolic Acid 5%	with JetCare Hydro	with Glycolic Acid 5%
with Scalp & Hair	with Scalp & Hair	with Boost Hydrate	with Scalp & Hair	with Scalp & Hair	with Scalp and Hair
Treatment Guideline 34	Treatment Guideline 34	Treatment Guideline 41	Treatment Guideline 39	Treatment Guideline 34	Treatment Guideline 39
with JetCare Hydro	with JetCare Hydro	with Glycolic Acid 5%	with Glycolic Acid 5%	with JetCare Hydro	with Glycolic Acid 5%
with Scalp & Hair	with Scalp & Hair	with Scalp & Hair	with Scalp & Hair	with Scalp & Hair	with Scalp and Hair

Treatment Guideline No. 34

	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
☐ Use	~12ml of JetCare Hydro	~6-7ml of of Selective Care Scalp & Hair



	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
(B) Use	~12ml of JetCare Hydro	~6-7ml of Boost Hydrate

Treatment Guideline No. 36

	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
Use	~12ml of JetCare Hydro	~6-7ml of Boost Tight

Treatment Guideline No. 37

	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
Use	~6ml of Glycolic Acid 5%	~6-7ml of Boost Hydrate
Premarks	After a few minutes wash with ~10ml of JetCare Hydro	

Treatment Guideline No. 38

	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
Use	~6ml of Glycolic Acid 5%	~6-7ml of Boost Tight
! Remarks	After a few minutes wash with ~10ml of JetCare Hydro	



	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
☐ Use	~6ml of Glycolic Acid 5%	~6-7ml of of Selective Care Scalp & Hair
Premarks	After a few minutes wash with ~10ml of JetCare Hydro	

Treatment Guideline No. 40

	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
Use	~6ml of Glycolic Acid 10%	~6-7ml of Boost Hydrate
Premarks	After a few minutes wash with ~10ml of JetCare Hydro	

Treatment Guideline No. 41

	MASSAGE AND EXFOLIATION	INFUSION
Handpiece	TripleJet	DoubleJet
Distance from skin	10 mm	5-7 mm
(Handpiece Angle	45-90°	90°
① Time	10 minutes	5-7 minutes
Use	~6ml of Glycolic Acid 5%,	~6-7ml of Boost Regenerate
Pemarks	After a few minutes wash with ~10ml of JetCare Hydro	



T: +972.3.536.0515 E: info@jetpeel.com

40 Haatzmaut St. Yehud, 5630417 Israel

FOLLOW US:







www.jetpeel.com

