

TREATMENT GUIDELINES





Feels More Beautiful Than Ever

Welcome to JetCare Treatment Guidelines.

JetPeel's entire line of cosmetic solutions has been renewed, with each formula improved and upgraded with the latest cutting-edge materials and ingredients, all from the highest quality cosmetic suppliers, and all especially suited to JetPeel's unique and patented handpieces, and to its enduring jet pressure-energy device.

Concentrations have been improved to optimally treat a wide range of skin types, body areas and customer concerns.

JetPeel brings this new and vastly improved line of JetCare solutions to its customers after many months of R&D and testing to ensure that it is the most compatible line of solutions to the JetPeel handpieces and devices, and after ensuring enhanced performance.

Developed exclusively by JetPeel in conjunction with a leading, renowned cosmetics formulation laboratory, JetCare products deliver powerfully effective JetPeel treatments in the clinic, spa and cosmetic salon, and are the only solutions sanctioned and approved by JetPeel for use with its handpieces and devices.

Together with these advances, JetPeel by TavTech has suited Treatment Guidelines to its exclusive line of JetCare solutions for the best treatment outcomes and customer satisfaction.

JetCare by TavTech is the first choice for cosmeticians and specialists worldwide, and the only choice for the ultimate treatment experience by JetPeel.

With JetCare formulas and Treatment Guidelines, the skin feels smoother, brighter and more beautiful than ever before.

CONTENTS

JetPeel Treatment Sequence.....4

Skin.....6-35

 Skin Characteristics.....6

 ● Anti-Aging Treatments by Skin Characteristics.....8

 ● Loss of Tone and Wrinkle Treatments by Skin Characteristics.....10

 ● Acne Treatments by Skin Characteristics.....12

 ● Post-Acne Treatments by Skin Characteristics.....14

 ● Couperose and Sensitive Skin Treatments by Skin Characteristics16

 ● Lightening and Pigmentation Treatments by Skin Characteristics.....18

 Skin Treatment Guidelines.....20

Scalp & Hair.....36-43

 Scalp & Hair Characteristics.....37

 ● Treatment Guidelines.....38

The Treatment Sequence

STEP 01


LYMPHATIC MASSAGE

Relaxing massage for detoxifying the skin and stimulating micro-circulation.

STEP 02

EXFOLIATION

Preps the skin to receive the nourishment from the following infusion treatment.

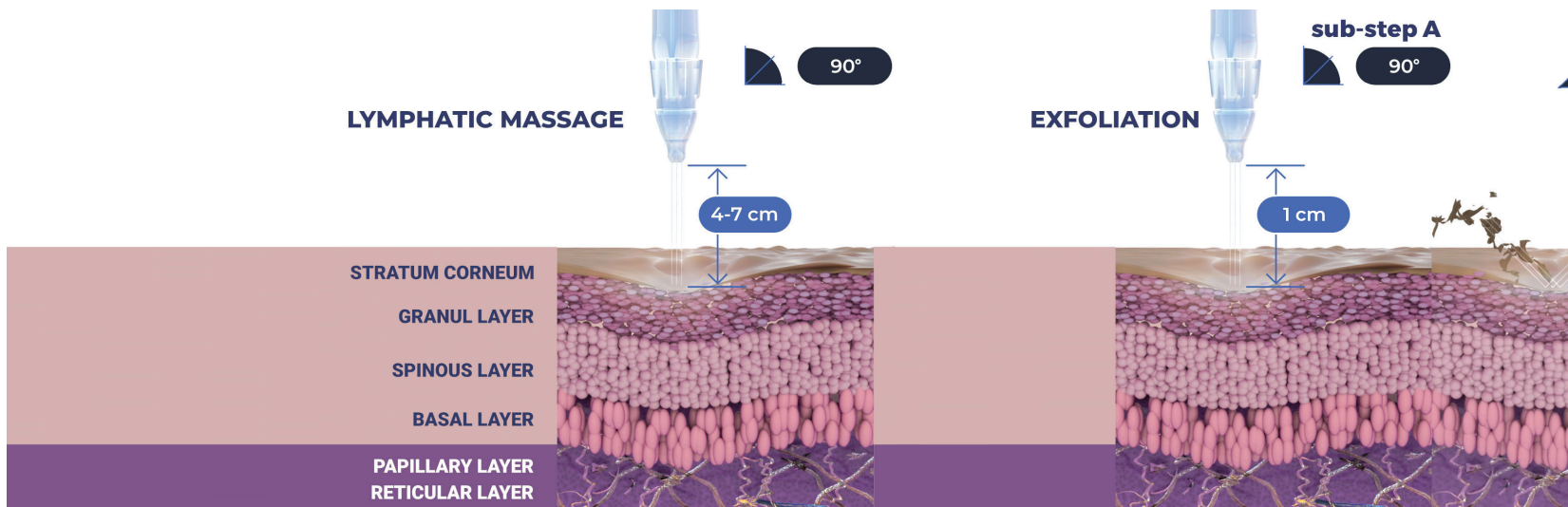

STEP 03

INFUSION

Deep, effective delivery of active ingredients into the skin.

STEP 04

BOOSTER INFUSION

Post-infusion step to amplify the treatment result.

sub-step B



8-10 mm

INFUSION



5-8 mm

EPIDERMIS
30 - 200µm

DERMIS
1000 - 4000µm



SKIN

Recommended Treatment Cycle



First 6 treatments:

1-2 times every week for 3-6 weeks



After 6 treatments:

Maintenance treatments every 3-4 weeks

Skin Characteristics

Maturity in the skin occurs when the skin's ability to retain its youthful glow is detrimentally affected by the natural process of aging and by adverse environmental influences such as pollution and sun exposure. This leads to a decrease in the production of collagen, elastin and sebum, and increased risk of dehydration, which may affect skin hyperpigmentation, dark spots and appearance.

Normal Skin Type

Normal skin, which is neither too dry nor too oily, has:

- No or few imperfections
- No severe sensitivity
- Barely visible pores
- A radiant complexion

Oily Skin Type

- Enlarged pores
- Dull or shiny, thick complexion
- Blackheads, pimples, or other blemishes

Dry Skin Type

- Almost invisible pores
- Dull, rough complexion
- Red patches
- Less elasticity
- More visible lines

Sensitive Skin Type

- Redness
- Itching
- Burning
- Dryness

Combination Skin Type

Skin can be dry or normal in some areas and oily in others, such as the T-zone (nose, forehead, and chin). Many people have this skin type. It may need different care in different areas

Combination skin can have:

- Pores that look larger than normal since they are more open
- Blackheads
- Shiny skin

Anti-Aging Treatments by Skin Characteristics

ANTI-AGING - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 4 with JetCare Boost Tight
Dry	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Sensitive	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

ANTI-AGING – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Dry	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 2 with JetCare Boost Glow	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Sensitive	Treatment Guideline 4 with JetCare Boost Glow	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 6 with JetCare Boost Glow	Treatment Guideline 10 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 10 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 6 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 6 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Treatment Guideline 10 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 7 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 8 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight



Loss of Tone and Wrinkles Treatments by Skin Characteristics

LOSS OF TONE & WRINKLES - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 4 with JetCare Boost Regenerate
Dry	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 1 with JetCare Boost Regenerate
Oily	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 2 with JetCare Boost Regenerate
Sensitive	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 1 with JetCare Boost Regenerate

LOSS OF TONE & WRINKLES – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 2 with JetCare Boost Tight	Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 6 with JetCare Boost Regenerate
Dry	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 5 with JetCare Boost Regenerate
Oily	Treatment Guideline 2 with JetCare Boost Tight	Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 7 with JetCare Boost Regenerate
Sensitive	Treatment Guideline 4 with JetCare Boost Tight	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 5 with JetCare Boost Regenerate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 5 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 6 with JetCare Boost Glow	Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 20 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 10 with JetCare Boost Hydrate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 19 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 11 with JetCare Boost Hydrate	Treatment Guideline 19 Boost with Wide Range HA
Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 17 Boost with Wide Range HA

Acne Treatments by Skin Characteristics

ACNE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 4 with JetCare Boost Tight
Dry	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 4 with JetCare Boost Tight
Oily	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 1 with JetCare Boost Tight
Sensitive	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 4 with JetCare Boost Tight

ACNE – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 26 with JetCare Boost Glow	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 27 with JetCare Boost Glow	Treatment Guideline 2 with JetCare Boost Tight
Sensitive	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 25 with JetCare Boost Glow	Treatment Guideline 1 with JetCare Boost Tight

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 28 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 32 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 32 with JetCare Boost Tight	Treatment Guideline 28 with JetCare Boost Hydrate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 30 with JetCare Boost Tight	Treatment Guideline 26 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 31 with JetCare Boost Tight	Treatment Guideline 27 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 28 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Tight	Treatment Guideline 25 with JetCare Boost Hydrate

Post-Acne Treatments by Skin Characteristics

POST-ACNE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Dry	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Oily	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Sensitive	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA

POST-ACNE – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Dry	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA
Oily	Treatment Guideline 2 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 19 Boost with Wide Range HA
Sensitive	Treatment Guideline 1 with JetCare Boost Tight	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 18 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 29 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 12 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 18 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 9 with JetCare Boost Hydrate	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 19 Boost with Wide Range HA
Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 17 Boost with Wide Range HA	Treatment Guideline 30 with JetCare Boost Regenerate	Treatment Guideline 17 Boost with Wide Range HA



Couperose and Sensitive Skin Treatments by Skin Characteristics

COUPEROSE AND SENSITIVE - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Dry	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Oily	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Sensitive	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 4 with JetCare Boost Glow

COUPEROSE & SENSITIVE – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 21 with JetCare Boost Regenerate	Treatment Guideline 22 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Dry	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow
Oily	Treatment Guideline 22 with JetCare Boost Regenerate	Treatment Guideline 23 with JetCare Boost Tight	Treatment Guideline 2 with JetCare Boost Glow
Sensitive	Treatment Guideline 24 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Tight	Treatment Guideline 1 with JetCare Boost Glow

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Regenerate	Treatment Guideline 8 with JetCare Boost Hydrate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 21 with JetCare Boost Hydrate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 24 with JetCare Boost Glow	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 11 with JetCare Boost Regenerate	Treatment Guideline 21 with JetCare Boost Hydrate
Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 24 with JetCare Boost Glow	Treatment Guideline 21 with JetCare Boost Hydrate



Lightening and Pigmentation Treatments by Skin Characteristics

LIGHTENING AND PIGMENTATION - YOUNG SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Dry	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 4 with JetCare Boost Tight
Oily	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Sensitive	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 4 with JetCare Boost Tight

LIGHTENING & PIGMENTATION – MATURE SKIN

SKIN TYPE	1st Treatment	2nd Treatment	3rd Treatment
Normal	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 3 with JetCare Boost Tight
Dry	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight
Oily	Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 15 with JetCare Boost Regenerate	Treatment Guideline 2 with JetCare Boost Tight
Sensitive	Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 1 with JetCare Boost Tight







4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 9 with JetCare Boost Regenerate	Treatment Guideline 13 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 10 with JetCare Boost Regenerate	Treatment Guideline 14 with JetCare Boost Hydrate
Treatment Guideline 16 with JetCare Boost Hydrate	Treatment Guideline 8 with JetCare Boost Glow	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate

4th Treatment	5th Treatment	6th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 10 with JetCare Boost Regenerate	Treatment Guideline 14 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Hydrate
Treatment Guideline 14 with JetCare Boost Regenerate	Treatment Guideline 31 with JetCare Boost Hydrate	Treatment Guideline 14 with JetCare Boost Hydrate	Treatment Guideline 15 with JetCare Boost Hydrate
Treatment Guideline 13 with JetCare Boost Regenerate	Treatment Guideline 29 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate	Treatment Guideline 13 with JetCare Boost Hydrate

Treatment Guideline No. 1

Indications: Anti-Aging
Loss of Tone & Wrinkles Acne
Post-Acne Couperose & Sensitive Skin
Lightening & Pigmentation







Glycolic Acid 5% + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight/ Regenerate/Glow	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 2

Indications: Anti-Aging
Loss of Tone & Wrinkles Acne
Post-Acne Couperose & Sensitive Skin
Lightening & Pigmentation







Glycolic Acid 10% + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight/ Regenerate/Glow	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 3

Indications: Lightening & Pigmentation







Glycolic Acid 16% + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°					

Treatment Guideline No. 4

Indications: Anti-Aging
Loss of Tone & Wrinkles
Couperose & Sensitive Skin
Lightening & Pigmentation
Acne







Mandelic & Salicylic + Wide Range Hyaluronic Acid

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic		~5ml of Anti-Aging Care Wide Range Hyaluronic Acid		~3ml of JetCare Boost: Tight/ Regenerate/Glow	
 Remarks			Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 5

Indications: Loss of Tone & Wrinkles







Glycolic Acid 5% + Peptide Complex

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 5%		~5ml of Anti-Aging Care Peptide Complex		~3ml of JetCare Boost: Regenerate/ Glow	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 6

Indications: Anti-Aging
Loss of Tone & Wrinkles







Glycolic Acid 10% + Peptide Complex

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		~5ml of Anti-Aging Care Peptide Complex		~3ml of JetCare Boost: Regenerate/ Glow	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 7

Indications: Anti-Aging







Glycolic Acid 16% + Peptide Complex

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		~5ml of Anti-Aging Care Peptide Complex		~3ml of JetCare Boost: Regenerate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°					

Treatment Guideline No. 8

Indications: Anti-Aging
Loss of Tone & Wrinkles
Couperose & Sensitive Skin
Lightening & Pigmentation
Post-Acne

Mandelic & Salicylic + Peptide Complex







	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Mandelic & Salicylic		~5ml of Anti-Aging Care Peptide Complex		~3ml of JetCare Boost: Regenerate/ Glow/ Hydrate	
 Remarks			Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 9

Indications: Anti-Aging
Lightening & Pigmentation

Post-Acne
Couperose & Sensitive Skin







Glycolic Acid 5% + Witch Hazel & Peptides

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%		~5ml of Anti-Aging Care Witch Hazel & Peptides		~3ml of JetCare Boost: Regenerate/ Glow/Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 10

Indications: Lightening & Pigmentation Loss of Tone & Wrinkles







Glycolic Acid 10% + Witch Hazel & Peptides

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%		~5ml of Anti-Aging Care Witch Hazel & Peptides		~3ml of JetCare Boost: Regenerate/ Glow/Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 11

Indications: Loss of Tone & Wrinkles Couperose & Sensitive Skin







Glycolic Acid 16% + Witch Hazel & Peptides

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%		~5ml of Anti-Aging Care Witch Hazel & Peptides		~3ml of JetCare Boost: Glow / Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 12

Indications: Loss of Tone & Wrinkles Post-Acne

Mandelic & Salicylic + Witch Hazel & Peptides







	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic		~5ml of Anti-Aging Care Witch Hazel & Peptides		~3ml of JetCare Boost: Hydrate	
 Remarks			Wash with ~8ml of JetCare Hydro if necessary. TripleJet handpiece at 45°- 60°					

Treatment Guideline No. 13

Indications: Anti-Aging
Post-Acne

Couperose & Sensitive Skin
Lightening & Pigmentation

Glycolic Acid 5% + Lightening

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%		~5ml of Selective Care Lightening		~3ml of JetCare Boost: Regenerate / Glow / Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	







Treatment Guideline No. 14

Indications: Anti-Aging

Acne
Post-Acne

Couperose & Sensitive Skin
Lightening & Pigmentation







Glycolic Acid 10% + Lightening

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%		~5ml of Selective Care Lightening		~3ml of JetCare Boost: Regenerate / Glow / Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 15

Indications: Anti-Aging
Lightening & Pigmentation







Glycolic Acid 16% + Lightening

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		~5ml of Selective Care Lightening		~3ml of JetCare Boost: Regenerate / Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 16

Indications: Anti-Aging
Acne
Couperose & Sensitive Skin
Lightening & Pigmentation







Mandelic & Salicylic + Lightening

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Mandelic & Salicylic		~5ml of Selective Care Lightening		~3ml of JetCare Boost: Regenerate / Glow / Hydrate	
 Remarks			Wash with ~8ml of JetCare Hydro if necessary. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 17

Indications: Anti-Aging Post-Acne Loss of Tone & Wrinkles







Glycolic Acid 5% + Argireline® Complex

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 5%		~5ml of Selective Care Argireline® Complex		~3ml of Anti-Aging Care Wide Range Hyaluronic Acid	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°					

Treatment Guideline No. 18

Indications: Anti-Aging Post-Acne Loss of Tone & Wrinkles

Glycolic Acid 10% + Argireline® Complex

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		~5ml of Selective Care Argireline® Complex		~3ml of Anti-Aging Care Wide Range Hyaluronic Acid	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°					





*Argireline® is a trademark of Lipotec S.A. or its affiliates

Treatment Guideline No. 19

Indications: Post-Acne

Loss of Tone & Wrinkles







Glycolic Acid 16% + Argireline® Complex

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		~5ml of Selective Care Argireline® Complex		~3ml of Anti-Aging Care Wide Range Hyaluronic Acid	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°					

Treatment Guideline No. 20

Indications: Loss of Tone & Wrinkles







Mandelic & Salicylic + Argireline® Complex

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Mandelic & Salicylic		~5ml of Selective Care Argireline® Complex		~3ml of Anti-Aging Care Wide Range Hyaluronic Acid	
 Remarks			Wash with ~8ml of JetCare Hydro if necessary. TripleJet handpiece at 45°- 60°					

Treatment Guideline No. 21

Indications: Couperose & Sensitive Skin







Glycolic Acid 5% + Soothing

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 5%		~5ml of Selective Care Soothing		~3ml of JetCare Boost: Regenerate / Glow / Hydrate / Tight	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 22

Indications: Couperose & Sensitive Skin







Glycolic Acid 10% + Soothing

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		~5ml of Selective Care Soothing		~3ml of JetCare Boost: Regenerate / Glow / Tight	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 23

Indications: Couperose & Sensitive Skin







Glycolic Acid 16% + Soothing

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		~5ml of Selective Care Soothing		~3ml of JetCare Boost: Tight	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°					

Treatment Guideline No. 24

Indications: Couperose & Sensitive Skin






Mandelic & Salicylic + Soothing

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Mandelic & Salicylic		~5ml of Selective Care Soothing		~3ml of JetCare Boost: Regenerate / Glow	
 Remarks			Wash with ~8ml of JetCare Hydro if necessary. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 25

Indications: Acne







Glycolic Acid 5% + Clear

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 5%		~5ml of Selective Care Clear		~3ml of JetCare Boost: Glow / Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 26

Indications: Acne







Glycolic Acid 10% + Clear

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		~5ml of Selective Care Clear		~3ml of JetCare Boost: Glow / Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 27

Indications: Acne







Glycolic Acid 16% + Clear

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		~5ml of Selective Care Clear		~3ml of JetCare Boost: Glow / Hydrate	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 28

Indications: Acne







Mandelic & Salicylic + Clear

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
 Use	~18ml of JetCare Hydro		~5ml of Renewal Care Mandelic & Salicylic		~5ml of Selective Care Clear		~3ml of JetCare Boost: Regenerate / Hydrate	
 Remarks			Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 29

Indications: Anti-Aging
Loss of Tone & Wrinkles Acne
Post-Acne Couperose & Sensitive Skin
Lightening & Pigmentation







Glycolic Acid 5% + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 5%		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost: Regenerate / Glow / Hydrate / Tight	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 30

Indications: Acne Post-Acne







Glycolic Acid 10% + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 10%		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost: Regenerate / Tight	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 31

Indications: Acne Lightening & Pigmentation







Glycolic Acid 16% + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Glycolic Acid 16%		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost: Hydrate / Tight	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45°- 60°				As per Treatments by Skin Characteristics	

Treatment Guideline No. 32

Indications: Acne

Mandelic & Salicylic + Growth Factor

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOSTER INFUSION	STEP 04
 Handpiece		TripleJet	TripleJet		TripleJet or SingleJet Narrow		TripleJet or SingleJet Narrow	
 Distance from skin		4-7 cm	1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle		90°	90°		90°		90°	
 Time		10-15 minutes	5-7 minutes		5-7 minutes		5-7 minutes	
 Use		~18ml of JetCare Hydro	~5ml of Renewal Care Mandelic & Salicylic		~5ml of Selective Care Growth Factor		~3ml of JetCare Boost:Tight	
 Remarks			Wash with ~8ml of JetCare Hydro if neccesary. TripleJet handpiece at 45°- 60°					



SCALP & HAIR

Recommended Treatment Cycle



10 Treatments

once every week for 10 weeks



After 10 treatments:

Maintenance treatments every 3-4 weeks

Scalp & Hair Characteristics

Itchy, Sensitive Scalp

The main reason for itchiness of scalp is inflammation and sensitivity. This may be due to environmental causes, such as pollution or sun exposure. Itchy scalp can also be caused by emotional issues such as stress and anxiety. In some cases, itchiness is caused by buildup of dry skin or from the use of harsh products that irritate the delicate skin of the scalp.

Dry Scalp

Dry scalp is sometimes genetically caused, and those who suffer from dry scalp are also often prone to other skin conditions, such as eczema. In other cases, dry skin of the scalp is caused by external or emotional factors, such as indoor heating or stress. The scalp has numerous oil glands, however, those with a tendency towards dry skin often suffer from dry scalp. Dry scalp may create a sensation of tightness and may also be accompanied by unwanted flaking.


Oily Scalp

Skin and scalp have a Hydrolipidic film consisting of sebum, sweat and moisture that forms a natural barrier to protect the skin from external elements. In those prone to oily skin, the Hydrolipidic film may over-perform, causing oiliness of the scalp. Oily skin can result from stress, hormonal changes, exposure to pollution or harsh products. An oily scalp can cause the hair roots to become greasy very quickly, which may cause acne spots.






Flaky Scalp

Flaky scalp is highly visible, as it produces small dry flakes that sit on the scalp or fall onto the clothing. The causes of flaky scalp are varied. It may occur in dry skin, oily and irritated skin, or sensitive skin. Flakiness may also be a result of contact dermatitis, which is caused by sensitivity to products used on the hair and scalp.

Scalp and Hair Treatment Guidelines

SCALP TYPE	1st Treatment	2nd Treatment	3rd Treatment	4th Treatment	5th Treatment
 Itchy, Sensitive	Treatment Guideline 33 with JetCare Hydro with Boost Regenerate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 35 with JetCare Hydro with Boost Hydrate
Dry	Treatment Guideline 33 with JetCare Hydro with Boost Regenerate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate
Oily	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 40 with Glycolic Acid 10% with Boost Hydrate
Flaky	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 33 with JetCare Hydro with Boost Regenerate	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 40 with Glycolic Acid 10% with Boost Hydrate

Treatment Guideline No. 33






		MASSAGE AND EXFOLIATION	INFUSION
	Handpiece	TripleJet	DoubleJet
	Distance from skin	10 mm	5-7 mm
	Handpiece Angle	45-90°	90°
	Time	10 minutes	5-7 minutes
	Use	~12ml of JetCare Hydro	~6-7ml of Boost Regenerate

6th Treatment	7th Treatment	8th Treatment	9th Treatment	10th Treatment	Maintenance Treatments (every 3-4 weeks)
Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 36 with JetCare Hydro with Boost Tight	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp and Hair
Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 38 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp and Hair
Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 37 with Glycolic Acid 5% with Boost Hydrate	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 39 with Glycolic Acid 5% with Scalp and Hair
Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 41 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 39 with Glycolic Acid 5% with Scalp & Hair	Treatment Guideline 34 with JetCare Hydro with Scalp & Hair	Treatment Guideline 39 with Glycolic Acid 5% with Scalp and Hair






Treatment Guideline No. 34

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~12ml of JetCare Hydro	~6-7ml of of Selective Care Scalp & Hair







Treatment Guideline No. 35

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~12ml of JetCare Hydro	~6-7ml of Boost Hydrate







Treatment Guideline No. 36

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~12ml of JetCare Hydro	~6-7ml of Boost Tight







Treatment Guideline No. 37

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~6ml of Glycolic Acid 5%	~6-7ml of Boost Hydrate
 Remarks	After a few minutes wash with ~10ml of JetCare Hydro	







Treatment Guideline No. 38

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~6ml of Glycolic Acid 5%	~6-7ml of Boost Tight
 Remarks	After a few minutes wash with ~10ml of JetCare Hydro	







Treatment Guideline No. 39

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~6ml of Glycolic Acid 5%	~6-7ml of of Selective Care Scalp & Hair
 Remarks	After a few minutes wash with ~10ml of JetCare Hydro	

Treatment Guideline No. 40

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~6ml of Glycolic Acid 10%	~6-7ml of Boost Hydrate
 Remarks	After a few minutes wash with ~10ml of JetCare Hydro	

Treatment Guideline No. 41

	MASSAGE AND EXFOLIATION	INFUSION
 Handpiece	TripleJet	DoubleJet
 Distance from skin	10 mm	5-7 mm
 Handpiece Angle	45-90°	90°
 Time	10 minutes	5-7 minutes
 Use	~6ml of Glycolic Acid 5%,	~6-7ml of Boost Regenerate
 Remarks	After a few minutes wash with ~10ml of JetCare Hydro	

TAVTECH LTD

T: +972.3.536.0515

E: info@jetpeel.com

40 Haatzmaut St.

Yehud, 5630417

Israel

FOLLOW US:



www.jetpeel.com

