


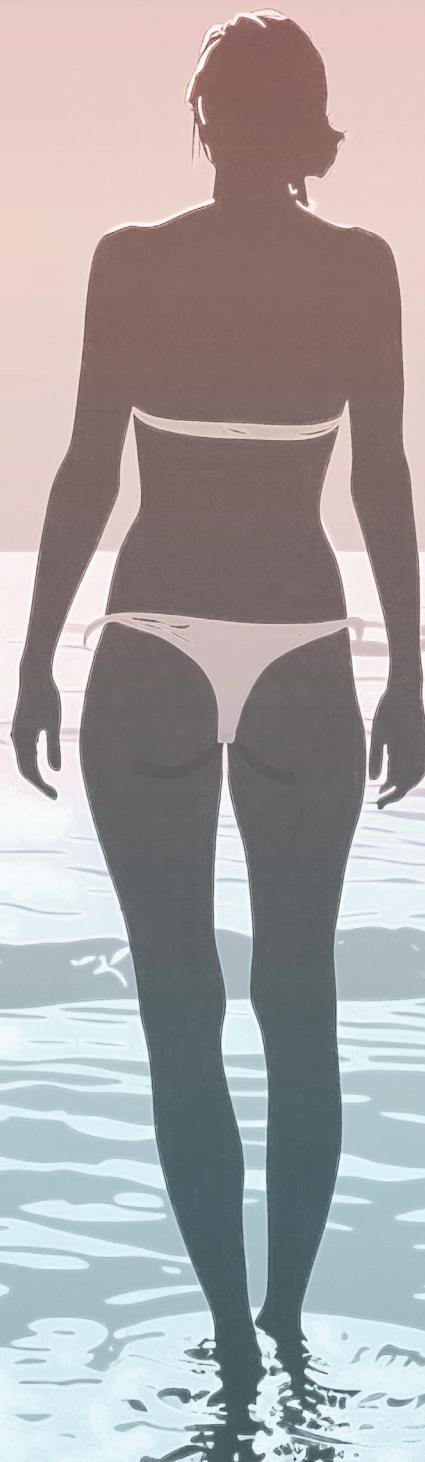

Beautiful Results In Exactly the Right Place

"Prejuvenation Buttocks" by JetPeel is the ultimate aesthetic skin protection and enhancement treatment for the perfect summer reveal.



JetCare Youth

Smooth, Youthful, Amazing in Bikinis

During beach season, every young woman wants to look her absolute best, and the focus is firmly on the buttocks. In fact, buttocks is the most popular and high demand area for aesthetic treatments among sun-loving bikini wearers.

Preserve What They Have

Prejuvenation is a fast-growing trend among young women who want to preserve and maintain the youthful tone and glow of their skin. Prejuvenation Buttocks by JetPeel is a treatment protocol aimed especially at younger women who want to show off their best buttocks in the summer.

Prejuvenation Buttocks by JetPeel:

The Summer Solution

Sand, sea and sun are highly stressful for the skin. Exposure can lead to loss of tone and accelerate the aging process. With the JetPeel Buttocks treatment, customers enjoy a non-invasive, pleasant procedure based on jet pressure technology that gets results. Prejuvenation Buttocks is especially developed by TavTech to prevent the impact of skin stressors, activating collagen stimulation and antioxidants, and helping to maintain the appearance of smooth, youthful buttocks.



JetPeel by TavTech's Beautiful Buttocks

3 Step Treatment Guideline

Prejuvenation Buttocks includes three pleasant and comfortable treatment steps with the JetCare Youth by TavTech line of liquid solutions which are enriched with a blend of 21 Dead Sea minerals, and especially formulated for preserving and protecting youthful skin, and for keeping it vibrant.

The total treatment session time is 30 minutes.

Prejuvenation Buttocks by JetPeel:

Treatment Schedule

Just 6 sessions cover the entire bikini season. After one treatment, expect skin refinement and a more even and radiant tone.






*For best results, monthly treatments are highly recommended.

Sessions 1 & 2	Week 1, 2nd Session 3-4 days after Session 1
Sessions 3	8-12 days after Session 2
Sessions 4	18-22 days after Session 3
Sessions 5	28-32 days after Session 4
Sessions 6	28-32 days after Session 5



JETPEEL 3D MESSAGE with Dead Sea Hydro

Pressurized air infused with JetCare Youth by TavTech Dead Sea Hydro stimulates micro-circulation and improves skin tone.






	Handpiece	JetPeel by TavTech TripleJet
	Distance from skin	4 cm
	Handpiece Angle	90°
	Time	8 minutes
	Use	Approx. 10ml JetCare Youth by TavTech Dead Sea Hydro

STEP 02



EXFOLIATION with Dead Sea Exfoliation

Gentle exfoliation of the delicate buttocks area reactivates new skin cells and helps preserve the appearance of youth.






	Sub-Step A	Sub-Step B
 Handpiece	JetPeel by TavTech TripleJet	JetPeel by TavTech TripleJet
 Distance from skin	1 cm	1 cm
 Handpiece Angle	90°	45°
 Time	Leave for 8 minutes	6 minutes
 Use	Approx. 6 to 8 ml of JetCare Youth by TavTech Dead Sea Exfoliation,	Wash off with approx. 8 ml of JetCare Youth by TavTech Dead Sea Hydro

STEP 03



INFUSION with Dead Sea Nourishment

Nourishing infusion delivered to the dermis to boost skin hydration and regeneration, slowing the effects of sun and sand and helping to preserve the buttocks' natural youth.

 Handpiece	JetPeel by TavTech DoubleJet,
 Distance from skin	5-8 mm
 Handpiece Angle	90°
 Time	8 minutes
 Use	Approx. 6 to 8 ml of JetCare Youth by TavTech Dead Sea Nourishment

*Use the JetPro Duo or JetPro ToGo devices for a simultaneous hydroporation and red LED photoporation boost that amplifies the JetPeel Buttocks results.