



JetCare Ayurveda for sensitive skin

TAVTECH LTD
T: +972.3.536.0515
E: info@jetpeel.com

40 Haatzmaut St.
Yehud, 5630417
Israel

FOLLOW US:



www.jetpeel.com



**Ancient Healing
Modern Science
Beautiful Glow**

The Science of Life, The JetPeel Results

JetCare by TavTech liquid solutions provide powerful and effective treatment for a wide range of skin care concerns. Used in conjunction with JetPeel's exclusive, world-renowned, non-invasive, non-ablative and non-contact jet technology, JetCare delivers smooth, healthy and glowing skin for customers worldwide.

JetCare Ayurveda formulated for sensitive skin combines the ancient Indian medicinal philosophy of Ayurveda, known as the "Science of Life", with the advanced technology and results of JetPeel.

The JetCare Ayurveda line includes three products – Exfoliation, Infusion and Boost Infusion – enriched with Ayurvedic ingredients, enabling aesthetic practitioners to extend their clinic potential with an alternative, clinically proven treatment for sensitive skin.

JetCare Ayurveda + JetPeel: A Holistic Approach

Originating in India thousands of years ago, Ayurveda is a holistic healing system that promotes balance in the body, helping to restore natural health. Today, it is the primary medicinal school throughout India, and is recognized by the World Health Organization as an effective alternative treatment. Ayurveda uses a selection of herbs and natural supplements to reduce inflammation and detoxify the body. It has gained a popular reputation among the public as a natural and effective healing treatment for a host of health and skin concerns.

Clinically Proven for Sensitive Skin

All the JetPeel benefits, minimal environmental footprint

The JetCare Ayurveda line includes three products to create an Ayurvedic-based JetPeel treatment that delivers smooth, balanced results:

- **JetCare Ayurveda Exfoliation** to cleanse, balance and prep the skin for the infusion stage.
- **JetCare Ayurveda Infusion** to deliver the power of Ayurvedic healing and hydration on and below the skin surface.
- **JetCare Ayurveda Boost Infusion** to amplify the effects of Ayurvedic treatment, reduce pores and restore the skin's natural smooth appeal.

Thanks to the careful selection of gentle, natural herbs known in Ayurveda medicine for their balancing and detoxifying properties, JetCare Ayurveda is ideal and is clinically proven for sensitive skin.



JetCare Ayurveda Formulas: Enriched with Healing Power

Each JetCare Ayurveda solution is enriched with a unique blend of four traditional Ayurvedic ingredients:

- **Bacocalmine:** Soothing herb that fights free radicals and oxidation, and reduces inflammation. Originates in India and Pakistan, and is one of the key anti-stress herbs in the Ayurvedic healing system.
- **Campo Vijaya Sara:** Provides an astringent effect and balances sebum production. Rich in flavonoids that stimulate blood flow and oxygenate the skin, as well as saponins and mucins whose hydrating and softening properties promote a smoother, healthier appearance.
- **Campo Jyotismati:** Traditional revitalizing ingredient, rich in tannins that provide a natural astringent effect, helping to close pores and create a smooth, even tone. Rich in saponins that promote softening action, and mineral salts that help regulate moisture balance and restore the skin's natural elasticity.
- **Campo Susunishak:** Rich in mucins and mineral salts that provide excellent hydrating and moisture-balancing properties. Also contains amino acids that promote skin tightening and toning.

In addition to these Ayurvedic ingredients, each JetCare Ayurveda product contains complementary ingredients to bring the best balancing and healing results for each step of the JetPeel treatment.



JetCare Ayurveda Treatment



STEP
01

LYMPHATIC MASSAGE

Micro-circulation and Detoxifying

Like all JetPeel treatment sessions, the JetPeel's Ayurveda treatment starts with a Lymphatic Massage using one of the JetCare Hydro liquid solutions – JetCare Hydro, JetCare Youth Dead Sea Hydro or JetCare Green Hydro.



STEP
02

EXFOLIATION

Smooth Natural Balance

The Lymphatic Massage is followed with JetCare Ayurveda Exfoliation, enriched with 5% Lactic Acid to enhance the exfoliating effects while remaining gentle enough for sensitive skin.

JetCare Ayurveda Exfoliation 50 ml (Cat. no: JC-AE) Contains:

- Bacocalmine
- Campo Vijaya Sara
- Campo Jyotismati
- Campo Susunishak
- Lactic Acid 5%



STEP
03

INFUSION

Hydrating, Healing, Gentle

JetCare Ayurveda Infusion is enriched with Primalhyal 3k, composed of 3 molecular sizes of Hyaluronic Acid, delivering basic level hydration, and a gentle infusion of Ayurvedic healing and smoothing effects.

JetCare Ayurveda Infusion 50 ml (Cat. no: JC-AI) Contains:

- Bacocalmine
- Campo Vijaya Sara
- Campo Jyotismati
- Campo Susunishak
- Primalhyal 3k



STEP
04

BOOST INFUSION

Amplify the Ayurvedic Impact

Formulated to enhance the JetPeel Ayurvedic treatment, the Boost Infusion is the final stage of the protocol, enriched with 5% Pore Reductyl, a mushroom extract with strong astringent properties to close pores and leave skin even and smooth.







JetCare Ayurveda Boost Infusion 50 ml (Cat. no: JC-ABI) Contains:

- Bacocalmine
- Campo Vijaya Sara
- Campo Jyotismati
- Campo Susunishak
- Pore Reductyl 5%

JetCare Ayurveda Treatment Recommendations

Indications	Anti-Aging Loss of tone and wrinkle Post-Acne Lightening	Ages	20's and upwards
LED add-on	Red / Blue	Skin Types	Sensitive
Frequency	<ul style="list-style-type: none">1st 6 Treatments: 1-2 times every week for 3-6 weeksAfter 6 Treatments: Maintenance treatments every 3-4 weeks	Time of Year	All year round

JetCare Ayurveda Treatment Guideline

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	INFUSION BOOST	STEP 04
 Handpiece	TripleJet		TripleJet		TripleJet		TripleJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	15 minutes		10 minutes		5 minutes		5 minutes	
 Use	18ml of any of the JetCare Hydro products (JetCare Hydro, JetCare Youth Dead Sea Hydro, JetCare Green Hydro)		5ml of JetCare Ayurveda Exfoliation		5ml of JetCare Ayurveda Infusion		5ml of JetCare Ayurveda Boost Infusion	
 Remarks			Leave the JetCare Ayurveda Exfoliation for 5 minutes then wash with 8ml of one of the JetCare Hydro products with TripleJet handpiece at 45° - 60° angle					



Traditional Ayurveda in Advanced Action

Launched in 2021, JetCare liquid solutions are world-renowned for their skin-enhancing effects as part of the JetPeel aesthetic treatment experience. TavTech, the JetCare developers, are continually expanding and updating the product collection to offer new, advanced treatment approaches. Following the launches of JetCare, JetCare Youth and JetCare Green, TavTech now introduces JetCare Ayurveda based on the natural, ancient and proven wisdom of holistic Ayurvedic medicine.

Clinically proven for sensitive skin, the JetCare Ayurveda delivers gentle, effective healing and smooth, glowing results, from the leading provider of advanced jet pressure technology for aesthetic clinicians worldwide.

The JetPeel System: Trio of Excellence

Three powerful components comprise the JetPeel system:

JetPro Devices


The iconic JetPro by TavTech devices generate pressurized air accelerated to manipulate liquid into fine misty jet streams, traveling at speeds of over 200 m/s and pleasantly impacting the skin at a pressure of 95 p.s.i.

The advanced models, JetPro, JetPro Duo by TavTech and the portable JetPro ToGo by TavTech combine the jet technology of JetPeel with Red (620-630 nm) and Blue (460-470 nm) LED's in a simultaneous treatment for enhanced and powerful results.



JetPeel Handpieces

The advanced, patented JetPeel handpieces used to apply the jet streams to the skin are available in several versions, suitable for different treatment types and skin concerns. The jet stream exfoliates the upper epidermal layer (stratum corneum) and then gently stretches the skin surface, opening microchannels that allow for non-invasive delivery of nutritive micro droplets deep into the skin.

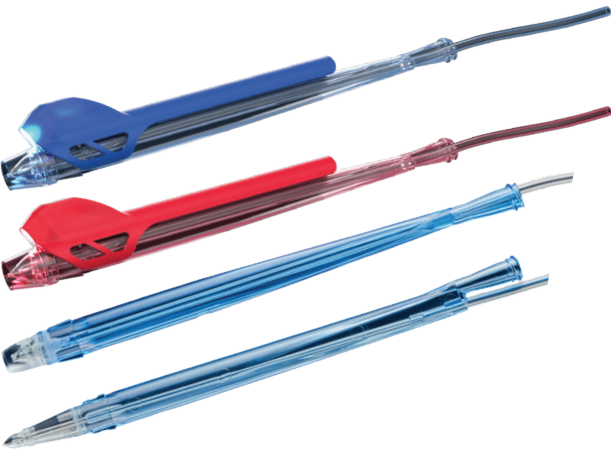

JetCare Liquid Solutions

JetCare solutions are uniquely formulated for a range of skin concerns and needs, including Anti-Aging, Renewal, Soothing, Lightening, Hydrate, Tightening, Glow, Scalp & Hair, as well as JetCare Youth for young skin, JetCare Green exclusively formulated with natural origin ingredients and raw materials, and more.

