

Management of Post-Acne Scarring with JetPeel by TavTech

Prepared by Laura Masini, Clinical and Global Trainer for JetPeel, TavTech

Case Description

Patient Profile: Female, Age 29.

This case demonstrates the clinical efficacy of JetPeel by TavTech for post-acne skin treatment over four sessions. The protocol includes lymphatic massage, exfoliation, and trans-epidermal infusion, targeting improvements in skin texture, scarring, hydration, and elasticity. Clinical observations highlight visible enhancements, demonstrating JetPeel's effectiveness as a non-invasive treatment.

During the initial consultation, the following concerns were presented:

- **Skin Texture:** Uneven with prominent pores and mild post-acne scarring, requiring epidermal support.
- **Redness and Pigmentation:** Generalized erythema and localized hyperpigmentation needing tone correction.
- **Periorbital Area:** Fine lines indicating early aging and dehydration, calling for rejuvenation and hydration.
- **Overall Skin Quality:** Dullness and uneven tone, emphasizing the need for improved hydration, radiance, and uniformity.

Treatment Process

This protocol addresses post-acne skin challenges through targeted, stepwise interventions with 5-day intervals.

STEP 01	Focuses on reducing inflammation and strengthening the skin barrier.	STEP 02	Targets epidermal support, improving texture and minimizing scarring while regulating pigmentation.	STEP 03	Promotes regeneration, supporting repair and enhancing skin vitality.	STEP 04	Refines the skin's surface, addressing residual imperfections, and boosting clarity, resilience, and elasticity.
----------------	--	----------------	---	----------------	---	----------------	--

JetCare by TavTech products used in the JetPeel treatments

	LYMPHATIC MASSAGE STEP 01	EXFOLIATION STEP 02	INFUSION STEP 03	BOOSTER STEP 04
TREATMENT 1	18ml JetCare Hydro	5ml Renewal Care Glycolic Acid 5%	5ml Anti-Aging Wide Range Hyaluronic Acid	3ml Boost Regenerate
TREATMENT 2	18ml JetCare Hydro	5ml Renewal Care Glycolic Acid 10%	5ml Selective Care Lightening	3ml Boost Glow
TREATMENT 3	18ml JetCare Hydro	5ml Renewal Care Glycolic Acid 10%	5ml Selective Care Argireline® Complex	3ml Anti-Aging Wide Range Hyaluronic Acid
TREATMENT 4	18ml JetCare Hydro	5ml Renewal Care Mandelic & Salicylic	5ml Anti-Aging Peptide Complex	3ml Boost Hydrate



To: BEFORE JETPEEL TREATMENTS



T4: AFTER 4 JETPEEL TREATMENT SESSIONS

Summary and Conclusion

The treatment protocol delivered significant improvements in post-acne skin care, including reduced inflammation, enhanced hydration, refined texture, and more uniform skin tone. Skin elasticity, firmness, and radiance were visibly improved.

To further enhance and sustain these results:

- **Extend Protocol:** Add two weekly sessions for additional texture refinement and structural strengthening.
- **Maintenance:** Schedule JetPeel treatments every 3–4 weeks to sustain hydration, circulation, and clarity, preserving long-term skin health and results.