

TAVTECH LTD

T: +972.3.536.0515

E: info@jetpeel.com

40 Haatzmaut St.
Yehud, 5630417
Israel

FOLLOW US:



www.jetpeel.com





JetCare Med by TavTech

Osmolyte enriched for invigorating
collagen and elastin | Re-ignite your
clinic result

Formulated for Clinicians, Designed for Results

JetCare Med by TavTech is an exclusive collection of osmolyte-enriched JetPeel infusion products, designed for dermatologists and aesthetic specialists to offer an advanced collagen and elastin invigorating JetPeel experience and clinically proven results.

The JetCare Med line includes four infusion products and one booster infusion:

- **JetCare Med Refill Infusion** for skin filling
- **JetCare Med Relight Infusion** for lightening & brightening
- **JetCare Med Relax Infusion** for muscle calming
- **JetCare Med Revitalize Infusion** for anti-aging
- **JetCare Med Refine Boost Infusion** for final lift & glow

All JetCare Med products are formulated with a unique and rich concentration of osmolytes. Osmolytes are organic molecules with lower molecular weights, making them effective carriers to protect peptides and improve protein stability, contributing to healthier, energized, and invigorated collagen and elastin.

The infusions are delivered with the MDJet by TavTech handpiece, purpose-designed for JetPeel treatments using JetCare Med serums. The infusion products are pre-packed in 5 ml syringes that quickly and easily attach to the MDJet handpiece for a convenient and seamless treatment flow.


JetCare Med: The Results Speak for Themselves

A clinical study* of JetCare Med and JetPeel treatment demonstrated outstanding results for skin density, firmness and elasticity.

Eight weeks after the first of 3 JetCare Med treatments during an 8 day period, the subjects showed**:

- **26% improvement in skin density**
- **16% improvement in skin firmness**
- **12% improvement in skin elasticity**

Summary of the improvement of skin elasticity, firmness, and density obtained after 3 sessions of the procedures by using JetPeel (D8), after 28 days (D28), and after 56 days (D56) from the first procedure by using JetPeel in %.

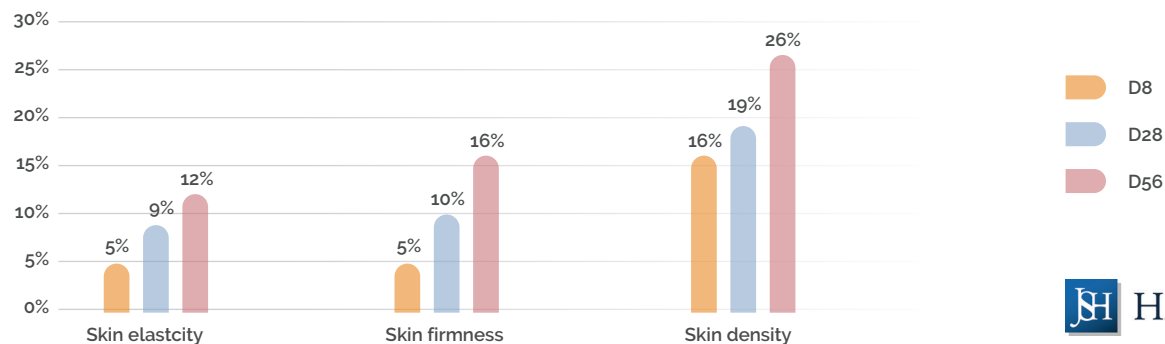

All skin types showed high toleration of the product, attesting to the safety and efficacy of JetCare Med treatment with JetPeel

With JetCare Med, it has never been easier to provide clinically proven, comfortable JetPeel care that leaves patients feeling rejuvenated, glowing, and beautiful.


*Source: JS Hamilton REPORT No. 416345-1/24/GDA/PK

** All findings show statistical significance

Skin Density Improvements

Measured with Dub® Skin Scanner manufactured by TPM Taberna Pro Medicum GmbH

JS HAMILTON REPORT No. 416345-1/24/GDA/PK | JETCARE MED TREATMENT

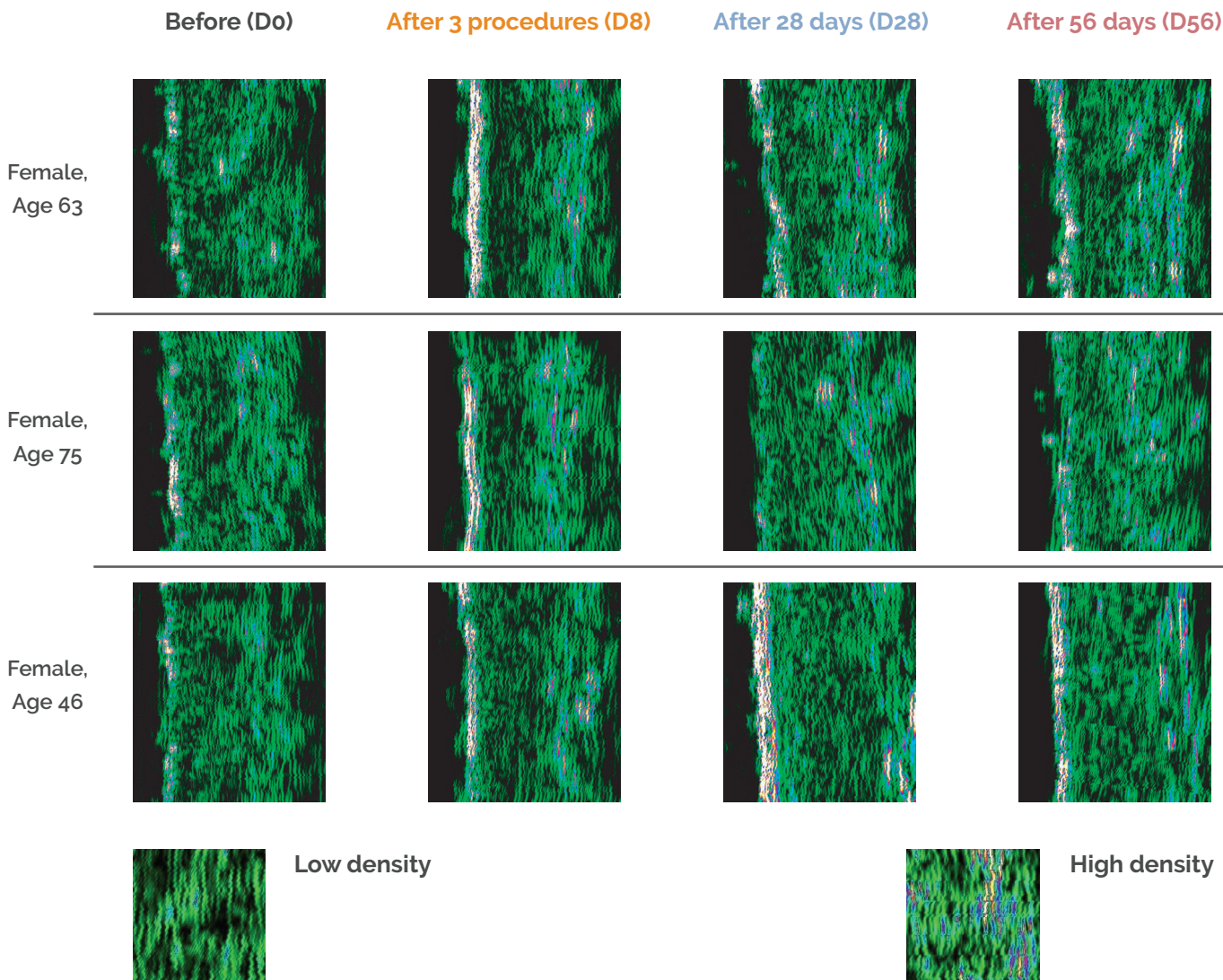

JetPeel + JetCare Med

Power Your Clinic

The JetPeel by TavTech experience consists of three core elements: the JetPro by TavTech pressure energy devices, original JetPeel by TavTech handpieces, and JetCare by TavTech liquid cosmetic solutions, the only products to ensure outstanding JetPeel results.

With JetCare Med, dermatologists and aesthetic specialists can elevate their JetPeel offering with lab-proven, cutting-edge infusion treatments, uniquely suited to the medical clinic setting.

Transform your JetPro potential with JetCare Med infusions and the MDJet handpiece, enhancing the JetPeel experience and results.

MDJet by TavTech Handpiece: Simply Invigorating

MDJet is the exclusive JetPeel handpiece specially designed for use in JetCare Med treatments.

JetCare Med liquid infusions come in 5 ml syringes that connect directly to the MDJet handpiece. For physicians and treatment providers, treatments with MDJet are smooth and easy. Simply attach the infusion syringe by screwing it onto the MDJet handpiece, and quickly replace it with a new one to continue working. JetCare Med is an uninterrupted experience for you and your patients, a more efficient and seamless way to get results.



JetCare Med by TavTech

Power in Concentration

Introducing JetCare Med, the power collection of highly-concentrated infusions for outstanding aesthetic performance in your clinic, and smooth, visible results for your patients.

JetCare Med products are formulated with over 5.5% concentration of osmolytes, low-weight organic molecules that hydrate and protect proteins for advanced collagen and elastin invigoration.

Unlock the skin's natural potential with JetCare Med's synergistic osmolytes blend, including:



Betaine

Extracted from non-GMO sugar beets, highly purified betaine is a natural humectant that helps strengthen the skin barrier, maintain moisture balance, and support long-lasting skin hydration.



Xylitol


A prebiotic moisturizer that enhances the skin's microbiome profile, xylitol helps to fortify the skin barrier and supports powerful hydration.



Inositol

A glucose isomer, inositol is a natural source of energy for the skin. It improves fibroblasts metabolism, helping to improve oxygenation, boost elasticity, and fight signs of fatigue.

JetCare Med infusions and boost infusion also contain propanediol, a 100% bio-based glycol alternative produced sustainably from corn sugar, helping to enhance moisture and soothe the skin.

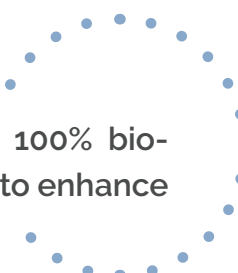

JetCare Med Series

Re-ignite your clinic results with osmolyte-enriched collagen and elastin energizers for clinically proven improved skin elasticity, skin firmness and skin thickness

Refill
+ Refine

Relax
+ Refine



Revitalize
+ Refine

Relight
+ Refine



JetCare Med Refill

JetCare Med Refill is a filler infusion to help smooth fine lines, soothe facial muscles, and invigorates the skin's natural collagen. Give your patients the ultimate alternative filler treatment for skin that appears firm, even and youthful.

In addition to the unique osmolytes blend and propanediol, JetCare Med Refill formula contains:

BioPeptide CL – 5%

Anti-wrinkle tripeptide of the collagen chain that helps to energize and invigorate collagen and reinforce the epidermis.

Argireline® Amplified Peptide Solution C – 5%

Next generation multi-active peptide that helps slow the natural aging process in the epidermal layers, while calming facial muscles for a smoother, even appearance. (Argireline® is a trademark of Lipotec S.A. or its affiliates)

Feminage – 2%

Helps protect skin elasticity and helps maintain the integrity of the dermal-epidermal junction, while soothing inflammatory effects of oxidative stress.

PrimalHyal™ UltraFiller

Patented Hyaluronic Acid derivative enhances penetration of HA, stimulating cell metabolism and fortifying natural moisture levels, improving the appearance of fine lines and expression lines.

PrimalHyal™-3K

Hyaluronic acid blend concentrated in 3 molecular weights, providing anti-aging effects at different levels of the skin.

- 100-300 kDA: Helps boost natural healing processes and the skin's protective defenses in the upper skin layers.
- 20-50 kDA: The ideal size to improve skin functioning and tone the skin for a firmer appearance.
- 5-20 kDA: The smallest HA component penetrates to the deeper skin layers to help rejuvenate from within.



JetCare Med Relight

JetCare Med Relight is a lightening infusion rich in vitamin C to help brighten the complexion and protect against the dulling effects of free radicals and UV damage. With JetCare Med Relight, your patients can experience the sensation of revitalized skin that is lighter, brighter, and beautifully even-toned.

In addition to the unique osmolytes blend and propanediol, JetCare Med Relight formula contains:

Ascorbyl Glucoside AA2G – 5%

Stable vitamin C derivative that lightens the skin, helping to fight free radicals, inhibit melanin synthesis, and enhance collagen production. It also protects against damage from UV radiation, for a powerful brightening and smoothing effect.



JetCare Med Relax

JetCare Med Relax is formulated to calm the muscles, helping to create a smooth, clear skin texture and appearance. Enriched with calming hydrating ingredients and age-defying peptides, JetCare Med Relax is the ideal infusion treatment to restore the relaxed look of youth.

In addition to the unique osmolytes blend and propanediol, JetCare Med Relax formula contains:

Argireline® Amplified Peptide Solution C – 10%

Next-generation multi-active peptide that helps slow the natural aging process in the epidermal layers while calming facial muscles for a smoother, even appearance.

(Argireline® is a trademark of Lipotec S.A. or its affiliates)

Neutrazen™ – 2.5%

Advanced neurocosmetic biomimetic lipopeptide, based on three amino acids that deliver vital calming action, formulated to soothe irritated skin and prevent inflammation.

Allantoin

Comfrey root extract helps hydrate, exfoliate, and smooth the skin surface to restore the skin's natural glow.

Dipotassium Glycyrrhizate

Anti-inflammatory ingredient extracted from liquorice root that helps to regulate sebum and soothe and calming the skin.



JetCare Med Revitalize

JetCare Med Revitalize is a peptide-rich anti-aging treatment, specially formulated for mature, fragile skin. The combination of wrinkle-smoothing and regenerating ingredients delivers age-defying effects for a powerful sensation of rejuvenation and restored youth.

In addition to the unique osmolytes blend and propanediol, JetCare Med Revitalize formula contains:

Matrixil® 3000 – 10%

Powerful double-peptide blend for anti-wrinkle action, helping to boost tone and elasticity. Rich in matikines that promote effective skin repair and regeneration.



JetCare Med Refine

For the final treatment step, JetCare Med Refine infusion boost provides a powerful lift and glow effect. Based on a power blend of advanced flower-based ingredients, vitamins, and JetCare Med's signature cocktail of osmolytes, JetCare Med Refine instantly refreshes the skin, enhancing the radiance impact of JetPeel treatment.

In addition to the unique osmolytes blend and propanediol, JetCare Med Refine formula contains:

Detoxi-Look® – 3%

Hydrating complex based on Everlasting and Daisy flowers, rich in flavonoids, polyphenols, and saponins, that *detoxifies* the enzymatic system and regulates microcirculation, reducing redness and boosting the skin's natural radiance.

Keratoline C

Enzymatic protease-based exfoliant that helps to remove dead skin cells, promoting cell renewal and leaving the skin surface smooth and refreshed.






D-Panthenol – 1.8%

Deeply nourishing vitamin B5 helps to lock in moisture, strengthen the skin's natural barrier, and protect against exposure and aging, for glowing hydrated skin.








JetCare Med Treatment Recommendations and Guidelines

Anti-Aging Treatment for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					






JetCare Med Treatment Recommendations and Guidelines

Loss of Tone and Wrinkles Treatment for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					






JetCare Med Treatment Recommendations and Guidelines

Post-Acne Treatments for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					

JetCare Med Treatment Recommendations and Guidelines

Lightening and Pigmentation Treatments for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					



Innovating Aesthetics

Enhancing Clinics Worldwide

Launched in 2021, JetCare liquid solutions are world-renowned for their skin-enhancing effects as part of the JetPeel aesthetic treatment experience. TavTech, the JetCare developers, are continually expanding and updating the product collection to offer new, advanced treatment approaches.

Following the launches of JetCare, JetCare Youth, JetCare Green, and JetCare Ayurveda, TavTech now introduces JetCare Med and the MDJet handpiece: a new approach to JetPeel treatment that is clinically proven to deliver outstanding results. Based on advanced cosmeceutical power, all JetCare collections are designed to suit the demands of dermatology and aesthetic clinics and support the delivery of exceptionally smooth and glowing results.

The Skin Rejuvenation Revolution

Based on over 20 years of research & development and clinical experience, TavTech's unique Jet technology is available in over 50 countries worldwide under three sought-after brands: **JetPeel** aesthetic skin rejuvenation, **JetCare** prestige cosmetic liquid solutions & **JetOx** medical wound care.

JetPeel Trio – Amplify the results

The JetPeel jet pressure energy system delivers trans-epidermal infusion of a rich variety of nutrients and solutions for the treatment of an array of skincare and aesthetic conditions. The treatment includes Lymphatic Massage, Exfoliation, and trans-epidermal Infusion.

The JetPeel system is made up of three powerful elements:

1. JetPro Devices

JetPro uses advanced jet propulsion technology harnessed to provide trans-epidermal treatment in a compact, easy-to-use device. The JetPro devices create high-pressure, high-velocity fine liquid jet streams applied to the skin with the patented JetPeel handpiece. The JetPro Duo and Pro ToGo models include LED add-ons for simultaneous Jet-LED treatment.

2. JetCare Solutions

The line of unique nutrient-rich cosmetic solutions developed to treat a wide variety of skin concerns.

3. JetPeel Handpieces

The unique patent protected handpieces feature various nozzle varieties to deliver the right JetPeel effect for the skin's needs. The pressurized jet streams gently stretch the skin into concave regions, opening trans-epidermal microchannels allowing nourishing JetCare solutions to penetrate deeper, amplifying their beneficial effects.

Every year, millions of people worldwide enjoy the benefits of JetPeel

JetPeel is a safe, non-invasive, and needle-free treatment, offering a painless, pleasant, and non-contact treatment. Suitable for all skin types, all year round, JetPeel delivers immediate results with no downtime or recovery needed so people everywhere can enjoy the instant rejuvenating glow that only JetPeel can provide.

